

Maxi Air



Transfer

1

Apply the brakes (A) on the bed.

2

Position caregiver on each side of the patient. Place Maxi Air mattress under the patient (labels upwards).

3

Center the patient (A) on the mattress. Place the patients head 6" (150mm) (B) from the top. Make sure the patient does not reach outside of the boundaries (C).

4

Attach the safety belts loosely.

5

Adjust the receiving surface to an ergonomic height. Apply the brakes on both beds.

6

Raise and lock the bed rail (A) on the receiving surface. Adjust the sending surface height slightly higher than receiving surface.

7

Hang the air supply on the receiving bed then plug in the power cord. Connect the air hose to the mattress.

8

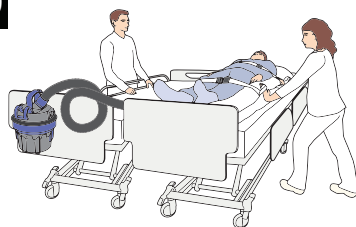
Sending caregiver: firmly hold the transfer handles during the inflation of the mattress.

Receiving caregiver: turn on the air supply and inflate the mattress.

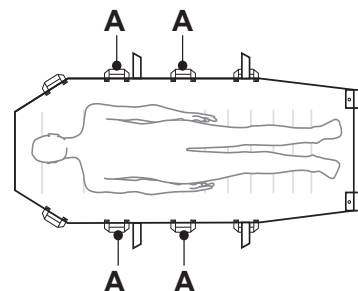
continue with the steps on the next page

9

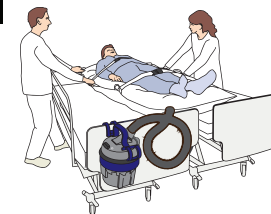
Make sure that the patient can't be felt by the caregiver when pushing under the mattress.

10

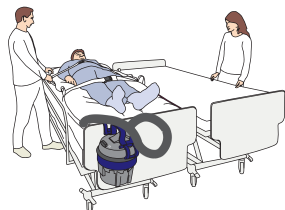
Sending caregiver: tighten the safety belts and push the mattress firmly towards the receiving surface.

11**NOTE**

Use the handles (A) that are closest to the widest part of the patient.

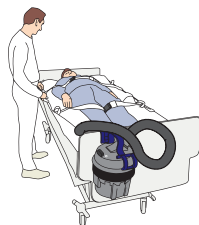
12

Receiving caregiver: meet up the mattress when it is halfway over the receiving surface. Make sure the mattress is centered.

13

Sending caregiver: turn off the air supply and detach the hose from the mattress.

Receiving caregiver: firmly hold the handles until the mattress is completely deflated.

14

After the transfer is completed:

- unbuckle the safety belts
- lock side rails (if possible)
- remove and disinfect the air supply.

