

# Maxi Sky 2 PLUS



## WARNING



To avoid eye and skin damage, always use protective glasses and gloves. If contact occurs rinse with plenty of water. If eyes or skin becomes irritated, contact a physician. Always read the instructions for use and material safety data sheet of the disinfectant.

Allowed disinfectant agents	Quaternary Ammonium ≤ 0.21% (2,100 ppm)	Isopropyl Alcohol ≤ 70%	Phenolic ≤ 1.56% (15,600 ppm)	Bleach ≤ 1% (10,000 ppm)	Peracetic Acid ≤ 0.25% (2,500 ppm)	Hydrogen Peroxide ≤ 1.5%
Standard model (woven nylon strap & Spreader Bar with padding)	X	X	X			
Infection Control model (coated strap & Spreader Bar without padding) *	X	X	X	X	X	X
Other Accessories **	X	X	X	X	X	X

\* Some parts on the spreader bar may discolour after repeated contact with bleach, peracetic acid or hydrogen peroxide.

\*\* Handset attachment cable may become slightly grimy after repeated contact with bleach or hydrogen peroxide.

Clean and disinfect before new patients.

Accessories needed:

- Protective glasses and gloves
- Disposable cloth

## Clean

1. Use a cloth soaked with water to remove excessive residues from the lift and its accessories from top to bottom.
2. Use a damp cloth with warm water mixed with a cleaning fluid.
3. Scrub the equipment from top to bottom to remove any deposits.
4. Use a clean damp cloth with clear water to wipe off all spots of cleaning fluids.
5. Then wipe clean with a dry cloth.

Areas listed are most likely to contain dirt and microbes, pay attention to:

- Edge of hand control
- Back and handle of hand control
- Joint between spreader bar and strap
- Frame and cover of spreader bar
- Emergency cord
- Strap

## Disinfect

6. Wipe the Maxi Sky 2 Plus and accessories using a solution containing one of the compatible disinfectants.
7. Disinfection methods should comply with the local or national guidelines, depending on the Healthcare Facility or country of use. If uncertain, consult local Infection Control specialist.

