



TECHNICAL REFERENCE GUIDE

Slings and slide sheets

Page intentionally left blank

Content

SLINGS

Introduction	9
Finding Arjo slings online	10
Sling fabric options	11
Pictograms definition	14

Seated transfer slings

Seated transfer slings	17
Clip sling – unpadded legs	18
Clip sling – padded legs	18
Clip sling – padded legs and extended legs	19
Clip bathing sling – unpadded legs	19
Clip bathing sling – padded legs	20
Clip bathing sling – unpadded and extended legs	20
Clip in situ sling – unpadded legs	21
Clip bathing in situ sling – unpadded legs	21
Clip comfort sling – padded legs	22
Clip toilet sling – padded legs	22
Loop sling – padded legs	23
Loop sling – padded legs, without head support	23
Loop bathing sling – unpadded legs	24
Loop in situ sling – padded legs	24
Loop toilet sling – padded legs	25
Loop toilet sling – padded legs, without head support	25
Loop hammock sling – 6 straps	26
Loop hammock sling – 4 straps	26
Loop quick fit sling – without head support	27
Disposable clip sling	27
Disposable loop sling	28
Bariatric loop comfort sling	28
Bariatric loop sling	29
Bariatric loop hammock sling - divided legs	29
Bariatric loop hammock sling	30
Disposable bariatric loop sling	30
Diposable bariatric loop sling	30
Amputee clip sling	31
Amputee clip sling - "hook & loop" fastener	31
Amputee right leg clip sling	32
Amputee left leg clip sling	32
Amputee clip bathing sling	33
Amputee loop hammock sling	33
Amputee clip in situ sling	34
Amputee clip bathing in situ sling	34
Amputee clip toilet sling	35
Paediatric clip sling	35

Standing and walking slings

Slings for standing and raising aids	39
Ambulation slings for ceiling and passive floor lifts	39
Sara Flex standing sling	40
Sara Flex standing sling – separate belt with one buckle	40
Sara Flex seated transfer sling	41
Sara clip standing sling – flat	41
Sara Flex disposable standing sling	42
Sara Flex wipeable standing sling	42
Sara Plus standing sling	43
Sara Plus standing sling – extra postural support	43
Sara Plus disposable standing sling	44
Sara Plus wipeable standing sling	44
Sara Plus walking sling	45
Loop walking sling	45
Loop walking jacket	46

In bed and lateral transfer slings

In bed and lateral transfer slings	49
Loop stretcher sling – commode aperture	50
Loop stretcher sling	50
Loop bathing stretcher sling – commode aperture	51
Disposable loop stretcher sling	51
Strap stretcher	52
Loop repositioning sling	52
Loop comfort repositioning sling	53
Disposable loop repositioning sling	53
Bariatric loop repositioning sling	54
Disposable bariatric loop repositioning sling	54
Limb Clip Sling	55
Loop limb sling	55
Bariatric loop limb sling	56
Loop turning sling	56
Disposable clip limb sling	57
Disposable loop limb sling	57
Disposable loop turning sling	58

SLIDE SHEETS

Arjo slide sheets	63
Arjo slide tubes	65
Arjo slide sheet kits	67
Arjo disposable slide sheets & tubes	69

Design Policy and Copyright

® and ™ are trademarks belonging to the Arjo group of companies.
© Arjo - January 2024.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice.

The content of this publication may not be copied either whole or in part without the consent of Arjo.



Slings



Introduction

The Technical Reference Guide for slings contains a full overview of the Arjo sling range. Providing the right sling for the individual patient type and need, is vital, to promote optimal safety and comfort.

You will find descriptions of each product accompanied by images and specifications. Whether you're working at a hospital or in a care home, our slings are designed to meet the highest standards of quality and performance. Thank you for choosing Arjo.

Finding Arjo slings online

Arjo.com


In the slings section on arjo.com you will find similar information to what is available in this document with detailed descriptions of each product, images and specifications. The Instructions for Use and marketing material are also available to download.

At the start page, there are several filtering options to support in locating a specific sling or group of slings in an easy and intuitive way. These are:

- Lift Systems - Standing and raising aids or ceiling and passive floor lifts
- Compatible Lifts - Maxi Sky, Maxi Move etc
- Attachment options - Clip or loop
- Patient characteristics - General, bariatric etc
- Care activity - Toileting, bathing, lateral transfer etc
- Fabric options - Mesh, disposable etc
- Extended application - Floor recovery, prone positioning etc

Search engines

If you know the name or article number of a specific Arjo sling and want to find more information, simply use the name or number as search terms. In most cases you will find the sling in the top of your search result with a direct link to the specific product page on arjo.com

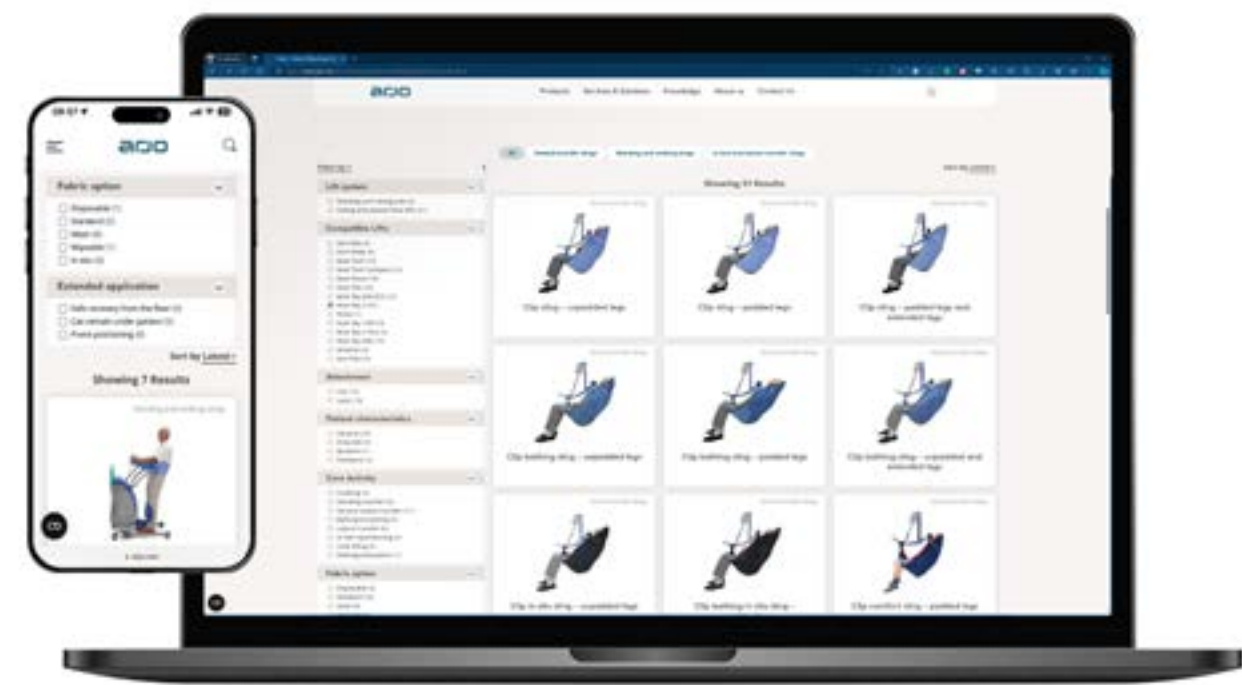

Sling fabric options



Launderable

- Great durability and large selection of sling models.
- Full cleaning recommendations and instructions can be found on the sling label and within the Instructions for Use.



Disposable for single patient use

- To be disposed of if soiled, or once the patient stay is over.
- Meets infection control requirements.
- A trouble-free solution as no laundering is required.



Wipeable

- Can be wiped clean and disinfected between patients.
- Minimises downtime as immediately available after wiping for use between patients.

LAUNDERABLE




Knitted polyester fabric

- General purpose fabric.
- Durable and strong material for everyday transfers.

white

grey

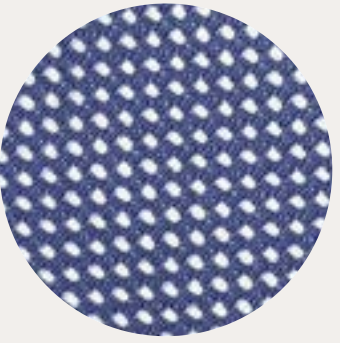


Knitted polyester fabric – mesh

- Durable and strong material for everyday transfers.
- Ideal choice for showering or bathing; the mesh allows optimal water dispersal.

white

grey



Premium knitted polyester fabric

- Softer fabric for extra comfort.
- Durable and strong material for everyday transfers.



High strength woven polycotton fabric

- Available with bariatric repositioning slings.



Woven polyester fabric (in situ)

- Suitable to remain under patients for a period of time between transfers*.
- Breathable
- Dissipates heat
- Drapable
- Antistatic


pale blue



Lightweight knitted polyester fabric – mesh (in situ)

- Softer fabric for extra comfort.
- Durable and strong material for everyday transfers.
- Suitable to remain under patients for a period of time between transfers*.

DISPOSABLE

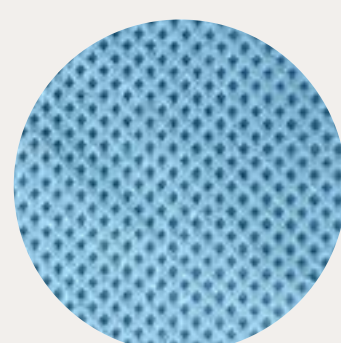

Non-woven polypropylene fabric

- Disposable single patient use option limits the spread of infection, improving infection control.




Woven polycotton fabric

- Disposable single patient use option limits the spread of infection, improving infection control.



High strength woven polycotton fabric

- Disposable single patient use option limits the spread of infection, improving infection control.
- For bariatric use.

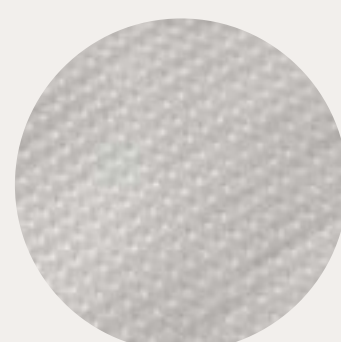

Stitch bond non-woven polypropylene fabric

- Disposable single patient use option limits the spread of infection, improving infection control.
- Reinforced non-woven suitable for slings with higher Safe Working Load (SWL)

WIPEABLE




Polyurethane coated fabric


- Designed to be wiped clean between patients or residents to prevent cross-contamination.


*An assessment by a caregiver should always be done before leaving a sling under a patient between transfers.

Pictograms definition

Morphology


 General population


 Amputee


 Paediatric


 Bariatric


Cleaning method

 Not washable

 Washable Max 70°C

 Washable Max 80°C

 Do not tumble dry

 Tumble dry at low temperature

Seated transfer slings





Seated transfer slings

A wide range of slings for every patient type and need. For use with passive lifting devices to transfer patients in a seated position to and from bed, chair, bathroom, wheelchair, and floor.

Attachment methods

- Clip
- Loop

Materials available

- Knitted polyester fabric (cache blue & white)
- Knitted polyester mesh fabric (cache blue, grey and white)
- Premium woven polyester fabric (black)
- Lightweight woven polyester fabric
- Non-woven polypropylene fabric

Morphologies

- Paediatric
- General population
- Amputee
- Bariatric

Options

- With or without head support
- Padded legs, unpadded legs or extended legs

Main activities

- Transfer and lift patients in seated position
- Toileting
- Bathing/showering
- Repositioning
- Lifting a patient from the floor

CLIP SLING – UNPADDED LEGS

The Clip sling – unpadded legs is suitable for everyday patient handling activities, such as bed and chair-related transfers.

PRODUCT INFORMATION	
Part number	MAA2000M
Safe working load	272 kg (600 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester knitted fabric

COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi MoveMaxi TwinMaxi Twin compactMaxi 500	<ul style="list-style-type: none">Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



CLIP SLING – PADDED LEGS

The Clip sling – padded legs is used for a range of everyday patient handling activities, such as bed and chair-related transfers. It has additional padding around the leg pieces to provide extra comfort.

PRODUCT INFORMATION	
Part number	MAA4000M
Safe working load	272 kg (600 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester knitted fabric

COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi MoveMaxi TwinMaxi Twin compactMaxi 500	<ul style="list-style-type: none">Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



CLIP SLING – PADDED LEGS AND EXTENDED LEGS

The Clip sling – padded legs and extended legs is used for a range of everyday patient handling activities, such as bed and chair-related transfers. It has additional padding around the leg pieces to provide extra comfort. The extended leg sling is suitable when a larger circumference is needed to reach around the thighs.

PRODUCT INFORMATION	
Part number	MAA4100M
Safe working load	272 kg (600 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester knitted fabric

COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi MoveMaxi TwinMaxi Twin compactMaxi 500	<ul style="list-style-type: none">Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



CLIP BATHING SLING – UNPADDED LEGS

The Clip bathing sling – unpadded legs is suitable for bathing and showering activities as the mesh material allows water to pass easily through the sling.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA4060M
Safe working load	272 kg (600 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Knitted polyester fabric – mesh

COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi MoveMaxi TwinMaxi Twin compactMaxi 500	<ul style="list-style-type: none">Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



CLIP BATHING SLING – PADDED LEGS

The Clip bathing sling – padded legs is suitable for bathing and showering as the mesh material allows water to pass easily through the sling. It has additional padding around the legs area to provide extra comfort.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA4061M
Safe working load	272 kg (600 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Knitted polyester fabric – mesh
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>• Maxi Move</div><div>• Maxi Twin</div><div>• Maxi Twin compact</div><div>• Maxi 500</div></div>	<div><div>• Maxi Sky 2 Series</div><div>• Maxi Sky 2 Plus (Single mode)</div></div>



CLIP BATHING SLING – UNPADDED AND EXTENDED LEGS

The Clip bathing sling – unpadded and extended legs is suitable for bathing and showering as the mesh material allows water to pass easily through the sling. The extended leg sling has been designed to create a longer reach around the thighs if needed.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA4160M
Safe working load	272 kg (600 lbs)
Size ⁽¹⁾	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Knitted polyester fabric – mesh
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>• Maxi Move</div><div>• Maxi Twin</div><div>• Maxi Twin compact</div><div>• Maxi 500</div></div>	<div><div>• Maxi Sky 2 Series</div><div>• Maxi Sky 2 Plus (Single mode)</div></div>



CLIP IN SITU SLING – UNPADDED LEGS

The Clip in situ sling – unpadded legs is suitable for everyday patient handling activities, such as bed and chair-related transfers. It may be left under the patient between transfers (if clinical assessment allows).

PRODUCT INFORMATION	
Part number	MAA2040M
Safe working load	190 kg (418 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester woven fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>• Maxi Move</div><div>• Maxi Twin</div><div>• Maxi Twin compact</div><div>• Maxi 500</div></div>	<div><div>• Maxi Sky 2 Series</div><div>• Maxi Sky 2 Plus (Single mode)</div></div>



CLIP BATHING IN SITU SLING – UNPADDED LEGS

The Clip bathing in situ sling – unpadded legs is suitable for bathing and showering activities as the mesh material allows water to pass easily through the sling.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA2070M
Safe working load	190 kg (418 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Lightweight knitted polyester fabric – mesh
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>• Maxi Move</div><div>• Maxi Twin</div><div>• Maxi Twin compact</div><div>• Maxi 500</div></div>	<div><div>• Maxi Sky 2 Series</div><div>• Maxi Sky 2 Plus (Single mode)</div></div>



CLIP COMFORT SLING – PADDED LEGS

The Clip comfort sling – padded legs is suitable for everyday patient handling activities, such as bed and chair-related transfers. It offers additional comfort throughout the sling, with additional padding for the legs and has no binding in the body of the sling.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA4050M
Safe working load	228 kg (500 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Premium knitted polyester fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>Maxi Move</div><div>Maxi Twin</div><div>Maxi Twin compact</div><div>Maxi 500</div></div>	<div><div>Maxi Sky 2 Series</div><div>Maxi Sky 2 Plus (Single mode)</div></div>



CLIP TOILET SLING – PADDED LEGS

The Clip toilet sling – padded legs is a special-purpose design intended for toileting, dressing and undressing patients/residents.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA4031M
Safe working load	272 kg (600 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>Maxi Move</div><div>Maxi Twin</div><div>Maxi Twin compact</div><div>Maxi 500</div></div>	<div><div>Maxi Sky 2 Series</div><div>Maxi Sky 2 Plus (Single mode)</div></div>



LOOP SLING – PADDED LEGS

The Loop sling – padded legs is suitable for everyday patient handling activities, such as bed and chair-related transfers. It has additional padding around the leg pieces to provide extra comfort.

PRODUCT INFORMATION	
ARTICLE NUMBER	MLA2000
Safe working load	272 kg (600 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>Maxi 500</div><div>Maxi Twin</div><div>Maxi Move</div><div>Minstrel</div></div>	<div><div>Maxi Sky 440</div><div>Maxi Sky 2 Series</div><div>Maxi Sky 2 Plus (Single mode)</div></div>



LOOP SLING – PADDED LEGS, WITHOUT HEAD SUPPORT

The loop sling – padded legs, without head support offers an alternative low-back design where the patient's arms can remain outside of the sling and head support is not required.

PRODUCT INFORMATION	
ARTICLE NUMBER	MLA3000
Safe working load	272 kg (600 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>Maxi 500</div><div>Maxi Twin</div><div>Maxi Move</div></div>	<div><div>Maxi Sky 440</div><div>Maxi Sky 2 Series</div><div>Maxi Sky 2 Plus (Single mode)</div></div>



LOOP BATHING SLING – UNPADDED LEGS

The Loop bathing sling – unpadded legs is suitable for bathing and showering activities as the mesh material allows water to pass easily through the sling.

PRODUCT INFORMATION	
ARTICLE NUMBER	MLA4060
Safe working load	272 kg (600 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Knitted polyester fabric – mesh

COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi 500Maxi TwinMaxi MoveMinstrel	<ul style="list-style-type: none">Maxi Sky 440Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



LOOP TOILET SLING – PADDED LEGS

The Loop toilet sling – padded legs is a special-purpose design intended for toileting, dressing and undressing patients.

PRODUCT INFORMATION	
ARTICLE NUMBER	MLA4031
Safe working load	272 kg (600 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester knitted fabric

COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi 500Maxi TwinMaxi MoveMinstrel	<ul style="list-style-type: none">Maxi Sky 440Maxi Sky 600Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



LOOP IN SITU SLING – PADDED LEGS

The Loop in situ sling – padded legs is suitable for everyday patient handling activities, such as bed and chair-related transfers. It may be left under the patient between transfers (if clinical assessment allows).

PRODUCT INFORMATION	
ARTICLE NUMBER	MLAAS2000
Safe working load	190 kg (418 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Woven polyester fabric

COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi 500Maxi TwinMaxi MoveMinstrel	<ul style="list-style-type: none">Maxi Sky 440Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



LOOP TOILET SLING – PADDED LEGS, WITHOUT HEAD SUPPORT

The Loop toilet sling – padded legs, without head support is a special-purpose design intended for toileting, dressing and undressing patients. The low-back design makes the sling easy to apply and remove. It is suitable for patients and residents who can support their head themselves during transfer.

PRODUCT INFORMATION	
ARTICLE NUMBER	MLA4531
Safe working load	272 kg (600 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester knitted fabric

COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi 500Maxi TwinMaxi MoveMinstrel	<ul style="list-style-type: none">Maxi Sky 440Maxi Sky 600Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



LOOP HAMMOCK SLING – 6 STRAPS

The Loop hammock sling – 6 straps is a total lift sling that provides full head and neck support and double thigh padding for comfort. The additional straps provide more support for patient/resident. This sling is made of a quick drying mesh fabric that makes it ideal for bathing as well as general transfers.

PRODUCT INFORMATION	
ARTICLE NUMBER	THA6i
Safe working load	272 kg (600 lbs)
Size	XXS XS S M L XL
Morphology	
Cleaning method	
Material	Knitted polyester fabric – mesh
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi MoveMaxi 500	<ul style="list-style-type: none">Maxi 440Maxi Sky 2Maxi Sky 1000Maxi Sky 2 Plus



LOOP QUICK FIT SLING – WITHOUT HEAD SUPPORT

The Loop quick fit sling – without head support is a general purpose total lift sling. It is used to lift residents/patients from the bed, wheelchair and off the floor. This sling can be used for residents/patients with limited upper body function, but with good head control as head support is not provided by the sling.

PRODUCT INFORMATION	
ARTICLE NUMBER	TIR
Safe working load	272 kg (600 lbs)
Size	XXS S M L XL XXL
Morphology	
Cleaning method	
Material	Knitted polyester fabric – mesh
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi 500Maxi TwinMaxi Move	<ul style="list-style-type: none">Maxi Sky 440Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



LOOP HAMMOCK SLING – 4 STRAPS

The Loop hammock sling – 4 straps is a total lift sling that provides full head and neck support and double thigh padding for comfort. This sling is made of a quick drying mesh fabric that makes it ideal for bathing as well as general transfers.

PRODUCT INFORMATION	
ARTICLE NUMBER	THAI
Safe working load	272 kg (600 lbs)
Size	XXS S M L XL XXL
Morphology	
Cleaning method	
Material	Knitted polyester fabric – mesh
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi 500Maxi TwinMaxi Move	<ul style="list-style-type: none">Maxi Sky 440Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



DISPOSABLE CLIP SLING

The Disposable clip sling is used for a range of everyday patient handling activities, such as bed and chair-related transfers. It is designed to accompany a patient throughout the hospital stay and can be used in exactly the same way as standard clip slings but it cannot be laundered.

PRODUCT INFORMATION	
ARTICLE NUMBER	MFA1000M
Safe working load	272 kg (600 lbs)
Size	XXS S M L LL XL XXL
Supplied in packs of 10 slings	
Morphology	
Cleaning method	Disposable for single patient use
Material	Polypropylene Non-woven fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi MoveMaxi TwinMaxi Twin compactMaxi 500	<ul style="list-style-type: none">Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



DISPOSABLE LOOP SLING

The Disposable loop sling is designed to support infection prevention, streamline safe patient handling tasks, and optimise patient comfort. It is always ready to use with no reprocessing requirements.

PRODUCT INFORMATION	
ARTICLE NUMBER	MFA2100
Safe working load	272 kg (600 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>XL</div><div>XXL</div></div>
Supplied in packs of 10 slings	
Morphology	<div><div>Person icon</div><div>Person icon</div><div>Person icon</div><div>Person icon</div></div>
Cleaning method	<div><div>Washing icon</div> Disposable for single patient use</div>
Material	Polypropylene Non-woven fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>• Maxi 500</div><div>• Maxi Twin</div><div>• Maxi Move</div><div>• Minstrel</div></div>	<div><div>• Maxi Sky 440</div><div>• Maxi Sky 600</div><div>• Maxi Sky 2 Series</div><div>• Maxi Sky 2 Plus</div></div>



BARIATRIC LOOP SLING

The Bariatric loop sling is suitable for assisted transfers of plus-size patients and residents that require full body support during the transfer. It has standard padding in the legs that enhance comfort for patients and residents.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA8010
Safe working load	454 kg (1000 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div>Person icon</div><div>Person icon</div><div>Person icon</div><div>Person icon</div></div>
Cleaning method	<div><div>Washing icon</div><div>Washing icon</div></div>
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>• Tenor</div></div>	<div><div>• Maxi Sky 1000</div><div>• Maxi Sky 2 Plus</div></div>



BARIATRIC LOOP COMFORT SLING

The Bariatric loop comfort sling is suitable for assisted transfers of plus-size patients and residents that require full body support during the transfer. It is designed with extra padding in the most stressed areas to enhance patient comfort.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA8000
Safe working load	454 kg (1000 lbs)
Size ⁽¹⁾	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div>Person icon</div><div>Person icon</div><div>Person icon</div><div>Person icon</div></div>
Cleaning method	<div><div>Washing icon</div><div>Washing icon</div></div>
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>• Tenor</div></div>	<div><div>• Maxi Sky 1000</div><div>• Maxi Sky 2 Plus</div></div>



BARIATRIC LOOP HAMMOCK SLING - DIVIDED LEGS

The Bariatric loop hammock sling - divided legs is suitable for assisted transfers of plus-size patients and residents that require full body support during the transfer. It is designed with additional support around the leg area - each leg is supported individually to prevent abduction and provides more space around the abdomen area.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA8020
Safe working load	454 kg (1000 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div>Person icon</div><div>Person icon</div><div>Person icon</div><div>Person icon</div></div>
Cleaning method	<div><div>Washing icon</div><div>Washing icon</div></div>
Material	Polyester Mesh Cache Blue
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>• Tenor</div></div>	<div><div>• Maxi Sky 1000</div><div>• Maxi Sky 2 Plus</div></div>



BARIATRIC LOOP HAMMOCK SLING

The Bariatric loop hammock sling is suitable for assisted transfers of plus-size patients and residents that require full body support during the transfer. It is designed for transfers when the patient is semi-reclined and does not tolerate leg straps positioned between the legs.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA8030
Safe working load	454 kg (1000 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester Mesh Cache Blue
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>Tenor</div></div>	<div><div>Maxi Sky 1000</div><div>Maxi Sky 2 Plus</div></div>



DISPOSABLE BARIATRIC LOOP SLING

The Disposable bariatric loop sling is suitable for assisted transfers of plus-size patients and residents that require full body support during the transfer. It is designed to accompany a patient throughout the hospital stay and can be used in exactly the same way as standard bariatric slings but it cannot be laundered.

PRODUCT INFORMATION	
ARTICLE NUMBER	MFA8010
Safe working load	454 kg (1000 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div>Disposable for single patient use</div></div>
Material	Stitch bond nonwoven fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>Tenor</div></div>	<div><div>Maxi Sky 1000</div><div>Maxi Sky 2 Plus</div></div>



AMPUTEE CLIP SLING

The Amputee clip sling is suitable for residents/patients with bilateral leg amputations above the knee or total hip.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA2090M
Safe working load	272 kg (600 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>Maxi Move</div><div>Maxi Twin</div><div>Maxi Twin compact</div><div>Maxi 500</div></div>	<div><div>Maxi Sky 600</div><div>Maxi Sky 2 Series</div><div>Maxi Sky 2 Plus (Single mode)</div></div>



AMPUTEE CLIP SLING - "HOOK & LOOP" FASTENER

The Amputee clip sling - "hook & loop" fastener is suitable for residents/patients with leg amputations above or below the knee. This sling is applied like a standard sling, without turning the patient onto their side.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA4040M
Safe working load	272 kg (600 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>Maxi Move</div><div>Maxi Twin</div><div>Maxi Twin compact</div><div>Maxi 500</div></div>	<div><div>Maxi Sky 600</div><div>Maxi Sky 2 Series</div><div>Maxi Sky 2 Plus (Single mode)</div></div>



AMPUTEE RIGHT LEG CLIP SLING

The Amputee right leg clip sling is designed for residents/ patients with right leg amputations above or below the knee or total hip. This sling is shaped to give additional support around the amputation site.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA4070M
Safe working load	272 kg (600 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi MoveMaxi TwinMaxi Twin compactMaxi 500	<ul style="list-style-type: none">Maxi Sky 600Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



AMPUTEE LEFT LEG CLIP SLING

The Amputee left leg clip sling is designed for residents/ patients with left leg amputations above or below the knee or total hip. This sling is shaped to give additional support around the amputation site.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA4080M
Safe working load	272 kg (600 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi MoveMaxi TwinMaxi Twin compactMaxi 500	<ul style="list-style-type: none">Maxi Sky 600Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



AMPUTEE CLIP BATHING SLING

The Amputee clip bathing sling has the same design as the amputee clip sling but with mesh fabric. It is suitable for bathing and showering residents/patients with bilateral leg amputations above the knee or total hip.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA7090M
Safe working load	272 kg (600 lbs)
Size ⁽¹⁾	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Knitted polyester fabric - mesh
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi MoveMaxi TwinMaxi Twin compactMaxi 500	<ul style="list-style-type: none">Maxi Sky 600Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



AMPUTEE LOOP HAMMOCK SLING

The Amputee loop hammock sling is intended for lifting single or bilateral amputee patients/residents supporting each leg individually during transfer.

PRODUCT INFORMATION	
ARTICLE NUMBER	MLA7000
Safe working load	272 kg (600 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi MoveMaxi TwinMaxi 500Minstrel	<ul style="list-style-type: none">Maxi Sky 440Maxi Sky 600Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



AMPUTEE CLIP IN SITU SLING

The Amputee clip in situ sling is suitable for residents/ patients with bilateral leg amputations above the knee or total hip. It is designed with the ability to remain under the patient between transfers following clinical assessment.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA2050M
Safe working load	190 kg (418 lbs)
Size	<div><div>XXS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester woven fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>• Maxi Move</div><div>• Maxi Twin</div><div>• Maxi Twin compact</div><div>• Maxi 500</div></div>	<div><div>• Maxi Sky 600</div><div>• Maxi Sky 2 Series</div><div>• Maxi Sky 2 Plus (Single mode)</div></div>



AMPUTEE CLIP BATHING IN SITU SLING

The Amputee clip bathing in situ sling is suitable for bathing and showering residents/patients with bilateral leg amputations above the knee or total hip. It is designed with the ability to remain under the patient between transfers following clinical assessment.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA2080M
Safe working load	190 kg (418 lbs)
Size	<div><div>XXS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Lightweight knitted polyester fabric - mesh
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>• Maxi Move</div><div>• Maxi Twin</div><div>• Maxi Twin compact</div><div>• Maxi 500</div></div>	<div><div>• Maxi Sky 600</div><div>• Maxi Sky 2 Series</div><div>• Maxi Sky 2 Plus (Single mode)</div></div>



AMPUTEE CLIP TOILET SLING

The Amputee clip toilet sling is designed with a commode hole to assist with toileting and hygiene. This sling is suitable for general transfers of residents/patients with bilateral leg amputations above the knee or total hip.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA2091M
Safe working load	272 kg (600 lbs)
Size	<div><div>XXS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>• Maxi Move</div><div>• Maxi Twin</div><div>• Maxi Twin compact</div><div>• Maxi 500</div></div>	<div><div>• Maxi Sky 600</div><div>• Maxi Sky 2 Series</div><div>• Maxi Sky 2 Plus (Single mode)</div></div>



PAEDIATRIC CLIP SLING

The Paediatric clip sling is suitable for everyday patient handling routines such as bed and chair related transfers and it is designed specifically for children with an additional buckle in the head area.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA4020M
Safe working load	125 kg (275 lbs)
Size	<div><div>XXS</div><div>XS</div><div>S</div><div>M</div><div>L</div><div>LL</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester woven fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>• Maxi Move</div><div>• Maxi Twin</div><div>• Maxi Twin compact</div><div>• Maxi 500</div></div>	<div><div>• Maxi Sky 2 Series</div><div>• Maxi Sky 2 Plus (Single mode)</div><div>• Maxi Sky 2 Series</div><div>• Maxi Sky 2 Plus (Single mode)</div></div>



Standing and walking slings

Page intentionally left blank





Slings for standing and raising aids

Slings to be used with Arjo standing and raising aids. Enable residents and patients to be raised up from a bed, chair, toilet or wheelchair for transfer and short distance transportation. The patient should be able to support their weight on at least one leg and have sufficient muscle strength in the upper and lower body.

For use with:

- Sara Flex, Sara Plus and Sara Stedy

Morphologies

- General population

Materials available

- Knitted polyester fabric (cache blue)
- Non-woven polypropylene fabric
- Polyurethane coated fabric

Main activities

- Standing transfers
- Toileting
- Dressing
- Repositioning in chair
- Sit to stand training/support
- Rehabilitation activities
- Ambulation

Ambulation slings for ceiling and passive floor lifts

Slings intended for assisted transfer and rehabilitation of patients/residents with limited ability to move. Gait training, rehabilitation, and ambulation activities can be performed with these slings.

For use with:

- Ceiling and passive floor lifts

Morphologies

- General population

Materials available

- Polyester knitted fabric

Main activities

- Sit to stand training/support
- Rehabilitation activities
- Ambulation

SARA FLEX STANDING SLING

The Sara Flex standing sling in conjunction with the lift enables a single caregiver to assist a patient or resident from a seated to a standing position, in one natural movement.

PRODUCT INFORMATION	
ARTICLE NUMBERS	TSS.500 (S) to TSS.504 (XXL)
Safe working load	200 kg (440 lbs)
Size	<div><div></div><div>S</div><div>M</div><div>L</div><div></div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>Sara Flex</div><div>Sara Steady</div></div>	Not compatible



SARA FLEX SEATED TRANSFER SLING

The Sara Flex seated transfer sling can provide additional support if required during transfer and supports the patient/resident if they were to sit or in case their standing ability reduces during the transfer.

PRODUCT INFORMATION	
ARTICLE NUMBER	TSS.511
Safe working load	200 kg (440 lbs)
Size	<div><div>One size</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>Sara Flex</div></div>	Not compatible



SARA FLEX STANDING SLING – SEPARATE BELT WITH ONE BUCKLE

The Sara Flex standing sling – separate belt with one buckle in conjunction with the lift enables a single caregiver to assist a patient or resident from a seated to a standing position, in one natural movement. The separate support belt is narrower than the fixed belt and attached with one belt clip instead of two.


PRODUCT INFORMATION	
ARTICLE NUMBERS	TSS.500S-1C (S) to TSS.503S-1C (XL)
Safe working load	200 kg (440 lbs)
Size	<div><div></div><div>S</div><div>M</div><div>L</div><div></div><div>XL</div><div></div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>Sara Flex</div><div>Sara Steady</div></div>	Not compatible



SARA CLIP STANDING SLING – FLAT

The Sara clip standing sling – flat in conjunction with the lift enables a single caregiver to assist a patient or resident from a seated to a standing position, in one natural movement. This sling has a more narrow design compared to the Sara Flex standing sling.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA3071M
Safe working load	180 kg (400 lbs)
Size	<div><div>One size</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div><div>Sara Flex</div><div>Sara Steady</div></div>	Not compatible

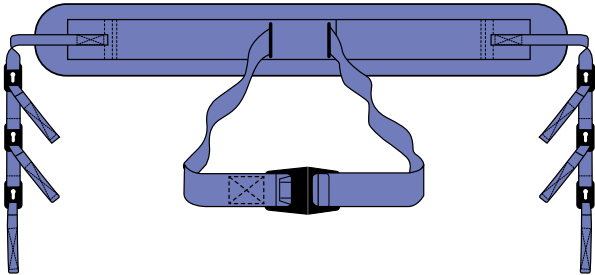

SARA FLEX DISPOSABLE STANDING SLING

The Sara Flex disposable standing sling in conjunction with the lift enables a single caregiver to assist a patient or resident from a seated to a standing position, in one natural movement. It supports infection prevention and safe patient handling routines.

PRODUCT INFORMATION	
ARTICLE NUMBER	MFA3000
Safe working load	200 kg (440 lbs)
Size	One size Supplied in packs of 10 slings
Morphology	
Cleaning method	Disposable for single patient use
Material	Polypropylene Non-woven fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Sara FlexSara Steady	Not compatible



SARA PLUS STANDING SLING

The Sara Plus standing sling in conjunction with the lift supports the patient through active standing transfer and rehabilitation activities by enabling a single caregiver/therapist to facilitate.

PRODUCT INFORMATION	
ARTICLE NUMBER	KKA5120
Safe working load	190 kg (418 lbs)
Size	S M L XL
Morphology	
Cleaning method	
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Sara Plus	Not compatible



SARA FLEX WIPEABLE STANDING SLING

The Sara Flex wipeable standing sling in conjunction with the lift enables a single caregiver to assist a patient or resident from a seated to a standing position, in one natural movement. Completely wipeable with no stitching, it can be wiped between patients or residents. The sling design helps to reduce the cross-contamination risks associated with shared patient handling devices with no laundering requirement.

PRODUCT INFORMATION	
ARTICLE NUMBERS	TSS.500W - TSS.503W
Safe working load	272 kg (600 lbs)
Size	S M L XL
Morphology	
Cleaning method	Wipe to clean and disinfect
Material	Polyurethane coated fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Sara FlexSara Steady	Not compatible



SARA PLUS STANDING SLING – EXTRA POSTURAL SUPPORT



The Sara Plus standing sling – extra postural support in conjunction with the lift supports the patient through active standing transfer and rehabilitation activities by enabling a single caregiver/therapist to facilitate. It helps patients to achieve full extension during sit to stand, transfer and toileting activities.

PRODUCT INFORMATION	
ARTICLE NUMBER	KKA5370
Safe working load	190 kg (418 lbs)
Size	XS S M L XL XXL
Morphology	
Cleaning method	
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Sara Plus	Not compatible



SARA PLUS DISPOSABLE STANDING SLING



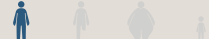

The Sara Plus disposable standing sling in conjunction with the lift supports the patient through active standing transfer and rehabilitation activities by enabling a single caregiver/therapist to facilitate. It supports infection prevention and safe patient handling routines.

PRODUCT INFORMATION	
ARTICLE NUMBER	MFA4000
Safe working load	190 kg (418 lbs)
Size	One size
Supplied in packs of 10 slings	
Morphology	
Cleaning method	 Disposable for single patient use
Material	Polypropylene Non-woven fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
▪ Sara Plus	Not compatible



SARA PLUS WALKING SLING




The Sara Plus walking sling in conjunction with the lift supports the patient through active transfer and rehabilitation activities such as balance, stepping and walking training by enabling a single caregiver/therapist to facilitate.

PRODUCT INFORMATION	
ARTICLE NUMBER	KKAS130M
Safe working load	140 kg (309 lbs)
Size	
Morphology	
Cleaning method	 
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
▪ Sara Plus	Not compatible



SARA PLUS WIPEABLE STANDING SLING





The Sara Plus wipeable standing sling in conjunction with the lift supports the patient through active standing transfer and rehabilitation activities by enabling a single caregiver/therapist to facilitate. Completely wipeable with no stitching, it can be wiped between patients or residents. The sling design helps to reduce the cross contamination risks associated with shared patient handling devices with no laundering requirement.

PRODUCT INFORMATION	
ARTICLE NUMBER	KKA6000
Safe working load	200 kg (440 lbs)
Size	
Morphology	
Cleaning method	 Wipe to clean and disinfect
Material	Polyurethane coated fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
▪ Sara Plus	Not compatible



LOOP WALKING SLING

The Loop walking sling is intended for assisted transfer and rehabilitation of patients/residents with limited ability to move. It is designed to support during standing, walking, and rehabilitation activities.

PRODUCT INFORMATION	
ARTICLE NUMBER	TEM
Safe working load	200 kg (440 lbs)
Size	
Morphology	
Cleaning method	 
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
Not compatible	▪ Maxi Sky 440 ▪ Maxi Sky 2



LOOP WALKING JACKET

The Loop walking jacket is intended for assisted transfer and rehabilitation of patients/residents with limited ability to move. Gait training, rehabilitation, and ambulation activities can be performed with this sling.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA5000
Safe working load	190 kg (418 lbs)
Size	<div><div>XS</div><div>S</div><div>M</div><div>L</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div></div><div></div><div></div><div></div></div>
Cleaning method	<div><div></div><div></div></div>
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<div>• Maxi Move</div>	<div>• Maxi Sky 440</div>



In bed and lateral transfer slings





In bed and lateral transfer slings

Used in combination with a passive ceiling or floor lift, these can help reduce the impact on caregiver workflow. It helps reposition the patient in bed, including turning to and from the prone position.

Attachment methods

- Clip
- Loop

Morphologies

- General population
- Amputee
- Bariatric

Materials available

- Knitted polyester fabric (cache blue)
- Woven polyester fabric (blue)
- Non-woven polypropylene fabric
- High strength woven polycotton fabric (blue and beige)
- Woven polycotton fabric

Main activities

- Repositioning patient in bed
- Lateral transfers
- Changing bed linen underneath the patient
- Transfer from the floor
- Sustained holding of a body part to perform patient care (wound care, dressing, bedside tasks, etc).
- Surgical procedure
- Turning to and from a prone position
- Applying an X-ray cassette
- Raising the trunk

LOOP STRETCHER SLING – COMMODE APERTURE

The Loop stretcher sling with commode aperture is suitable for residents/patients who require full body support in a supine position during transfers.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA1150
Safe working load	272 kg (600 lbs)
Size	<div><div>XSS</div><div>S</div><div>L</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div>Person icon</div><div>Person icon</div><div>Person icon</div><div>Person icon</div></div>
Cleaning method	<div><div>Washing icon</div><div>Drying icon</div></div>
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi Move	<ul style="list-style-type: none">Maxi Sky 600Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



LOOP BATHING STRETCHER SLING – COMMODE APERTURE

With mesh fabric and commode opening, the Loop bathing stretcher sling with commode aperture is suitable for hygiene care of residents/patients who require full body support in a supine position during transfers.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA1152
Safe working load	272 kg (600 lbs)
Size	<div><div>XSS</div><div>S</div><div>L</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div>Person icon</div><div>Person icon</div><div>Person icon</div><div>Person icon</div></div>
Cleaning method	<div><div>Washing icon</div><div>Drying icon</div></div>
Material	Polyester Mesh Cache Blue
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi Move	<ul style="list-style-type: none">Maxi Sky 600Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



LOOP STRETCHER SLING

The Loop stretcher sling is suitable for residents/patients who require full body support in a supine position during transfers.

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA1151
Safe working load	272 kg (600 lbs)
Size	<div><div>XSS</div><div>S</div><div>L</div><div>XL</div><div>XXL</div></div>
Morphology	<div><div>Person icon</div><div>Person icon</div><div>Person icon</div><div>Person icon</div></div>
Cleaning method	<div><div>Washing icon</div><div>Drying icon</div></div>
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi Move	<ul style="list-style-type: none">Maxi Sky 600Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



DISPOSABLE LOOP STRETCHER SLING


The Disposable loop stretcher sling is suitable for residents/patients who require full body support in a supine position during transfers.

PRODUCT INFORMATION	
ARTICLE NUMBER	MFA5000
Safe working load	272 kg (600 lbs)
Size	<div><div>One size</div></div> Supplied in packs of 10 slings
Morphology	<div><div>Person icon</div><div>Person icon</div><div>Person icon</div><div>Person icon</div></div>
Cleaning method	<div><div>Washing icon</div></div> Disposable for single patient use
Material	Polypropylene Non-woven fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi Move	<ul style="list-style-type: none">Maxi Sky 600Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



STRAP STRETCHER




With the Strap Stretcher, patients can be raised in a fully supine position, without needing to move or turn them. It gives full access and visual contact for hygiene and wound care.

PRODUCT INFORMATION	
ARTICLE NUMBER	700-14945
Safe working load	272kg (600lbs)
Size	One size
Morphology	
Cleaning method	Not applicable
Material	Not applicable
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi Move	<ul style="list-style-type: none">Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



LOOP COMFORT REPOSITIONING SLING




The Loop comfort repositioning sling assists the caregiver to move or reposition the patient in bed including turning to and from the prone position, lateral transfer from one flat surface to another and lifting from the floor. The sling can also be used as bed linen and may be left under the patient between transfers (if clinical assessment allows).

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA7000
Safe working load	272 kg (600 lbs)
Size	Regular <input type="radio"/> Wide <input type="radio"/>
Morphology	
Cleaning method	 
Material	Woven polyester fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi Move	<ul style="list-style-type: none">Maxi Sky 600Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



LOOP REPOSITIONING SLING



The Loop repositioning sling assists the caregiver to move or reposition the patient in bed including turning to and from the prone position and lateral transfer from one flat surface to another. It may be left under the patient between transfers (if clinical assessment allows).

PRODUCT INFORMATION	
ARTICLE NUMBER	MAA6000
Safe working load	272kg (600lbs)
Size	One size
Morphology	
Cleaning method	 
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi 500Maxi TwinMaxi Move	<ul style="list-style-type: none">Maxi Sky 600Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



DISPOSABLE LOOP REPOSITIONING SLING

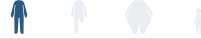

The Disposable loop repositioning sling assists the caregiver to move or reposition the patient in bed including turning to and from the prone position and lateral transfer from one flat surface to another. It can reduce the risk of cross-contamination as it stays with the patient. It may be left under the patient between transfers (if clinical assessment allows).

PRODUCT INFORMATION	
ARTICLE NUMBER	AHD001
Safe working load	272 kg (600 lbs)
Size	One size <input type="radio"/> Supplied in packs of 5 slings
Morphology	
Cleaning method	 Disposable for single patient use
Material	Woven polycotton fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi 500Maxi TwinMaxi Move	<ul style="list-style-type: none">Maxi Sky 600Maxi Sky 2 SeriesMaxi Sky 2 Plus (Single mode)



BARIATRIC LOOP REPOSITIONING SLING



The Bariatric loop repositioning sling assists the caregiver with in-bed repositioning and transferring plus-size patients in a supine position from one flat surface to another. The sling dimensions fit with the Arjo bariatric beds.

PRODUCT INFORMATION	
ARTICLE NUMBER	VIG11004-L1
Safe working load	454 kg (1000 lbs)
Size	One size
Morphology	
Cleaning method	
Material	High strength woven polycotton fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Tenor	<ul style="list-style-type: none">Maxi Sky 1000Maxi Sky 2 Plus



DISPOSABLE BARIATRIC LOOP REPOSITIONING SLING



The Disposable bariatric loop repositioning sling assists the caregiver with in-bed repositioning and transferring plus-size patients in a supine position from one flat surface to another. The sling dimensions fit with the Arjo bariatric beds. It is designed to accompany a patient throughout the hospital stay and can be used in exactly the same way as a standard bariatric repositioning sling but it cannot be laundered.

PRODUCT INFORMATION	
ARTICLE NUMBER	VIG22004-L1
Safe working load	454 kg (1000 lbs)
Size	One size Supplied in packs of 5 slings
Morphology	
Cleaning method	 Disposable for single patient use
Material	High strength woven polycotton fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Tenor	<ul style="list-style-type: none">Maxi Sky 1000Maxi Sky 2 Plus



LIMB CLIP SLING



The Clip limb sling is designed to support a limb to reduce the need for manual lifting and holding for activities like wound dressing or hygiene procedures.

PRODUCT INFORMATION	
ARTICLE NUMBER	VIG11000
Safe working load	70 kg (154 lbs)
Size	One size
Morphology	
Cleaning method	
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi MoveMaxi 500	<ul style="list-style-type: none">Maxi Sky 2 Series



LOOP LIMB SLING




The Loop limb sling is designed to assist the caregiver with lifting, supporting and positioning the limb for easier access when performing activities such as dressing, foot care, wound care and leg dressing.

PRODUCT INFORMATION	
ARTICLE NUMBER	VIG11001
Safe working load	70 kg (154 lbs)
Size	One size
Morphology	
Cleaning method	
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi MoveMaxi 500	<ul style="list-style-type: none">Maxi Sky 440Maxi Sky 2 SeriesMaxi Sky 1000



BARIATRIC LOOP LIMB SLING




The Bariatric loop limb sling is designed with a wider surface and higher safe working load than the standard limb slings.

PRODUCT INFORMATION	
ARTICLE NUMBER	VIG11002
Safe working load	120 kg (264 lbs)
Size	One size
Morphology	
Cleaning method	 
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">• Tenor	<ul style="list-style-type: none">• Maxi Sky 2 Series• Maxi Sky 1000• Maxi Sky 2 Plus



LOOP TURNING SLING



The Loop turning sling is designed to be applied without having to log roll or move the patient and assists the caregiver with turning/rolling a patient in a supine position in bed.

PRODUCT INFORMATION	
ARTICLE NUMBER	VIG11003
Safe working load	100 kg (220 lbs)
Size	One size
Morphology	
Cleaning method	 
Material	Polyester knitted fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">• Maxi Move• Maxi 500	<ul style="list-style-type: none">• Maxi Sky 440• Maxi Sky 2 Series• Maxi Sky 1000



DISPOSABLE CLIP LIMB SLING



The Disposable clip limb sling assists the caregiver with lifting, supporting and positioning the limb for easier access when performing activities such as dressing, foot care, wound care and leg dressings. It is a patient specific product and should be used for a limited period only.

PRODUCT INFORMATION	
ARTICLE NUMBER	VIG22000
Safe working load	70 kg (154 lbs)
Size	One size Supplied in packs of 5 slings
Morphology	
Cleaning method	 Disposable for single patient use
Material	Polypropylene Non-woven fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">• Maxi Move• Maxi 500	<ul style="list-style-type: none">• Maxi Sky 2 Series



DISPOSABLE LOOP LIMB SLING



The Disposable loop limb sling assists the caregiver with lifting, supporting and positioning the limb for easier access when performing activities such as dressing, foot care, wound care and leg dressing. It is designed for single patient use.

PRODUCT INFORMATION	
ARTICLE NUMBER	VIG22001
Safe working load	70 kg (154 lbs)
Size	One size Supplied in packs of 5 slings
Morphology	
Cleaning method	 Disposable for single patient use
Material	Polypropylene Non-woven fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">• Maxi Move• Maxi 500	<ul style="list-style-type: none">• Maxi Sky 440• Maxi Sky 2 Series• Maxi Sky 1000



DISPOSABLE LOOP TURNING SLING

The Disposable loop turning sling is designed to be applied without having to log roll or move the patient and assists the caregiver with turning/rolling a patient in a supine position in bed. It is a patient specific product and should be used for a limited period only.

PRODUCT INFORMATION	
ARTICLE NUMBER	VIG22003
Safe working load	100 kg (220 lbs)
Size	<div>One size</div> Supplied in packs of 5 slings
Morphology	<div></div>
Cleaning method	<div> Disposable for single patient use</div>
Material	Polypropylene Non-woven fabric
COMPATIBLE LIFTS	
Floor lifts	Ceiling lifts
<ul style="list-style-type: none">Maxi MoveMaxi 500	<ul style="list-style-type: none">Maxi Sky 440Maxi Sky 2 SeriesMaxi Sky 1000





Slide sheets

Arjo slide sheets

Arjo slide sheets and tubes are designed to help caregivers carry out in-bed repositioning and transfers with less of the physical stress and strain associated with manual handling.

Slide sheets can also be used to help with sling application and fitting to reduce caregiver effort, help positioning and reduce risk of friction and shear on the skin.

With the patient in a lying position, a sling can be introduced between two flat slide sheets or be fitted with the patient in a side lying position using a slide sheet.

A slide sheet can be used to introduce a seated transfer sling behind the patient. It can also be used to guide the leg sections of the sling underneath /around each thigh.



The Arjo slide sheets are made of a premium low friction fabric that can be laundered and re-used multiple times. The sewn in handles gives a comfortable grip for pulling and the Arjo pull straps can easily be attached to the handles when extended reach is necessary.



Arjo slide sheet	
ARTICLE #	DIMENSION
NSA0500	730 mm (28 3/4") x 1730 mm (68")
	

Arjo slide sheet - long	
ARTICLE #	DIMENSION
NSAV5310	840 mm (33") x 2000 mm (78 3/4")
	

Arjo slide sheet - XL	
ARTICLE #	DIMENSION
NSAV5260	1095 mm (43") x 1730 mm (68")
	

Arjo slide sheet - long XL	
ARTICLE #	DIMENSION
NSAV5320	1095 mm (43") x 2000 mm (78 3/4")
	


Arjo slide sheet - XXL	
ARTICLE #	DIMENSION
NSAV5000	1460 mm (57 1/2") x 1730 mm (68")



Arjo slide sheet - long XXL	
ARTICLE #	DIMENSION
NSAV5330	1460 mm (57 1/2") x 2000 mm (78 3/4")



Arjo pull straps for slide sheets	
ARTICLE #	DIMENSION
NSA0510	660 mm (26")



8 pieces per bag

Arjo slide tubes



The Arjo slide tubes are made of a premium low friction fabric that can be laundered and re-used multiple times. The products have two surfaces that have been sewn together into a tubular design, which means that only one product is needed rather than two flat slide sheets to achieve the same friction reduction.

Arjo slide tube - small	
ARTICLE #	DIMENSION
NSA0600	585 mm (23") ø433 mm (17")



A small extension tube to be used with the standard bed length slide sheets when the patient is taller than the sheets.

Arjo slide tube radiology	
ARTICLE #	DIMENSION
NSAV5060	670 mm (26 3/8") ø580 mm (22 13/16")



A small tube that fits an x-ray cassette.

Arjo slide tube - half bed	
ARTICLE #	DIMENSION
NSA0900	900 mm (35 1/2") ø1100 mm (43 3/8")



Arjo slide tube - long	
ARTICLE #	DIMENSION
NSA0700	1960 mm (77") ø433 mm (17")



An easy to use lateral slide tube in full bed length.

Arjo slide tube - long XL	
ARTICLE #	DIMENSION
NSA0800	2100 mm (82 11/16") ø900 mm (35 1/2")
	

Arjo slide sheet kits

ARTICLE #	DESCRIPTION
NSA1000	2x Arjo Slide sheet (#NSA0500) 1x Arjo slide tube - small (#NSA0600)
	

ARTICLE #	DESCRIPTION
NSA1100	2x Arjo Slide sheet - long (#NSAV5310) 1x Arjo Slide tube - small (#NSA600)
	

ARTICLE #	DESCRIPTION
NSA1200	2x Arjo Slide sheet - long (#NSAV5310)
	

ARTICLE #	DESCRIPTION
NSA1300	100x Arjo Slide sheet - long (#NSAV5310)
	

ARTICLE #	DESCRIPTION
NSA2000	2x Arjo Slide sheet (#NSA0500)
	

ARTICLE #	DESCRIPTION
NSA4000	100x Arjo Slide sheet (#NSA0500)
	

ARTICLE #	DESCRIPTION
NSA1400	2x Arjo Slide sheet - long XL (#NSAV5320)
	

ARTICLE #	DESCRIPTION
NSA6000	2x Arjo slide sheet - XL (#NSAV5260)
	

ARTICLE #	DESCRIPTION
NSA7000	2x Arjo Slide sheet - XXL (#NSAV5000)
	

ARTICLE #	DESCRIPTION
NSA1500	2x Arjo slide sheet - long XXL (#NSAV5330)
	

ARTICLE #	DESCRIPTION
NSA3000	2x Arjo Slide tube - long (#NSA0700)
	

ARTICLE #	DESCRIPTION
NSA510	8x Arjo pull straps for slide sheets
	

Arjo disposable slide sheets & tubes

Arjo disposable slide sheets for single patient use are designed to accompany a patient throughout the hospital stay. They are available in different designs and sizes both as flat slide sheets with handles and slide tubes.

The flat sheets have sewn in handles giving a comfortable grip for pulling and the disposable pull straps can easily be attached to the handles when extended reach is necessary.

The disposable slide tubes have two surfaces that have been sewn together into a tubular design, which means that only one product is needed rather than two flat slide sheets to achieve the same friction reduction.

Disposable pull straps

The disposable pull straps can be attached to the sheets for lateral transfers avoiding caregivers having to overreach, thereby reducing the risk of injury.

Strong built-in handles

The built-in reinforced handles offer something to hold during patient transfers



Patient identification tag

Each product has a tag to write the patient's name on and a green water-soluble label, which will dissolve if the sheet is laundered, offering you a visible indication when not to use the sheet.



Arjo disposable slide sheet



ARTICLE #	DIMENSION
NPA0500	760 mm (30") x 1960 mm (77")

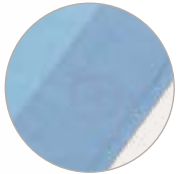

20 pieces per box

Arjo disposable slide sheet - XL


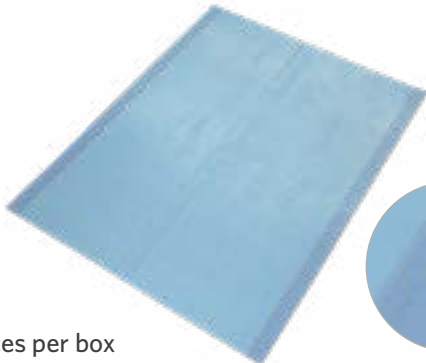
ARTICLE #	DIMENSION
NPAV5260	1140 mm (45") x 1960 mm (77")

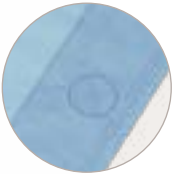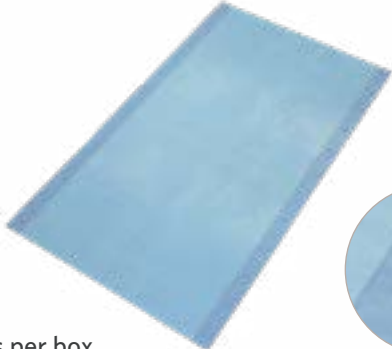

20 pieces per box

Arjo disposable slide sheet - XXL



ARTICLE #	DIMENSION
NPAV5000	1520 mm (60") x 1960 mm (77")



20 pieces per box

Arjo disposable slide tube

ARTICLE #	DIMENSION
NPA0700	760 mm (30") x 1960 mm (77")

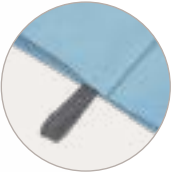

20 pieces per box

Arjo disposable slide tube - XL

ARTICLE #	DIMENSION
NPA0750	1140 mm (45") x 1960 mm (77")



20 pieces per box

Arjo disposable slide tube - half bed

ARTICLE #	DIMENSION
NPA0900	990 mm (39") x 990 mm (39")



50 pieces per box

Arjo disposable pull straps for slide sheets

ARTICLE #	DIMENSION
NPA0510	Length 800 mm (26") x 40 mm (1 ½")



50 pieces per bag

At Arjo, we believe that empowering movement within healthcare environments is essential to quality care. Our products and solutions are designed to promote a safe and dignified experience through patient handling, medical beds, personal hygiene, disinfection, diagnostics, and the prevention of pressure injuries and venous thromboembolism. With over 6000 people worldwide and 60 years caring for patients and healthcare professionals, we are committed to driving healthier outcomes for people facing mobility challenges.

Arjo International HQ · Hans Michelsensgatan 10 · 211 20 Malmö · Sweden · +46 10 335 4500

www.arjo.com