
ACCESSORY INSTRUCTIONS

# Evenda EV-ACC09

3/4 Folding bed rails

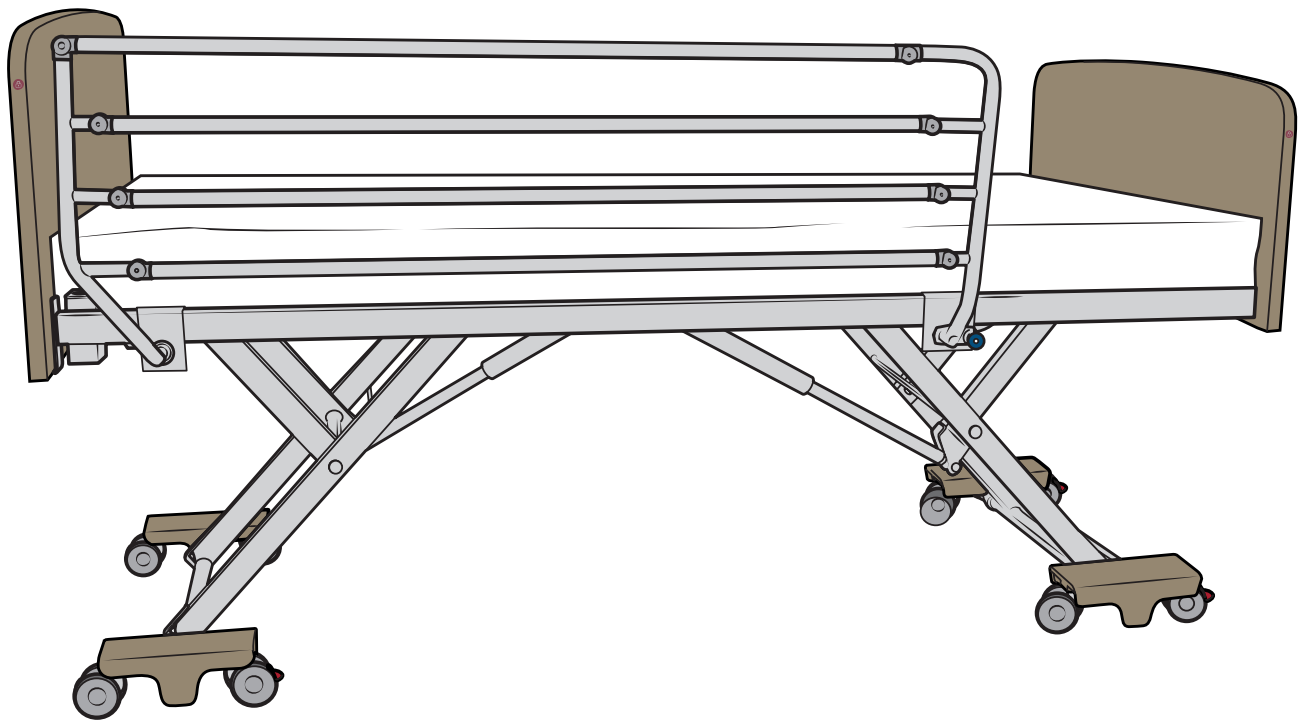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Zubehöranleitung · Instructions relatives aux accessoires · Istruzioni per gli accessori · Accessoire-instructies ·  
付属品取扱説明書

<b>English</b> .....	<b>4</b>
<b>Deutsch</b> .....	<b>8</b>
<b>Français</b> .....	<b>12</b>
<b>Italiano</b> .....	<b>16</b>
<b>Nederlands</b> .....	<b>20</b>
<b>日本語</b> .....	<b>24</b>

**WARNING**

To avoid injury, always read these Accessory Instructions and all accompanying documents before using the product.

Refer to instruction manual/ booklet – Accessory instructions should be read.

Design Policy and Copyright

® and ™ are trademarks belonging to the Arjo group of companies.

© Arjo 2023.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice.

The content of this publication may not be copied either whole or in part without the consent of Arjo.

# Foreword

Thank you for purchasing the 3/4 Folding bed rails for the Evenda® long term care bed.

## Customer contact information

For questions regarding this product, supplies, maintenance, or additional information about Arjo products and service, contact Arjo, an Arjo authorised representative or visit [www.arjo.com](http://www.arjo.com).

## Read and fully understand these Accessory Instructions before using the product

The information contained in this manual is required for the proper installation of the equipment. It will help you protect your product and make sure that the equipment performs to your satisfaction. The information in this manual is important for your safety and must be read and understood to help prevent possible injury. Unauthorised modifications on any Arjo equipment can affect its safety. Arjo cannot be held responsible for any accidents, incidents or loss of performance resulting from any unauthorised modification to its products.

### WARNING

To prevent injury, the Evenda 3/4 folding bed rails should always be installed by a qualified technician.

### WARNING

To avoid injury, only use bed rails that have been specifically designed for the Evenda bed.

## Definitions used in this manual

### WARNING

Safety warning. Failure to understand and obey this warning may result in injury to you or others.

### NOTE

This is important information for the correct use of this system or equipment.

# Installation

## WARNING

To prevent body parts from getting caught or pinched, be very careful when operating the bed rails.

## NOTE

It is easier and more comfortable to keep the bed rails in folded position during the installation.

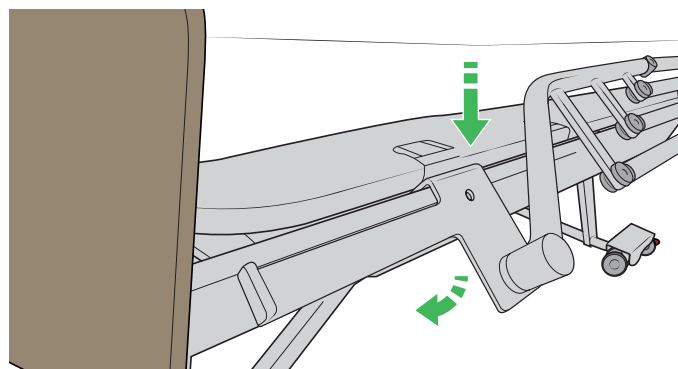
Figure 1

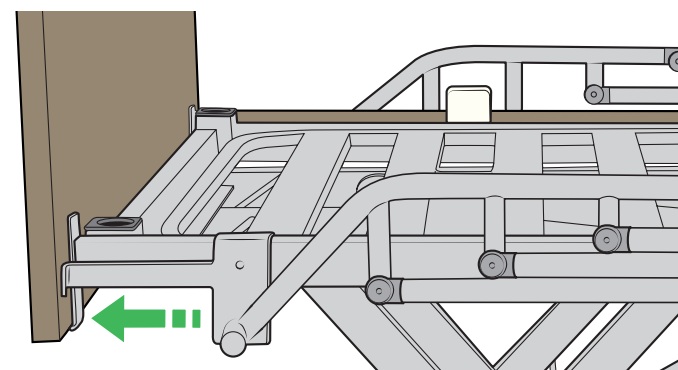
Figure 2

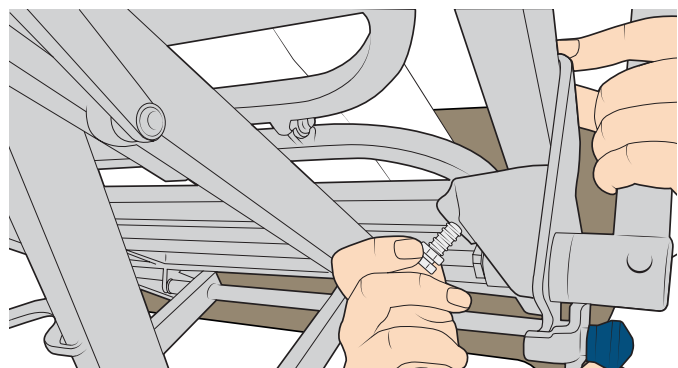
Figure 3

## Installing the bed rails on the bed frame

1. If the bed has side panels, remove them.
- 2.

### NOTE

Make sure to install the bed rails on the correct side of the bed. The spacer bar must point towards the headboard. See Figure 2.

Position the bed rail onto the side of the bed frame. Make sure that the two brackets are properly hooked over the top of the frame. See Figure 1.

3. Push the bed rail towards the head of the bed so that the end of the spacer bar rests against the headboard. See Figure 2.
4. Tighten the screws on both brackets with your fingers. Make sure that the bed rail is still firmly pushed against the headboard. See Figure 3.

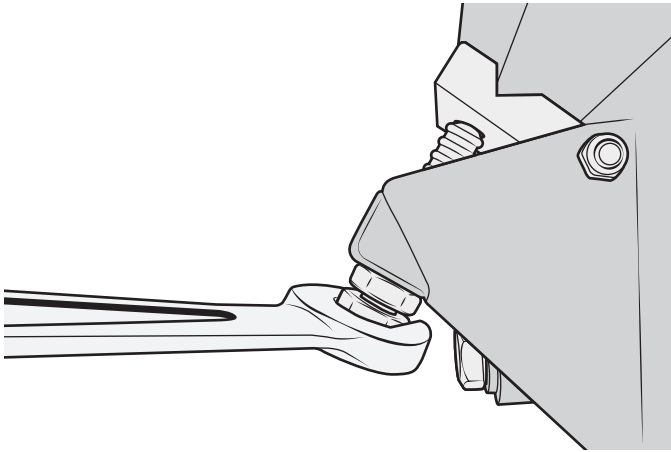
Figure 4

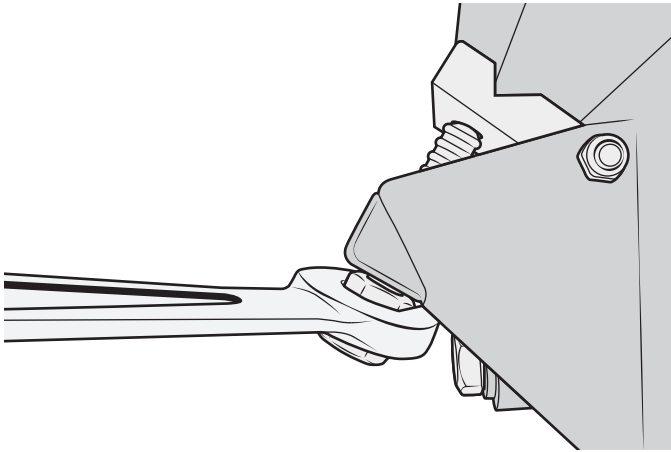
Figure 5

5. Tighten the screws on both brackets with an S17 wrench. See Figure 4.
6. Tighten the lock nuts on both brackets with an S17 wrench. See Figure 5. Make sure the bed rail is firmly secured.
7. Repeat steps 1 to 6 for the other side of the bed.

**WARNING**

To avoid injury, make sure that the bed rail is firmly secured to the bed.

**WARNING**

To avoid injury, refer to the Evenda bed IFU for the complete use and safety instructions.

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Intentionally left blank

**AUSTRALIA**

Arjo Australia  
 Building B, Level 3  
 11 Talavera Road  
 Macquarie Park, NSW, 2113,  
 Australia  
 Phone: 1800 072 040

**BELGIQUE / BELGIË**

Arjo Belgium  
 Evenbroekveld 16  
 9420 Erpe-Mere  
 Phone: +32 (0) 53 60 73 80  
 Fax: +32 (0) 53 60 73 81  
 E-mail: info.belgium@arjo.com

**BRASIL**

Arjo Brasil Equipamentos Médicos Ltda  
 Rua Marina Ciufuli Zanfelice, 329 PB02  
 Galpão - Lapa  
 São Paulo – SP – Brasil  
 CEP: 05040-000  
 Phone: 55-11-3588-5088  
 E-mail: vendas.latam@arjo.com  
 E-mail: servicios.latam@arjo.com

**CANADA**

Arjo Canada Inc.  
 90 Matheson Boulevard West  
 Suite 350  
 CA-MISSISSAUGA, ON, L5R 3R3  
 Tel/Tél: +1 (905) 238-7880  
 Free: +1 (800) 665-4831  
 Fax: +1 (905) 238-7881  
 E-mail: info.canada@arjo.com

**ČESKÁ REPUBLIKA**

Arjo Czech Republic s.r.o.  
 Na Strži 1702/65  
 140 00 Praha  
 Czech Republic  
 Phone No: +420225092307  
 E-mail: info.cz@arjo.com

**DANMARK**

Arjo A/S  
 Vassingerødvej 52  
 DK-3540 LYNGBE  
 Tel: +45 49 13 84 86  
 Fax: +45 49 13 84 87  
 E-mail: dk\_kundeservice@arjo.com

**DEUTSCHLAND**

Arjo GmbH  
 Peter-Sander-Strasse 10  
 DE-55252 MAINZ-KASTEL  
 Tel: +49 (0) 6134 186 0  
 Fax: +49 (0) 6134 186 160  
 E-mail: info-de@arjo.com

**ESPAÑA**

ARJO IBERIA S.L.  
 Polígono Can Salvatella  
 c/ Cabanyes 1-7  
 08210 Barberà del Valles  
 Barcelona - Spain  
 Telefono 1: +34 900 921 850  
 Telefono 2: +34 931 315 999

**FRANCE**

Arjo SAS  
 2 Avenue Alcide de Gasperi  
 CS 70133  
 FR-59436 RONCQ CEDEX  
 Tél: +33 (0) 3 20 28 13 13  
 Fax: +33 (0) 3 20 28 13 14  
 E-mail: info.france@arjo.com

**HONG KONG**

Arjo Hong Kong Limited  
 Room 411-414, 4/F, Manhattan Centre,  
 8 Kwai Cheong Road, Kwai Chung, N.T.,  
 HONG KONG  
 Tel: +852 2960 7600  
 Fax: +852 2960 1711

**ITALIA**

Arjo Italia S.p.A.  
 Via Giacomo Peroni 400-402  
 IT-00131 ROMA  
 Tel: +39 (0) 6 87426211  
 Fax: +39 (0) 6 87426222  
 E-mail: Italy.promo@arjo.com

**MIDDLE EAST**

Arjo Middle East FZ-LLC  
 Office 908, 9th Floor,  
 HQ Building, North Tower,  
 Dubai Science Park,  
 Al Barsha South  
 P.O. Box 11488, Dubai,  
 United Arab Emirates  
 Direct +971 487 48053  
 Fax +971 487 48072  
 Email: Info.ME@arjo.com

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At Arjo, we believe that empowering movement within healthcare environments is essential to quality care. Our products and solutions are designed to promote a safe and dignified experience through patient handling, medical beds, personal hygiene, disinfection, diagnostics, and the prevention of pressure injuries and venous thromboembolism. With over 6500 people worldwide and 65 years caring for patients and healthcare professionals, we are committed to driving healthier outcomes for people facing mobility challenges.

ArjoHuntleigh AB  
Hans Michelsensgatan 10  
211 20 Malmö, Sweden  
[www.arjo.com](http://www.arjo.com)

**arjo**

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Arjo Inc.