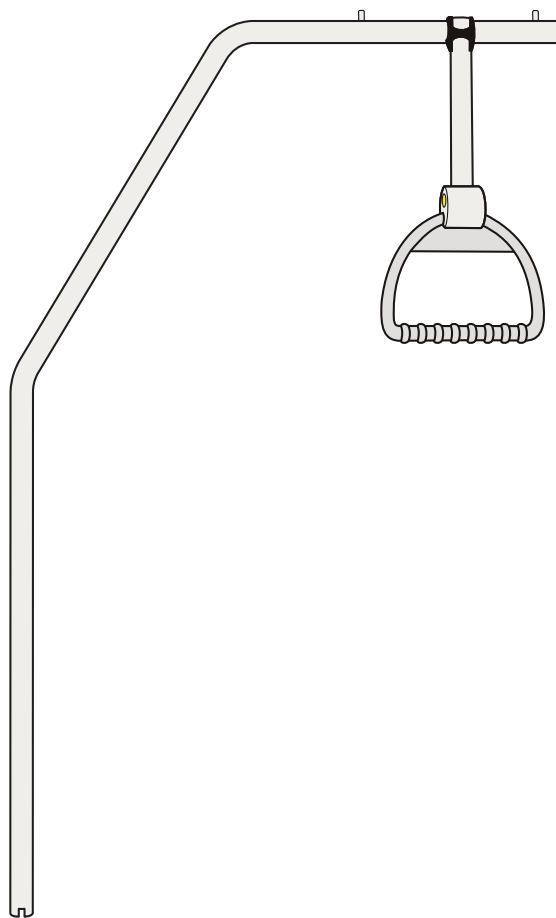
ACCESSORY INSTRUCTIONS

Evenda EV-ACC01

Lifting pole with strap and handle

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Zubehöranleitung · Instructions relatives aux accessoires · Istruzioni per gli accessori · Accessoire-instructies ·
付属品取扱説明書

English	4
Deutsch	10
Français	16
Italiano	22
Nederlands	28
日本語	34

**WARNING**

To avoid injury, always read these **Accessory Instructions** and accompanied documents before using the product.

Refer to instruction manual/ booklet – **Accessory instructions** should be read.

Design Policy and Copyright

® and ™ are trademarks belonging to the Arjo group of companies.

© Arjo 2023.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice.

The content of this publication may not be copied either whole or in part without the consent of Arjo.

Foreword

Thank you for purchasing the Lifting pole with strap and handle for the Evenda® long term care bed.

Customer contact information

For questions regarding this product, supplies, maintenance, or additional information about Arjo products and service, contact Arjo, an Arjo authorised representative or visit www.arjo.com.

Read and fully understand these Accessory Instructions before using the product

The information contained in this manual is required for the proper installation of the equipment. It will help you protect your product and make sure that the equipment performs to your satisfaction. The information in this manual is important for your safety and must be read and understood to help prevent possible injury. Unauthorised modifications on any Arjo equipment can affect its safety. Arjo cannot be held responsible for any accidents, incidents or loss of performance resulting from any unauthorised modification to its products.

Serious incident

If a serious incident occurs in relation to this medical device, affecting the user, or the patient, then the user or patient should report the serious incident to the medical device manufacturer or the distributor. In the European Union, the user should also report the serious incident to the Competent Authority in the member state where they are located.

About EV-ACC01

EV-ACC01 is a lifting pole intended to assist patients in positioning themselves on the bed. The lifting pole is supplied with a strap and handle. EV-ACC01 is suitable for use with Arjo Evenda long term care bed.

Definitions used in this manual

Warning means: Safety warning. Failure to understand and obey this warning may result in injury to you or others.

Safety instructions

 **WARNING**

Do not exceed the Safe Working Load of 75 kg (165 lb).

 **WARNING**

Do not use the lifting pole to push or pull the bed.

 **WARNING**

Do not suspend other equipment from the lifting pole.

 **WARNING**

Refer to the bed Instructions for Use to confirm compatibility before using this accessory.

 **WARNING**

The lifting pole must be positioned so that the handle hangs over the middle of the bed.

 **WARNING**

Do not allow the strap and handle to hang outside the perimeter of the bed.

 **WARNING**

To reduce the risk of patient entrapment, the head end panel must remain fitted at all times while the lifting pole is installed on the bed.

 **WARNING**

The bed should not be occupied by a patient while the lifting pole is being fitted.

 **WARNING**

Before installing the lifting pole, check that the bed's lifting pole sockets are not damaged in any way.

 **WARNING**

Make sure that the lifting pole and handle are positioned to suit the size and medical needs of the patient.

 **WARNING**

The strap and handle must not be used by persons under twelve years of age.

 **WARNING**

The operational life of the strap and handle is five years when used and maintained in accordance with the manufacturer's instructions. After this time the complete unit should be replaced.

 **WARNING**

Inspect the strap and handle regularly. If any sign of wear or damage is found, remove it from use immediately and replace the complete unit.

Installation and use

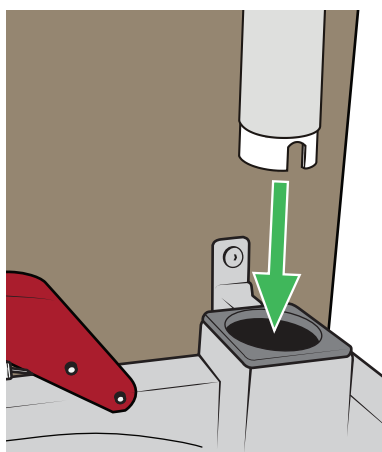
Figure 1

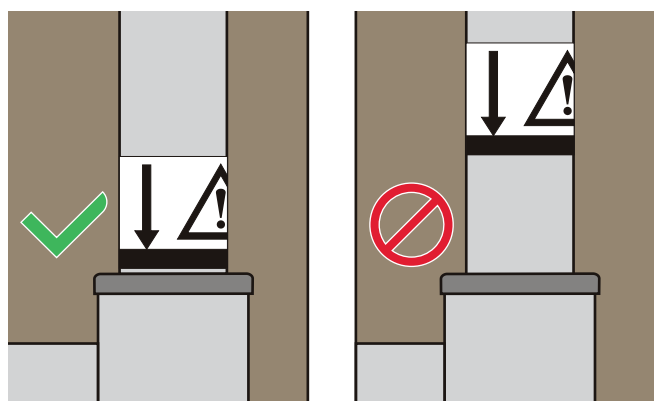
Figure 2

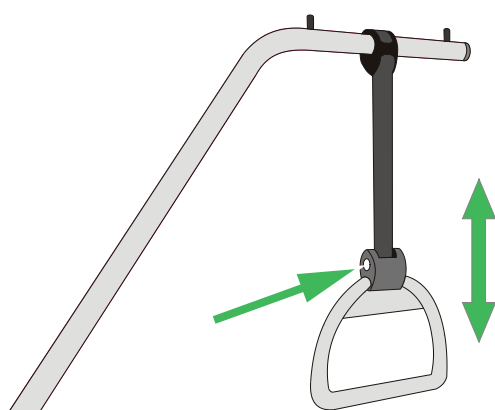
Figure 3

Installation and use

1. Insert the lifting pole into either socket at the head end of the bed, with the horizontal part of the pole above the centre of the bed. See Figure 1.
2. Twist the pole so that the notch locates over the pin at the bottom of the socket.

3. When the lifting pole is correctly installed the indicator line is level with the top of the socket. See Figure 2.

4. Fit the lifting strap loop over the horizontal part of the lifting pole. Make sure that the loop lies between the projecting pegs at all times.

5. To adjust the height of the handle above the bed, press the release button and adjust the strap to the required length. See Figure 3.

⚠ WARNING

Do not adjust the strap while a patient is holding the handle as this may damage the mechanism. After adjusting the strap, make sure it locks with an audible “click”.

Do not twist the strap more than 180 as this may cause premature wear.

Removal

1. Remove the strap and handle.
2. Lift the lifting pole out of the socket.

Cleaning

WARNING

Remove the strap and handle from the lifting pole before cleaning.

WARNING

Only use approved disinfectants and cleaning methods. Other products may cause degradation of the equipment and possibly premature failure.

WARNING

Use all cleaners and disinfectants with care. Follow the manufacturer's instructions.

WARNING

Do not disinfect the strap and handle in a pressure chamber, autoclave or steam cleaner as this may damage the product.

WARNING

Before cleaning the handle, extend the strap fully and leave it extended until the handle is completely dry.

Lifting pole: Refer to the bed instructions for use.

Strap and handle: Follow the cleaning and disinfection procedures recommended by the manufacturer and your local institutional cleaning and disinfection protocols.

Care and preventive maintenance

Refer to the bed Instructions for use.

Product weight

The weight of EV-ACC01 including strap and handle is 8 kg (18 lb).

Safe working load

The safe working load for EV-ACC01 is 75 kg (165 lb).

End of life disposal

Components that are primarily made up of different kinds of metal (containing more than 90% metal by weight) for example sling bars, rails, upright supports etc., should be recycled as metals.

Symbol explanation

SYMBOL EXPLANATION	
	CE marking indicating conformity with European Community harmonised legislation
	Refer to instruction manual/ booklet – Accessory instructions should be read
	Indicates the product is a Medical Device according to EU Medical Device Regulation 2017/745

UK SYMBOL EXPLANATION	
This section is only applicable to United Kingdom (UK) market when UK marking is applied to the Arjo medical device labelling.	
	UK marking indicating conformity with UK Medical Devices Regulations 2002 (SI 2002 No 618, as amended)
<p>UK Responsible Person & UK Importer:</p> <p>Arjo (UK) Ltd, ArjoHuntleigh House, Houghton Regis. LU5 5XF</p> <p>Is the appointed UK Responsible Person as defined in UK Medical Devices Regulations 2002 (SI 2002 No 618, as amended).</p> <p>For Northern Ireland (NI) CE marking will still apply until further amendment to applicable regulations.</p>	

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Intentionally left blank

AUSTRALIA

Arjo Australia
Building B, Level 3
11 Talavera Road
Macquarie Park, NSW, 2113,
Australia
Phone: 1800 072 040

BELGIQUE / BELGIË

Arjo Belgium
Evenbroekveld 16
9420 Erpe-Mere
Phone: +32 (0) 53 60 73 80
Fax: +32 (0) 53 60 73 81
E-mail: info.belgium@arjo.com

BRASIL

Arjo Brasil Equipamentos Médicos Ltda
Rua Marina Ciufuli Zanfelice, 329 PB02
Galpão - Lapa
São Paulo – SP – Brasil
CEP: 05040-000
Phone: 55-11-3588-5088
E-mail: vendas.latam@arjo.com
E-mail: servicios.latam@arjo.com

CANADA

Arjo Canada Inc.
90 Matheson Boulevard West
Suite 350
CA-MISSISSAUGA, ON, L5R 3R3
Tel/Tél: +1 (905) 238-7880
Free: +1 (800) 665-4831
Fax: +1 (905) 238-7881
E-mail: info.canada@arjo.com

ČESKÁ REPUBLIKA

Arjo Czech Republic s.r.o.
Na Strži 1702/65
140 00 Praha
Czech Republic
Phone No: +420225092307
E-mail: info.cz@arjo.com

DANMARK

Arjo A/S
Vassingerødvej 52
DK-3540 LYNGE
Tel: +45 49 13 84 86
Fax: +45 49 13 84 87
E-mail: dk_kundeservice@arjo.com

DEUTSCHLAND

Arjo GmbH
Peter-Sander-Strasse 10
DE-55252 MAINZ-KASTEL
Tel: +49 (0) 6134 186 0
Fax: +49 (0) 6134 186 160
E-mail: info-de@arjo.com

ESPAÑA

ARJO IBERIA S.L.
Polígono Can Salvatella
c/ Cabanyes 1-7
08210 Barberà del Valles
Barcelona - Spain
Telefono 1: +34 900 921 850
Telefono 2: +34 931 315 999

FRANCE

Arjo SAS
2 Avenue Alcide de Gasperi
CS 70133
FR-59436 RONCQ CEDEX
Tél: +33 (0) 3 20 28 13 13
Fax: +33 (0) 3 20 28 13 14
E-mail: info.france@arjo.com

HONG KONG

Arjo Hong Kong Limited
Room 411-414, 4/F, Manhattan Centre,
8 Kwai Cheong Road, Kwai Chung, N.T.,
HONG KONG
Tel: +852 2960 7600
Fax: +852 2960 1711

ITALIA

Arjo Italia S.p.A.
Via Giacomo Peroni 400-402
IT-00131 ROMA
Tel: +39 (0) 6 87426211
Fax: +39 (0) 6 87426222
E-mail: Italy.promo@arjo.com

MIDDLE EAST

Arjo Middle East FZ-LLC
Office 908, 9th Floor,
HQ Building, North Tower,
Dubai Science Park,
Al Barsha South
P.O. Box 11488, Dubai,
United Arab Emirates
Direct +971 487 48053
Fax +971 487 48072
Email: Info.ME@arjo.com

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At Arjo, we believe that empowering movement within healthcare environments is essential to quality care. Our products and solutions are designed to promote a safe and dignified experience through patient handling, medical beds, personal hygiene, disinfection, diagnostics, and the prevention of pressure injuries and venous thromboembolism. With over 6500 people worldwide and 65 years caring for patients and healthcare professionals, we are committed to driving healthier outcomes for people facing mobility challenges.

ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö, Sweden
www.arjo.com

arjo

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Arjo Inc.