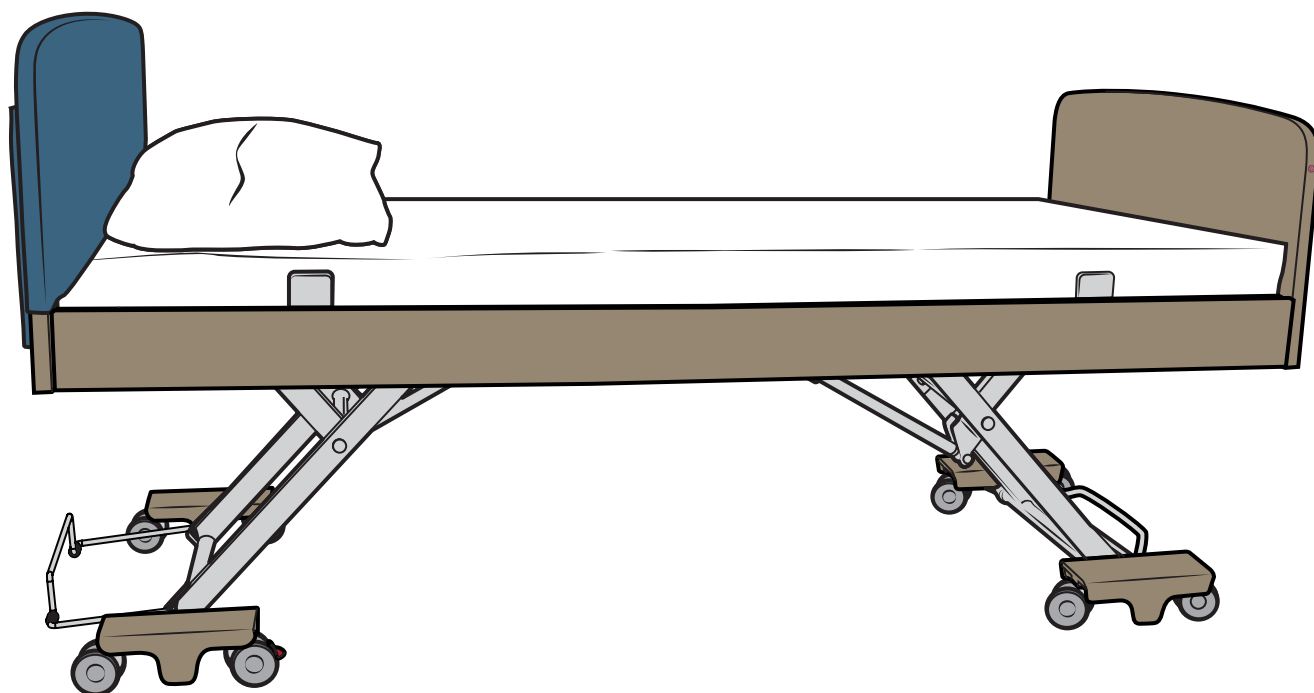
ACCESSORY INSTRUCTIONS

Evenda EV-ACC12, EV-ACC13, EV-ACC17

Textile accessories

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Zubehöranleitung · Instructions relatives aux accessoires · Istruzioni per gli accessori · Accessoire-instructies ·
付属品取扱説明書

English	4
Deutsch	10
Français	16
Italiano	22
Nederlands	28
日本語	34

**WARNING**

To avoid injury, always read these **Accessory Instructions** and accompanied documents before using the product.

Operating instructions - Consult Accessory Instructions.

Design Policy and Copyright

® and ™ are trademarks belonging to the Arjo group of companies.

© Arjo 2023.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice.

The content of this publication may not be copied either whole or in part without the consent of Arjo.

Foreword

Thank you for purchasing the Textile accessories for the Evenda® long term care bed.

Customer contact information

For questions regarding this product, supplies, maintenance, or additional information about Arjo products and service, contact Arjo, an Arjo authorised representative or visit www.arjo.com.

Read and fully understand these Accessory Instructions before using the product

The information contained in this manual is required for the proper installation of the equipment. It will help you protect your product and make sure that the equipment performs to your satisfaction. The information in this manual is important for your safety and must be read and understood to help prevent possible injury. Unauthorised modifications on any Arjo equipment can affect its safety. Arjo cannot be held responsible for any accidents, incidents or loss of performance resulting from any unauthorised modification to its products.

Products covered by this manual

PRODUCT NUMBER	DESCRIPTION
EV-ACC12	Textile pocket for personal items
EV-ACC13	Padded textile cover for the headboard (regular bed)
EV-ACC17	Padded textile cover for the headboard (wide bed)

Definitions used in this manual

WARNING

Warning means: Safety warning. Failure to understand and obey this warning may result in injury to you or others.

CAUTION

Caution means: Failure to follow these instructions may cause damage to all or parts of the system or equipment.

NOTE

Note means: This is important information for the correct use of this system or equipment.

Installation

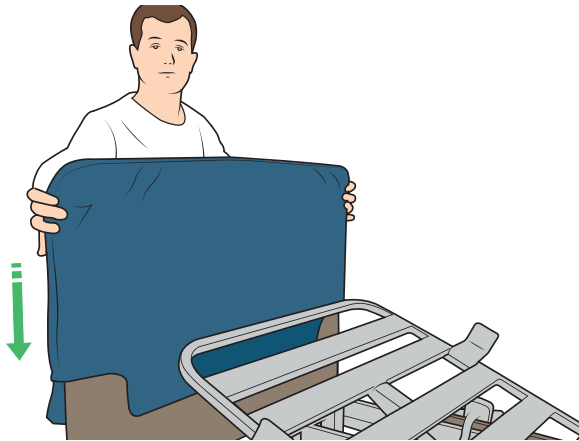
Figure 1

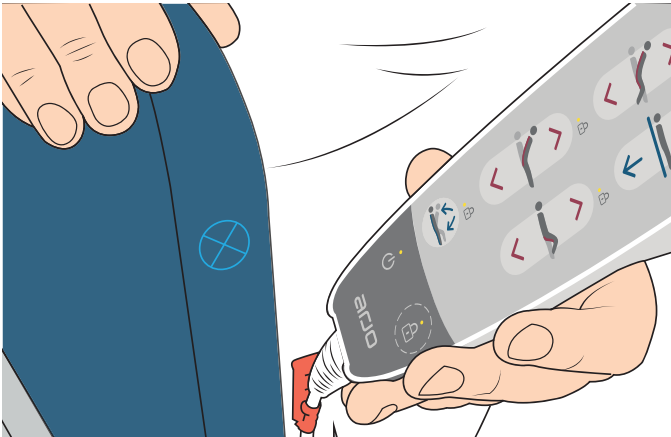
Figure 2

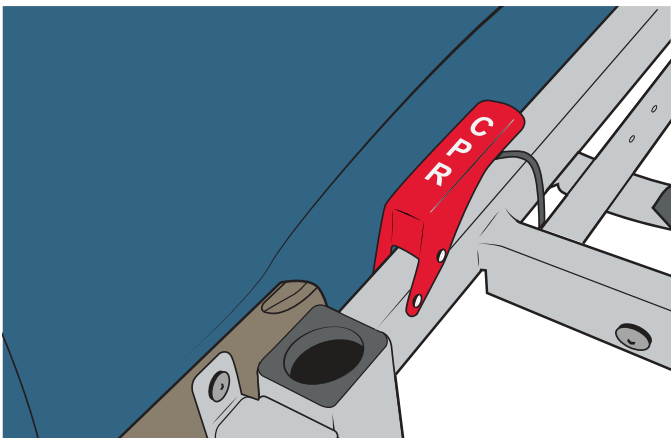
Figure 3

WARNING

To avoid risks of fire, keep any heat source or open flame away from the fabrics at all times.

CAUTION

To prevent loss of functionality, regularly inspect your textile accessories for damage. The fabric, seams and buttons may wear down with time. Do not use a product if it is damaged.

Headboard cover (EV-ACC13, EV-ACC17)

WARNING

To avoid severe injury, keep the CPR handle (if equipped) accessible at all times.

NOTE

The cover must be stretched tightly over the headboard for the magnets to work properly.

1. Slide the cover onto the headboard. If the cover has a flap for the headboard pocket, make sure it is on the back of the headboard. See Figure 1.
2. Make sure that the crosshair markings on the sides of the covers align with the magnets that unlock the hand control. See Figure 2.
3. Tuck the inner flap between the headboard and the frame. Make sure that it does not cover the CPR handle. See Figure 3.

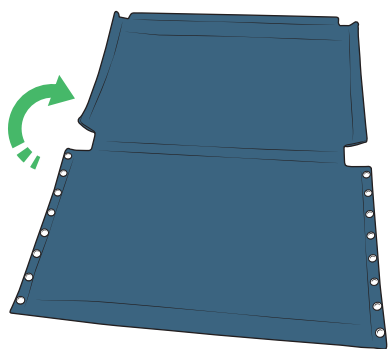
Figure 4

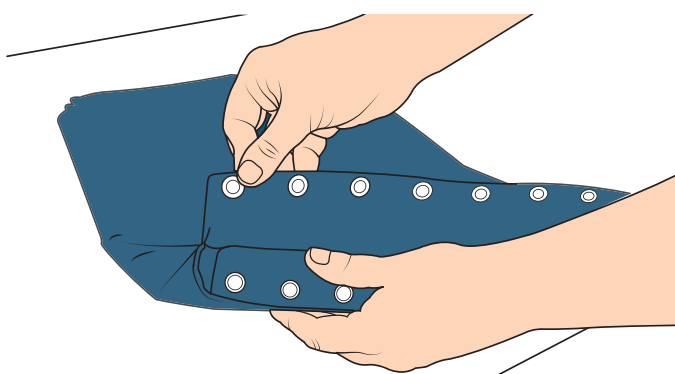
Figure 5

Figure 6

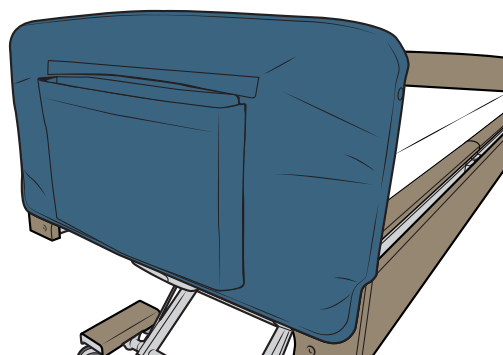
Figure 7

Textile pocket for the headboard cover (EV-ACC12)

The headboard pocket can be used to store a sling or other personal items.

CAUTION

To avoid damage to your personal items, be careful not to overload the pocket with heavy objects. Repeated removals and installations may wear down the buttons and cause the pocket to detach more easily.

1. Lay the textile pocket out flat with the buttons on the top flap facing down. See Figure 4.
2. Fold the pocket in half and fasten the snap buttons to close both sides. See Figure 5.
3. Attach the pocket to the back of the headboard cover using the snap buttons located under the flap. See Figure 6 and Figure 7.

Cleaning and disinfection

 **WARNING**

To avoid risks of infection and cross-contamination, follow the Cleaning and Disinfection instructions presented in this manual.

 **WARNING**

To avoid risks of cross-contamination, inspect the fabric and seams of the textile accessories every time you disinfect them between patients. Stop using the product if any part is damaged or worn out.

CAUTION

To avoid damage to your products, never machine wash or tumble dry the textile accessories.

The textile accessories should be cleaned and disinfected with the rest of the bed before a new patient is placed on the bed. Frequent touch points such as the bed rails should be cleaned daily.

Cleaning

- Wipe the fabrics with a solution of water and a mild detergent (mixing ratio approximately 1:10). Wipe with clean water.
- Vacuum the fabrics regularly.
- Treat stains as quickly as possible. Always remove excess liquids and moisture immediately.

Disinfection

- Wipe with Propanol (65%), Ethanol (65%); or Helipur® (1%) to disinfect the textile surfaces.

Labels

SYMBOL	EXPLANATION
	Reference number
	Serial number
	Name and address of the manufacturer
	Manufacturing date
	Do not wash
	Do not bleach
	Do not dry clean
	Do not tumble dry
	Do not iron
	Wipe surfaces with cleaning solution
	Operating instructions - Consult Accessory Instructions

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Intentionally left blank

Intentionally left blank

AUSTRALIA
Arjo Australia
Building B, Level 3
11 Talavera Road
Macquarie Park, NSW, 2113,
Australia
Phone: 1800 072 040

BELGIQUE / BELGIË
Arjo Belgium
Evenbroekveld 16
9420 Erpe-Mere
Phone: +32 (0) 53 60 73 80
Fax: +32 (0) 53 60 73 81
E-mail: info.belgium@arjo.com

BRASIL
Arjo Brasil Equipamentos Médicos Ltda
Rua Marina Ciufuli Zanfelice, 329 PB02
Galpão - Lapa
São Paulo – SP – Brasil
CEP: 05040-000
Phone: 55-11-3588-5088
E-mail: vendas.latam@arjo.com
E-mail: servicios.latam@arjo.com

CANADA
Arjo Canada Inc.
90 Matheson Boulevard West
Suite 350
CA-MISSISSAUGA, ON, L5R 3R3
Tel/Tél: +1 (905) 238-7880
Free: +1 (800) 665-4831
Fax: +1 (905) 238-7881
E-mail: info.canada@arjo.com

ČESKÁ REPUBLIKA
Arjo Czech Republic s.r.o.
Na Strži 1702/65
140 00 Praha
Czech Republic
Phone No: +420225092307
E-mail: info.cz@arjo.com

DANMARK
Arjo A/S
Vassingerødvej 52
DK-3540 LYNGBE
Tel: +45 49 13 84 86
Fax: +45 49 13 84 87
E-mail: dk_kundeservice@arjo.com

DEUTSCHLAND
Arjo GmbH
Peter-Sander-Strasse 10
DE-55252 MAINZ-KASTEL
Tel: +49 (0) 6134 186 0
Fax: +49 (0) 6134 186 160
E-mail: info-de@arjo.com

ESPAÑA
ARJO IBERIA S.L.
Polígono Can Salvatella
c/ Cabanyes 1-7
08210 Barberà del Valles
Barcelona - Spain
Telefono 1: +34 900 921 850
Telefono 2: +34 931 315 999

FRANCE
Arjo SAS
2 Avenue Alcide de Gasperi
CS 70133
FR-59436 RONCQ CEDEX
Tél: +33 (0) 3 20 28 13 13
Fax: +33 (0) 3 20 28 13 14
E-mail: info.france@arjo.com

HONG KONG
Arjo Hong Kong Limited
Room 411-414, 4/F, Manhattan Centre,
8 Kwai Cheong Road, Kwai Chung, N.T.,
HONG KONG
Tel: +852 2960 7600
Fax: +852 2960 1711

ITALIA
Arjo Italia S.p.A.
Via Giacomo Peroni 400-402
IT-00131 ROMA
Tel: +39 (0) 6 87426211
Fax: +39 (0) 6 87426222
E-mail: Italy.promo@arjo.com

MIDDLE EAST
Arjo Middle East FZ-LLC
Office 908, 9th Floor,
HQ Building, North Tower,
Dubai Science Park,
Al Barsha South
P.O. Box 11488, Dubai,
United Arab Emirates
Direct +971 487 48053
Fax +971 487 48072
Email: Info.ME@arjo.com

NEDERLAND
Arjo Nederland BV
Biezenwei 21
4004 MB TIEL
Postbus 6116
4000 HC TIEL
Tel: +31 (0) 344 64 08 00
Fax: +31 (0) 344 64 08 85
E-mail: info.nl@arjo.com

NEW ZEALAND
Arjo Ltd
34 Vestey Drive
Mount Wellington
NZ-AUCKLAND 1060
Tel: +64 (0) 9 573 5344
Free Call: 0800 000 151
Fax: +64 (0) 9 573 5384
E-mail: nz.info@Arjo.com

NORGE
Arjo Norway AS
Olaf Helsets vei 5
N-0694 OSLO
Tel: +47 22 08 00 50
Faks: +47 22 08 00 51
E-mail: no.kundeservice@arjo.com

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At Arjo, we believe that empowering movement within healthcare environments is essential to quality care. Our products and solutions are designed to promote a safe and dignified experience through patient handling, medical beds, personal hygiene, disinfection, diagnostics, and the prevention of pressure injuries and venous thromboembolism. With over 6500 people worldwide and 65 years caring for patients and healthcare professionals, we are committed to driving healthier outcomes for people facing mobility challenges.

ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö, Sweden
www.arjo.com

arjo

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Arjo Inc.