

INSTRUCTIONS FOR USE

Walker



WARNING

To avoid injury, always read this Instruction for Use and accompanying documents before using the product.

Design Policy and Copyright

® and ™ are trademarks belonging to the Arjo group of companies.

© Arjo 2023.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either whole or in part without the consent of Arjo.

Contents

Foreword	4
Intended Use	5
Safety Instructions	6
Preparations	7
Parts Designation	8
Product Description/Functions	10
Using the Walker	13
Cleaning and Disinfection Instructions	15
Battery Instructions	17
Troubleshooting	18
Care and Preventive Maintenance	19
Technical Specifications	22
Overall Dimensions	24
Labels	26
Electromagnetic Compatibility	28
Parts and Accessories	30

Foreword

Thank you for Purchasing Arjo Equipment

Your Walker aid is a part of a series of quality products designed especially for hospitals, nursing homes and other health care use.

Please contact us if you have any questions about the operation or maintenance of your Arjo equipment.

Please Read this Instructions for Use Thoroughly!

Please read this *Instructions for Use (IFU)* in its entirety before using your Walker aid. Information in this *IFU* is crucial to the proper operation and maintenance of the equipment, will help protect your product, and make sure that the equipment performs to your satisfaction. Some of the information in this *IFU* is important for your safety and must be read and understood to help prevent possible injury.

Unauthorized modifications on any Arjo equipment may affect its safety. Arjo will not be held responsible for any accidents, incidents or lack of performance that occur as a result of any unauthorized modification to its products.

Serious incident

If a serious incident occurs in relation to this medical device, affecting the user, or the patient then the user or patient should report the serious incident to the medical device manufacturer or the distributor. In the European Union, the user should also report the serious incident to the Competent Authority in the member state where they are located.

Service and Support

A service routine has to be done on your Walker aid to ensure the safety and operating procedures of your product. See section *Care and Preventive Maintenance* on page 19.

If you require further information, please contact Arjo for comprehensive support and service programs to maximize the long-term safety, reliability and value of the product.

Contact Arjo for replacement parts. The contact information is found on the last page of this *IFU*.

Definitions in this IFU

WARNING

Means:

Safety warning. Failure to understand and obey this warning may result in injury to you or to others.

CAUTION

Means:

Failure to follow these instructions may cause damage to all or parts of the system or equipment.

NOTE

Means:

This is important information for the correct use of this system or equipment.



Means:

The name and address of the manufacturer.

Intended Use

This equipment is intended for therapeutic raising and walking training of adult residents under the supervision of trained caregivers with the adequate knowledge of the care environment, its common practises and procedures, and in accordance with the guidelines in *Instructions for Use*.

The Walker should only be used for the purpose specified in its Instructions for Use. Any other use is prohibited.

Resident Assessment

We recommend that facilities establish regular assessment routines. Caregivers should assess each resident according to the following criteria prior to use:

- The resident's weight should not exceed 136 kg / 300 lbs.
- The resident must be able to stand and sit in an upright position, normally defined as active
- The resident must understand and be able to respond to instructions to stand and stay seated in an upright position.
- The resident's length must be in the range of 140 and 195 cm (4 feet 7 inches - 6 feet 4 inches).

If the resident does not meet these criteria an alternative equipment/system shall be used.

Installation and Service Requirements

The expected lifetime of this equipment, unless otherwise stated, is ten (10) years, subject to preventive maintenance being carried out in accordance with the instructions for care and maintenance found in the *Instructions for Use*.

Safety Instructions

WARNING

To avoid explosion or fire, never use the equipment in oxygen rich environment, in presence of heat source or flammable anaesthetic gases.

WARNING

To avoid injury, make sure that the patient is not left unattended at any time.

WARNING

To prevent cross-contamination, always follow the disinfection instructions in this IFU.

WARNING

To avoid entrapment, make sure to keep the patient's hair, arms and feet close to the body and use designated grab supports during any movement.

WARNING

To avoid the device from tipping and the patient from falling, do not use the equipment on floor with recessed drains, holes or slopes exceeding a ratio of 1:50 (1.15°).

WARNING

To avoid falling, make sure that the patient is positioned correctly and that the safety belt is being used, properly fastened and tightened.

WARNING

To avoid falling during transfer, always make sure that the brakes are applied on all equipment being used.

WARNING

To avoid falling during patient transfer, always make sure that the brakes on the equipment receiving the patient are applied.

WARNING

To avoid the device in use from tipping, do not raise or lower other equipment close to it and be aware of stationary object when lowering.

WARNING

To avoid entrapment of the patient's or caregiver's legs or feet, make sure that they are kept clear of all obstacles.

Preparations

Actions Before First Use

(9 steps)

- 1 Visually check the package for damage.
- 2 The package should be recycled according to local regulations.
- 3 Check that all parts of the product are supplied. Compare with *Parts Designation* on page 8. If any part is missing or damaged - do NOT use the product!
- 4 Unpack the battery charger. Select and attach the adapter correctly suited to your power outlet. See *Battery Charger IFU*.
- 5 Charge the battery until indicator indicates full charge.
- 6 Disinfect the product according to *Cleaning and Disinfection Instructions* on page 15.
- 7 Prepare an area, dry with good ventilation for storage of the lift.
- 8 Choose a designated area where the *IFU* should be kept, accessible at all times for the users.
- 9 If you have any questions, please contact your local Arjo representative for support and service. The telephone number appears on the last page of this *IFU*.
- 6 If you have any questions, please contact your local Arjo representative for support and service. The telephone number appears on the last page of this *IFU*.

Actions Before Every Use

(6 steps)

- 1 Check that all parts of the lift are in place.
- 2 Carry out a thorough inspection of the sling for damage.
- 3 If any part is missing or damaged - do NOT use the lift!

4 **WARNING**

To prevent cross-contamination, always follow the disinfection instructions in this IFU.

The lift shall be disinfected and wiped dry before every use to prevent cross-contamination.

- 5 Make sure the battery is fully charged.

Walker Directions

The directions *right*, *left*, *rear* and *front* given in this *IFU* refers to the directions in below illustration.

(See Fig. 1)

Avoid to move the lift backwards when using it together with a patient. See illustration below for correct walking direction, (See Fig. 2).

Fig. 1


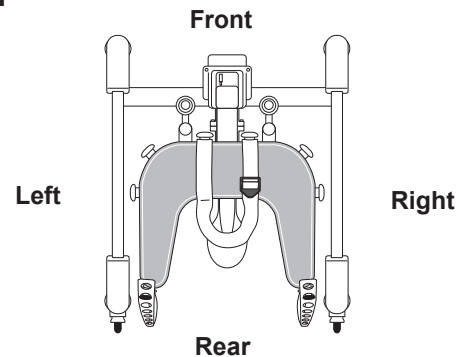




Fig. 2


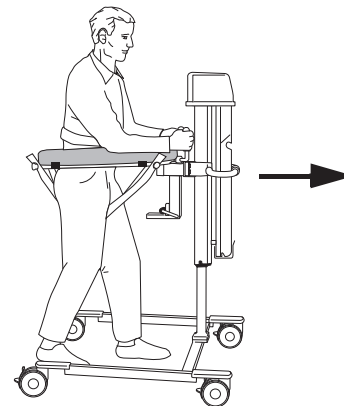


Parts Designation

- 1 Castor with brake (all four castors)
- 2 Pillar (Electrical)
- 3 Hand control (Electrical)
- 4 Battery (Electrical)
- 5 Hand grip
- 6 Support table 
- 7 Safety belt
- 8 Suspension straps
- 9 Spade seat 
- 10 Straight steering device
- 11 Quick coupling
- 12 Lowering button (Hydraulic)
- 13 Hand pump for raising (Hydraulic)
- 14 Pillar (Hydraulic)



Type B. Applied part: protection
against electric shock in accordance
with EN 60601-1.

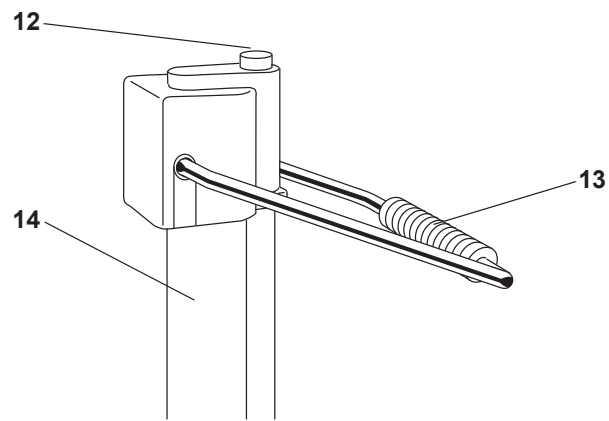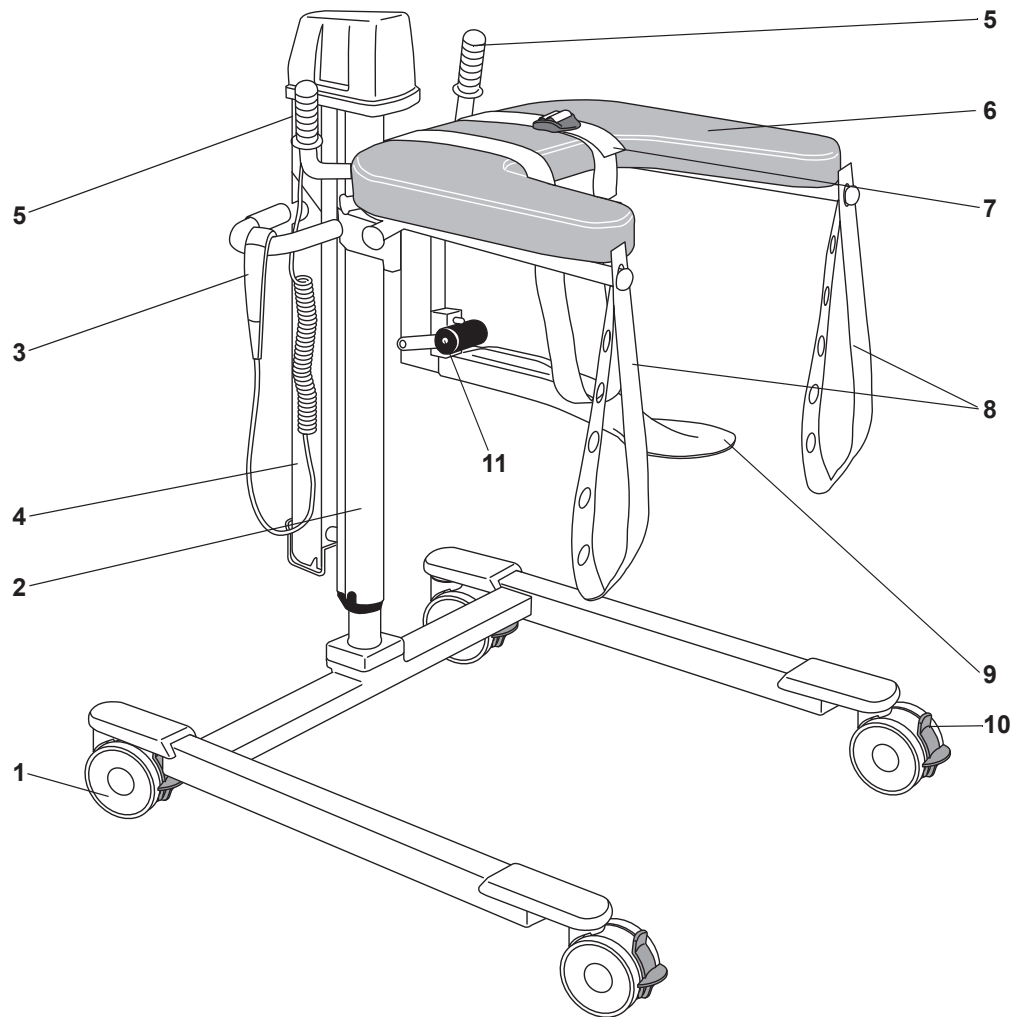

Product Description/Functions

Fig. 1

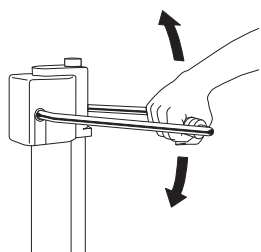


Fig. 2

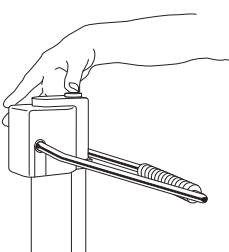


Fig. 3

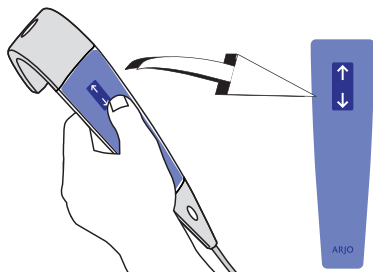


Fig. 4

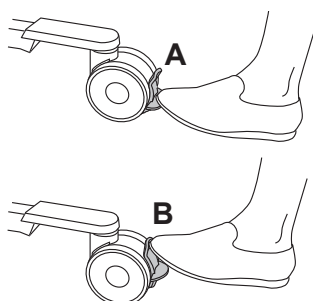


Fig. 5

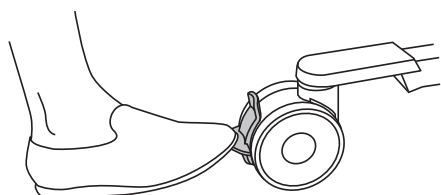


Fig. 6


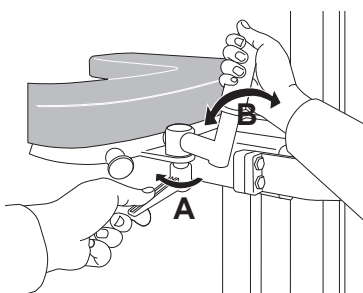


Fig. 7

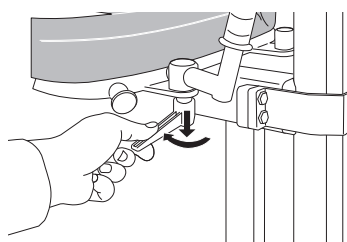

Manual Raising and Lowering

(2 steps)

- 1 The support table is raised by using the hand pump. Pump evenly, using the full range of movement. (See Fig. 1)
- 2 The support table is lowered by pressing the Lowering button. (See Fig. 2)

Electric Raising and Lowering

(2 steps)

- 1 Unhook the hand control from its holder. (See Fig. 3)
- 2 To raise the support table, press the Up arrow button on the hand control. To lower the support table press the Down arrow button.

Castor Brake

(2 steps)

- 1 Apply brakes on the castors by pressing the lock tabs (A) with your foot. (See Fig. 4)
- 2 The castors brakes are released by pressing down the centre tabs (B).

Straight Steering

(1 steps)

To facilitate a transfer, for example in a corridor, the rear castors on the Walker aid are fitted with straight steering devices.

- 1 Press the green tab with your foot to activate straight steering. (See Fig. 5)

Hand Grips

(2 steps)

The two hand grips are ergonomically designed to provide a firm grip for the patient.

- 1 Loosen the locking lever (A) and adjust the hand grips (B) in both position and angle. Fasten the locking lever. (See Fig. 6)
- 2 Place the locking lever (A) in desired position by pushing it down while turning it. (See Fig. 7)

Safety Belt and Suspension Straps

As standard equipment the Walker aid has an individually adjustable safety belt and two suspension straps for body weight reduction and added patient security.

Safety Belt (4 steps)

Depending on what fits the patient the most, the safety belt can be straight across or crossed over the support table.

- 1 The safety belt is attached to the knob on the front of the support table (A). (See Fig. 1 & Fig. 2)
- 2 Pass the safety belt over the support table, behind the patient's back and over the support table again.
- 3 Attach to the other knob on the front of the support table (B).
- 4 Make sure the padding is placed in a convenient position. Adjust safety belt by releasing the hook and loop strap in the padding.

Fig. 1

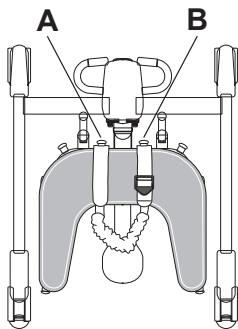


Fig. 2

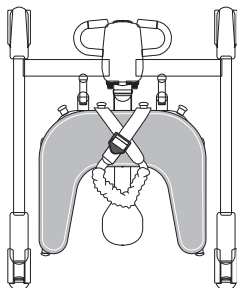


Fig. 3


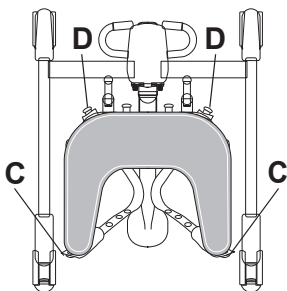


Fig. 4

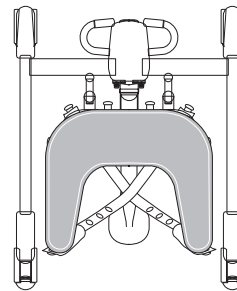

Suspension Straps, Male Patient (3 steps)

- 1 Attach the suspension strap to the knob (C). (See Fig. 3)

NOTE

The suspension strap is attached with its adjustment holes on the rear knobs (C), behind the patient.

- 2 Pass it under the patient and attach it to the knob in front (D).
- 3 Repeat on the other side of the Walker aid.

Suspension Straps, Female Patient (3 steps)

- 1 Attach the suspension strap to the knob (C). (See Fig. 4)

NOTE

The suspension strap is attached with its adjustment holes on the rear knobs (C), behind the patient.


- 2 Pass it under the patient and attach it to the knob on the other side (D).
- 3 Repeat on the other side of the Walker aid.

Spade Seat (3 steps)

The spade seat is used when lifting the patient.

- 1 Attach the spade seat to the support table by gently pushing it into the quick coupling. (See Fig. 5)

Fig. 5

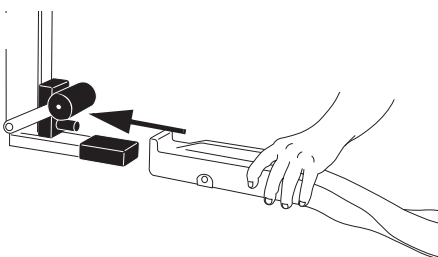

Continue with the steps on the next page.

Fig. 6

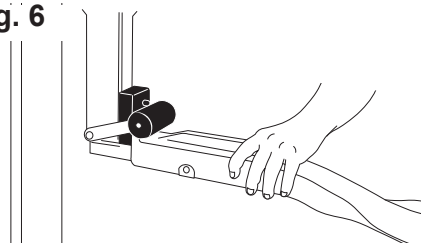


Fig. 7

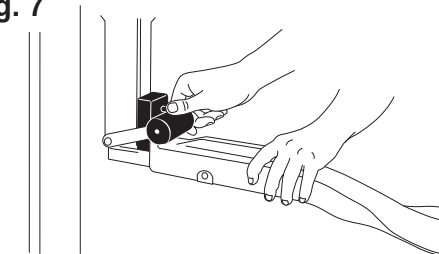


Fig. 1

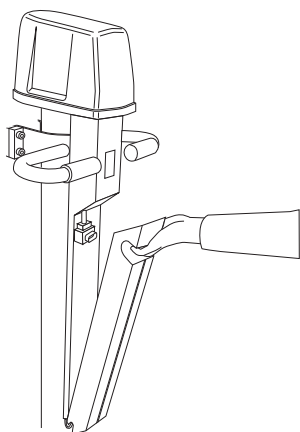


Fig. 2


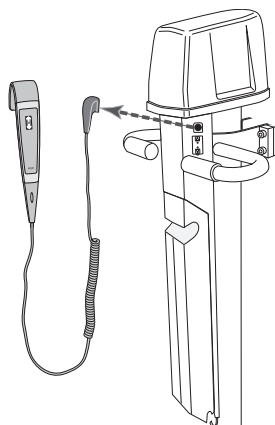


Fig. 3

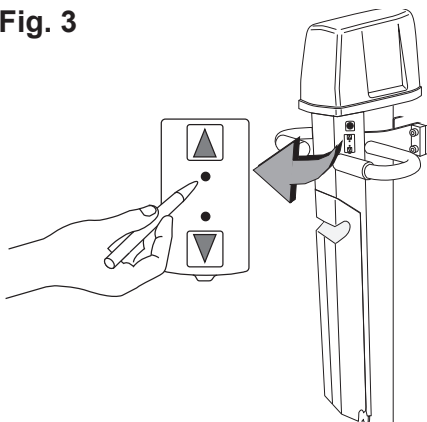

- 2 Before lifting the patient, make sure the spade is securely supported and locked by the quick coupling. **(See Fig. 6)**
- 3 Loosen the spade seat by pressing the ejector and lift the quick coupling upwards, holding the spade seat with the other hand. **(See Fig. 7)**

Emergency Stop

If, for any reason, the Walker aid does not respond to the control buttons, you can quickly stop lift movement by pressing the button on the battery and disconnect it. **(See Fig. 1)**

Emergency High/Low (4 steps)

If, for any reason, the Walker aid does not respond to the control buttons, raise or lower the Walker aid by using the Emergency High/Low.

- 1 Unplug the hand control. **(See Fig. 2)**
- 2 Control the Walker aid by pressing a blunt, thin object into the holes at the panel. (i.e. a pen). **(See Fig. 3)**

CAUTION

Do not use an object with sharp tip when pressing in the holes.

- 3 Press it into the hole next to the “up arrow” in order to raise the Walker aid.
- 4 Press it into the hole next to the “down arrow” in order to lower the Walker aid.

NOTE

A continuous beeping sound is heard during action “up” and “down”.

Contact your local Arjo representative before re-using the Walker aid.

Using the Walker

Fig. 1


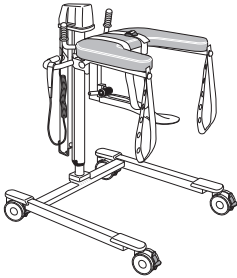


Fig. 2

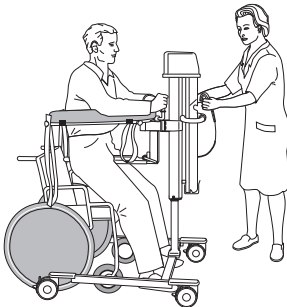

From Wheelchair or Bed Without Heavy Lifting

(19 steps)

- 1 Attach the spade seat and the safety belts to the Walker aid. (Let the belts hang on the knob from where you attach it.) (See Fig. 1)
- 2 Brake the wheels on the wheelchair/bed.
- 3 Make sure the patient is sitting safely in the wheelchair or on the bedside.
- 4 Move the Walker aid so the spade seat is next to the wheelchair seat/bedside.
- 5 Adjust the spade seat height so it is in level with the wheelchair seat /bedside.
- 6 If possible, move the spade seat gently in over the wheelchair/ bed.
- 7 Apply brakes on all castors of the Walker aid.
- 8 Let the patient grip the hand grips. If necessary, adjust these.
- 9 Move the patient over onto the spade seat. (See Fig. 2)

NOTE

Be carefully while transferring patient to/from the spade or attaching the suspension straps. Make sure the patient's skin is not pinched by the spade seat or suspension straps

- 10 Attach and adjust the safety belt.
- 11 Raise the patient to suitable height.


WARNING

To avoid the device in use from tipping, do not raise or lower other equipment close to it and be aware of stationary object when lowering.

- 12 Release the castor brakes and pull the Walker aid out from the wheelchair/bed.
- 13 Apply brakes on all castors of the Walker aid.

Continue with the steps on the next page.

Fig. 3

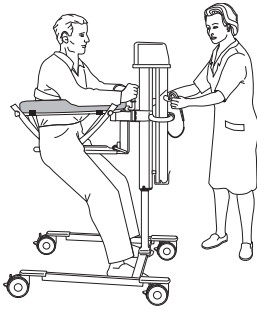


14 Attach and adjust the suspension straps. (See Fig. 3)

NOTE


The suspension strap is attached with its adjustment holes on the rear knobs, behind the patient.

Fig. 4

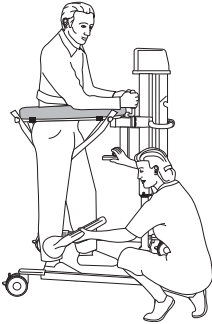

15 Loosen the spade seat. (See Fig. 4)


Fig. 5

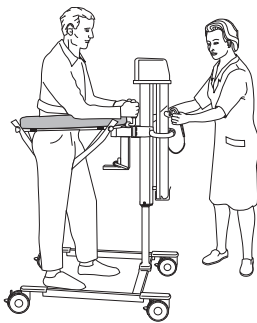

16 Adjust the height of the Walker aid. The height adjustment permits variable body weight reduction. (See Fig. 5)

17 Readjust the hand grips.

Fig. 6

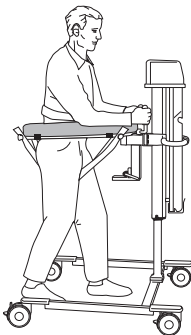

18 The training may start. (See Fig. 6)

19 If desired, apply straight steering.

WARNING

To avoid entrapment, make sure to keep the patient's hair, arms and feet close to the body and use designated grab supports during any movement.

Cleaning and Disinfection Instructions

Arjo Disinfectant

For optimal performance use only Arjo Disinfectant.

For any questions regarding the proper disinfection procedure or for liquid product ordering information (see *Parts and Accessories on page 30*), contact Arjo Customer Service. Disinfection will minimize cross-contamination and removing tissue residue.

WARNING



To avoid eye and skin damage, always use protective glasses and gloves. If contact occurs rinse with plenty of water. If eyes or skin becomes irritated, contact a physician. Always read the IFU and the Material Safety Data Sheet (MSDS) of the disinfectant.

CAUTION

To avoid damage on the equipment only use Arjo branded disinfectants.

WARNING

To prevent cross-contamination, always follow the disinfection instructions in this IFU.

WARNING

To avoid eye or skin irritation, never disinfect in the presence of a patient.

Accessories for Disinfecting Walker

- Protective gloves
- Protective glasses
- Spray bottle with disinfectant
- Spray bottle with water
- Cloth – wet and dry
- Disposable towels
- Soft bristled brush

Always Follow these 11 Steps for Proper Disinfection


If *Arjo Shower Panels* is used, see respective *IFU*.

NOTE

Remove any removable parts, such as battery and suspension straps, the safety belt and its cover and clean these separately according to the cleaning instructions.

Removing visual residue (Step 1 of 11)

- 1 Clean the Walker aid and its removable parts from visible residue using a cloth soaked in water, alternatively spray with water and wipe with a clean cloth. Start from top and move downwards.

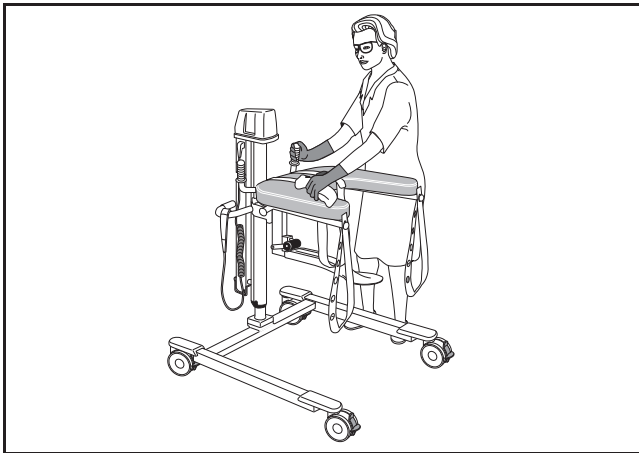

Cleaning (Step 2 to 5 of 11)

- 2 Spray disinfectant on the Walker aid and its removed parts (including the suspension straps, the safety belt and its cover).
- 3 Use a brush or a cloth if needed for cleaning to remove any deposits.
- 4 Use a new wet cloth to wipe off all traces of disinfectant or if more suitable by spraying water and wipe with a clean cloth. Pay attention to the care and washing symbols on the seat belt cover and perform a gentle cleaning.
- 5 Repeat previous step until all of the disinfectant has been removed.

Disinfection (Step 6 to 11 of 11)

- 6 Spray disinfectant on the Walker aid and its removed parts.
- 7 Allow appropriate contact time of the disinfectant according to the instructions on the bottle.

- 8 Use a new wet cloth to wipe off all traces of disinfectant or if more suitable by spraying water and wipe with a clean cloth. Pay attention to the care and washing symbols on the seat belt cover and perform a gentle disinfection.
- 9 If disinfectant cannot be removed, spray water on the affected part and wipe off with disposable towels.
- 10 Repeat previous step until all of the disinfectant has been removed.
- 11 Let all parts dry.

Battery Instructions

WARNING

To avoid bodily injury, do NOT crush, puncture, open, dismantle or otherwise mechanically interfere with the battery.

- **Should the battery casing crack and cause contents to come in contact with skin or clothing, rinse immediately with plenty of water.**
- **If contents come in contact with the eyes, rinse immediately with plenty of water and seek medical attention.**

Inhalation of the contents can cause respiratory irritation. Provide fresh air and medical attention.

- Be careful not to drop the battery.
- Contact the appropriate local authority for advice when disposing batteries.
- Check label on the battery.

Low Charge Warning

If the battery needs charging a buzzing tone is heard when the lift is activated. At this time, there will be enough battery power to complete the immediate task.

Storage of Battery

- The battery is delivered charged but we recommend you to recharge the battery when received due to a slow self discharge.
- The battery will slowly self discharge when not used.
- A battery during storage and transport should be in a temperature range of -0° C to +30 °C (32 °F to 86 °F), cooler temperatures will extend its service life.
- For maximum battery performance do not store the battery above 50 °C (122 °F).

Installation of Charger

See Battery Charger IFU.

How to Charge the Battery

See Battery Charger IFU.

Troubleshooting

PROBLEM	ACTION
The lift continues to raise or descend after the hand control button is released.	Activate the red emergency Stop button and pull the battery out - remove the patient from the lift and contact qualified personnel.
The lift does not raise or descend when the hand control button is pressed.	<ul style="list-style-type: none"> • Make sure that the battery is fully engaged in the battery compartment. • Make sure that the hand control connection plug is properly engaged in the socket. • Make sure that the battery is charged. <div style="border: 1px solid black; padding: 5px; margin-top: 10px;"> <p>WARNING</p> <p>To avoid injury, make sure that the patient is not left unattended at any time.</p> </div> <ul style="list-style-type: none"> • Use the emergency lowering to descend the patient, see <i>Emergency High/Low on page 12</i>. Contact qualified personnel as soon as the patient is removed from the lift.
The lift is hard to manoeuvre during the transfer.	<ul style="list-style-type: none"> • Make sure all brakes are released. • Make sure all castors role and swivel freely. • If the problem persists, remove the patient from the lift and contact qualified personnel.
The lift is making abnormal sounds during lifting or transfer.	Remove the patient from the lift and contact qualified personnel.
The lift beeps shortly once every minute when not operated.	Contact qualified personnel.
It is only possible to lower the lift.	The lifting device of this equipment is equipped with a safety nut. If the safety nut will be activated it will only be possible to lower the equipment into a safe position. In case of this scenario stop using the equipment and contact qualified personnel for repair.

Care and Preventive Maintenance

The Walker aid is subject to wear and tear, and the following actions must be performed when specified to make sure the product remains within its original manufacturing specification.

WARNING

To avoid malfunction resulting in injury, make sure to conduct regular inspections and follow the recommended maintenance schedule. In some cases due to heavy use of the product and exposure to aggressive environment more frequent inspections should be carried out. Local regulations and standards may be more stringent than the recommended maintenance schedule.

NOTE

Product cannot be maintained and serviced while in use with the patient

Preventive Maintenance Schedule: Walker aid

CAREGIVER OBLIGATIONS Action/Check	Every DAY	Every WEEK	Every YEAR	Every 2nd YEAR
Cleaning/Disinfection	X			
Visually check all exposed parts		X		
Visually check mechanical attachments		X		
Visually check battery and battery charger (Electrical version)		X		
Check hand control and cable (Electrical version)		X		
Check the safety belt and suspension straps		X		
Perform functionality test		X		
Check and clean castors		X		
Yearly checks by qualified personnel only			X	
Replace safety belt and suspension strap				X

WARNING

To avoid injury to both the patient and the caregiver, never modify the equipment or use incompatible parts.

Fig. 1

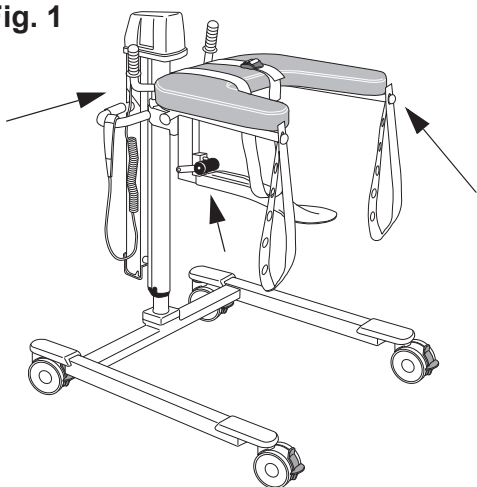


Fig. 2

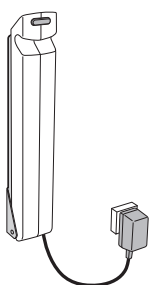


Fig. 3


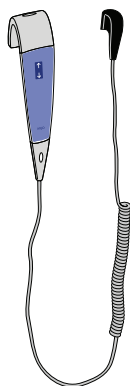


Fig. 4

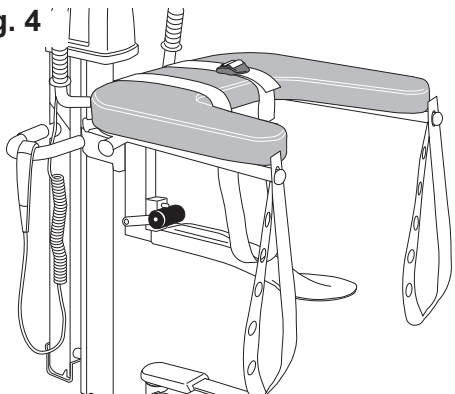

Caregiver Obligations

Caregiver obligations shall be carried out by personnel with sufficient the Walker aid knowledge following the instructions in this *IFU*.

Daily

- **Disinfect:** The Walker aid has to be cleaned immediately after every use. Arjo disinfectant should be used in recommended concentrations labelled on the disinfectant bottle.

CAUTION


Do not use petroleum based solvents, trichlorethylene etc., since they may damage the plastic.

Every week

- **Visually check all exposed parts:** Examine the Walker aid for damage. Visually check all exposed parts, especially where personal contact is made by either the patient or caregiver.
- Make sure no cracks, sharp edges or unhygienic areas that could cause the patient or user injury.
- Replace damaged parts.
- **Visually check mechanical attachments:** Visual check that all screws and nuts are tightened and that there are no gaps. (See Fig. 1)
- Check the hydraulic system is working smoothly (raising and lowering).
- **Visually check the battery and the battery charger:** Check the cable condition as well. Withdraw and replace immediately if damaged. (Electric version) (See Fig. 2)
- **Check hand control and cable:** Visually check the condition of the hand control and its cable. If found cut or damaged, replace. (Electric version) (See Fig. 3)
- **Visually check the safety belt and suspension straps:** Check the complete length for fraying, cuts, crack, tears, that no inner fabric is showing and that no other damage is visible. If the safety belt or suspension strap is found damaged, replace it. (See Fig. 4)
- **Perform functionality test:** Test up/down motion by pressing up/down. Test emergency High/Low.

Continue with the steps on the next page.

Fig. 5

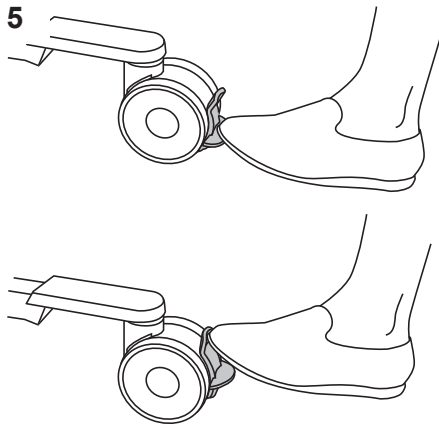

- **Check and clean the castors:** Make sure the castors are properly fixed and are rolling and swivelling freely (the function can be affected by soap, hair, dust and chemicals from floor cleaning). (See Fig. 5)
- If the function is disturbed, clean the castors with water.
- Visual check that the castors are tightened to the chassis.

Yearly Checks by Qualified Personnel Only

The Walker aid must be serviced once a year in accordance with the *Maintenance and Repair Manual*.

Contact your local Arjo service representative to sign for a service agreement.



WARNING

To avoid injury and/or unsafe product, the maintenance activities must be carried out at the correct frequency by qualified personnel using correct tools, parts and knowledge of procedures. Qualified personnel must have documented training in maintenance of this device.

NOTE

All Caregiver Obligations are to be checked when performing the Qualified Personnel Service. For details, see separate service instructions.

Technical Specifications

Walker Hydraulic	
Safe Working Load (SWL) (Max patient weight)	136 kg (300 lbs.)
Maximum weight of lift	44 kg (97 lbs.)
Maximum total weight of lift (Lift + Patient)	180 kg (397 lbs)
Medical equipment	Type 
Walker Electric	
Lifting capacity	136 kg (300 lbs.)
Maximum weight of lift	53 kg (117 lbs.)
Battery weight	4.4 kg (9.7 lbs.)
Maximum total weight of lift (Lift + Patient)	189 kg (417 lbs)
Medical equipment	Type 
Power Source	24V DC
Protection class	IP X4
Fuse	F1 F10AL 250 V
Insulation Class	Class II Equipment
Operating forces of controls	2.7 N
Sound level	65 dB
Transport, Storage and Operation (Applies to all Walker models except charges)	
Temperature range	-20 °C to +70 °C (-4 °F to +158 °F) Transport and Storage +10 °C to +40 °C (+50 °F to +104 °F) Operation
Relative humidity range	10% - 80% Transport and Storage 30% - 75% Operation
Atmospheric pressure range	500 - 1100 hPa Transport and Storage 800 - 1060 hPa Operation
Mode of operation	ED maximum 10%; Maximum 1 minute ON; Minimum 9 minutes OFF
Start force	85 N
Rolling force	20 N
Push button force	<5 N
Brake force	105 N
Lever force	20 N
Environmental	
Pollution degree	2

Recycling

Package	Wood and corrugated cardboard, recyclable
---------	---

End of Life Disposal

- All batteries in the product must be recycled separately. Batteries are to be disposed in accordance with national or local regulations.
- Slings including stiffeners/stabilizers, padding material, any other textiles or polymers or plastic materials etc. should be sorted as combustible waste.
- Lift systems having electrical and electronic components or an electrical cord should be disassembled and recycled per Waste of Electrical and Electronic Equipment (WEEE) or in accordance with local or national regulation.

Components that are primarily made up of different kinds of metal (containing more than 90% metal by weight) for example sling bars, rails, upright supports, etc., should be recycled as metals.

WARNING

To avoid injury, always follow the allowed combinations listed in this IFU. No other combinations are allowed

Allowed Combinations

Power	Model
Battery 24 V	NDA0100-03
Battery charger	NDA1200-EU, NDA2200-EU, NDA4200-US, NDA6200-AU and NDA8200-INT

Overall Dimensions

Walker Electric

mm (inches)


A minimum 400 (15 3/4") gives

B minimum 1165 (45 7/8")

A maximum 965 (38") gives

B maximum 1730 (68 1/8")

Stroke maximum 565 (22 1/4")

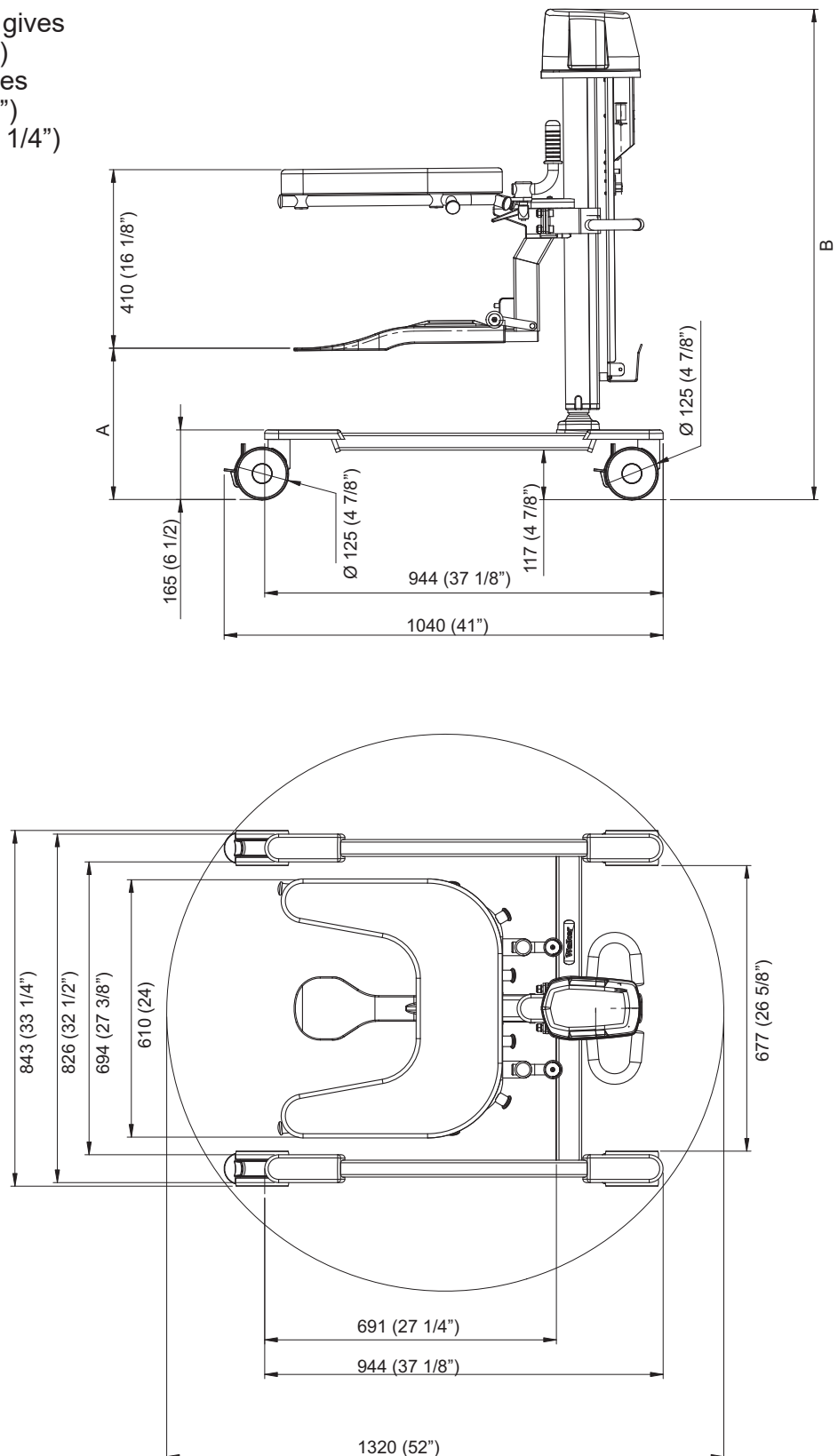

Walker Hydraulic

mm (inches)


A minimum. 350 (14 1/8") gives

B minimum 1140 (44 7/8")

A maximum 850 (33 7/8") gives

B maximum 1640 (64 5/8")

Stroke maximum 500 (19 5/8")

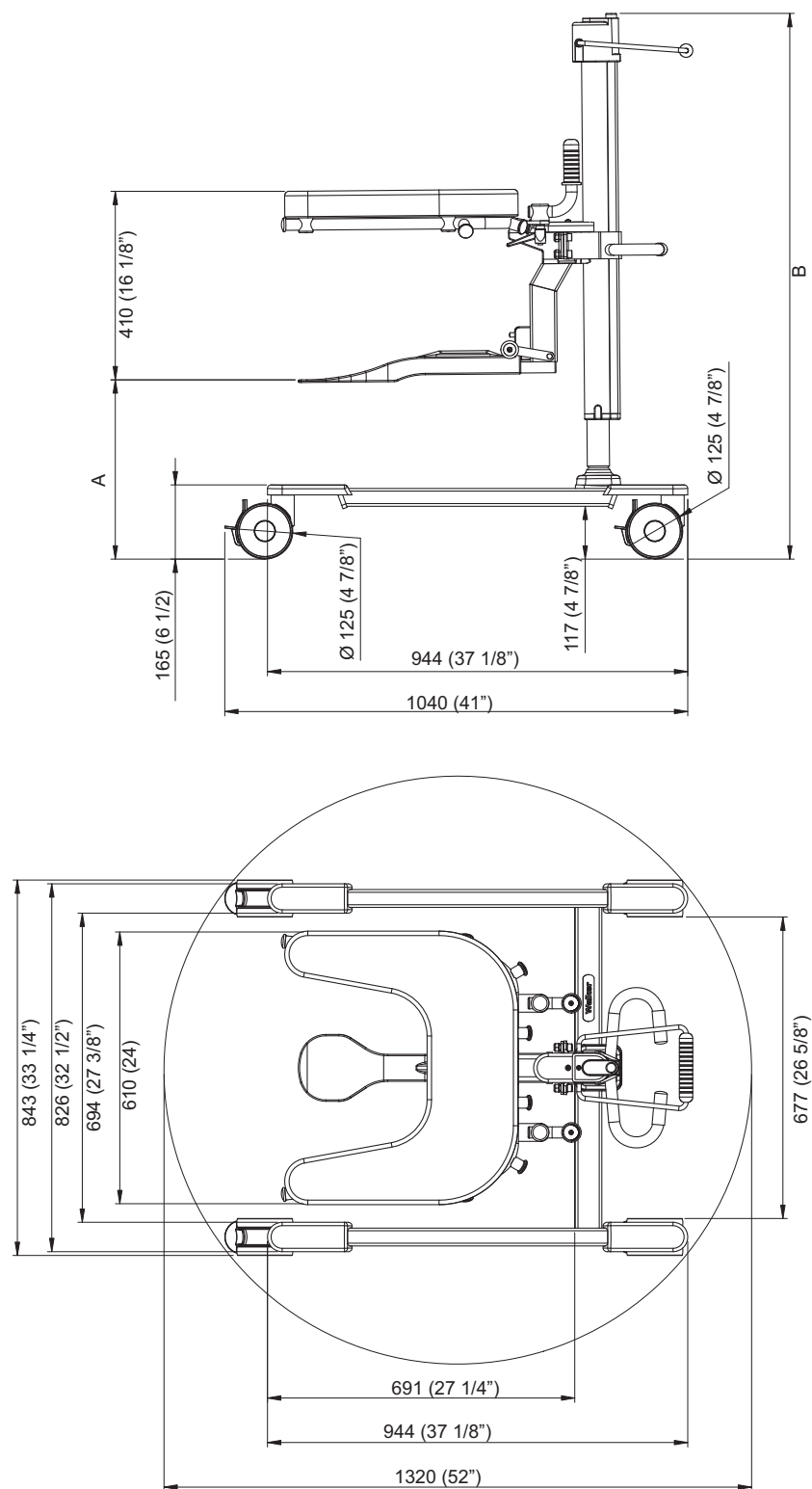

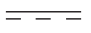











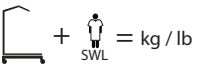



Labels

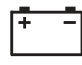


Label Explanation




Battery Label	States safety and environmental information for the battery.
Identification Label	States the product identification; serial number, year and month of manufacturing.
Data Label	States technical performance and requirements, e.g. the Input Power, Input Voltage and maximum Patient Weight etc.
Tested with Weight Label	States the date and the weight that was used when validating the product.


Symbol Explanation

SWL	Safe Working Load (max total load)
	Total mass of equipment including its safe working load.
	Lifting stroke
	Supply voltage
24V DC	Supply voltage
MAX: 150 VA	Maximum power
	Fuse FA - F10AL 250V
IP X4	Degree of protection (i.e. the product is protected against splashing water)
NDA0100-XX	Product no. of the battery
ED max 10%	Mode of operation for Hi/Lo function: ED maximum 10%; Maximum 1 minute ON; Minimum 9 minutes OFF
	Type B, applied part: protection against electrical shock in accordance with IEC 60601-1.
	Read the <i>IFU</i> before use.

	Separate electrical and electronic components for recycling in accordance with the European Directive 2012/19/EU (WEEE)
	A battery is the power source of this equipment.
	Environmental danger - contains lead. Not for disposal.
	Recyclable
	CE marking indicating conformity with European Community harmonised legislation
	Indicates the product is a Medical Device according to EU Medical Device Regulation 2017/745
	Classified by Underwriters Laboratories Inc. with respect to electrical shock, fire, mechanical hazards and other specified hazards only in accordance with: AAMI ES60601-1:2005, ES60601-1:2005/AMD1 1:2012, ES60601-1:2005/AMD2:2021 CSA-C22.2 No. 60601-1:14, Amendment 2:2022 (MOD) to CAN/CSA-C22.2 No. 60601-1:14.
	Name and address of the manufacturer
	Manufacturing date
	Unique device identifier

	Recommended wash temperature
	Do not dry clean
	Do not iron

	Do not bleach
	Tumble dry on low heat
 <ul style="list-style-type: none"> • Cl 1000 ppm • 70% IPA 	Wipe clean with either/or: <ul style="list-style-type: none"> • Chlorine, up to 1000 ppm • Isopropyl Alcohol (IPA), up to 70%

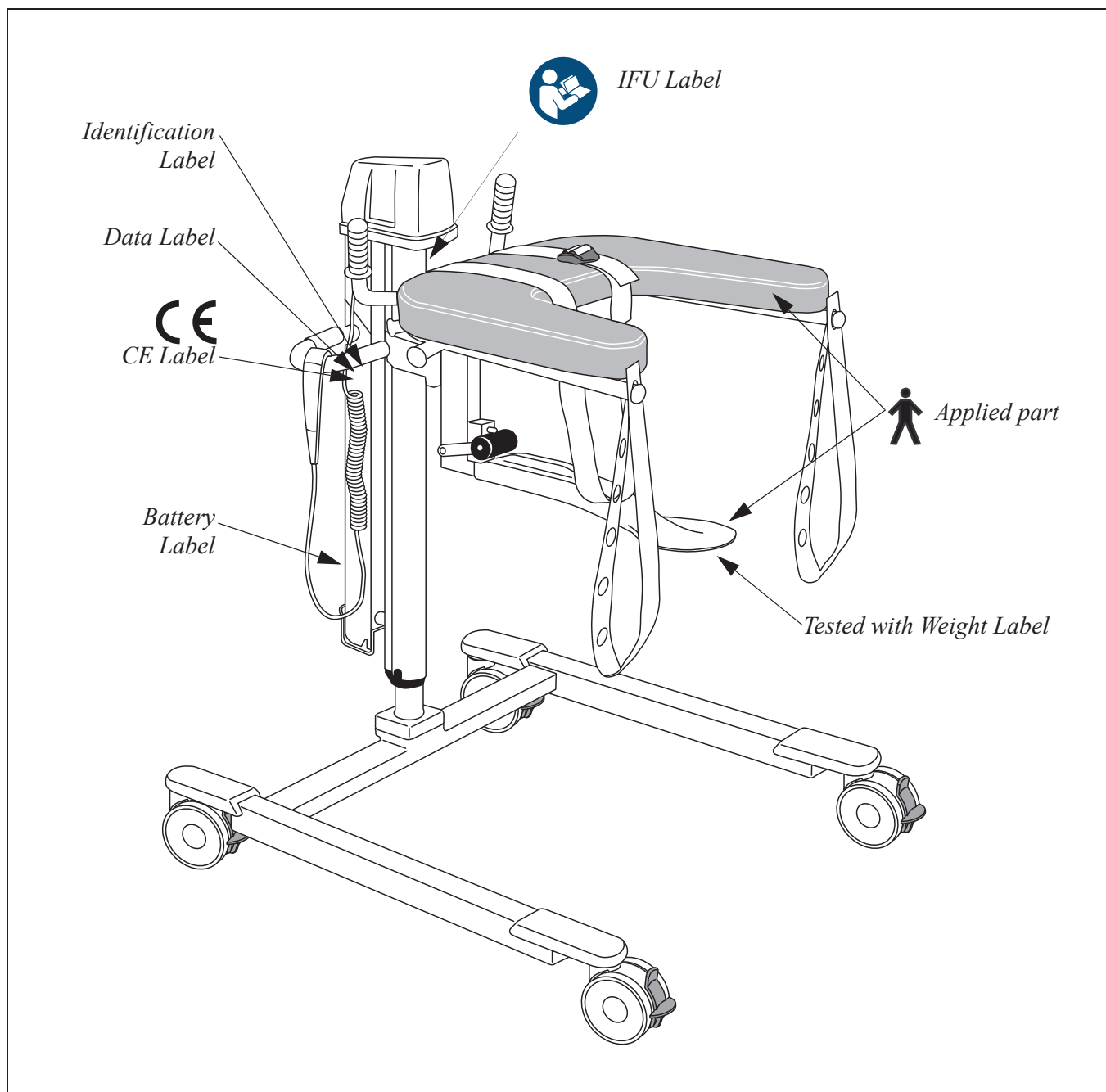

Electromagnetic Compatibility (EMC)

Product has been tested for compliance with current regulatory standards regarding its capacity to block EMI (electromagnetic interference) from external sources.

Some procedures can help reduce electromagnetic interferences:

Ensure that other devices in patient-monitoring and/or life-support areas comply to accepted emissions standards.

WARNING

Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

WARNING

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Walker, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

WARNING

Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.


Intended Environment: Professional Healthcare Facility Environment

Exceptions: HF Surgical Equipment and the RF Shielded room of an ME SYSTEM for magnetic resonance imaging


Guidance and manufacturer's declaration – electromagnetic emission		
Emission Test	Compliance	Guidance
RF emissions CISPR 11	Group 1	This equipment uses RF energy only for its internal functions. Therefore its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	This equipment is suitable for use in all establishments, including domestic establishments and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes.

Guidance and manufacturer's declaration – electromagnetic immunity		
Immunity test	IEC 60601-1-2 test level	Compliance level
Electrostatic discharge (ESD) EN 61000-4-2	±2kV, ±4kV, ±8kV, ±15kV air ±8kV contact	±2kV, ±4kV, ±8kV, ±15kV air ±8kV contact
Conducted disturbances induced by RF fields EN 61000-4-6	3V in 0,15 MHz to 80 MHz 6V in ISM and amateur radio bands between 0,15 MHz and 80 MHz 80% AM at 1 kHz	3V in 0,15 MHz to 80 MHz 6V in ISM and amateur radio bands between 0,15 MHz and 80 MHz 80% AM at 1 kHz
Radiated RF electromagnetic field EN 61000-4-3	Professional Healthcare environment 3 V/m 80 MHz to 2,7 GHz 80% AM at 1 kHz	Professional Healthcare environment 3 V/m 80 MHz to 2,7 GHz 80% AM at 1 kHz
Proximity fields from RF wireless communications equipment EN 61000-4-3	385 MHz - 27 V/m 450 MHz - 28 V/m 710, 745, 780 MHz - 9V/m 810, 870, 930 MHz - 28 V/m 1720, 1845, 1970, 2450 MHz - 28 V/m 5240,5500, 5785 MHz - 9V/m	385 MHz - 27 V/m 450 MHz - 28 V/m 710, 745, 780 MHz - 9V/m 810, 870, 930 MHz - 28 V/m 1720, 1845, 1970, 2450 MHz - 28 V/m 5240,5500, 5785 MHz - 9V/m
Electrical fast transient/burst EN 61000-4-4	±1kV SIP/SOP ports 100 kHz repetition frequency	±1kV SIP/SOP ports 100 kHz repetition frequency
Power frequency Magnetic field EN 61000-4-8	30 A/m 50 Hz or 60 Hz	30 A/m 50 Hz
Proximity magnetic fields EN 61000-4-39	134,2 kHz - 65 A/m 13,56 MHz - 7,5 A/m	134,2 kHz - 65 A/m 13,56 MHz - 7,5 A/m


Parts and Accessories

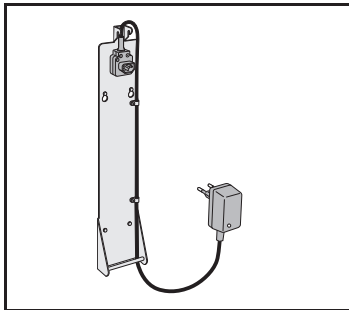


Battery Charger
NDAX000–XX

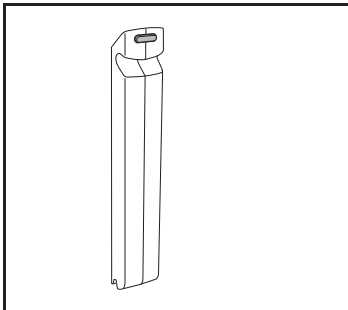

Extra battery
NDA 0100

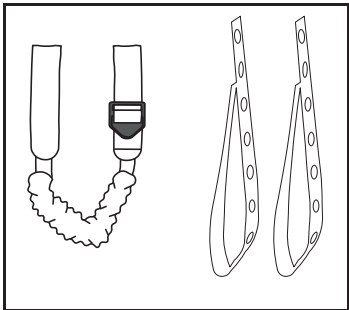


GCA0010–031
Safety belt, padding and
suspension straps

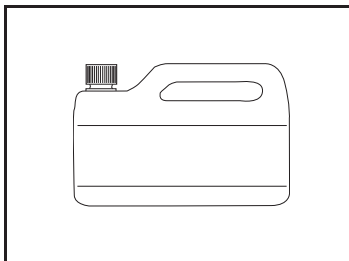

For disinfectant contact
your local representative

UK Symbol explanation

This section is only applicable to United Kingdom (UK) market when UK marking is applied to the Arjo medical device labelling.	
	UK marking indicating conformity with UK Medical Devices Regulations 2002 (SI 2002 No 618, as amended)
<p>UK Responsible Person & UK Importer:</p> <p>Arjo (UK) Ltd, ArjoHuntleigh House, Houghton Regis. LU5 5XF</p> <p>Is the appointed UK Responsible Person as defined in UK Medical Devices Regulations 2002 (SI 2002 No 618, as amended).</p> <p>For Northern Ireland (NI) CE marking will still apply until further amendment to applicable regulations.</p>	

AUSTRALIA

Arjo Australia
Building B, Level 3
11 Talavera Road
Macquarie Park, NSW, 2113,
Australia
Phone: 1800 072 040

BELGIQUE / BELGIË

Arjo Belgium
Evenbroekveld 16
9420 Erpe-Mere
Phone: +32 (0) 53 60 73 80
Fax: +32 (0) 53 60 73 81
E-mail: info.belgium@arjo.com

BRASIL

Arjo Brasil Equipamentos Médicos Ltda
Rua Marina Ciufuli Zanfelice, 329 PB02
Galpão - Lapa
São Paulo - SP - Brasil
CEP: 05040-000
Phone: 55-11-3588-5088
E-mail: vendas.latam@arjo.com
E-mail: servicios.latam@arjo.com

CANADA

Arjo Canada Inc.
90 Matheson Boulevard West
Suite 350
CA-MISSISSAUGA, ON, L5R 3R3
Tel/Tél: +1 (905) 238-7880
Free: +1 (800) 665-4831
Fax: +1 (905) 238-7881
E-mail: info.canada@arjo.com

ČESKÁ REPUBLIKA

Arjo Czech Republic s.r.o.
Na Strži 1702/65
140 00 Praha
Czech Republic
Phone No: +420225092307
E-mail: info.cz@arjo.com

DANMARK

Arjo A/S
Vassingerødvej 52
DK-3540 LYNGE
Tel: +45 49 13 84 86
Fax: +45 49 13 84 87
E-mail: dk_kundeservice@arjo.com

DEUTSCHLAND

Arjo GmbH
Peter-Sander-Strasse 10
DE-55252 MAINZ-KASTEL
Tel: +49 (0) 6134 186 0
Fax: +49 (0) 6134 186 160
E-mail: info-de@arjo.com

ESPAÑA

ARJO IBERIA S.L.
Poligono Can Salvatella
c/ Cabanyes 1-7
08210 Barberà del Valles
Barcelona - Spain
Telefono 1: +34 900 921 850
Telefono 2: +34 931 315 999

FRANCE

Arjo SAS
2 Avenue Alcide de Gasperi
CS 70133
FR-59436 RONCQ CEDEX
Tél: +33 (0) 3 20 28 13 13
Fax: +33 (0) 3 20 28 13 14
E-mail: info.france@arjo.com

HONG KONG

Arjo Hong Kong Limited
Room 411-414, 4/F, Manhattan Centre,
8 Kwai Cheong Road, Kwai Chung, N.T.,
HONG KONG
Tel: +852 2960 7600
Fax: +852 2960 1711

ITALIA

Arjo Italia S.p.A.
Via Giacomo Peroni 400-402
IT-00131 ROMA
Tel: +39 (0) 6 87426211
Fax: +39 (0) 6 87426222
E-mail: Italy.promo@arjo.com

MIDDLE EAST

Arjo Middle East FZ-LLC
Office 908, 9th Floor,
HQ Building, North Tower,
Dubai Science Park,
Al Barsha South
P.O. Box 11488, Dubai,
United Arab Emirates
Direct +971 487 48053
Fax +971 487 48072
Email: Info.ME@arjo.com

NEDERLAND

Arjo Nederland BV
Biezenwei 21
4004 MB TIEL
Postbus 6116
4000 HC TIEL
Tel: +31 (0) 344 64 08 00
Fax: +31 (0) 344 64 08 85
E-mail: info.nl@arjo.com

NEW ZEALAND

Arjo Ltd
34 Vestey Drive
Mount Wellington
NZ-AUCKLAND 1060
Tel: +64 (0) 9 573 5344
Free Call: 0800 000 151
Fax: +64 (0) 9 573 5384
E-mail: nz.info@Arjo.com

NORGE

Arjo Norway AS
Olaf Helsets vei 5
N-0694 OSLO
Tel: +47 22 08 00 50
Faks: +47 22 08 00 51
E-mail: no.kundeservice@arjo.com

ÖSTERREICH

Arjo Austria GmbH
Lemböckgasse 49 / Stiege A / 4.OG
A-1230 Wien
Tel: +43 1 8 66 56
Fax: +43 1 866 56 7000

POLSKA

Arjo Polska Sp. z o.o.
ul. Ks Piotra Wawrzyniaka 2
PL-62-052 KOMORNIKI (Poznań)
Tel: +48 61 662 15 50
Fax: +48 61 662 15 90
E-mail: arjo@arjo.com

PORTUGAL

Arjo em Portugal
MAQUET Portugal, Lda.
(Distribuidor Exclusivo)
Rua Poeta Bocage n.º 2 - 2G
PT-1600-233 Lisboa
Tel: +351 214 189 815
Fax: +351 214 177 413
E-mail: Portugal@arjo.com

SUISSE / SCHWEIZ

Arjo Switzerland AG
Fabrikstrasse 8
Postfach
CH-4614 HÄGENDORF
Tél/Tel: +41 (0) 61 337 97 77
Fax: +41 (0) 61 311 97 42

SUOMI

Arjo Scandinavia AB
Riihitontuntie 7 C
02200 Espoo
Finland
Puh: +358 9 6824 1260
E-mail: asiakaspalvelu.finland@arjo.com

SVERIGE

Arjo International HQ
Hans Michelsensgatan 10
SE-211 20 MALMÖ
Tel: +46 (0) 10 494 7760
Fax: +46 (0) 10 494 7761
E-mail: kundservice@arjo.com

UNITED KINGDOM

Arjo UK and Ireland
Houghton Hall Park
Houghton Regis
UK-DUNSTABLE LU5 5XF
Tel: +44 (0) 1582 745 700
Fax: +44 (0) 1582 745 745
E-mail: sales.admin@arjo.com

USA

Arjo Inc.
2349 W Lake Street Suite 250
US-Addison, IL 60101
Tel: +1 (630) 307-2756
Free: +1 (800) 323-1245
Fax: +1 (630) 307 6195
E-mail: us.info@arjo.com

JAPAN

Arjo Japan K.K.
東京都港区虎ノ門三丁目7番8号
ランディック第2 虎ノ門ビル9 階
Tel: +81 (0)3-6435-6401
Fax: +81 (0)3-6435-6402
E-mail: info.japan@arjo.com

At Arjo, we believe that empowering movement within healthcare environments is essential to quality care. Our products and solutions are designed to promote a safe and dignified experience through patient handling, medical beds, personal hygiene, disinfection, diagnostics, and the prevention of pressure injuries and venous thromboembolism. With over 6500 people worldwide and 65 years caring for patients and healthcare professionals, we are committed to driving healthier outcomes for people facing mobility challenges.



ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö, Sweden
www.arjo.com

arjo

