


AtmosAir Velaris

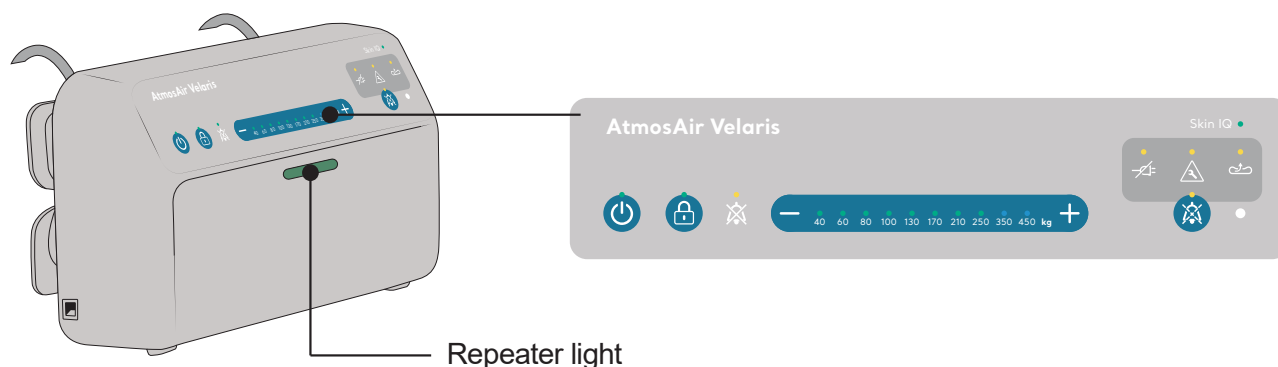

Run/Standby button and light

The Run/Standby button switches the pump between Run mode and Standby mode.

In Run mode the light is lit.

For Standby mode, press and hold the button for two seconds. The light turns off.



Lock button and light

To lock the control panel, press and hold the Lock button for 2 seconds. When locked the light turns on.

To unlock all control panel buttons, press and hold the Lock button for two seconds. When unlocked the light turns off.



Audio off light

The Audio off light turns on when the Audio notifications and alarms have been permanently disabled



Weight select buttons and light

The default weight is set and lit at 100 kg (220 lb).

- Press the - button to reduce the patient weight.
- Press the + button to increase the patient weight.



Audio alarm pause button and light

Press the Audio alarm pause button to silence the audible alarm for 15 minutes, the light turns on.

Press the button again to cancel the alarm pause.



Power fail light

If a power failure is detected the Power failure light turns on and an alarm sounds.

Active (alternating) therapy is not possible during power failure conditions.



System fault light

If an internal fault of the pump is detected during Built-In Self-Test (BIST) or during therapy, the System fault light turns on and an alarm sounds.



Low pressure light

If the mattress fails to achieve the target pressure, the Low pressure light turns on and an alarm sounds.



Skin-IQ light

The Skin-IQ light turns on when a Skin-IQ coverlet is connected to the Skin IQ power outlet port.