

Freedom Bath



Bathe and shower patient

1

Press the door down button to lower the door to its lowest position.
Help the patient enter the bathtub.

2

Risk of entrapment

For bathing: raise the door to the top position using the door up button.
For showering: leave the door down at least 25 mm (1 in) from the top position to keep drain open.

3

Press the door seal/drain button, the LED will flash a green light.

4

At proper inflation the LED has a steady green light.

5

Risk of scalding

Feel the water with a bare hand. Adjust temperature if needed.

For bathing: fill bathtub using the bathtub filling button.
For showering: use the shower button.

End bath

6

Press the door seal/drain button once to open drain.

7

Press again to deflate the seal. During the deflating the LED will flash a green light.

8

Optional. Shower the patient.
Lower door for more convenient access to the patient.

9

Press the door down button to lower the door to the lowest position.
Help the patient leave the bathtub.