

Freedom Bath



WARNING



To avoid eye and skin damage, always use protective glasses and gloves. If contact occurs rinse with plenty of water. If eyes or skin becomes irritated, contact a physician. Always read the material safety data sheet of the disinfectant.

To prevent cross-contamination, always follow the disinfection instructions in the Instructions for Use.

To avoid eye or skin irritation, never disinfect in the presence of a patient.

To avoid injury, only use the emergency lowering when normal lowering is impossible. Use extreme care during lowering.

Caregiver obligations

Between patients

Clean and disinfect Freedom bath before every use.

Every day

Clean door strip/seal and under the bathtub.

For non lift accessible bathtubs, follow these steps to make the area at the lower part of the door and under the bathtub accessible for cleaning:

1. Press down both the door up and door down button simultaneously for 30 seconds, until the temperature display starts flashing.
2. Press the door down button. The door will move under the bathtub, leaving the lower part of the strip/seal and the area under the bathtub free for cleaning.

Take caution at this operation as the crush protection is off. The protection is automatically activated again after one minute.

Without disinfectant/rinse inlet

Use a spray bottle or a MultiClean unit. For the MultiClean unit, refer to the MultiClean unit Instructions for use.

Remove visual residue

1. Put on protective gloves and eye wear.
2. Make sure all functions are turned off.
3. Open the drain to empty any left over water.
4. Detach and place accessories such as neck pillow and safety belt on the bottom of the bathtub.
5. Shower and rinse the bathtub and the accessories with water to remove visible residue. Turn the accessories and rinse again.

Clean

6. Spray disinfectant in the drain. Use a long handled brush to scrub the drain.
7. Close the drain by raising the door to the top position.
8. Spray disinfectant on all internal bath surfaces and the accessories at the bottom of the bathtub. Turn the accessories and spray on the other side. Pay attention to parts, knobs and sensors attached to the bath surface such as safety belt knobs and surface overflow outlet.
9. Scrub all the surfaces of the bathtub and the accessories with a soft bristle brush or cloth.
10. Open the drain by pressing the door seal/drain button.
11. Rinse all parts clean with water with a temperature of approximately 25 °C (77 °F) to remove the disinfectant.
12. Soak a cloth in disinfectant.
13. Wipe all other contact areas such as the outside of the bathtub (particularly the surface were the head is), metal sensor knobs and door seals.
14. Wet a new cloth with water and remove all traces of disinfectant on the contact areas.

Disinfect

15. Spray disinfectant in the drain.
16. Close the drain.
17. Spray disinfectant on all internal bath surfaces and the accessories at the bottom of the bathtub. Turn the accessories and spray on the other side. Pay attention to parts, knobs and sensors attached to the bath surface such as safety belt knobs and surface overflow outlet.
18. Soak a cloth in disinfectant.
19. Wipe all other contact areas such as the outside of the bathtub (particularly the surface were the head is), metal sensor knobs and door seals.
20. Allow an appropriate contact time of the disinfectant according to the instructions on the disinfectant bottle.
21. Open the drain.
22. Rinse all parts clean with water with a temperature of approximately 25 °C (77 °F) to remove the disinfectant. Use the shower to remove disinfectant.
23. Wet a new cloth with water and remove all traces of disinfectant on the contact areas.
24. Raise the door to the top position.
25. Leave to air dry.

Freedom Bath



With disinfectant/rinse inlet and a MultiClean unit

For the MultiClean unit, refer to the MultiClean unit Instructions for use.

Remove visual residue

1. Put on protective gloves and eye wear.
2. Make sure all functions are turned off.
3. Open the drain to empty any left over water.
4. Detach and place accessories such as neck pillow and safety belt on the bottom of the bathtub.
5. Shower and rinse the bathtub and accessories to remove visible residue.

Clean

6. Place the disinfection spray handle into the disinfectant/rinse inlet and turn either way to secure it.
7. Press the trigger handle and hold it until the disinfectant flows from all Air Spa system nozzles.
8. Let it run for another 30 seconds.
9. Release the trigger and remove the disinfection spray handle from the disinfectant/rinse inlet.
10. Place the water spray handle into the disinfectant/rinse inlet and turn either way to secure it.
11. Press the trigger handle and hold it until clean water flows from all nozzles.
12. Let it run for another 30 seconds.
13. Remove the shower handle from the disinfectant/rinse inlet.
14. Spray disinfectant in the drain. Use a long handled brush to scrub the drain.
15. Close the drain by raising the door to the top position.
16. Spray disinfectant on all internal bath surfaces and accessories. Turn accessories and spray on the other side. Pay attention to parts, knobs and sensors attached to the bath surface such as safety belt knobs and surface overflow outlet.
17. Scrub all the surfaces of the bathtub and accessories with a soft bristle brush or cloth (to remove any deposits, e.g. skin flakes).
18. Open the drain by pressing the door seal/drain button.
19. Rinse all parts clean with water with a temperature of approximately 25 °C (77 °F) to remove the disinfectant.
20. Soak a cloth in disinfectant.
21. Wipe all other contact areas such as the outside of the bathtub, metal sensor knobs and door seals.
22. Wet a new cloth with water and remove all traces of disinfectant on the contact areas.

Disinfect

23. Place the disinfection spray handle into the disinfectant/rinse inlet and turn either way to secure it.
24. Press the trigger handle and hold it until the disinfectant flows from all Air Spa system nozzles.
25. Let it run for 30 seconds.
26. Release the trigger and remove the disinfection spray handle from the disinfectant/rinse inlet.
27. Allow an appropriate contact time of the disinfectant according to the instructions on the disinfectant bottle.

28. Spray disinfectant in the drain.
29. Close the drain.
30. Spray disinfectant on all internal bath surfaces and the accessories at the bottom of the bathtub. Turn the accessories and spray on the other side. Pay attention to parts, knobs and sensors attached to the bath surface such as safety belt knobs and surface overflow outlet.
31. Soak a cloth in disinfectant.
32. Wipe all other contact areas such as the outside of the bathtub, metal sensor knobs and door seals.
33. Allow an appropriate contact time of the disinfectant according to the instructions on the disinfectant bottle.
34. Open the drain.
35. Place the water spray handle into the disinfectant/rinse inlet and turn either way to secure it.
36. Press the trigger handle and hold it until clean water flows from all nozzles.
37. Let it run for another 30 seconds.
38. Remove the shower handle from the disinfectant/rinse inlet.
39. Rinse all parts clean with water with a temperature of approximately 25 °C (77 °F) to remove the disinfectant. Use the shower to remove disinfectant.
40. Wet a new cloth with water and remove all traces of disinfectant on the contact areas.
41. Raise the door to the top position.
42. Press the button Air Spa system – ON/OFF to start the air blower, which then will be activated for one minute.
43. Leave to air dry.