



# Auralis

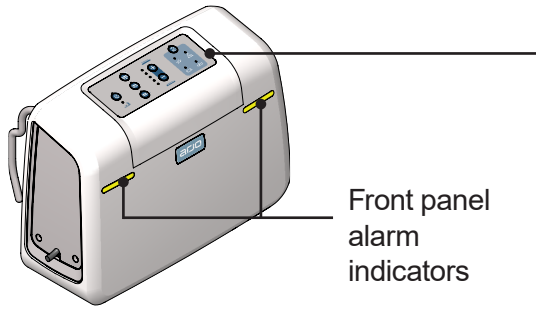

Front panel alarm indicators

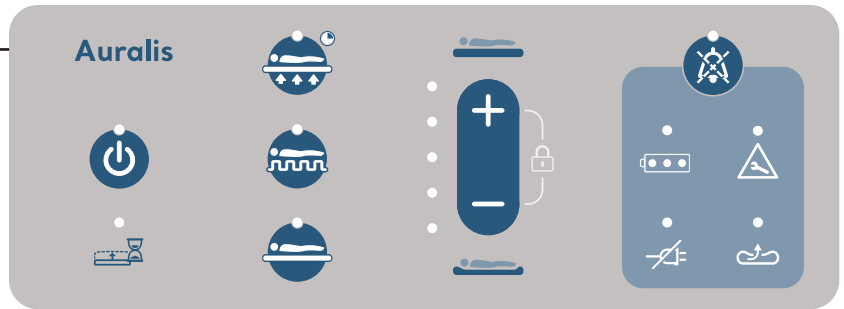

### Run/Standby button

**Run mode** – press the button once. The indicator changes from flashing to constant.

**Standby mode** – press the button for two seconds. The indicator changes from constant to flashing.



### Wait indicator

Shown when the mattress or seat cushion is being inflated. Remains on 15 minutes until the mattress or seat cushion is fully inflated.



### Autofirm mode button

Press and hold 2 seconds for Autofirm mode. Mattress inflates to a temporary firm surface. Autofirm mode lasts for 15 minutes but may be increased in 5 minute steps to a maximum of 30 minutes. Indicator turned on when active.

Not for seat cushion.



### Alternating mode button

Press for Alternating mode (default). The Alternating mode inflates or deflates different cells to create a fluctuating surface. Indicator turned on when active.



### Reactive CLP (Constant Low Pressure) mode button

Press for Reactive CLP mode. Maintains constant lower pressure in all the cells. Indicator turned on when active.

Not for seat cushion.



Risk of injury or discomfort

**ALWAYS** set the comfort level at maximum for patients weighing over 360 kg (793 lb). (Auralis Plus only)

### Comfort control buttons

Press to adjust the pressure in the support surface to set the comfort level. The indicators show the comfort setting.



### Mute alarm button

Press to silence the audible alarm for 15 minutes, the indicator is turned on.

The alarm is cancelled when the alarm condition has been corrected.



### Low battery indicator

Indicator yellow and audible alarm – 1 hour battery life left. The alarm can be muted.

When less than 15 minutes battery life left, the pump enters the Reactive CLP mode. The audible alarm can no longer be muted.



### Battery charge indicators

Blue indicators show the charge level of the battery. Flashing indicator – battery is charging.



### Service indicator

- Service indicator yellow, front panel alarm indicators green – the pump is ready for service. Information only, not an alarm.
- Service indicator yellow, front panel alarm indicators yellow, audible alarm – the pump has an internal fault or error. The alarm can be muted.



### Mains power failure indicator

Mains power failure has been detected. Information only, the pump continues to operate normally by using the battery. There is no audible alarm.



### Low pressure indicator

- The indicator is shown, there is a single warning tone – the pump detects low pressure within mattress or seat cushion.
- The indicator is shown, there is an audible alarm – mattress or seat cushion disconnected while in use. The alarm can be muted.
- The indicator is shown, there is a single warning tone – the pump detects that a mattress is not connected at start up.