

Prefill footwell

1

Make sure the bathtub is in upright position.

2

Close the drain.

3

Risk of scalding

Press auto fill button. Feel the water with a bare hand, adjust temperature if needed.

4

The filling stops when the footwell is filled.

Seat patient

5

Open the bathtub door fully.

6

Adjust bathtub height to suit the patient.

7

Risk of falling

Help the patient enter the bathtub.

8

Risk of entrapment

Close and lock the bathtub door.

Bathe patient

9

Recline the bathtub.

10

Adjust bathtub height to suit caregiver.

11

Risk of scalding

Press auto fill button to add water. Feel the water with a bare hand, adjust temperature if needed.

Shower patient

12

Lift off the shower handle.

13

Activate the shower button.

14

Risk of scalding

Press the shower handle trigger. Feel the water with a bare hand. Adjust temperature if needed.

15

Let the patient feel the water. Adjust temperature if needed.

16

Wash and shower the patient.

End bath

17

Deactivate the shower button.

18

Place the shower handle in the holder.

19

Open the drain.

20

With intervals, tilt the bathtub to upright position.

21

Keep the patient warm using the shower or towels.

22

Lower the bathtub.

23

Make sure the water is below the door opening.

24

Open the bathtub door fully.

25

Risk of falling

Dry the patient. Help the patient leave the bathtub.

26

Risk of contamination

After each patient: clean and disinfect the bathtub according to the instructions in the IFU or Cleaning and disinfection QRG.