

Disinfect with integrated spray handle

Remove visual residue





The bathtub should be in upright position with all functions turned off.



Place all accessories at the bottom.



Determine the disinfectant mixing ratio



Clean



Step 13–23 for Air spa only

13 Recline the bathtub.	Place the disinfection spray handle into the inlet. Turn left or right to secure tightly.	15 Press and hold the trigger. Disinfectant must flow from all nozzles for at least 30 seconds.	16 Release the trigger and remove the spray handle.	17 Put back the spray handle.
18 5 seconds	19 Place the shower handle	20	0° 00	23



Continue with the steps on the next page



08.AL.01_7EN • 03/2023 • Page 1/4





Continue with the steps on the next page



08.AL.01_7EN • 03/2023 • Page 2/4









08.AL.01_7EN • 03/2023 • Page 3/4



Disinfect with spray bottle

6

Remove visual residue





The bathtub should be in upright position with all functions turned off.



12

Place all accessories at the bottom.















Wipe all external surfaces with disinfectant.



Remove all disinfectant with a new cloth and water.

Disinfect



accessories with water.

disinfectant manufacturers IFU.





ArjoHuntleigh AB Hans Michelsensgatan 10 211 20 Malmö, Sweden www.arjo.com

EMPOWERING MOVEMENT

08.AL.01_7EN • 03/2023 • Page 4/4