

Auto fill

Press the auto fill button again to continue filling for 60 seconds.

Stop filling at any time by pressing the auto fill button.

Press auto fill button. The filling stops automatically at correct water level.

Shower handle

Press to activate shower function.

Press to regulate water flow

Press to lock water flow

Unlock by pressing the trigger.

Temperature display

Shows the temperature of the active filling, shower or disinfectant mixture.

Shows current bathtub water temperature.

Scalding protection

The temperature displays flashes and there is a beeping sound. Water shuts off automatically after 10 seconds.

Get the patient out safely and stop using the bath.

45 °C (113 °F)

Emergency stop

Press button to stop all functions. Turn clockwise to release.

Functions lock

Press to lock all functions except scalding protection and emergency stop. Press again to unlock.

Door lock

Lock bathtub door.

Unlock bathtub door.

Air spa system (optional feature)

Whole bathtub

Footwell only

Press to turn on Air spa and to select intensity.

Use for maximum 15 minutes.

Press to turn off.

Sound & Vision system (optional feature)

Unscrew cap and plug in USB stick.

Use hand control to regulate sound and light.

Disinfection unit (optional feature)

The disinfection bottle is located between the bathtub legs.

The disinfection handle is located behind the cover.

Determine disinfection mix ratio: spray disinfectant into the bathtub while checking the flowmeter inside cover, adjust knob on suction hose if needed.

Flashes when the disinfection bottle is empty.