

System 2000

Autofill

Autofill ON/OFF button

The filling stops automatically at preset water level.

Press the Autofill button to stop the filling at any time.

P300 P220

Shower button and Shower handle

Shower ON/OFF button

Lock button (A) locks water flow.

Trigger (B) regulates and unlocks water flow.

P300 display

No water in the bathtub. No functions activated.

Water temperature for: active filling, shower or filled bathtub.

P300 has three Autofill levels. Preset level is Normal.

Autofill Normal level Autofill Low level Autofill High level

To change level:
Press the (-) or the (+) within 8 seconds after the autofill starts.

Scalding protection

If scalding protection is activated, move the patient safely from the bath.

If dangerous temperature level of 45 °C (113 °F) is reached, the scalding protection shuts off the water in less than 10 s.

P300

The display shows four dashes and a beeping sound is heard. During this time the mixer recalibrates automatically.

P220

Both LEDs flash and a beeping sound is heard.

P220 display

Water temperature for: active filling, shower or filled bathtub.

Hydromassage or Hydrosound (optional)

Hydromassage or Hydrosound ON/OFF button

Intensity level adjustment

Hydromassage Hydrosound

P300 P220

Sound & Vision system (optional)

USB connector is located under the Control panel lid.

Use the hand control to regulate sound and light.

Disinfection unit (optional)

Disinfection ON/OFF button

The On/OFF button flashes when the disinfection bottle is empty. (P300 only)

The disinfection unit is located under the control panel lid.

Disinfection spray handle (A)

Flow meter (B)
Disinfection bottle (C)
(under shampoo/bath oil)

Determine disinfection mix ratio:
Spray disinfectant into the bathtub. Check that the mix ratio on the flowmeter is correct. Adjust knob on suction hose if needed.