

# System 2000 Rhapsody, Primo, Harmonie



**Clean and disinfect:**

- with disinfection unit (Still Bath and Hydrosound System) page 1-2
- with a spray bottle (Still Bath and Hydrosound System), page 3
- the Hydromassage System with automatic disinfection, page 4
- the Hydromassage System without automatic disinfection, page 5

## Clean and disinfect with disinfection unit (Still Bath and Hydrosound System)

### Remove visual residue

1

Check the level of the disinfectant.

2

Raise the bathtub to convenient working position.

3

Make sure all functions are turned off.

4

Open the drain. Detach and place all accessories at the bottom of the bathtub.

5

Shower and rinse the bathtub and the accessories with water to remove visible residue. Turn the accessories and rinse again.

### Clean

6

Put on protective gloves and eye wear.

7

Lift out the disinfection spray handle.

8

Press the disinfection button.

9

Spray disinfectant into the bathtub until the flow meter (A) shows correct mixing value according to the Instructions for use. Adjust if needed (B).

10

°C (°F)

Adjust water temperature to approximately 25 °C (77 °F) to eliminate disinfectant fumes.

11

Spray disinfectant in the drain. Scrub the drain with a brush.

12

Close the drain.

13

Spray disinfectant on all internal bathtub surfaces and on all sides of the accessories.

14

Spray into the surface the overflow outlet.

15

Press the disinfection button to turn it off.

16

Scrub the surfaces of the bathtub, loose parts and accessories thoroughly with a soft bristle brush.

17

Open the drain.

18

Rinse all parts with plenty of water with a temperature of approximately 25 °C (77 °F).

19

Soak a cloth in disinfectant and wipe all other contact areas:  
the outside of the bathtub, hand control and metal sensory knobs.


20

Wet a new cloth with water and remove the disinfectant. Occasionally rinse the cloth in running water.

Continue with Disinfection for *Still Bath and Hydrosound System* on page 2.


Disinfect

21

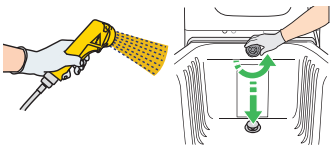

Spray disinfectant in the drain. Close the drain.


22

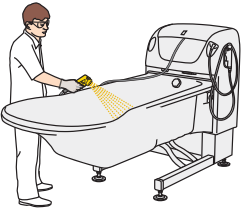

Spray disinfectant on all internal bathtub surfaces and on all sides of the accessories.

Allow appropriate contact time according to the instructions of the disinfectant.

23



Push the disinfection button to end the disinfection cycle.


**Automatic Disinfection only:**  
If the preset disinfection time has not elapsed other functions are disabled for 2-10 minutes depending on model.


24



Put back the disinfection spray handle.


25



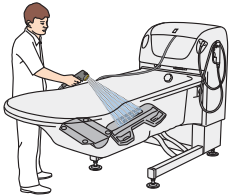

Open the drain.  
Rinse all parts with water having a temperature of 25 °C (77 °F).


26



Soak a cloth in disinfectant and wipe all other contact areas: the outside of the bathtub, hand control and metal sensory knobs.


Allow appropriate contact time according to the instructions of the disinfectant.

27



Wet a new cloth with water and remove the disinfectant. Occasionally rinse the cloth in running water.

28

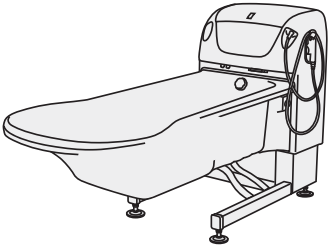



Leave to air dry. Attach any accessories.

# Clean and disinfect with a spray bottle

## Remove visual residue

1

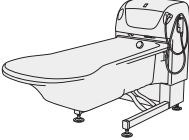


Raise the bathtub to convenient working position.

2



Make sure all functions are turned off.

3

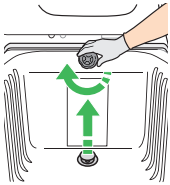

Open the drain.  
Detach and place all accessories at the bottom of the bathtub.


4

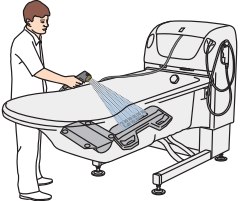

Shower and rinse the bathtub and the accessories with water to remove visible residue.  
Turn the accessories and rinse again.


## Clean

5




Put on protective gloves and eye wear.

6




Spray disinfectant in the drain. Scrub the drain with a brush.

7

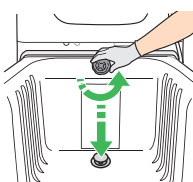


Close the drain.

8

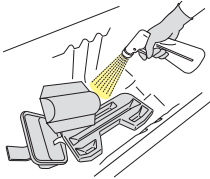


Spray disinfectant on all internal bathtub surfaces and on all sides of the accessories.

9

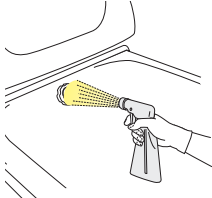


Spray into the surface of the overflow outlet.

10

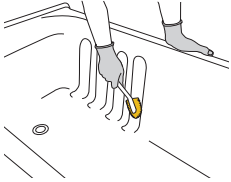



Scrub the surfaces of the bathtub, loose parts and accessories thoroughly with a soft bristle brush.

11

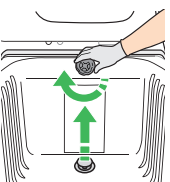


Open the drain.

12

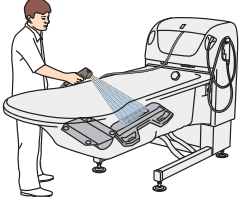


Rinse all parts with plenty of water having a temperature of 25 °C (77 °F).

13



Soak a cloth in disinfectant and wipe all other contact areas:  
the outside of the bathtub, hand control and metal sensory knobs.


14



Wet a new cloth with water and remove the disinfectant. Occasionally rinse the cloth in running water.


## Disinfect

15



Spray disinfectant in the drain. Close the drain.

16

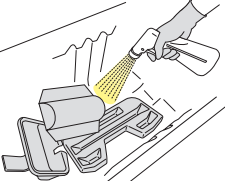


Spray disinfectant on all internal bathtub surfaces and on all sides of the accessories.

17



Allow appropriate contact time according to the instructions of the disinfectant.

18

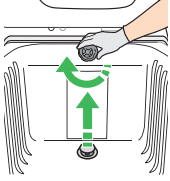


Open the drain.

19

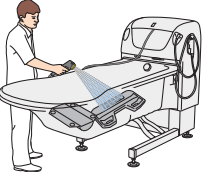

Rinse all parts with plenty of water having a temperature of 25 °C (77 °F).

20




Soak a cloth in disinfectant and wipe all other contact areas:  
the outside of the bathtub, hand control and metal sensory knobs.

21


Allow appropriate contact time according to the instructions of the disinfectant.

22



Wet a new cloth with water and remove the disinfectant. Occasionally rinse the cloth in running water.


23

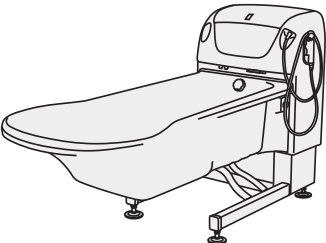

Leave to air dry. Attach any accessories.


# Clean and disinfect the Hydromassage System with automatic disinfection

1




Check the level of the disinfectant.

2




Raise the bathtub to convenient working position.

3

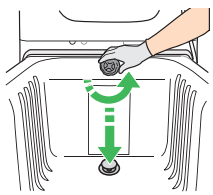

Close the drain.

4

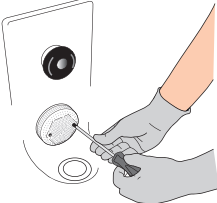

Unscrew the screws and remove the suction inlet screen. Clean the suction inlet screen separately.

5



Press the disinfection button to start the disinfection cycle.

6

Disinfectant comes out from the Hydromassage system nozzle.

The disinfection cycle ends with the Hydromassage system being rinsed with water.

7



Spray disinfectant into the surface overflow outlet.

8

Continue with the remaining disinfection instructions according to *Clean and disinfect with disinfection unit (Still bath and Hydrosound System)*, on page 1 and 2 of this QRG.

9

Allow appropriate contact time according to the instructions of the disinfectant.


10



Press the shower button. Rinse the surface overflow outlet.


11

After completed disinfection, attach the suction inlet screen with the screws and tighten them gently.

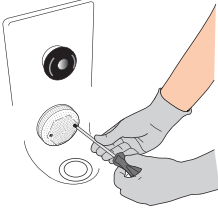


# Clean and disinfect the Hydromassage System without automatic disinfection


- 1**




Check the level of the disinfectant.
- 2**

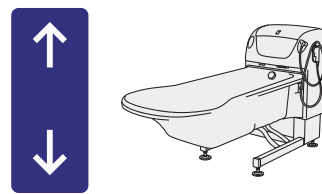


Raise the bathtub to convenient working position.
- 3**

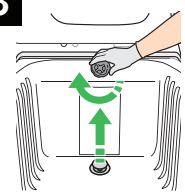


Open the drain.
- 4**

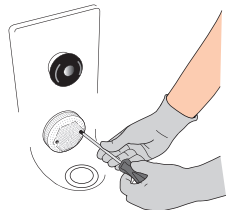


Unscrew the screws and remove the suction inlet screen. Clean the suction inlet screen separately.
- 5**

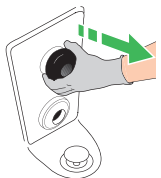


If a hose is going to be connected, pull out the Hydromassage system nozzle.
- 6**

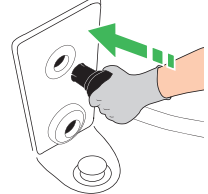

Insert the hose, if any, in to the Hydromassage system outlet.
- 7**

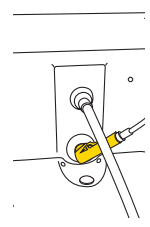


Spray disinfectant into the suction inlet until disinfectant comes out of the hose.
- 8**

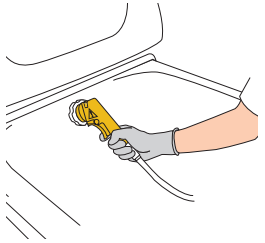


Spray disinfectant into the surface overflow outlet.


Allow appropriate contact time according to the instructions of the disinfectant.
- 9**




Press the disinfection button to turn off the disinfection.
- 10**




Press the shower button.
- 11**

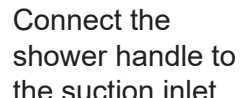


Connect the shower handle to the suction inlet.
- 12**

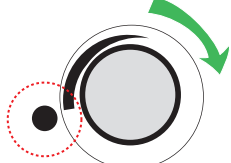


Turn the Hydromassage system knob to its highest intensity.
- 13**

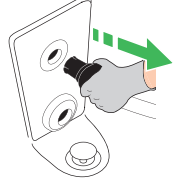


Press the trigger and rinse until the water comes out **clear** from the hose.
- 14**

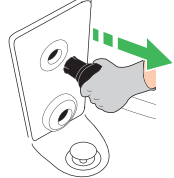


Remove the hose, if any.
- 15**

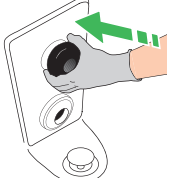


Put back the nozzle into the Hydromassage system outlet.
- 16**

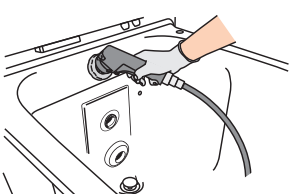

Rinse the surface overflow outlet with water.
- 17**

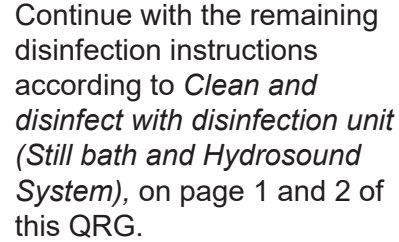

Continue with the remaining disinfection instructions according to *Clean and disinfect with disinfection unit (Still bath and Hydrosound System)*, on page 1 and 2 of this QRG.
- 18**

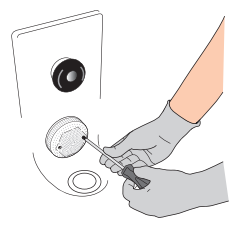

After completed disinfection, attach the suction inlet screen with the screws and tighten them gently.