

# Carevo

## Transfer from bed



**1**

Apply brakes on bed.

**2**

Place sliding aid under patient.  
Place a pillow under patient's head.  
Fold down bed rail on transfer side.

**3**

Position shower trolley next to the bed's transfer side. Fold down the shower trolley mattress on transfer side.

**4**

Press down and fixate the shower trolley mattress corners between head/foot panel and support plate.

**5**

Adjust the shower trolley height slightly lower than bed.

**6**

**! Risk of tipping**

Do not raise or lower other equipment.  
Push the shower trolley against bed.

**7**

**! Risk of falling**

Apply brakes on the shower trolley.

**8**

**! Risk of tipping**

Do not raise or lower other equipment.  
Transfer patient.  
Keep an eye on patient's head.


*Continue with steps on next page*

## Transfer from bed continued

9



Risk of slipping out

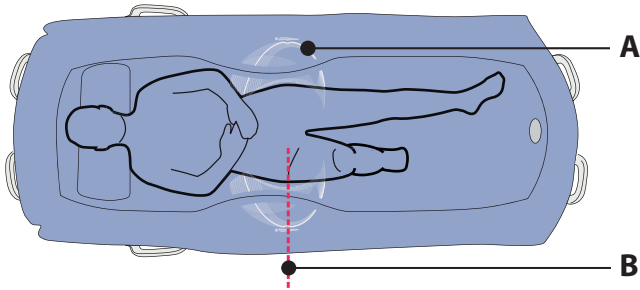


Keep mattress sides up, not folded.  
Position patient correctly on the Flexi zone.  
Patient's buttocks within graphic outline (A).  
Patient's hip creases aligned with human figure (B).

10



Replace the bed pillow with the shower trolley head pillow.  
Remove the sliding aid.

11

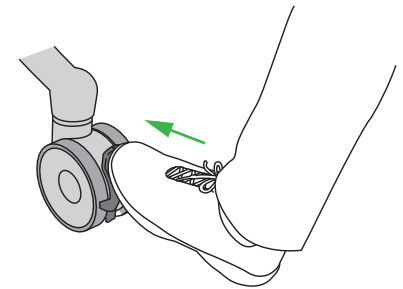


Release brakes on the shower trolley.

12




Risk of entrapment  
Risk of falling

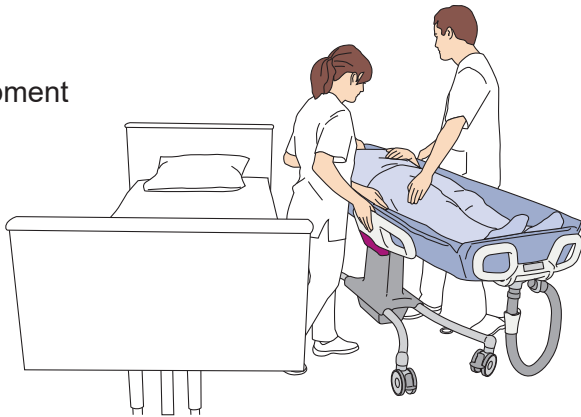

Keep patient's hair, arms and feet close to the body.  
Carefully move the shower trolley away from bed enough to fold up and lock side support on the shower trolley.


13

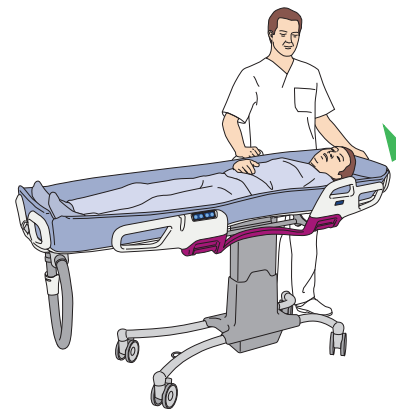

Raise the shower trolley adjustable back support, if available.  
Transport patient.


## Transfer to bed

1

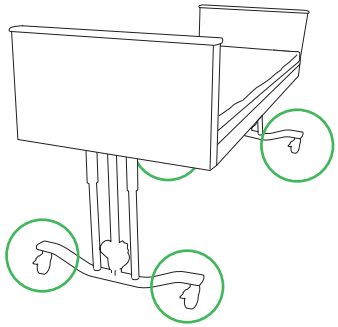

Apply brakes on the bed.  
Fold down the bed rail on transfer side.

2

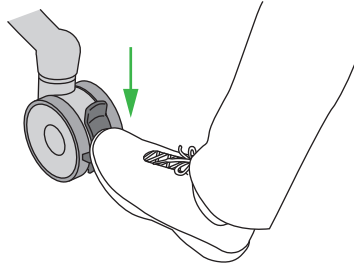


Apply brakes on the shower trolley.  
Lower the shower trolley adjustable  
back support, if available.

3



Place sliding aid under patient.

4

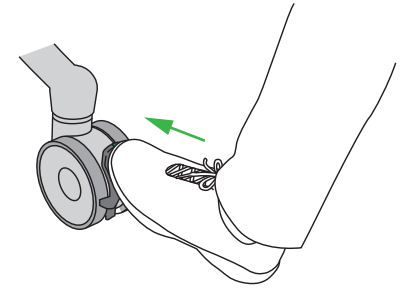

Release brakes on the shower  
trolley.

5



Risk of entrapment




Keep patient's hair, arms and feet close to the body.  
Position the shower trolley close to bed.  
Fold down the shower trolley side support on transfer side.

6



Risk of tipping

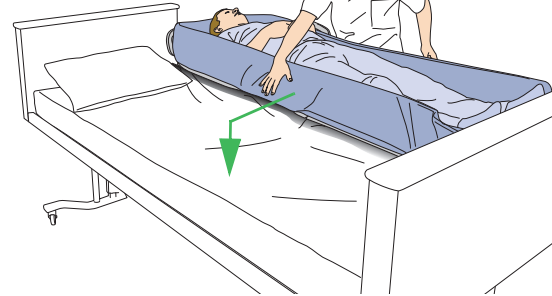


Immediately place the shower trolley so it overlaps the bed.  
Lower the shower trolley on the bed for an even transfer.

7



Risk of falling

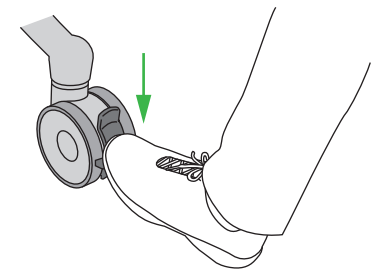


Apply brakes on the shower  
trolley.

*Continue with steps on next page*


## Transfer to bed continued

8

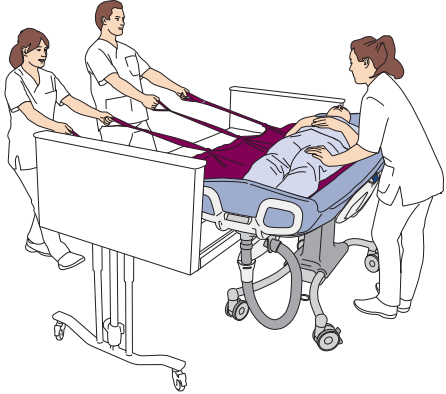


Check that the shower trolley mattress is on top of the bed.  
Transfer patient with sliding aid.  
Keep an eye on patient's head.

9

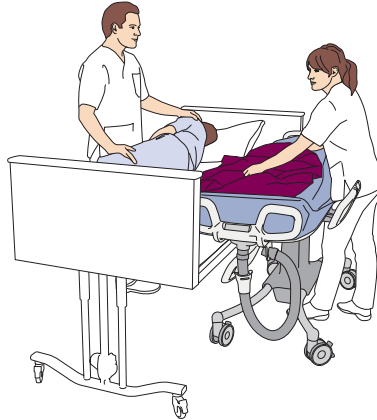


Replace the shower trolley head pillow with the bed pillow.  
Remove sliding aid.

10

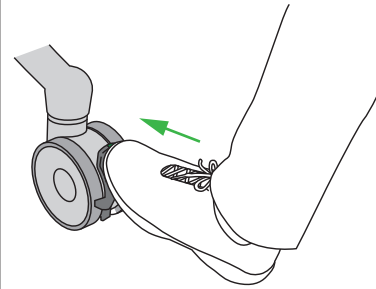

Release brakes on the shower trolley.


11

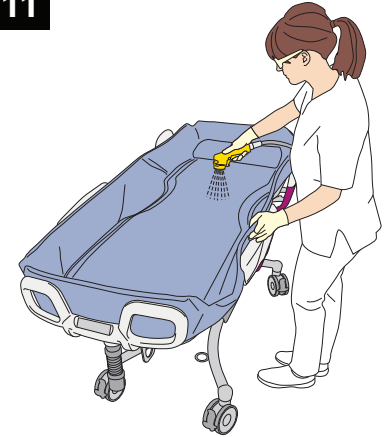

Move the shower trolley away from bed.  
Disinfect the shower trolley.

## Transfer from sling lift

1

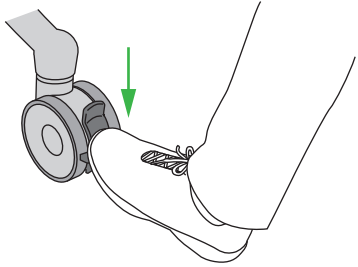

Shower trolley:

- Lower to lowest height
- Apply brakes
- Fold up and lock side supports
- Place head pillow

2



Risk of entrapment  
Risk of falling




Keep patient's hair, arms and feet close to the body.  
Raise the shower trolley adjustable back support to a seated position, if available. Lower patient onto the shower trolley.

3




Risk of slipping out

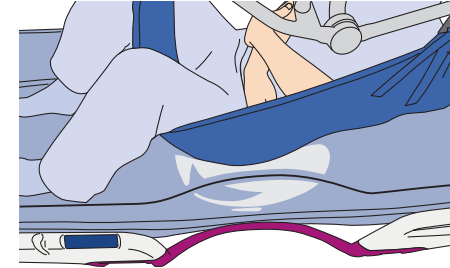


Keep mattress sides up, not folded.  
Position patient correctly on the Flexi zone.  
Patient's buttocks within graphic outline.  
Patient's hip creases aligned with human figure.

4

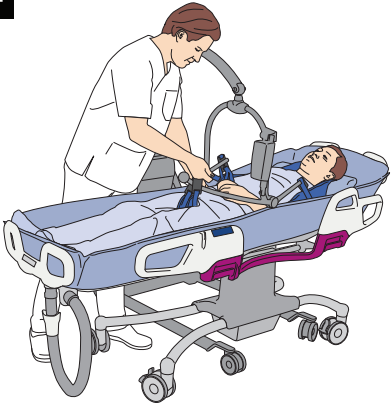

Detach sling from spreader bar.  
Move lift away from the shower trolley.

5

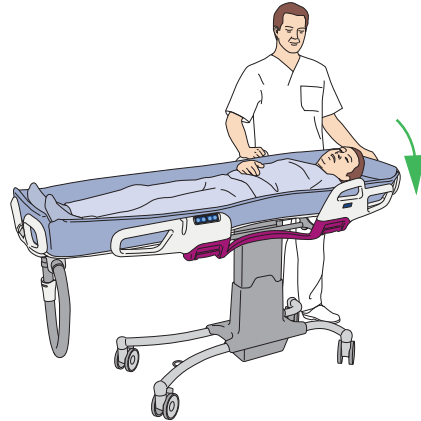


Adjust the shower trolley to an ergonomic height.  
Lower the shower trolley adjustable back support, if available.

6



Remove sling from underneath patient.


7

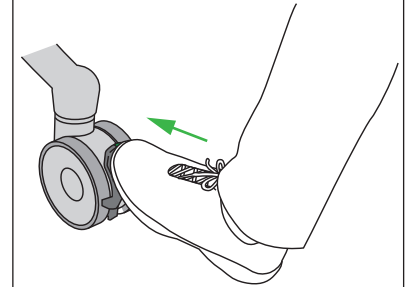

Release brakes on the shower trolley.  
Transport patient.


## Transfer to sling lift

1

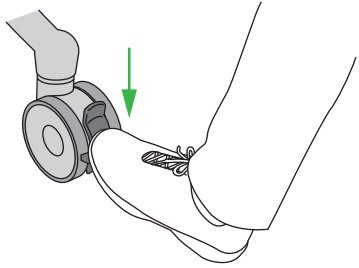

Apply brakes on the shower trolley.

2

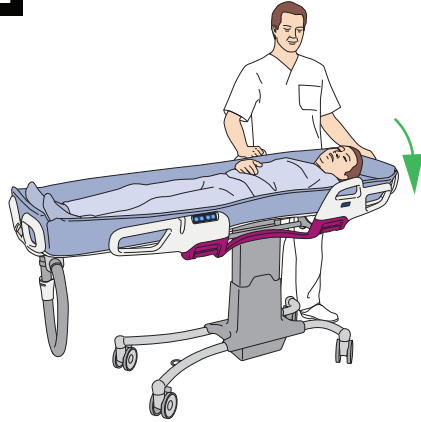

Lower the shower trolley adjustable back support, if available.

3



Apply sling on patient.

4




Lower the shower trolley to lowest height. Position lift's spreader bar over the shower trolley. Attach sling to spreader bar.

5




Risk of entrapment  
Risk of falling

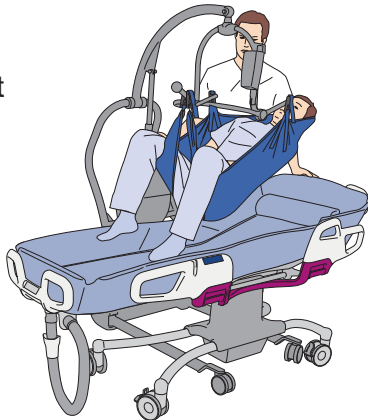

Keep patient's hair, arms and feet close to the body.  
Lock the side supports.  
Lift the patient out of the shower trolley.


6

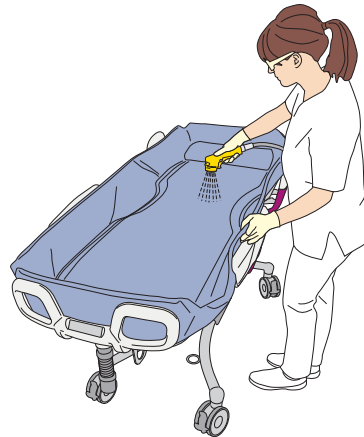

Disinfect the shower trolley.

## Shower the patient

1

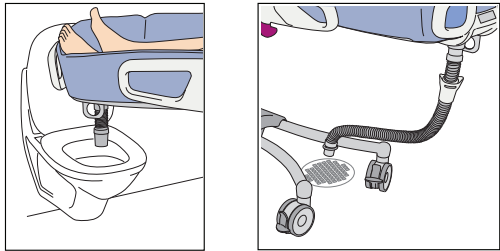


A

B

Position the shower trolley upper drainage over toilet/flush sink (A) or position lower drainage outlet in drainage hose holder, above floor drain (B).

2

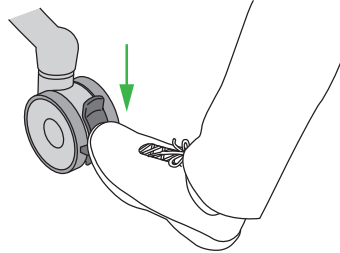


Apply brakes on the shower trolley.

3




Risk of scalding

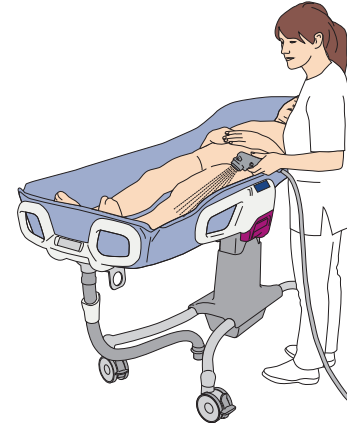


Undress patient.  
Set water temperature.  
Direct the shower handle away from patient.  
Turn water on.  
Check temperature with your naked hand.

4

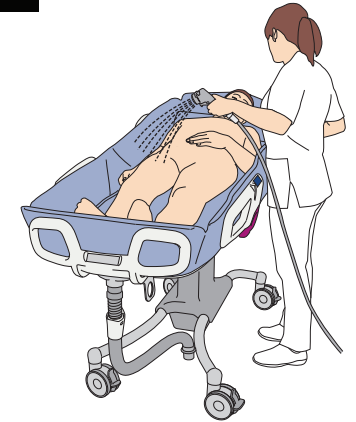


Shower patient.  
For shallow bathing, plug the mattress drain.

5

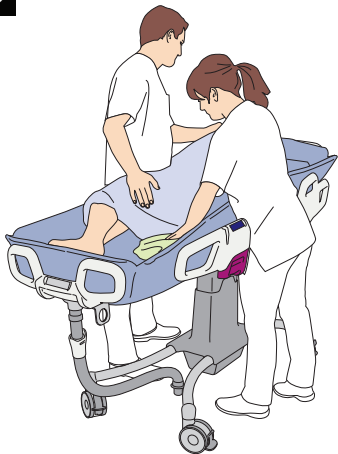

Drain the water and dry patient and mattress.

6

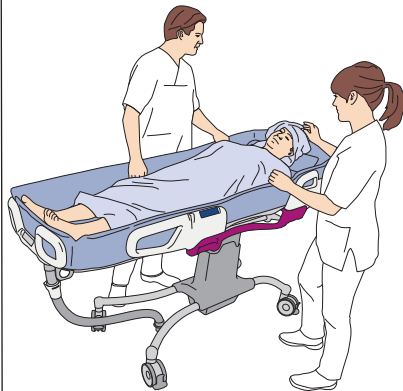


Cover or dress patient.

7




Place lower drainage outlet back in holder or attach lower drainage to upper drainage.

8

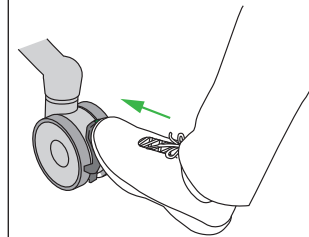

Release brakes on the shower trolley.

9

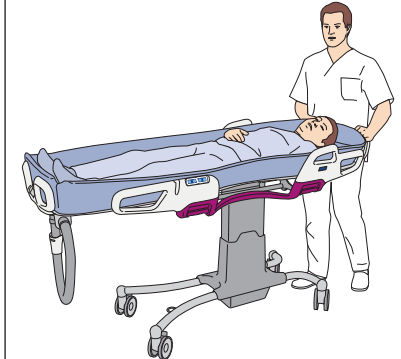

Transport patient.

