


Calypso


Transfer from bed

1

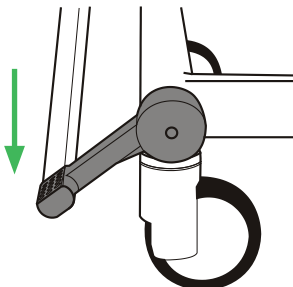


Apply the brakes on the bed, if applicable.

2

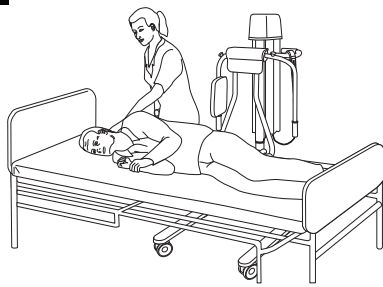


Fold down the bedrail. Turn the patient onto their side, away from Calypso lift and hygiene chair.

3

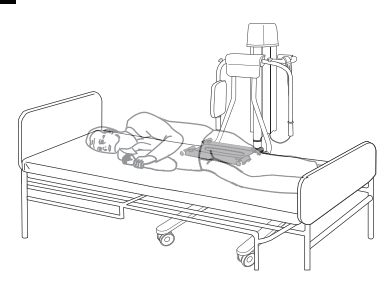


Position the seat over the bed as close to the patient as possible.

4

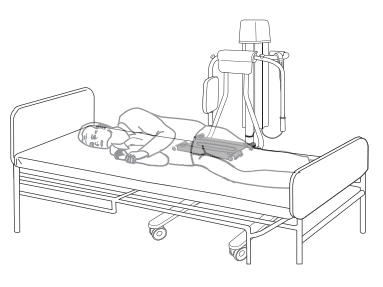


Lower the seat so that it is pressed against the bed.

5

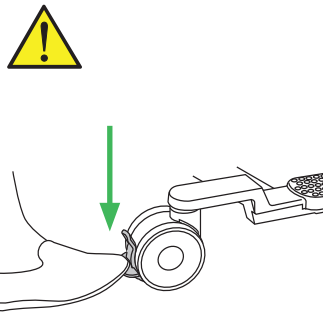


Apply brakes on all castors.

6

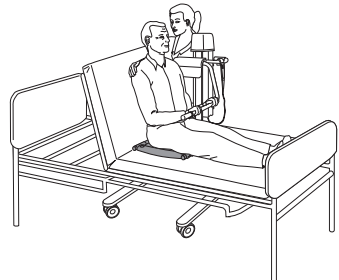


Turn the patient on their back. Sit the patient upright, manually or by raising the head end of the bed.


7




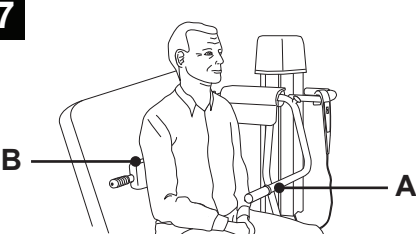


Fold down the hand rest (A). Lean the patient forward and fold down the backrest (B).





8








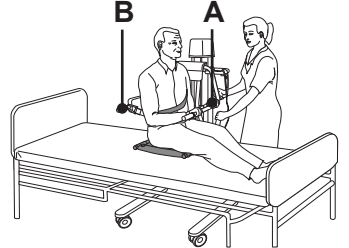


 Fasten safety belt. Position the patient in the centre of the seat, back against the backrest (B) and hands on the hand rest (A).

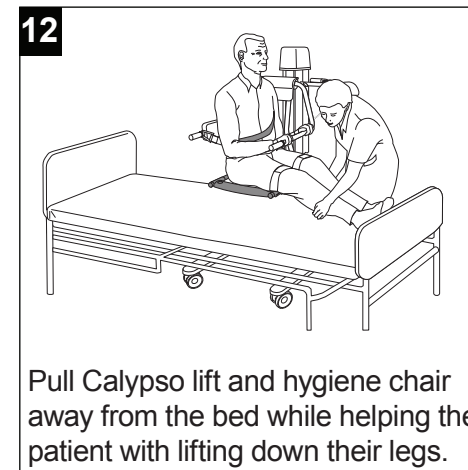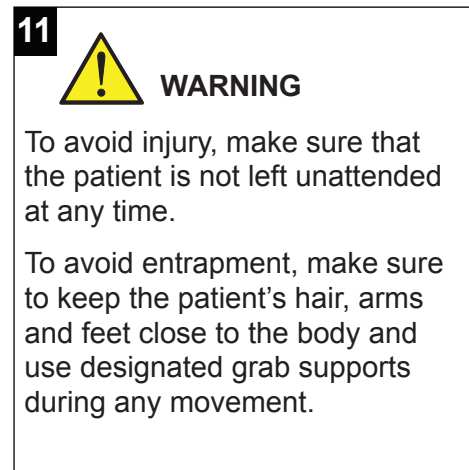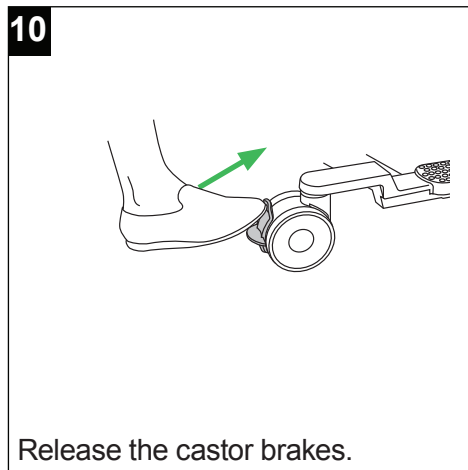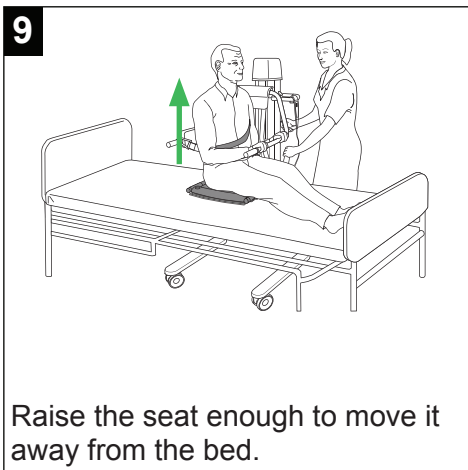

Transfer to bed

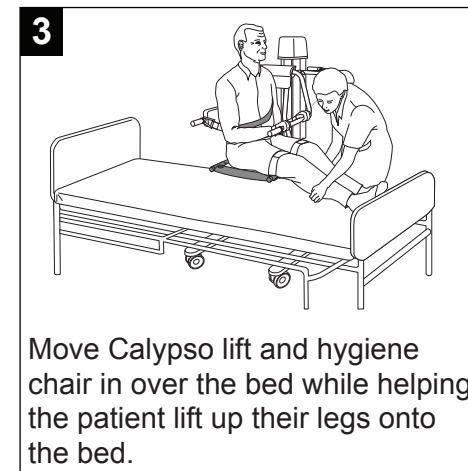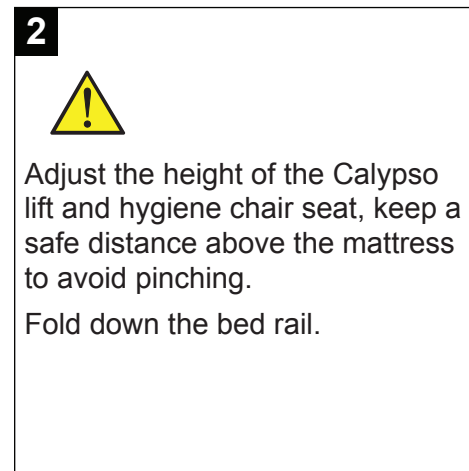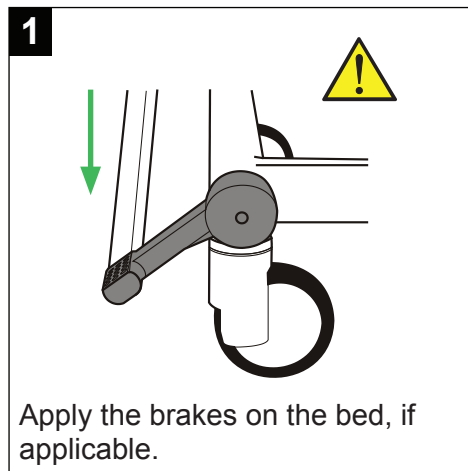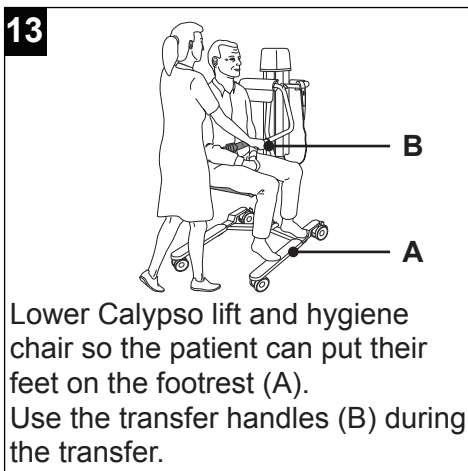

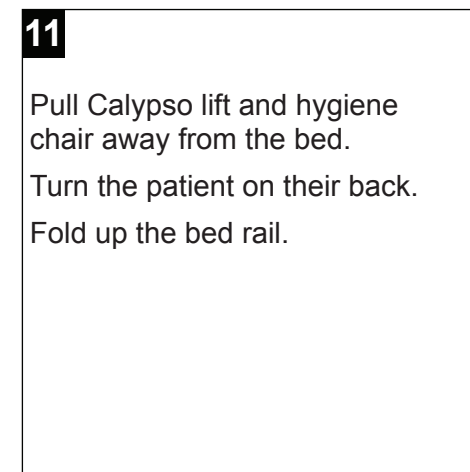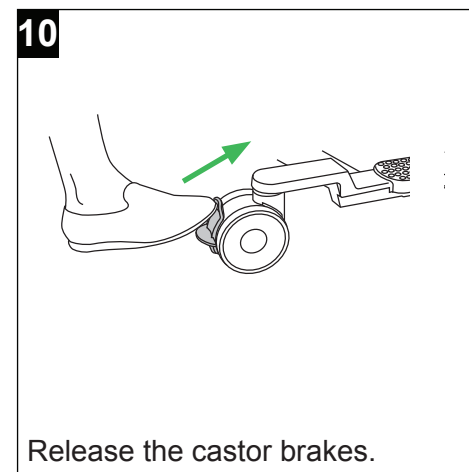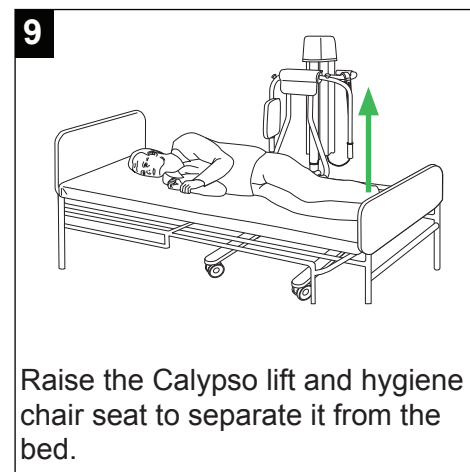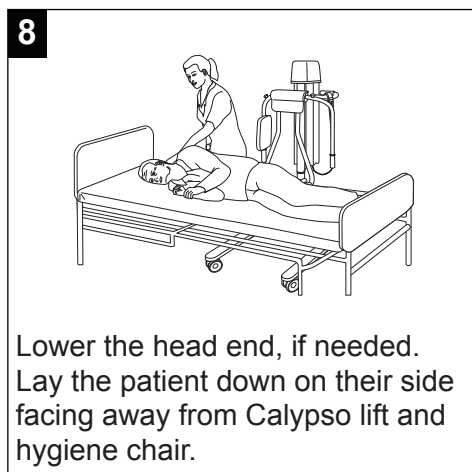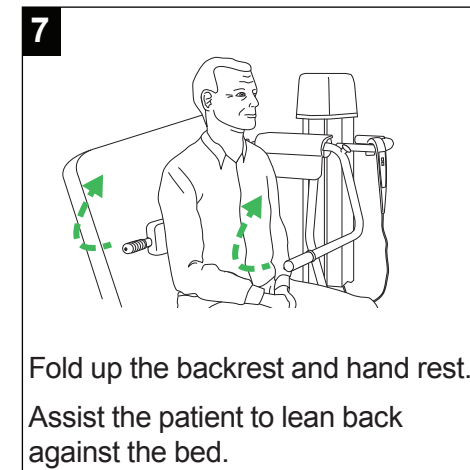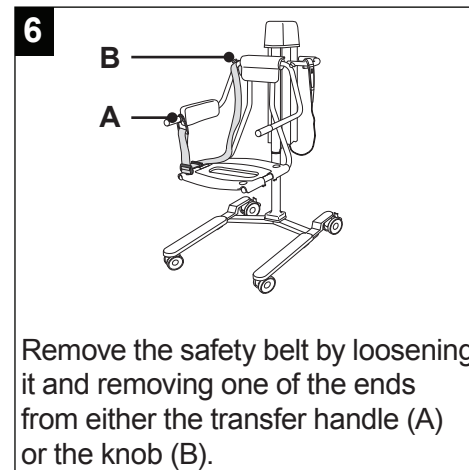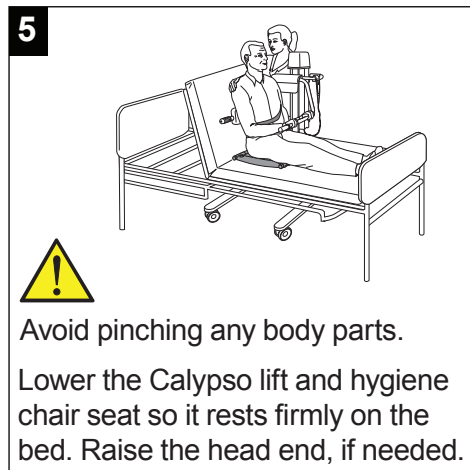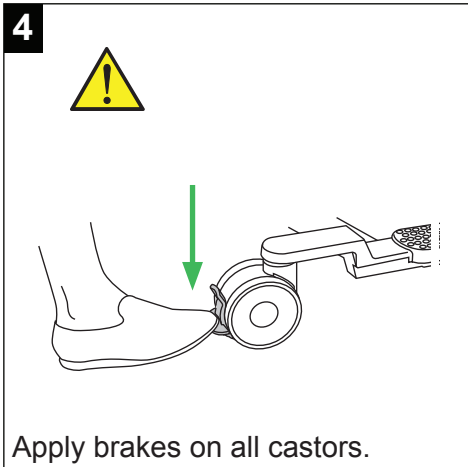

Transfer from wheelchair

1

NOTE

For the patient to be able to move from a wheelchair to Calypso lift and hygiene chair they must be strong enough to move sideways and/or be able to stand up.

2




Apply the brakes on the wheelchair.

Position Calypso lift and hygiene chair next to the wheelchair.

3


Adjust Calypso lift and hygiene chair so the seat is slightly higher than the wheelchair seat. Overlap the wheelchair seat if possible.

4

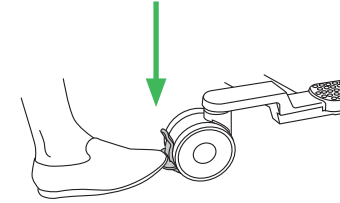


Apply the brakes on Calypso lift and hygiene chair.

5

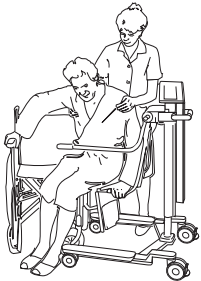


Help move the patient to Calypso lift and hygiene chair.

6

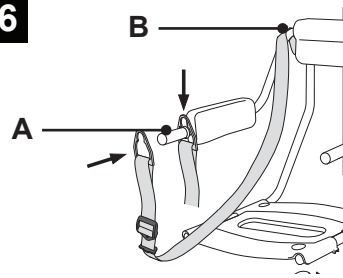


Attach the safety belt to the transfer handle (A).
Fasten the other end of the safety belt to the knob (B).

7

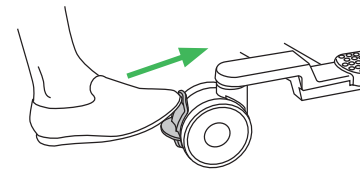

Release the brakes on Calypso lift and hygiene chair.

8

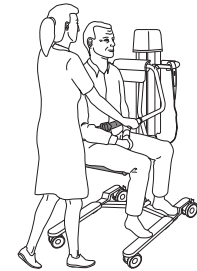

Hold the transfer handles during transfer.


Transfer to wheelchair

1

NOTE

For the patient to be able to move from Calypso lift and hygiene chair to a wheelchair they must be strong enough to move sideways and/or be able to stand up.

2

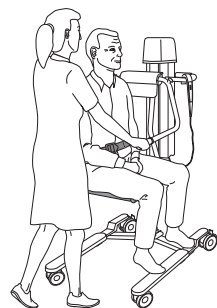

Make sure the patient has the feet on the footrest and the hands on the hand rest.

3



Apply the brakes on the wheelchair.


Remove the wheelchair armrest and fold up the wheelchair foot rests.

Position Calypso lift and hygiene chair next to the wheelchair.

4


Adjust Calypso lift and hygiene chair so the seat is slightly higher than the wheelchair seat. Overlap the wheelchair seat if possible.

5

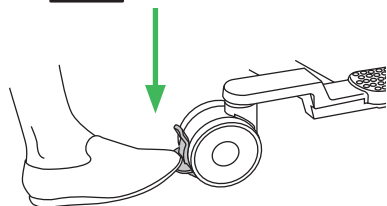


Apply the brakes on Calypso lift and hygiene chair.

6

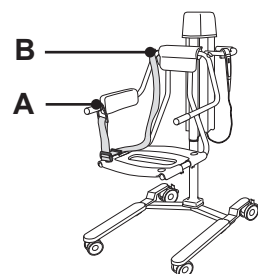

Remove the safety belt by loosening it and removing one of the ends from either the transfer handle (A) or the knob (B).

7

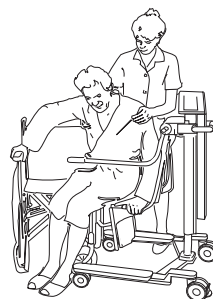

Help move the patient to the wheelchair.

8

Make sure the patient sits comfortably and safely in the wheelchair.

Bathe the patient

1



WARNING

Be aware of stationary objects when lowering.

Make sure to keep patient's arms and feet close to the body.

Make sure safety belt is properly fastened.

Make sure not to leave the patient unattended.

2




Avoid entrapment of body parts.

Raise the Calypso lift and hygiene chair to clear the edge of the bathtub.

Move the patient over the bathtub.

3

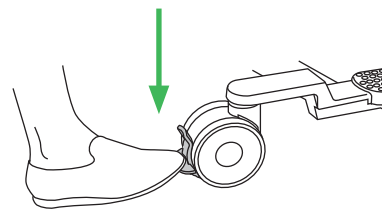


Apply the brakes on Calypso lift and hygiene chair.

4



Raise the bathtub to a comfortable height.

5

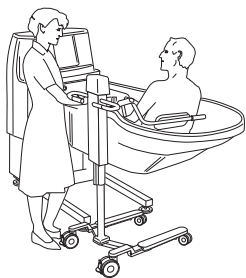


Lower Calypso lift and hygiene chair seat so that it rest against the bottom of the bathtub.
Remove the safety belt.

6




Pull the hand rest forward.
Recline the back rest slightly.
Bathe the patient.

7

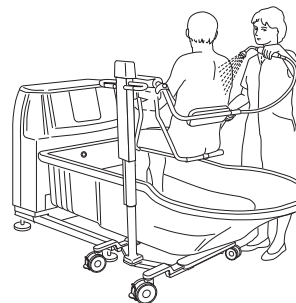

Lower the hand rest.
Raise the backrest.
Fasten safety belt.
Shower patient.

8

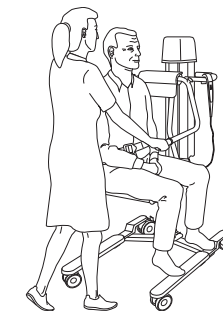

Dry patient.
Cover or dress patient.
Return patient to their room.

