


Miranti


Transfer patient from bed

1

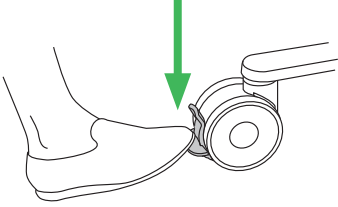


Apply the brakes on the bed, if applicable.
Fold down the bed rail on the caregiver side.
Undress the patient.

2

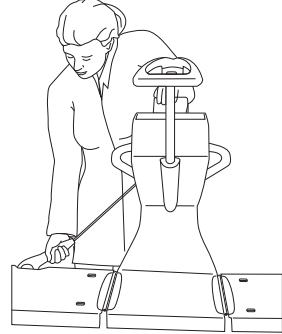


Fasten the hook rope to the side of the lift bath trolley that will function as backrest.

3

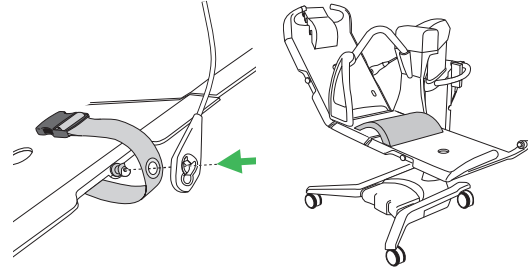


Attach safety belts, if required.
Put the thigh support cushion in place.

4

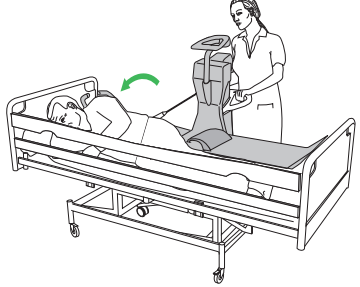


Turn the patient on their side, facing away from the lift bath trolley.
Move the lift bath trolley over the bed as close as possible to the patient.
Lower the lift bath trolley onto the bed.

5




Apply brakes

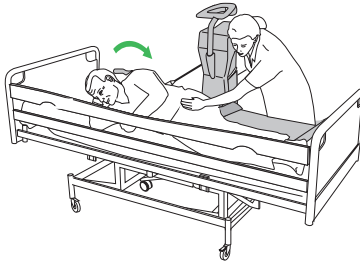


Apply the central brake on the lift bath trolley.
Turn the patient onto their back on the stretcher.

6

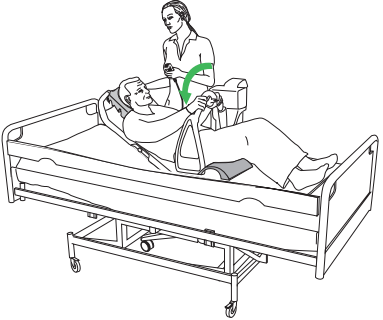


Lower the patient security grip and side guard.
Put the head rest in place.
Adjust the backrest to a comfortable position for the patient.

7




Release brakes

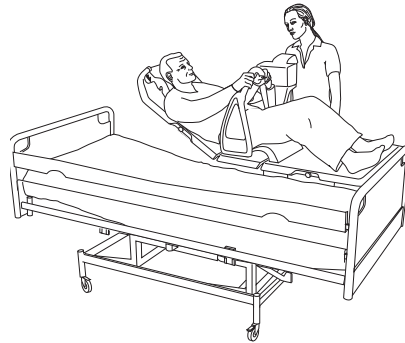

Release the central brake on the lift bath trolley.
Raise the lift bath trolley and pull it away from the bed.


Transfer patient to bed

1

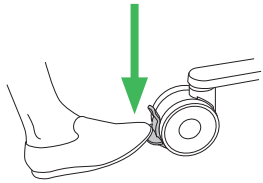

Apply the brakes on the bed, if applicable. Fold down the bed rail on the caregiver side.

2

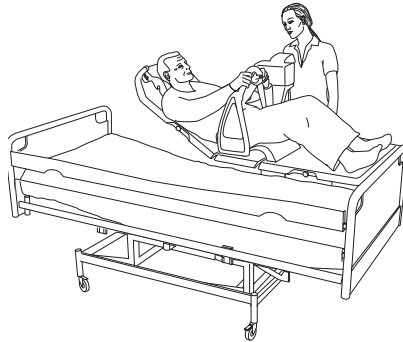


Adjust the lift bath trolley height and move it as far as possible over the bed.

3




Apply brakes

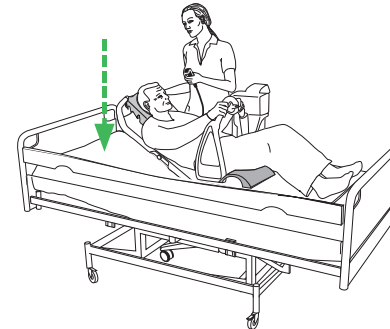

Apply the central brake on the lift bath trolley. Lower the lift bath trolley onto the bed. Lower the backrest until the stretcher is flat.

4

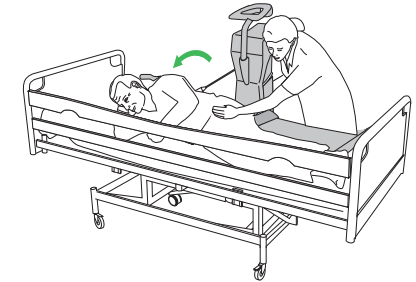


Raise the patient security grip and side guard. Remove the safety belts. Turn the patient on their side, facing away from the lift bath trolley.

5



Release brakes

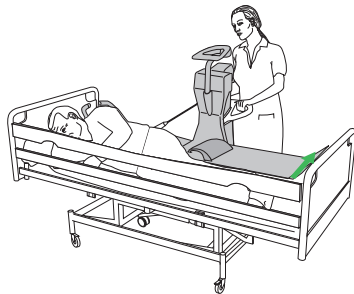

Raise the stretcher off the bed. Release the central brake on the lift bath trolley. Pull the lift bath trolley away from the bed.

Bath the patient



Read the separate instructions for your bathing equipment.

