

Miranti, Bolero, Alenti, Calypso

Clean and disinfect daily.
Clean and disinfect before next patient.

Accessories needed:

- Protective gloves
- Protective glasses
- Spray bottle with disinfectant (if integrated disinfection system is not available)
- Spray bottle (or shower handle) with water
- Cloths - wet and dry
- Disposable towels
- Soft-bristled brush
- Long-handled, soft-bristled brush

NOTE

The cleaning and disinfection procedure is illustrated with Miranti and an integrated disinfection system as an example.



WARNING




To avoid eye and skin damage, always use protective glasses and gloves. If contact occurs rinse with plenty of water. If eyes or skin becomes irritated, contact a physician. Always read the Instructions for Use and the Safety Data Sheet (SDS) of the disinfectant.

To prevent cross-contamination, always follow the disinfection instructions in the Instructions for Use.

To avoid eye or skin irritation, never disinfect in the presence of a patient.

Removing visual residue


1

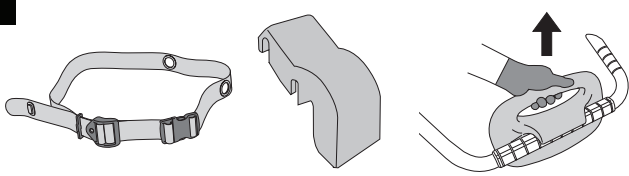

Miranti: Remove headrest, thigh support cushion, patient security grip and safety belts.
Bolero: Remove mattress and safety belts.
Alenti: Remove backrest and safety belt.
Calypso: Remove seats, seat insert, backrest and safety belt.


Place the accessories at the bottom of the bath tub.

2

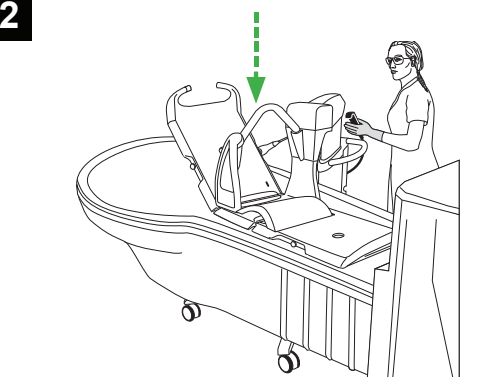

Position the hygiene lift over the bathtub and lower it.

3

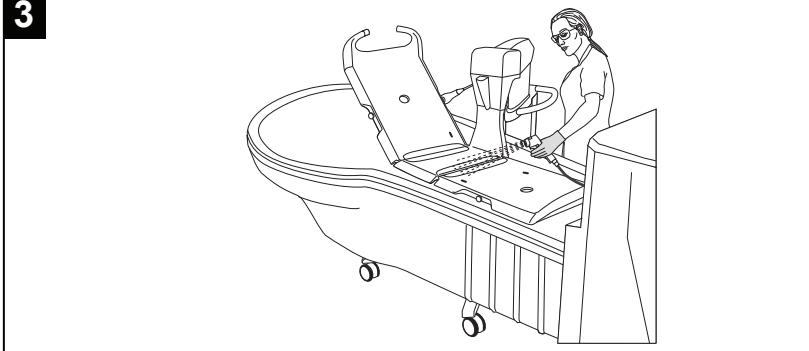

Clean from visible residue using a cloth soaked in water or by showering with water and scrubbing with a brush. Clean the hygiene lift from the top and move downwards.
Clean all removed parts.

Continue with the steps on the next page

Cleaning

4




Put on protective gloves and glasses.

5




Read the separate instructions for disinfection system and disinfectant used

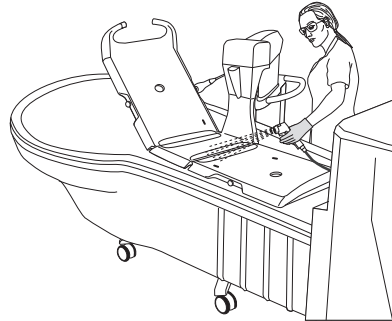


Spray disinfectant on the hygiene lift and on all removed parts.
Use a spray bottle with disinfectant or use the bath's integrated disinfection system.

6

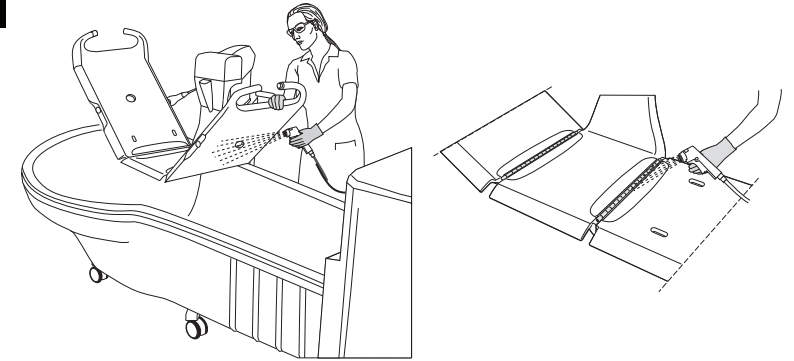


Scrub thoroughly with a brush soaked in disinfectant, especially on handles, hand control, hinges and the underside of the hygiene lift.

7

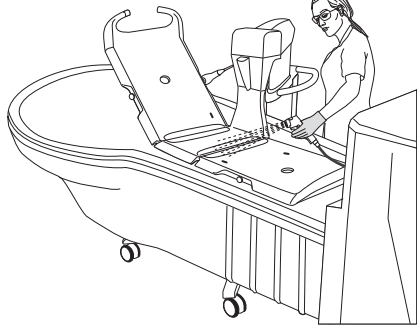


Shower with water (approx. 25 °C (77 °F)) or spray with water and wipe with a clean cloth.

8

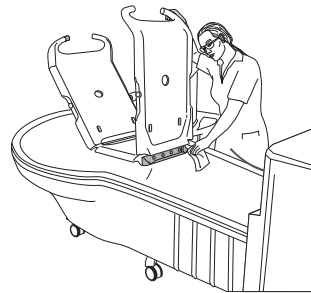


Soak a cloth in disinfectant and wipe all other areas.

9

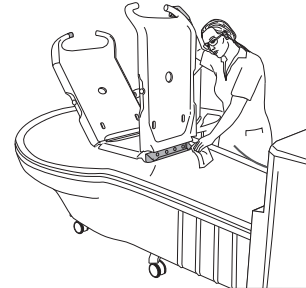

Wet a new cloth with water and remove all traces of disinfectant on other areas. The cloth should occasionally be rinsed in running water.

10

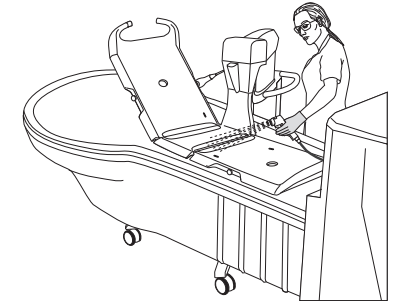


If all disinfectant cannot be removed, spray with water on the affected part and wipe off with disposable towel until all disinfectant is removed.

Continue with the steps on the next page


Disinfection

11

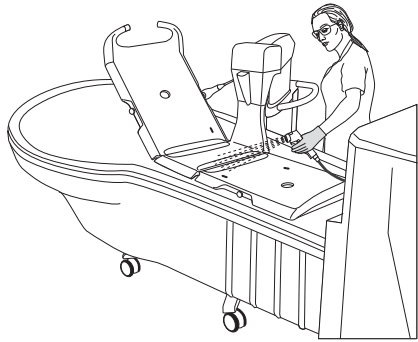


Spray disinfectant on the hygiene lift and on all removed parts.

12

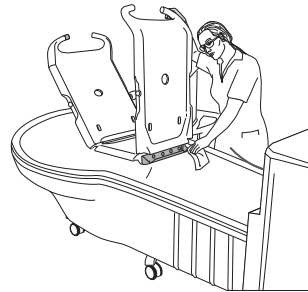


Soak a cloth in disinfectant and wipe all other areas.

13

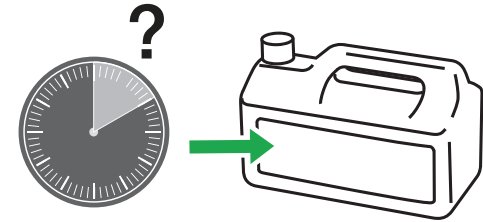


Allow a disinfection time according to the instructions of disinfectant.

14

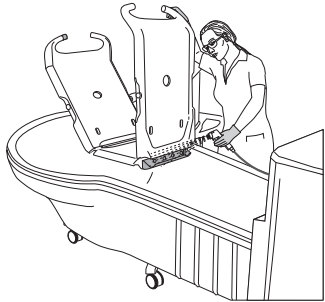


Shower or spray with water and wipe with a clean cloth to remove all traces of disinfectant from the hygiene lift, chassis and all removed parts. The cloth should occasionally be rinsed in running water.

15

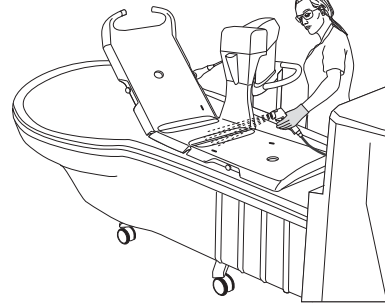

If all disinfectant cannot be removed, spray with water on affected parts and wipe off with disposable towel until all disinfectant is removed.

16

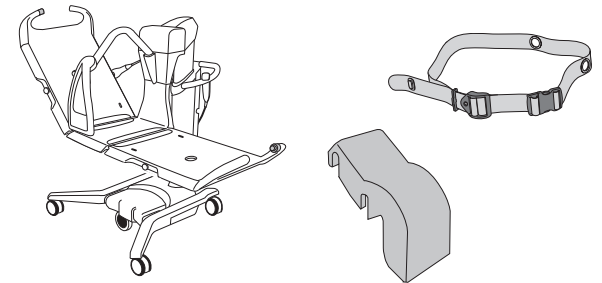

Let all parts dry.