

Sara Combilizer



WARNING



To avoid eye and skin damage, always use protective glasses and gloves.

If contact occurs rinse with plenty of water. If eyes or skin becomes irritated, contact a physician.

Always read the instructions for use and material safety data sheet of the disinfectant.

Clean and disinfect between patients.

Accessories needed

- Protective glasses
- Protective gloves
- Disposable towels
- Spray bottle with water
- Spray bottle with mild neutral detergent
- Spray bottle with approved disinfectant
- Protection plug
(for hand control connection)

Verified Detergents/Disinfectants

- Arjo Clean
- Arjo General Purpose Disinfectant
- Arjo Disinfectant Cleanser IV
- Arjo All Purpose Disinfectant
- Arjo CenKleen IV
- Alcohol (Isopropanol 70%)
- Sodium Hypochlorite 0,1-0,5%
- Mild neutral detergent

Cleaning

1. Place Sara Combilizer in chair position (normal mode).
2. Adjust the height of the chair to an ergonomic working height.
3. Insert either the protection plug or the hand control contact plug correctly in the connection point on the driving handle.
4. Remove pillow and all safety belts.
5. Wet a disposable towel with water and remove visible stains and soil.
6. Spray detergent on all parts including transport handle, control panel and hand control.
7. Spray detergent on pillow and safety belts separately.
8. Wet a new disposable towel with water and remove all detergent.
9. If the detergent cannot be removed, spray with water and wipe off with a disposable towel.
10. Repeat until all detergent is removed.
11. Let all parts dry.

Disinfection

12. Spray approved disinfectant on all parts including transport handle, control panel and hand control.
13. Spray approved disinfectant on pillow and safety belts separately.
14. Allow the disinfectant to stay on the parts according to the recommended disinfection time labelled on the bottle, or at least 2 minutes.
15. Wipe off all traces of disinfectant using disposable towels.