



BUTTON/LAMP	FUNCTION
1	When the yellow light comes on: The machine is running a program. The temperature and cleaning phase can be seen on the display.
2	When the green light comes on: The program is complete. The green lamp goes out when the door is opened and the maximum temperature of the process shows on the display.
3	When the red light comes on: The machine has not reached the disinfection temperature or another error has occurred (Error code F).
4	Display.
5	Open/close door.
6	Sensor to open/close door and to start preselected program.

	U1/U4	Agent low level
	U7	Warning of high temperature for items

For type F error message, see Instructions for Use.