22004 Bariatric Disposable Repositioning Sling

Patient Specific Disposable
 Jetable, patient spécifique
 Paciente desechable específico
 Patientenspezifische Einweg
 Specifico paziente monouso
 Patiënt Specifieke Disposable
 患者一次性专用产品


EN Instruction for use
 FR Instructions d'utilisation
 ES Instrucciones de uso
 GB Gebrauchsanweisung
 IT Istruzioni per l'uso
 NL Instructies voor gebruik
 CN 使用說明書

Attachment straps

Sangle
 Correa
 Gurtband
 Inturino
 Lussen
 錶帶

Body

Pièce de corps
 Pieza del cuerpo
 Körperstück
 Piece corpo
 Lijf
 本体

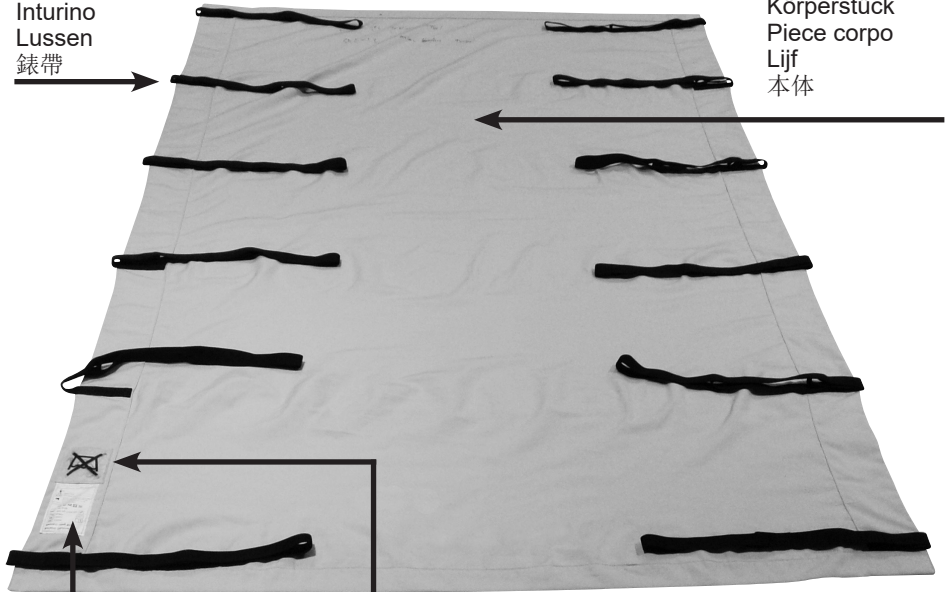Product label
 L'étiquette du produit
 Etiqueta del producto
 Produktetikett
 Etichetta del prodotto
 Etiket van het product
 產品標籤

Do not wash label
 Étiquette, Ne pas laver
 Etiqueta, No lave
 Etikett , Nicht waschen
 Etichetta, Non lavare
 Label, niet wassen
 標籤

Warning! - Avertissement! - Advertencia! - Attenzione! - Warnung! - Waarschuwing - 警告! 請

Always read this instruction and the instruction for the lifter to be used with the sling
 Toujours lire cette instruction et l'instruction pour l'élevateur à être utilisés avec la fronde
 Siempre lea esta Instrucción y la Instrucción para el elevador que se utilizarán con la Honda
 Lesen Sie immer diese Anleitung und die Anleitung für die Heber mit der Schlinge verwendet werden
 Leggere sempre questa Istruzione e l'Istruzione per il sollevatore da utilizzare con la fionda
 Lees altijd deze instructie en de instructie voor de lifter te gebruiken met de draagdoek
 务必阅读本使用说明书以及使用吊兜的升降机的说明书。

Technical Specifications

Part no.	Description	Size	SWL	Material	Dimension
22004	Bariatric Disposable Repositioning Sling	Onesize	Max: 454 kg / 1000 lbs.	Polypropylene (PP) Polyester (PES) Cotton (CO)	1450 x 2100 CM 57 x 82"

Definition in this IFU

Warning: Means Safety warning. Failure to understand or obey this warning may result in injury to you or to others.

Note means: Important information for the correct use of this system or equipment.

Intended Use

The Bariatric Repositioning Sling is intended for lateral transfer of patients between adjacent support surfaces, moving patient up in the bed and turning.

Note:The Bariatric Repositioning Sling must not be used for moving and lifting in sitting position.

Safety instructions

- Always read the label and check that the sling is in good condition without signs of tear and wear such as loose seams, frays and material inconsistency.
- Always make sure not to exceed the SWL (Safe Working Load) defined for the product.
- Always read the IFU for the lifting equipment to perform the lifting

Warning! Do not use the product if any signs of deterioration.

Do Not Wash - this product is designed for single patient use only and should be disposed if in contact with liquid of any kind.

Before and after every use

1. Always perform a risk assessment before starting the specific lifting session in order to make sure the product model is correct and that the lifting procedure is fully understood.
2. Before using the sling, read this IFU and the IFU for the lifting equipment to be used
3. When soiled or stained the sling must be replaced immediately.
4. Check the sling for any fraying, loose stitching, attachments or any deteriorations.
5. Warning: Remember to mark patient name on the care label in order to avoid cross infection between patients.
6. After use, check sling according to the step 3 and 4 above.
7. Store in a dry area.

Attachments

The Repositioning Sling will fit most lifts with standard hangerbars with loop attachment hooks. Check the Lift manufacturers documentation for details.

Service & Maintenance

The sling shall be checked by authorized personnel every 6 months according to ISO10535

Customer support/Trouble shooting

- Please refer to the manual for the lifting equipment used
- In doubt please contact your dealer who supplied the product

Applying the Sling: Moving up in bed or Lateral transfer

1. Position the Repositioning Sling centrally on the bed with the label pointing at the head end of the bed. The label side faced down towards the mattress.
2. Position the hangerbar carefully over the patient (across the patients chest); connect the straps to each side of the hanger bar starting with the head end straps, then the patients' hips straps and leg straps. For Lateral transfer and Moving up in bed use a minimum of five straps on each side.
3. Check patient position and carefully make the required adjustment to achieve the best comfort for the patient, especially that head is supported and legs and knee's position is not causing any unattended force on knee joints before lifting.
4. Raise the hanger bar until all straps are tense to make sure the straps are securely attached.
5. Carefully perform the lift, repositioning or lateral transfer, whilst keeping eye on the patient position and comfort.
6. At desired position, slowly lower the hanger bar until all straps can be released from the hanger bar hooks.

Applying the Sling: Turning in bed

1. Position the Repositioning Sling centrally on the bed with the label pointing at the head end of the bed. The label side faced down towards the mattress.
2. Position the Hanger bar carefully in parallel position over the patient.
3. Attach a minimum of four straps on one side of the sling to the hangerbar. Use the straps at the patients shoulder and hip area. Use a minimum of two hooks on the hangerbar.
4. Check patient position and carefully make the required adjustment to achieve the best comfort for the patient.
5. Check the support of the head and that arms and legs are in the right position.
6. Raise the hanger bar until all straps are tense to make sure all straps are securely attached.
7. Carefully perform the turning, whilst keeping eye on the patient position and comfort.
8. At desired position, slowly lower the hanger bar and release the straps from the hanger bar hooks.

Expected service life

The service life is expected to be 56 performed lifts, however this will be subject to the actual conditions during use.

The Sling is disposable and designed for single patient use and should be disposed after patient hospital stay or when soiled or subjected to liquid.

Storing and Shelf life

The sling should be stored in a dry and clean environment and the shelf lifetime is 5 years in original packaging. Store in temperatures between 0°-40° C (+32°-104°F)

Safe Working Load (SWL)

Max 454kg / 1000 lbs.

Product care & Maintenance

- Always follow the instructions on the product label and check the sling for deteriorations.
- Mark the sling with the specific patient name and keep in dry and clean environment.
- This is a patient specific disposable sling, that must be disposed after the patient has been discharged.

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