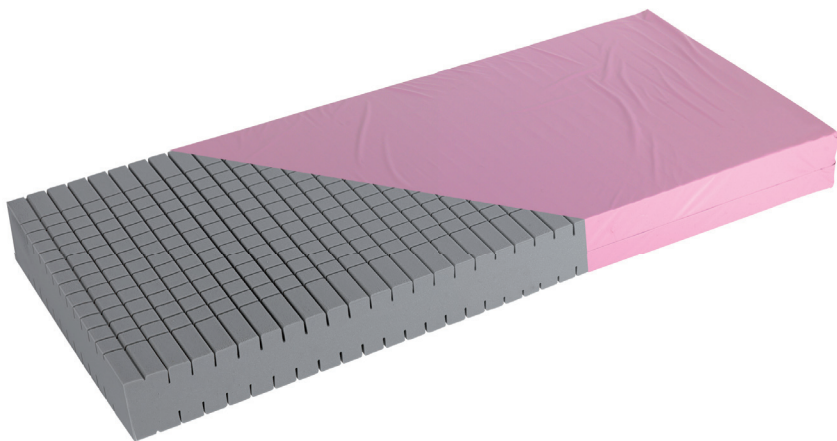
INSTRUCTIONS FOR USE

# Pentaflex

2-way turn mattress replacement system

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**WARNING**

**To avoid injury, always read this Instructions For Use and accompanied documents before using the product.**

Design Policy and Copyright

® and ™ are trademarks belonging to the Arjo group of companies.

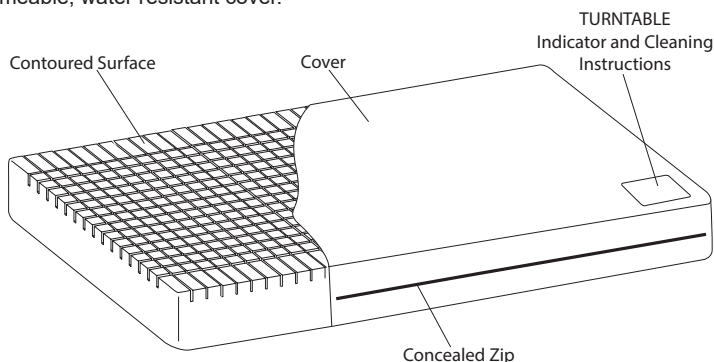
© Arjo 2024.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either whole or in part without the consent of Arjo.

## Product Descriptions

### Mattress replacement system

The Pentaflex® mattress replacement design comprises of a 2-way turn, high resilience polyurethane core with 5 anatomical pressure redistributing zones, protected by a 2-way stretch, vapour permeable, water resistant cover.

A *Pentaflex* foam squab is available for use with in-built bed extensions, for more details refer to the user leaflet 609903.

#### WARNING

Mattress handles are for mattress transport only.

## Clinical applications

### Indications for Use

*Pentaflex 150* is indicated for users up to 150 kg (330 lb / 24 stones).

*Pentaflex 250* is indicated for users up to 250 kg (550 lb / 39 stones). Suitable for a wide range of patients including those with superficial ulcers (in association with an individualised plan of care).

**Note:** These are guidelines only and should not replace clinical experience or judgment. If there is no improvement in the patient's condition, specialist advice should be sought.

#### WARNING

The cover of this product is vapour permeable but not air permeable and may present a suffocation risk. It is the responsibility of the care giver to ensure that the user can use this product safely.

If a serious incident occurs in relation to this medical device, affecting the user, or the patient then the user or patient should report the serious incident to the medical device manufacturer or the distributor. In the European Union, the user should also report the serious incident to the Competent Authority in the member state where they are located.

## Instructions for Use

1. Remove all packaging.
2. Lay the *Pentaflex* mattress replacement onto the bed frame, ensuring that there are no sharp edges which might rip the cover.

#### CAUTION

Do not use the mattress without a cover. It provides a protective barrier against biological contamination of the foam and helps extend the life of the product.

3. Ensure the correct month is showing on the Turntable™ indicator at the foot of the cover. If the incorrect month is showing, rotate the mattress to show correct month at the foot end. Months are indicated as follows:  
1/12 = January  
2/12 = February, etc.
4. Use the *Turntable* indicator as a guide for the monthly rotation and turning of the mattress. This helps to even out wear of the mattress and to extend the life of the product.

## Decontamination

### To Clean the Cover

Wipe all exposed surfaces with a cloth which has been moistened in simple detergent and water. Ensure to remove any organic debris that may be adherent to the surfaces. Dry thoroughly.

### To Disinfect the Cover

For disinfection we recommend a chlorine-releasing agent such as NaOCl or NaDCC diluted to 1,000 ppm (range may vary between 250 & 10,000 ppm depending on contamination status and local policy).

Wipe with a cloth moistened with clean water and dry thoroughly.

Alternatively, use a cloth moistened with alcohol (70%) all allow to air dry.

### Routine checks

The foam core and cover should be inspected regularly for any signs of damage, soiling or staining.

## Technical Data

Product Description	Reliant IS2 MR		Premium MR	Reliant IS2 Cover		Premium Cover	Dimensions		
	Mattress SKU (Sewn Part No.)	Mattress SKU (Welded Part No.)	Mattress SKU (Welded Part No.)	Sewn Cover Part No.	Welded Cover Part No.	Welded Cover Part No.	Length (cm)	Width (cm)	Height (cm)
MR Acute	150SL 198/086	150SL 198086W	150SL 198086P	150SLC 198/086	150SLC 198086W	150SLC 198086P	198	86	15
	250SL 198/086	250SL 198086W	250SL 198086P	250SLC 198/086	250SLC 198086W	250SLC 198086P			
MR Standard	150SL 190/092	150SL 190092W	150SL 190092P	150SLC 190/092	150SLC 190092W	150SLC 190092P	190	92	
	250SL1 90/092	250SL 190092W	250SL 190092P	250SLC 190/092	250SLC 190092W	250SLC 190092P			
MR Double	150SL 190/138	150SL 190138W	150SL 190138P	150SLC 190/138	150SLC 190138W	150SLC 190138P	190	138	
	250SL 190/138	250SL 190138W	250SL 190138P	250SLC 190/138	250SLC 190138W	250SLC 190138P			
MR Euro Wide	150SL 204/120	150SL 204120W	150SL 204120P	150SLC 204/120	150SLC 204120W	150SLC 204120P	204	120	
	250SL 204/120	250SL 204120W	250SL 204120P	250SLC 204/120	250SLC 204120W	250SLC 204120P			
MR Euro Narrow	150SL 200/080	150SL 200080W	150SL 200080P	150SLC 200/080	150SLC 200080W	150SLC 200080P	200	80	
	250SL 200/080	250SL 200080W	250SL 200080P	250SLC 200/080	250SLC 200080W	250SLC 200080P			
Material	High resilience polyurethane foam								
Cover Material	Polyurethane coated polyester fabric with enhanced durability.								

*Pentaflex* mattresses are available in other custom made sizes, e.g. 150SLYYY/ZZZ, where YYY specifies the length (between 140 and 216 cm) and ZZZ specifies the width (between 66 and 140 cm). All sizes must be specified in increments of 2 cm.

When combining beds and mattresses outside the table given above, it is the customer's responsibility to ensure compatibility of mattress and bed frame. Please refer to your bed manufacturers recommended mattress sizes. For further details contact Arjo.

## Fire retardancy:

System meets the requirements of BS EN 597-1: 1995, BS EN 597-2: 1995, and BS 6807: 2006, 5.

### FIRE WARNING

**Cigarettes, lighters, naked flames and exposed heating elements e.g. in electric fires (and other electrical appliances) are sources of fire hazard and could ignite bed linen, blankets and clothing. It is essential that these fire sources are kept away from the mattress.**

**NB: It is highly dangerous to smoke in bed.**

## End of Life Disposal

The following advice is recommended:

- Fabric material used on the mattresses or any other textiles, polymers or plastic materials etc. should be sorted as combustible waste.
- Mattresses at the end of life should be disposed of as waste according to the national or local requirements which may be landfill or combustion.

**Note:** Follow all local regulations and, if applicable, healthcare facility standard operating procedures.

## Warranty and Service

Standard terms and conditions apply to all sales. A copy is available upon request. These contain full details of warranty terms and do not limit the statutory rights of the consumer.

For service, maintenance, and any questions regarding this, or any other product, please contact your local distributor.

## Symbols

Recommended wash temperature:  
15 min at 60°C  
Maximum wash temperature:  
15 min at 95°C

Do not iron

Do not use phenol-based cleaning solutions

Hospital name

Tumble dry at 60°C  
Maximum drying temperature 80°C

Wipe all surfaces with cleaning solution, then wipe with a cloth moistened with water and dry thoroughly.

Date of first use

Operating instructions - Consult Instructions for use.

Routine disinfection: 1,000 ppm.

Fire retardancy

Safe Working Load (SWL)

150: 150kg / 330 lb  
250: 250 kg / 550 lb

CE marking indicating conformity with European Community harmonised legislation

Indicates the product is a Medical Device according to EU Medical Device Regulation 2017/745

Unique device identifier.

**UK Symbol explanation**

This section is only applicable to United Kingdom (UK) market when UK marking is applied to the Arjo medical device labelling.

UK marking indicating conformity with UK Medical Devices Regulations 2002 (SI 2002 No 618, as amended)

**UK Responsible Person & UK Importer:**

**Arjo (UK) Ltd, ArjoHuntleigh House, Houghton Regis. LU5 5XF**

Is the appointed UK Responsible Person as defined in UK Medical Devices Regulations 2002 (SI 2002 No 618, as amended).

For Northern Ireland (NI) CE marking will still apply until further amendment to applicable regulations.

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Intentionally left blank

Intentionally left blank

AUSTRALIA  
Arjo Australia  
Building B, Level 3  
11 Talavera Road  
Macquarie Park, NSW, 2113, Australia  
Phone: 1800 072 040

BELGIQUE / BELGIË  
Arjo Belgium  
Evenbroekveld 16  
9420 Erpe-Mere  
Phone: +32 (0) 53 60 73 80  
Fax: +32 (0) 53 60 73 81  
E-mail: info.belgium@arjo.com

BRASIL  
Arjo Brasil Equipamentos Médicos Ltda  
Rua Marina Ciufulli Zanfelice, 329 PB02  
Galpão - Lapa  
São Paulo – SP – Brasil  
CEP: 05040-000  
Phone: 55-11-3588-5088  
E-mail: vendas.latam@arjo.com  
E-mail: servicios.latam@arjo.com

CANADA  
Arjo Canada Inc.  
90 Matheson Boulevard West  
Suite 350  
CA-MISSISSAUGA, ON, L5R 3R3  
Tel/Tél: +1 (905) 238-7880  
Free: +1 (800) 665-4831  
Fax: +1 (905) 238-7881  
E-mail: info.canada@arjo.com

ČESKÁ REPUBLIKA  
Arjo Czech Republic s.r.o.  
Na Strži 1702/65  
140 00 Praha  
Czech Republic  
Phone No: +420225092307  
E-mail: info.cz@arjo.com

DANMARK  
Arjo A/S  
Vassingerødvej 52  
DK-3540 LYNGBE  
Tel: +45 49 13 84 86  
Fax: +45 49 13 84 87  
E-mail: dk\_kundeservice@arjo.com

DEUTSCHLAND  
Arjo GmbH  
Peter-Sander-Strasse 10  
DE-55252 MAINZ-KASTEL  
Tel: +49 (0) 6134 186 0  
Fax: +49 (0) 6134 186 160  
E-mail: info-de@arjo.com

ESPAÑA  
ARJO IBERIA S.L.  
Poligono Can Salvatella  
c/ Cabanyes 1-7  
08210 Barberà del Valles  
Barcelona - Spain  
Telefono 1: +34 900 921 850  
Telefono 2: +34 931 315 999

FRANCE  
Arjo SAS  
2 Avenue Alcide de Gasperi  
CS 70133  
FR-59436 RONCQ CEDEX  
Tél: +33 (0) 3 20 28 13 13  
Fax: +33 (0) 3 20 28 13 14  
E-mail: info.france@arjo.com

HONG KONG  
Arjo Hong Kong Limited  
Room 411-414, 4/F, Manhattan Centre,  
8 Kwai Cheong Road, Kwai Chung,  
N.T.,  
HONG KONG  
Tel: +852 2960 7600  
Fax: +852 2960 1711

ITALIA  
Arjo Italia S.p.A.  
Via Giacomo Peroni 400-402  
IT-00131 ROMA  
Tel: +39 (0) 6 87426211  
Fax: +39 (0) 6 87426222  
E-mail: Italy.promo@arjo.com

MIDDLE EAST  
Arjo Middle East FZ-LLC  
Office 908, 9th Floor,  
HQ Building, North Tower,  
Dubai Science Park,  
Al Barsha South  
P.O. Box 11488, Dubai,  
United Arab Emirates  
Direct +971 487 48053  
Fax +971 487 48072  
Email: Info.ME@arjo.com

NEDERLAND  
Arjo Nederland BV  
Biezenwei 21  
4004 MB TIEL  
Postbus 6116  
4000 HC TIEL  
Tel: +31 (0) 344 64 08 00  
Fax: +31 (0) 344 64 08 85  
E-mail: info.nl@arjo.com

NEW ZEALAND  
Arjo Ltd  
34 Vestey Drive  
Mount Wellington  
NZ-AUCKLAND 1060  
Tel: +64 (0) 9 573 5344  
Free Call: 0800 000 151  
Fax: +64 (0) 9 573 5384  
E-mail: nz.info@Arjo.com

NORGE  
Arjo Norway AS  
Olaf Helsetts vei 5  
N-0694 OSLO  
Tel: +47 22 08 00 50  
Faks: +47 22 08 00 51  
E-mail: no.kundeservice@arjo.com

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At Arjo, we believe that empowering movement within healthcare environments is essential to quality care. Our products and solutions are designed to promote a safe and dignified experience through patient handling, medical beds, personal hygiene, disinfection, diagnostics, and the prevention of pressure injuries and venous thromboembolism. With over 6500 people worldwide and 65 years caring for patients and healthcare professionals, we are committed to driving healthier outcomes for people facing mobility challenges.

ArjoHuntleigh AB  
Hans Michelsensgatan 10  
211 20 Malmö, Sweden  
[www.arjo.com](http://www.arjo.com)

**arjo**

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Arjo Inc.