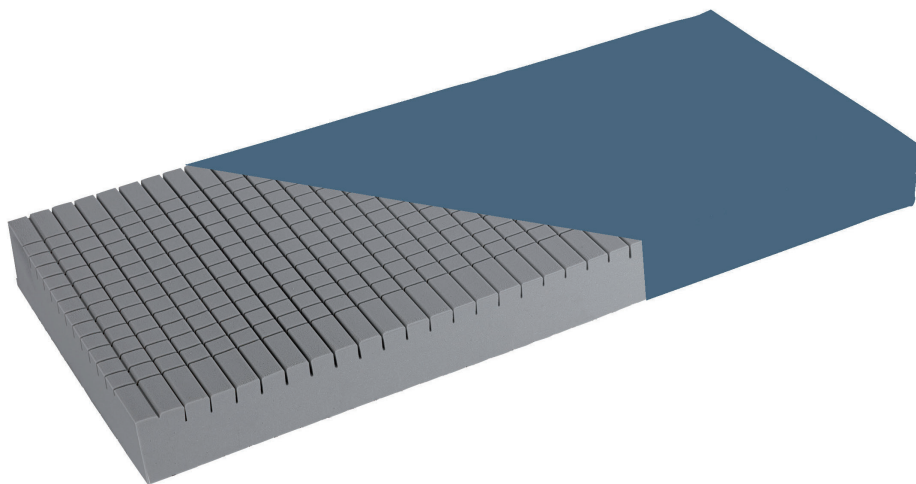
INSTRUCTIONS FOR USE

# Pentaflex Premium

2-way turn mattress replacement

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Brugsvejledning · Bedienungsanleitung · Instrucciones de uso · Mode d'emploi  
Istruzioni per l'uso · Gebruiksaanwijzing · Bruksanvisning · คำแนะนำการใช้งาน

## **WARNING**

**To avoid injury, always read this Instructions For Use and accompanied documents before using the product.**

**Mandatory to read the Instructions For Use.**

### Design Policy and Copyright

® and ™ are trademarks belonging to the Arjo group of companies.

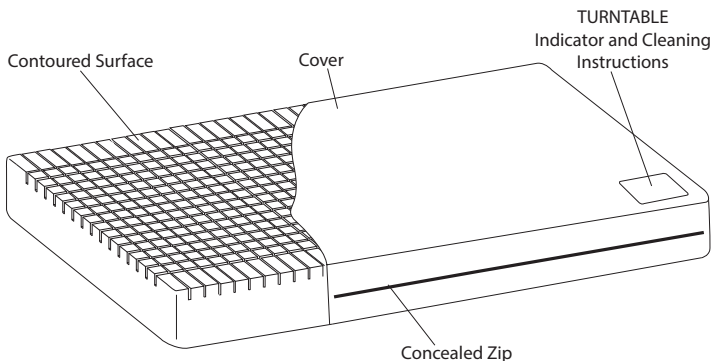
© Arjo 2019.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either whole or in part without the consent of Arjo.

## Product Descriptions

### ***Mattress replacement system***

The Pentaflex® Premium mattress replacement design comprises of a 2-way turn, high resilience polyurethane core with 5 anatomical pressure redistributing zones, protected by a 2-way stretch, vapour permeable, water resistant cover.

A *Pentaflex* Premium foam squab is available for use with in-built bed extensions, for more details refer to the user leaflet 609903.

## Clinical applications

### ***Indications for Use***

*Pentaflex* Premium 150 is indicated for users up to 150 kg (330 lb / 24 stones).

*Pentaflex* Premium 250 is indicated for users up to 250 kg (551 lb / 39 stones). Suitable for low risk patients including those with superficial ulcers (in association with an individualised plan of care).

**Note:** These are guidelines only and should not replace clinical experience or judgment. If there is no improvement in the patient's condition, specialist advice should be sought.

#### **WARNING**

The cover of this product is vapour permeable but not air permeable and may present a suffocation risk. It is the responsibility of the care giver to ensure that the user can use this product safely.

If a serious incident occurs in relation to this medical device, affecting the user, or the patient then the user or patient should report the serious incident to the medical device manufacturer or the distributor. In the European Union, the user should also report the serious incident to the Competent Authority in the member state where they are located.

## Instructions for Use

### ***Mattress replacement system:***

1. Remove all packaging.
2. Lay the *Pentaflex* Premium mattress replacement system onto the bed frame, ensuring that there are no sharp edges which might rip the cover.

#### **CAUTION**

Do not use the mattress without a cover. It helps extend the life of the product and provides a protective barrier against biological contamination of the foam.

3. Ensure the correct month is showing on the Turntable™ indicator at the foot of the cover. If the incorrect month is showing, rotate the mattress to show correct month at the foot end. Months are indicated as follows:  
1/12 = January  
2/12 = February, etc.
4. Use the *Turntable* indicator as a guide for the monthly rotation and turning of the mattress. This helps to even out wear of the mattress and to extend the life of the product.

## Decontamination

### To Clean the Cover

Wipe all exposed surfaces with a cloth which has been moistened in simple detergent and water. Ensure to remove any organic debris that may be adherent to the surfaces. Dry thoroughly.

### To Disinfect the Cover

For disinfection we recommend a chlorine-releasing agent such as NaOCl or NaDCC diluted to 1,000 ppm (range may vary between 250 & 10,000 ppm depending on contamination status and local policy).

Wipe with a cloth moistened with clean water and dry thoroughly.

Alternatively, use a cloth moistened with alcohol (70%) and allow to air dry.

### Routine checks

The foam core and cover should be inspected regularly for any signs of damage, soiling or staining.

## Technical Data

Typical Bed size	-	-	Acute Single Size	-	-	Community Standard Size	Community Double Size
Suitable for use on Arjo Bedframe	20050C Child Anniversary™	Contura™ 420 and 430	Contura 350, 360, 460, 480, 560 and Minuet® 2	Minuet 1	Contura 880 and 980	-	-
Size (cm) L x W x D	170 x 86 x 15	182 x 78 x 15	198 x 86 x 15	200 x 90 x 15	202 x 88 x 15	190 x 92 x 15	190 x 138 x 15
<i>PENTAFLEX</i> 150 Product Code	150SLX 170/086	150SLX 182/078	150SLX 198/086	150SLX 200/090	150SLX 202/088	150SLX 190/092	150SLX 190/138
<i>PENTAFLEX</i> 250 Product Code	250SLX 170/086	250SLX 182/078	250SLX 198/086	250SLX 200/090	250SLX 202/088	250SLX 190/092	250SLX 190/138
Material	High resilience polyurethane foam						
Cover Material	Polyurethane coated polyester fabric with enhanced durability						

*Pentaflex* Premium mattresses are available in other custom made sizes, e.g. 150SLXYYY/ZZZ, where YYY specifies the length (between 140 and 216 cm) and ZZZ specifies the width (between 66 and 140 cm). All sizes must be specified in increments of 2 cm.

When combining beds and mattresses outside the table given above, it is the customer's responsibility to ensure compatibility of mattress and bed frame. Please refer to your bed manufacturer's recommended mattress sizes. For further details contact Arjo.

### Fire retardancy:

System meets the requirements of BS EN 597-1: 1995, BS EN 597-2: 1995, and BS 6807: 2006, 5.

#### FIRE WARNING

**Cigarettes, lighters, naked flames and exposed heating elements e.g. in electric fires (and other electrical appliances) are sources of fire hazard and could ignite bed linen, blankets and clothing. It is essential that these fire sources are kept away from the mattress.**

**NB: It is highly dangerous to smoke in bed.**

## End of Life Disposal

The following advice is recommended:

- Fabric material used on the mattresses or any other textiles, polymers or plastic materials etc. should be sorted as combustible waste.
- Mattresses at the end of life should be disposed of as waste according to the national or local requirements which may be landfill or combustion.

**Note:** Follow all local regulations and, if applicable, healthcare facility standard operating procedures.

## Warranty and Service

Standard terms and conditions apply to all sales. A copy is available upon request. These contain full details of warranty terms and do not limit the statutory rights of the consumer.

For service, maintenance, and any questions regarding this, or any other product, please contact your local distributor.

## Symbols

Recommended wash temperature:  
15 min at 60°C  
Maximum wash temperature: 15 min at 95°C

Do not iron

Do not use phenol-based cleaning solutions

Hospital name

Tumble dry at 60°C  
Maximum drying temperature 80°C

Wipe all surfaces with cleaning solution, then wipe with a cloth moistened with water and dry thoroughly.

Date of first use

Refer to user instructions

Routine disinfection: 10,000 ppm.

Fire retardancy

Safe Working Load (SWL)  
150: 150 kg / 330 lb  
250: 250 kg / 551 lb

CE marking indicating conformity with European Community harmonised legislation

Indicates the product is a Medical Device according to EU Medical Device Regulation 2017/745

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TH

Intentionally left blank

AUSTRALIA  
Arjo Australia  
Building B, Level 3  
11 Talavera Road  
Macquarie Park, NSW, 2113, Australia  
Phone: 1800 072 040

BELGIQUE / BELGIË  
Arjo Belgium  
Evenbroekveld 16  
9420 Erpe-Mere  
Phone: +32 (0) 53 60 73 80  
Fax: +32 (0) 53 60 73 81  
E-mail: info.belgium@arjo.com

BRASIL  
Arjo Brasil Equipamentos Médicos Ltda  
Rua Marina Ciufuli Zanfelice, 329 PB02  
Galpão - Lapa  
São Paulo – SP – Brasil  
CEP: 05040-000  
Phone: 55-11-3588-5088  
E-mail: vendas.latam@arjo.com  
E-mail: servicios.latam@arjo.com

CANADA  
Arjo Canada Inc.  
90 Matheson Boulevard West  
Suite 300  
CA-MISSISSAUGA, ON, L5R 3R3  
Tel/Tél: +1 905 238 7880  
Free: +1 800 665 4831 Institutional  
Free: +1 800 868 0441 Home Care  
Fax: +1 905 238 7881  
E-mail: info.canada@arjo.com

ČESKÁ REPUBLIKA  
Arjo Czech Republic s.r.o.  
Na Strži 1702/65  
140 00 Praha  
Czech Republic  
Phone No: +420225092307  
e-mail: info.cz@arjo.com

DANMARK  
Arjo A/S  
Vassingerødvej 52  
DK-3540 LYNGE  
Tel: +45 49 13 84 86  
Fax: +45 49 13 84 87  
E-mail:  
dk\_kundeservice@arjo.com

DEUTSCHLAND  
Arjo GmbH  
Peter-Sander-Strasse 10  
DE-55252 MAINZ-KASTEL  
Tel: +49 (0) 6134 186 0  
Fax: +49 (0) 6134 186 160  
E-mail: info-de@arjo.com

ESPAÑA  
ARJO IBERIA S.L.  
Poligono Can Salvatella  
c/ Cabanyes 1-7  
08210 Barberà del Valles  
Barcelona - Spain  
Telefono 1: +34 900 921 850  
Telefono 2: +34 931 315 999

FRANCE  
Arjo SAS  
2 Avenue Alcide de Gasperi  
CS 70133  
FR-59436 RONCQ CEDEX  
Tél: +33 (0) 3 20 28 13 13  
Fax: +33 (0) 3 20 28 13 14  
E-mail: info.france@arjo.com

HONG KONG  
Arjo Hong Kong Limited  
Room 411-414, 4/F, Manhattan Centre,  
8 Kwai Cheong Road, Kwai Chung,  
N.T.,  
HONG KONG  
Tel: +852 2960 7600  
Fax: +852 2960 1711

ITALIA  
Arjo Italia S.p.A.  
Via Giacomo Peroni 400-402  
IT-00131 ROMA  
Tel: +39 (0) 6 87426211  
Fax: +39 (0) 6 87426222  
E-mail: Italy.promo@arjo.com

MIDDLE EAST  
Arjo Middle East FZ-LLC  
Office 908, 9th Floor,  
HQ Building, North Tower,  
Dubai Science Park,  
Al Barsha South  
P.O Box 11488, Dubai,  
United Arab Emirates  
Direct +971 487 48053  
Fax +971 487 48072  
Email: Info.ME@arjo.com

NEDERLAND  
Arjo Nederland BV  
Biezenwei 21  
4004 MB TIEL  
Postbus 6116  
4000 HC TIEL  
Tel: +31 (0) 344 64 08 00  
Fax: +31 (0) 344 64 08 85  
E-mail: info.nl@arjo.com

NEW ZEALAND  
Arjo Ltd  
34 Vestey Drive  
Mount Wellington  
NZ-AUCKLAND 1060  
Tel: +64 (0) 9 573 5344  
Free Call: 0800 000 151  
Fax: +64 (0) 9 573 5384  
E-mail: nz.info@Arjo.com

NORGE  
Arjo Norway AS  
Olaf Helsetts vei 5  
N-0694 OSLO  
Tel: +47 22 08 00 50  
Faks: +47 22 08 00 51  
E-mail: no.kundeservice@arjo.com

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At Arjo, we are committed to improving the everyday lives of people affected by reduced mobility and age-related health challenges. With products and solutions that ensure ergonomic patient handling, personal hygiene, disinfection, diagnostics, and the effective prevention of pressure ulcers and venous thromboembolism, we help professionals across care environments to continually raise the standard of safe and dignified care. Everything we do, we do with people in mind.

ArjoHuntleigh AB  
Hans Michelsensgatan 10  
211 20 Malmö, Sweden  
[www.arjo.com](http://www.arjo.com)

**arjo**

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Arjo Inc.