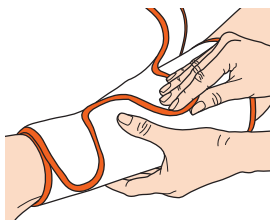
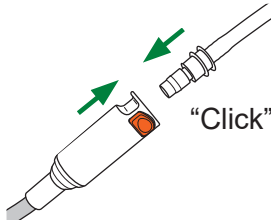
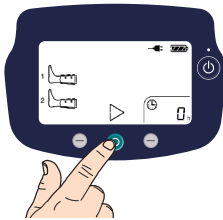


Flowtron ACS900

Start therapy

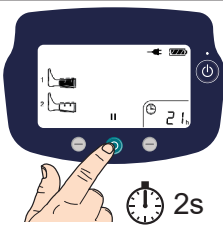
- 

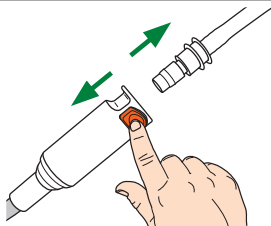
Fit the garment to the patient.
- 

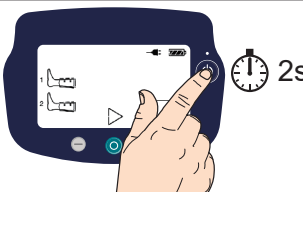
Connect the garment to the pump.
- 

Press Middle Control button to start.

Stop therapy

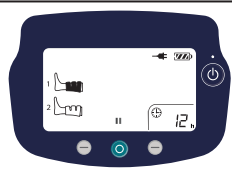
- 

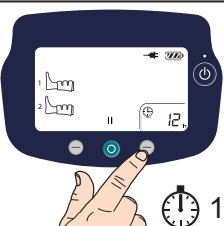
Press and hold Middle Control button.
- 

Disconnect the garment from the pump.
- 

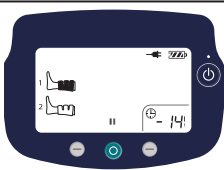
Press and hold Power button to turn off the pump.

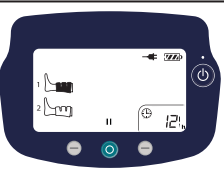
Read patient run timer – hours and minutes (from software version 2.210)

- 

Hours are displayed when the pump is powered on. (12 h)
- 

Press Right Control button to display the minutes.

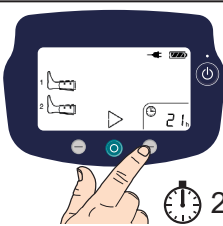


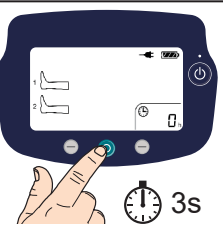
Minutes are displayed for 5 seconds. (-14 = 14 min)
- 

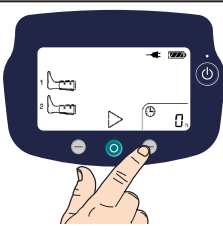
Hours resume being displayed automatically.

Compression time = 12 hours, 14 minutes.

Reset patient run timer – hours and minutes

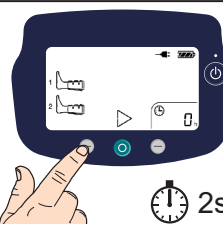
- 

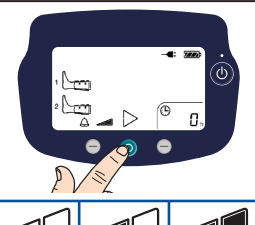
Press hold the Right Control button.
- 

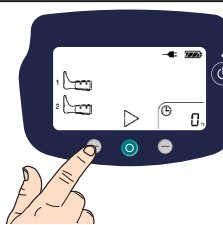
Press and hold Middle Control button to reset to 0.
- 

Press Right Control button to save the new setting.

Change audible alarm volume setting

- 

Press Left Control button.
- 

Press Middle Control button to change volume.
- 

Press Left Control button to save.