


Flowtron ACS900

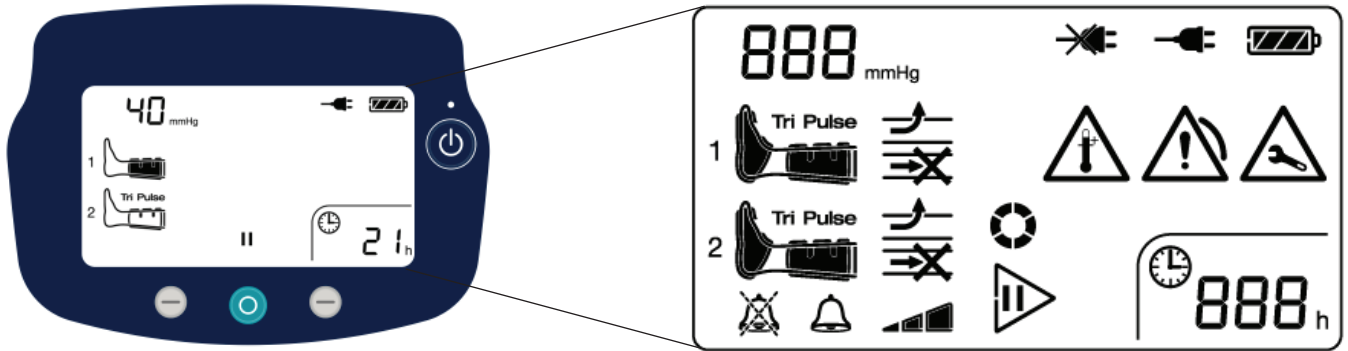

Mains indicator

Not illuminated: mains power is disconnected.
Illuminated green: mains power is connected.

Power button

To switch on: Press and hold for about 2 seconds, the pump goes directly to Stand by.
To switch off: Press and hold for about 2 seconds.



Middle Control button

Press to start therapy.
Press and hold to stop therapy.



Left Control button

Press to mute audible alarm.
Press and hold for Alarm Volume Settings.



Right Control button

Press and hold to enter Reset Patient Run Time mode.



Garment pressure indicator



Garment type and inflation indicators

Connected not inflated	Connected inflated
------------------------	--------------------



Foot



Uniform calf (or calf & thigh)



Sequential (Tri Pulse) calf (or calf & thigh)

1
2

Tuberset connector numbers



Audible alarm muted indicator



Audible alarm status indicator

During alarm: indicates that alarm can be muted.



Warning and alarm indicators

Alarm detected which may cause an interruption in therapy.



Low pressure or garment leak.



Kinked or blocked tubeset.



High temperature.



Hardware fault detected or periodic service required.



Volume indicator



Wait - initializing indicator



Start therapy indicator

Displayed in Standby when at least one garment is connected.



Stop therapy indicator

Displayed in Run mode.



Mains power indicators

Pump connected to mains power supply.



Pump NOT connected to mains power supply.



Battery indicators



Battery empty



Battery 1/4 full



Battery 1/2 full



Battery 3/4 full



Battery full



888_h Therapy time indicator