



BUTTON/LAMP	FUNCTION
1	When the yellow light comes on: The machine is running a program. The temperature and cleaning phase can be seen on the display.
2	When the green light comes on: The program is complete. The green lamp goes out when the door is opened and the maximum temperature of the process shows on the display.
3	When the red light comes on: The machine has not reached the disinfection temperature or another error has occurred (Error code F).
4	Display.
5	Button to start rim flushing Without disinfection!
6	Button for manually adding or removing process agent in a process.
7	Open/close door.
8	Sensor to open/close door and to start preselected program.
9	Button to start the economy program for urine bottles.
10	Button to start the normal program for normally soiled items.
11	Button to start the intensive program for heavily soiled items.
6+11	The button combination for selecting programs with cleaning agents for heavily soiled items.
6+10	The button combination for selecting the program with cleaning agents for general circulation items.

	U1/U4	Agent low level
	U7	Warning of high temperature for items

For type F error message, see Instructions for Use.