



	FUNCTION
1	Flashing yellow: The program can be terminated. Yellow: A program is running. The display shows the current status, for example the cleaning phase and temperature.
2	Green: The program is complete. The light goes out when the door is opened and for a short time the display shows a program summary in the window, for example maximum temperature.
3	Red: Error indication. The display shows an error message.
4	Display. A service technician can enter the information to be shown in the display.
5	Button for choosing economy program for lightly soiled items.
6	Button for choosing normal program for normally soiled items.
7	Button for starting a program.

CODE	DESCRIPTION	MEASURE
U1	Lime descaler low level	Top up descaler agent.
U2	Process agent low level	Top up process agent.
U3	Rinse-aid low level	Top up rinse-aid
U7	Warning of high temperature for goods	Let the goods cool down.
U9	Inactivity warning	The machine has been inactive for 72 hours. A program with an empty chamber must be run before using the machine to clean goods.
H6	Start attempt with door open	Close the door (the machine makes a new start attempt). If the error recurs, contact authorized service personnel.
H7	Cannot lock door	Acknowledge the error. Close the door correctly. Make a fresh start attempt. If the error recurs, contact authorized service personnel.
H8	Cannot unlock door	Acknowledge the error. If the error recurs, contact authorized service personnel.

For type F error codes, see "Instructions for Use".