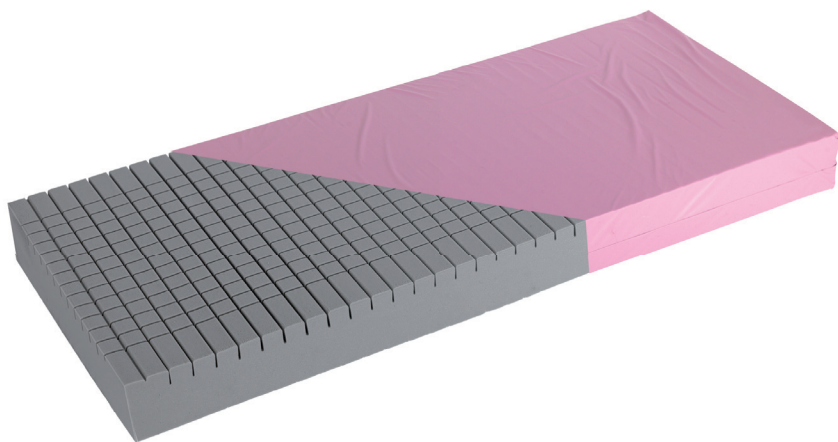
INSTRUCTIONS FOR USE

Pentaflex

4-way turn mattress replacement

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**WARNING**

To avoid injury, always read this Instructions For Use and accompanied documents before using the product.

Design Policy and Copyright

® and ™ are trademarks belonging to the Arjo group of companies.

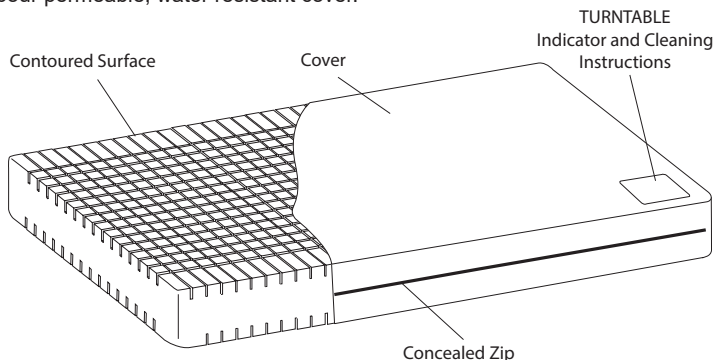
© Arjo 2024.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either whole or in part without the consent of Arjo.

Product Descriptions

Mattress replacement system

The Pentaflex® mattress replacement design comprises of a reversible 4-way turn, high resilience polyurethane core with 5 anatomical pressure redistributing zones per side, protected by a 2-way stretch, vapour permeable, water resistant cover.

A *Pentaflex* foam squab is available for use with in-built bed extensions, for more details refer to the user leaflet 609903.

WARNING

Mattress handles are for mattress transport only.

Clinical applications

Indications for Use

Pentaflex 120 is indicated for users up to 120 kg (264 lb / 19 stones).

Pentaflex 250 is indicated for users up to 250 kg (550 lbs / 39 stones). Suitable for a wide range of patients including those with superficial ulcers (in association with an individualised plan of care).

Note: These are guidelines only and should not replace clinical experience or judgment. If there is no improvement in the patient's condition, specialist advice should be sought.

WARNING

The cover of this product is vapour permeable but not air permeable and may present a suffocation risk. It is the responsibility of the care giver to ensure that the user can use this product safely.

If a serious incident occurs in relation to this medical device, affecting the user, or the patient then the user or patient should report the serious incident to the medical device manufacturer or the distributor. In the European Union, the user should also report the serious incident to the Competent Authority in the member state where they are located.

Instructions for Use

Mattress replacement system:

1. Remove all packaging.
2. Lay the *Pentaflex* mattress replacement onto the bed frame, ensuring that there are no sharp edges which might rip the cover.

CAUTION

Do not use the mattress without a cover. It provides a protective barrier against biological contamination of the foam.

3. Ensure the correct month is showing on the Turntable™ indicator at the foot of the cover. If the incorrect month is showing, rotate the mattress to show correct month at the foot end. Months are indicated as follows:
1/12 = January
2/12 = February, etc.
4. Use the *Turntable* indicator as a guide for the monthly rotation and turning of the mattress. This helps to even out wear of the mattress and to extend the life of the product.

Decontamination

To Clean the Cover

Wipe all exposed surfaces with a cloth which has been moistened in simple detergent and water. Ensure to remove any organic debris that may be adherent to the surfaces. Dry thoroughly.

To Disinfect the Cover

For disinfection we recommend a chlorine-releasing agent such as NaOCl or NaDCC diluted to 1,000 ppm (range may vary between 250 & 10,000 ppm depending on contamination status and local policy).

Wipe with a cloth moistened with clean water and dry thoroughly.

Technical Data

Typical Bed size	-	-	Acute Single Size	-	-	Community Standard Size	Community Double Size
Suitable for use on Arjo Bedframe	20050C Child Anniversary™	Contura™ 420 and 430	Contura 350, 360, 460, 480, 560 and Minuet® 2	Minuet 1	Contura 880 and 980	-	-
Size (cm) L x W x D	170 x 86 x 17	182 x 78 x 17	198 x 86 x 17	200 x 90 x 17	202 x 88 x 17	190 x 92 x 17	190 x 138 x 17
<i>PENTAFLEX</i> 120 Product Code	120PM 170/086	120PM 182/078	120 PM 198/086	120PM 200/090	120PM 202/088	120PM 190/092	120PM 190/138
<i>PENTAFLEX</i> 250 Product Code	250PM 170/086	250PM 182/078	250PM 198/086	250PM 200/090	250PM 202/088	250PM 190/092	250PM 190/138
Material	High resilience polyurethane foam						
Cover Material	Polyurethane coated polyester fabric with enhanced durability						

Pentaflex mattresses are available in other custom made sizes, e.g. 120PMYYY/ZZZ, where YYY specifies the length (between 140 and 216 cm) and ZZZ specifies the width (between 66 and 140 cm). All sizes must be specified in increments of 2 cm.

When combining beds and mattresses outside the table given above, it is the customer's responsibility to ensure compatibility of mattress and bed frame. Please refer to your bed manufacturers recommended mattress sizes. For further details contact Arjo.

Fire retardancy:

System meets the requirements of BS EN 597-1: 1995, BS EN 597-2: 1995, and BS 6807: 2006, 5.

FIRE WARNING

Cigarettes, lighters, naked flames and exposed heating elements e.g. in electric fires (and other electrical appliances) are sources of fire hazard and could ignite bed linen, blankets and clothing. It is essential that these fire sources are kept away from the mattress.

NB: It is highly dangerous to smoke in bed.

Alternatively, use a cloth moistened with alcohol (70%) all allow to air dry.

Routine checks

The foam core and cover should be inspected regularly for any signs of damage, soiling or staining.

End of Life Disposal

The following advice is recommended:

- Fabric material used on the mattresses or any other textiles, polymers or plastic materials etc. should be sorted as combustible waste.
- Mattresses at the end of life should be disposed of as waste according to the national or local requirements which may be landfill or combustion.

Note: Follow all local regulations and, if applicable, healthcare facility standard operating procedures.

Symbols

Recommended wash temperature:
15 min at 60°C
Maximum wash temperature: 15 min
at 95°C

Do not iron

Do not use phenol-based cleaning
solutions

Hospital name

Tumble dry at 60°C
Maximum drying temperature 80°C

Wipe all surfaces with cleaning solution,
then wipe with a cloth moistened with
water and dry thoroughly.

Date of first use

Warranty and Service

Standard terms and conditions apply to all sales. A copy is available upon request. These contain full details of warranty terms and do not limit the statutory rights of the consumer.

For service, maintenance, and any questions regarding this, or any other product, please contact your local distributor.

Operating instructions - Consult
Instructions for use.

Routine disinfection: 1,000 ppm.

Fire retardancy

Safe Working Load (SWL)
250-450 kg / 551-992 lb

CE marking indicating conformity with
European Community harmonised
legislation

Indicates the product is a Medical
Device according to EU Medical Device
Regulation 2017/745

Unique device identifier.

UK Symbol explanation

This section is only applicable to United Kingdom (UK) market when UK marking is applied to the Arjo medical device labelling.

**UK
CA**

UK marking indicating conformity with UK Medical Devices Regulations
2002 (SI 2002 No 618, as amended)

UK Responsible Person & UK Importer:

Arjo (UK) Ltd, ArjoHuntleigh House, Houghton Regis. LU5 5XF

Is the appointed UK Responsible Person as defined in UK Medical Devices Regulations 2002 (SI 2002 No 618, as amended).

For Northern Ireland (NI) CE marking will still apply until further amendment to applicable regulations.

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Intentionally left blank

AUSTRALIA
Arjo Australia
Building B, Level 3
11 Talavera Road
Macquarie Park, NSW, 2113, Australia
Phone: 1800 072 040

BELGIQUE / BELGIË
Arjo Belgium
Evenbroekveld 16
9420 Erpe-Mere
Phone: +32 (0) 53 60 73 80
Fax: +32 (0) 53 60 73 81
E-mail: info.belgium@arjo.com

BRASIL
Arjo Brasil Equipamentos Médicos Ltda
Rua Marina Ciufulli Zanfelice, 329 PB02
Galpão - Lapa
São Paulo – SP – Brasil
CEP: 05040-000
Phone: 55-11-3588-5088
E-mail: vendas.latam@arjo.com
E-mail: servicios.latam@arjo.com

CANADA
Arjo Canada Inc.
90 Matheson Boulevard West
Suite 350
CA-MISSISSAUGA, ON, L5R 3R3
Tel/Tél: +1 (905) 238-7880
Free: +1 (800) 665-4831
Fax: +1 (905) 238-7881
E-mail: info.canada@arjo.com

ČESKÁ REPUBLIKA
Arjo Czech Republic s.r.o.
Na Strži 1702/65
140 00 Praha
Czech Republic
Phone No: +420225092307
E-mail: info.cz@arjo.com

DANMARK
Arjo A/S
Vassingerødvej 52
DK-3540 LYNGE
Tel: +45 49 13 84 86
Fax: +45 49 13 84 87
E-mail: dk_kundeservice@arjo.com

DEUTSCHLAND
Arjo GmbH
Peter-Sander-Strasse 10
DE-55252 MAINZ-KASTEL
Tel: +49 (0) 6134 186 0
Fax: +49 (0) 6134 186 160
E-mail: info-de@arjo.com

ESPAÑA
ARJO IBERIA S.L.
Poligono Can Salvatella
c/ Cabanyes 1-7
08210 Barberà del Valles
Barcelona - Spain
Telefono 1: +34 900 921 850
Telefono 2: +34 931 315 999

FRANCE
Arjo SAS
2 Avenue Alcide de Gasperi
CS 70133
FR-59436 RONCQ CEDEX
Tél: +33 (0) 3 20 28 13 13
Fax: +33 (0) 3 20 28 13 14
E-mail: info.france@arjo.com

HONG KONG
Arjo Hong Kong Limited
Room 411-414, 4/F, Manhattan Centre,
8 Kwai Cheong Road, Kwai Chung,
N.T.,
HONG KONG
Tel: +852 2960 7600
Fax: +852 2960 1711

ITALIA
Arjo Italia S.p.A.
Via Giacomo Peroni 400-402
IT-00131 ROMA
Tel: +39 (0) 6 87426211
Fax: +39 (0) 6 87426222
E-mail: Italy.promo@arjo.com

MIDDLE EAST
Arjo Middle East FZ-LLC
Office 908, 9th Floor,
HQ Building, North Tower,
Dubai Science Park,
Al Barsha South
P.O. Box 11488, Dubai,
United Arab Emirates
Direct +971 487 48053
Fax +971 487 48072
Email: Info.ME@arjo.com

NEDERLAND
Arjo Nederland BV
Biezenwei 21
4004 MB TIEL
Postbus 6116
4000 HC TIEL
Tel: +31 (0) 344 64 08 00
Fax: +31 (0) 344 64 08 85
E-mail: info.nl@arjo.com

NEW ZEALAND
Arjo Ltd
34 Vestey Drive
Mount Wellington
NZ-AUCKLAND 1060
Tel: +64 (0) 9 573 5344
Free Call: 0800 000 151
Fax: +64 (0) 9 573 5384
E-mail: nz.info@Arjo.com

NORGE
Arjo Norway AS
Olaf Helsetts vei 5
N-0694 OSLO
Tel: +47 22 08 00 50
Faks: +47 22 08 00 51
E-mail: no.kundeservice@arjo.com

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At Arjo, we believe that empowering movement within healthcare environments is essential to quality care. Our products and solutions are designed to promote a safe and dignified experience through patient handling, medical beds, personal hygiene, disinfection, diagnostics, and the prevention of pressure injuries and venous thromboembolism. With over 6500 people worldwide and 65 years caring for patients and healthcare professionals, we are committed to driving healthier outcomes for people facing mobility challenges.

ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö, Sweden
www.arjo.com

arjo

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Arjo Inc.