INSTRUCTIONS FOR USE

Pentaflex Squab

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Brugsvejledning · Bedienungsanleitung · Návod k použití · Instrucciones de uso
Mode d'emploi · Istruzioni per l'uso · Gebruiksaanwijzing · Instrucțiuni de utilizare
Uputstvo za upotrebu · Instrukcja obsługi

WARNING

To avoid injury, always read this Instructions For Use and accompanied documents before using the product.

Mandatory to read the Instructions For Use.

Design Policy and Copyright

® and ™ are trademarks belonging to the Arjo group of companies.

© Arjo 2019.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either whole or in part without the consent of Arjo.

Product Description

The foam squab is for use with in-built bed extensions providing an extra 180 mm (7") for the taller patient.

Intended use

The foam squab comprises a high resilience polyurethane foam core with a 2-way stretch, water resistant, vapour permeable cover.

Note: These are guidelines only and should not replace clinical experience or judgement. If there is no improvement in the patient's condition, specialist advice should be sought.

WARNING

The cover of this product is vapour permeable but not air permeable and may present a suffocation risk. It is the responsibility of the care giver to ensure that the user can use this product safely.

Instructions for Use

1. Remove all packaging.
2. ENSURE THAT THE SQUAB IS FITTED WITH ITS COVER; It provides a protective barrier which helps to prolong the product life.
3. Extend the in-built bed extension.
4. Decide at which end (foot or head) to fit the foam squab; if fitting the squab at the head end, pull the Mattress toward the foot end.
5. Place the squab into the extended space, ensuring that there are no sharp edges which may rip the cover.
6. Ensure that the zip is NOT uppermost.

CAUTION

Do not use the squab without a cover. It provides a protective barrier against biological contamination of the foam.

Decontamination

To Clean the Cover

Wipe all exposed surfaces with a cloth which has been moistened in simple detergent and water. Ensure to remove any organic debris that may be adherent to the surfaces.

Dry thoroughly.

Recommended wash temperature: 15 min at 60°C. Maximum wash temperature: 15 min at 95°C. Tumble dry at 60°C. Maximum drying temperature 80°C.

To Disinfect the Cover

For disinfection we recommend a chlorine-releasing agent such as NaOCl or NaDCC diluted to 1,000 ppm (range may vary between 250 & 10,000 ppm depending on contamination status and local policy).

Wipe with a cloth moistened with clean water and dry thoroughly.

Alternatively, use a cloth moistened with alcohol (70%) allow to air dry.

Routine checks

The foam core and cover should be inspected regularly for any signs of damage, soiling or staining.

	Pentaflex Foam Squab	Contoura 1000 Foam Squab	Foam Squab
Size (mm)	180 X 180 X 860	180 X 180 X 920	150 X 180 X 860
Product code	SQB1	SQB6	SQB2
Material	High resilience polyurethane foam		
Cover material	Polyurethane coated knitted fabric		

Fire retardancy:

System meets the requirements of BS EN 597-1: 1995, BS EN 597-2: 1995 and BS 6807: 2006, 5.

FIRE WARNING

Cigarettes, lighters, naked flames and exposed heating elements e.g. in electric fires (and other electrical appliances) are sources of fire hazard and could ignite bed linen, blankets and clothing. It is essential that these fire sources are kept away from the mattress.

NB: It is highly dangerous to smoke in bed.

Note: Follow all local regulations and, if applicable, healthcare facility standard operating procedures.

Warranty and Service

Standard terms and conditions apply to all sales. A copy is available upon request. These contain full details of warranty terms and do not limit the statutory rights of the consumer.

For service, maintenance, and any questions regarding this, or any other product, please contact your local distributor.

Symbols

Recommended wash temperature:
15 min at 60°C
Maximum wash temperature:
15 min at 95°C

Wipe all surfaces with cleaning solution, then wipe with a cloth moistened with water and dry thoroughly.

Do not iron

Refer to user instructions

Do not use phenol-based cleaning solutions

Routine disinfection: 1,000 ppm.

Tumble dry at 60°C
Maximum drying temperature 80°C

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HONG KONG
Arjo Hong Kong Limited
Room 411-414, 4/F, Manhattan Centre,
8 Kwai Cheong Road, Kwai Chung, N.T.,
HONG KONG
Tel: +852 2960 7600
Fax: +852 2960 1711

ITALIA
Arjo Italia S.p.A.
Via Giacomo Peroni 400-402
IT-00131 ROMA
Tel: +39 (0) 6 87426211
Fax: +39 (0) 6 87426222
E-mail: Italy.promo@arjo.com

MIDDLE EAST
Arjo Middle East FZ-LLC
Office 908, 9th Floor,
HQ Building, North Tower,
Dubai Science Park,
Al Barsha South
P.O Box 11488, Dubai,
United Arab Emirates
Direct +971 487 48053
Fax +971 487 48072
Email: Info.ME@arjo.com

NEDERLAND
Arjo BV
Biezenwei 21
4004 MB TIEL
Postbus 6116
4000 HC TIEL
Tel: +31 (0) 344 64 08 00
Fax: +31 (0) 344 64 08 85
E-mail: info.nl@arjo.com

NEW ZEALAND
Arjo Ltd
34 Vestey Drive
Mount Wellington
NZ-AUCKLAND 1060
Tel: +64 (0) 9 573 5344
Free Call: 0800 000 151
Fax: +64 (0) 9 573 5384
E-mail: nz.info@Arjo.com

NORGE
Arjo Norway AS
Olaf Helsetts vei 5
N-0694 OSLO
Tel: +47 22 08 00 50
Faks: +47 22 08 00 51
E-mail: no.kundeservice@arjo.com

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Address page - REV 24: 04/2019

At Arjo, we are committed to improving the everyday lives of people affected by reduced mobility and age-related health challenges. With products and solutions that ensure ergonomic patient handling, personal hygiene, disinfection, diagnostics, and the effective prevention of pressure ulcers and venous thromboembolism, we help professionals across care environments to continually raise the standard of safe and dignified care. Everything we do, we do with people in mind.

ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö, Sweden
www.arjo.com

arjo

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Arjo Inc.