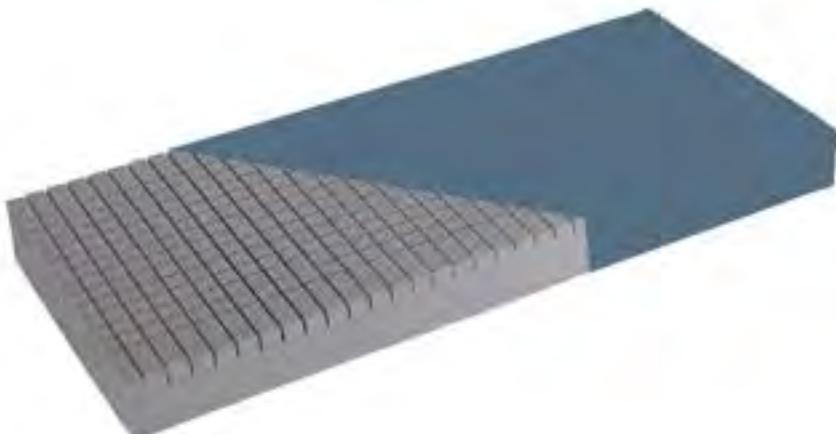
## INSTRUCTIONS FOR USE

# Pentaflex Plus

2-way turn mattress replacement system

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## **WARNING**

**To avoid injury, always read this Instructions For Use and accompanied documents before using the product.**

### **Design Policy and Copyright**

® and ™ are trademarks belonging to the Arjo group of companies.

© Arjo 2022.

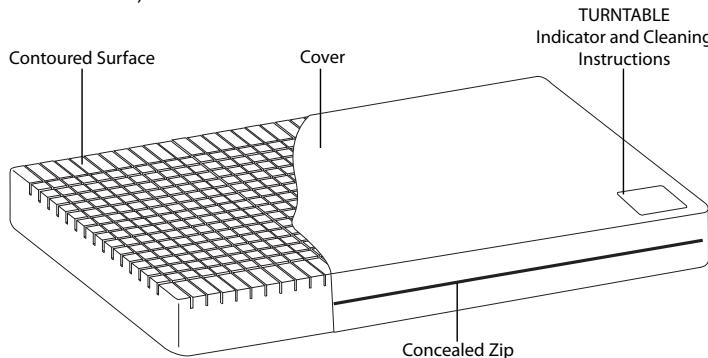
As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either whole or in part without the consent of Arjo.

## Product Descriptions

### **Mattress replacement system**

The Pentaflex® Plus 2-way mattress replacement system is a support system designed for the bed range formally known as the Contoura 1000. The *Pentaflex* Plus 2-way mattress replacement design comprises a high resilience polyurethane foam with a 2 way stretch, water resistant, vapour permeable cover.

**Note:** The *Pentaflex* Plus mattress replacement is not recommended for Patients/Users who are below 250 kg (551 lb / 39 stones) in weight. Maximum Safe Working Load (SWL) is 450 kg (992 lb / 70 stones).

#### **WARNING**

Mattress handles are for mattress transport only.

### **Mattress replacement side covers**

The mattress replacement side covers are designed to complement the mattress replacement system. The side covers comprise a high resilience polyurethane foam with a 2-way stretch, water-resistant, vapour permeable cover.

### **Clinical applications**

#### **Indications for Use**

The *Pentaflex* Plus 2-way mattress replacement system mattress is an aid to the prevention of pressure ulcers.

**Note:** These are guidelines only and should not replace clinical experience or judgment. If there is no improvement in the patient's condition, specialist advice should be sought.

#### **WARNING**

The cover of this product is vapour permeable but not air permeable and may present a suffocation risk. It is the responsibility of the care giver to ensure that the user can use this product safely.

If a serious incident occurs in relation to this medical device, affecting the user, or

the patient then the user or patient should report the serious incident to the medical device manufacturer or the distributor. In the European Union, the user should also report the serious incident to the Competent Authority in the member state where they are located.

### **Instructions for Use**

#### **Mattress replacement system:**

1. Remove all packaging.
2. Lay the *Pentaflex* Plus 2-way mattress replacement system onto the bed frame, ensuring that there are no sharp edges which might rip the cover. Ensure that the contoured side of the mattress is uppermost.
3. Ensure the correct month is showing on the Turntable™ indicator at the foot of the cover. If the incorrect month is showing, rotate the mattress to show correct month at the foot end. Months are indicated as follows:  
1/12 = January  
2/12 = February, etc.

**Mattress replacement side covers:**

1. Remove all packaging.
  2. Ensure bed side supports are extended and in the upright closed position.
  3. With the storage pocket facing outwards, slip the cover over the side support.
  4. Repeat for the opposite side.
- Note:** If the side supports are to be used in the open position, proceed as follows.
5. Remove the additional support piece from the storage pocket.
  6. Position the support piece in the gap created by opening the side support. Bring the support piece over the top of the side support and locate between the mattress and side support to form a continuation of the mattress.
  7. To return the side supports to the upright (closed) position, reverse Steps 5 and 6.

**CAUTION**

Do not use the mattress without a cover. It provides a protective barrier against biological contamination of the foam.

**Decontamination****Mattress replacement system and side covers:**

**Note:** The side covers are a finished assembly and must not be machine washed.

**To Clean the Cover**

Wipe all exposed surfaces with a cloth which has been moistened in simple detergent and water. Ensure to remove any organic debris that may be adherent to the surfaces. Dry thoroughly.

**To Disinfect the Cover**

For disinfection we recommend a chlorine-releasing agent such as NaOCl or NaDCC diluted to 1,000 ppm (range may vary between 250 & 10,000 ppm depending on contamination status and local policy).

Wipe with a cloth moistened with clean water and dry thoroughly.

Alternatively, use a cloth moistened with alcohol (70%) all allow to air dry.

**Routine checks**

The foam core and cover should be inspected regularly for any signs of damage, soiling or staining.

**Technical Data****Pentaflex Plus 2-way mattress replacement system and side covers**

	Mattress replacement system	Side covers
Size (mm)	2060 X 920 X 172	1535 X 565 X 160
Product code	450PM000/000	PMSR PBSR
Material	High resilience polyurethane foam	High resilience polyurethane foam
Cover material	Polyurethane coated knitted fabric	Coated knitted fabric

**Patient weight:**

Over 250 kg (551 lb / 39 stones)

**Fire retardancy:**

System meets the requirements of BS EN 597-1: 1995, BS EN 597-2: 1995, and BS 6807: 2006, 5.

**FIRE WARNING**

Cigarettes, lighters, naked flames and exposed heating elements e.g. in electric fires (and other electrical appliances) are sources of fire hazard and could ignite bed linen, blankets and clothing. It is essential that these fire sources are kept away from the mattress.

**NB: It is highly dangerous to smoke in bed.**

## End of Life Disposal

The following advice is recommended:

- Fabric material used on the mattresses or any other textiles, polymers or plastic materials etc. should be sorted as combustible waste.
- Mattresses at the end of life should be disposed of as waste according to the national or local requirements which may be landfill or combustion.

**Note:** Follow all local regulations and, if applicable, healthcare facility standard operating procedures.

## Symbols

	Recommended wash temperature: 15 min at 60°C Maximum wash temperature: 15 min at 95°C		Operating instructions - Consult Instructions for use.
	Do not iron		Routine disinfection: 1,000 ppm. 1000 ppm NaOCl NaDCC
	Do not use phenol-based cleaning solutions		Fire retardancy
	Hospital name		Safe Working Load (SWL) 250-450 kg / 551-992 lb
	Tumble dry at 60°C Maximum drying temperature 80°C		CE marking indicating conformity with European Community harmonised legislation
	Wipe all surfaces with cleaning solution, then wipe with a cloth moistened with water and dry thoroughly.		Indicates the product is a Medical Device according to EU Medical Device Regulation 2017/745
	Date of first use		Unique device identifier.

## UK Symbol explanation

This section is only applicable to United Kingdom (UK) market when UK marking is applied to the Arjo medical device labelling.

UK marking indicating conformity with UK Medical Devices Regulations 2002 (SI 2002 No 618, as amended)

### UK Responsible Person & UK Importer:

**Arjo (UK) Ltd, ArjoHuntleigh House, Houghton Regis, LU5 5XF**

Is the appointed UK Responsible Person as defined in UK Medical Devices Regulations 2002 (SI 2002 No 618, as amended).

For Northern Ireland (NI) CE marking will still apply until further amendment to applicable regulations.

## Warranty and Service

Standard terms and conditions apply to all sales. A copy is available upon request. These contain full details of warranty terms and do not limit the statutory rights of the consumer.

For service, maintenance, and any questions regarding this, or any other product, please contact your local distributor.

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**Intentionally left blank**

**AUSTRALIA**

Arjo Australia  
Building B, Level 3  
11 Talavera Road  
Macquarie Park, NSW, 2113, Australia  
Phone: 1800 072 040

**BELGIQUE / BELGIË**

Arjo Belgium  
Evenbroekveld 16  
9420 Erpe-Mere  
Phone: +32 (0) 53 60 73 80  
Fax: +32 (0) 53 60 73 81  
E-mail: info.belgium@arjo.com

**BRASIL**

Arjo Brasil Equipamentos Médicos Ltda  
Rua Marina Ciufoli Zanfelice, 329 PB02  
Galpão - Lapa  
São Paulo – SP – Brasil  
CEP: 05040-000  
Phone: 55-11-3588-5088  
E-mail: vendas.latam@arjo.com  
E-mail: servicios.latam@arjo.com

**CANADA**

Arjo Canada Inc.  
90 Matheson Boulevard West  
Suite 350  
CA-MISSISSAUGA, ON, L5R 3R3  
Tel/Tél: +1 (905) 238-7880  
Free: +1 (800) 665-4831  
Fax: +1 (905) 238-7881  
E-mail: info.canada@arjo.com

**ČESKÁ REPUBLIKA**

Arjo Czech Republic s.r.o.  
Na Strzi 1702/65  
140 00 Praha  
Czech Republic  
Phone No: +420225092307  
E-mail: info.cz@arjo.com

**DANMARK**

Arjo A/S  
Vassingerødvej 52  
DK-3540 LYNGE  
Tel: +45 49 13 84 86  
Fax: +45 49 13 84 87  
E-mail: dk\_kundeservice@arjo.com

**DEUTSCHLAND**

Arjo GmbH  
Peter-Sander-Strasse 10  
DE-55252 MAINZ-KASTEL  
Tel: +49 (0) 6134 186 0  
Fax: +49 (0) 6134 186 160  
E-mail: info-de@arjo.com

**ESPÀÑA**

ARJO IBERIA S.L.  
Polígono Can Salvatella  
c/ Cabanyes 1-7  
08210 Barberà del Valles  
Barcelona - Spain  
Telefono 1: +34 900 921 850  
Telefono 2: +34 931 315 999

**FRANCE**

Arjo SAS  
2 Avenue Alcide de Gasperi  
CS 70133  
FR-59436 RONCQ CEDEX  
Tél: +33 (0) 3 20 28 13 13  
Fax: +33 (0) 3 20 28 13 14  
E-mail: info.france@arjo.com

**HONG KONG**

Arjo Hong Kong Limited  
Room 411-414, 4/F, Manhattan Centre,  
8 Kwai Cheong Road, Kwai Chung,  
N.T.,  
HONG KONG  
Tel: +852 2960 7600  
Fax: +852 2960 1711

**ITALIA**

Arjo Italia S.p.A.  
Via Giacomo Peroni 400-402  
IT-00131 ROMA  
Tel: +39 (0) 6 87426211  
Fax: +39 (0) 6 87426222  
E-mail: Italy.promo@arjo.com

**MIDDLE EAST**

Arjo Middle East FZ-LLC  
Office 908, 9th Floor,  
HQ Building,North Tower,  
Dubai Science Park,  
Al Barsha South  
P.O. Box 11488, Dubai,  
United Arab Emirates  
Direct +971 487 48053  
Fax +971 487 48072  
Email: Info.ME@arjo.com

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At Arjo, we believe that empowering movement within healthcare environments is essential to quality care. Our products and solutions are designed to promote a safe and dignified experience through patient handling, medical beds, personal hygiene, disinfection, diagnostics, and the prevention of pressure injuries and venous thromboembolism. With over 6500 people worldwide and 65 years caring for patients and healthcare professionals, we are committed to driving healthier outcomes for people facing mobility challenges.

ArjoHuntleigh AB  
Hans Michelsensgatan 10  
211 20 Malmö, Sweden  
[www.arjo.com](http://www.arjo.com)

**arjo**

CE

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Arjo Inc.