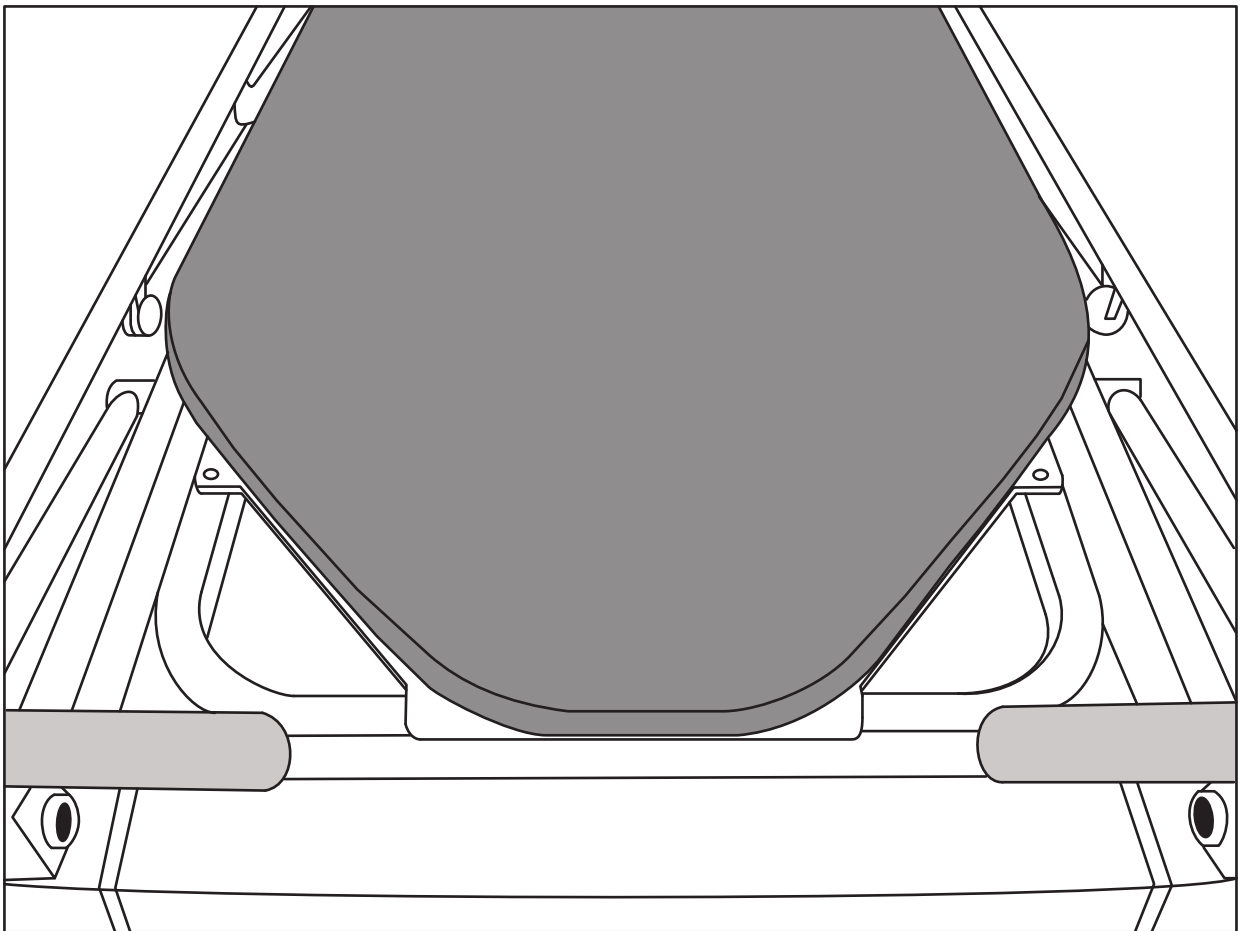
INSTRUCTIONS FOR USE

LGMAT/13 & LGMAT/13PN

EN · DE · NL · FR · JA

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WARNING

To avoid injury, always read this Instructions For Use and accompanied documents before using the product.

Mandatory to read the Instructions For Use.

Design Policy and Copyright

® and ™ are trademarks belonging to the Arjo group of companies.

© Arjo 2019.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either whole or in part without the consent of Arjo.

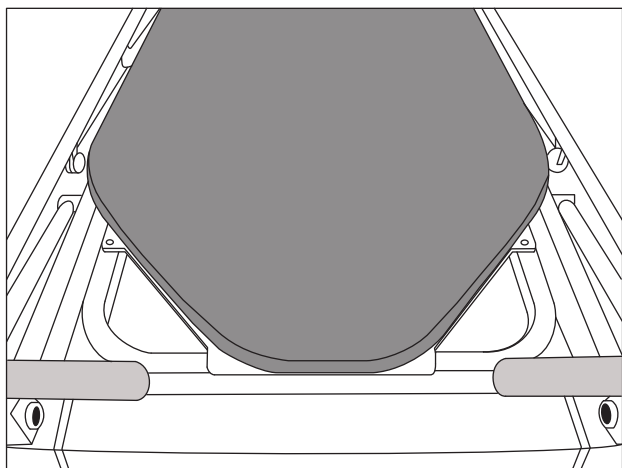
The LGMAT/13 transfer mattress is designed to allow patients to be moved easily and with minimal discomfort from a Lifeguard® trolley to an adjacent surface.

The mattress is supplied in two parts; the lower section is secured to the trolley and the upper section is free to slide over and off it during transfer, while still supporting the patient.

Fitting

Thoroughly clean the top surface of the mattress platform with a suitable volatile degreasing solvent, e.g. methylated spirits or isopropyl alcohol. Allow it to dry completely.

Turn the lower mattress section over and remove the backing tape from the two strips of hook and loop tape to expose the self-adhesive surface; do not separate the hook and loop tape at this stage.

Lower mattress section fitted to mattress platform

Position the lower mattress section on the mattress platform, with the tapered part at the head end of the trolley. Press down firmly to secure the self-adhesive hook and loop tape strips to the trolley.

Place the upper mattress section on the lower mattress section with the blue sliding surfaces together.

WARNING

The clinically qualified person responsible should consider the size, age and condition of the patient before any movement is attempted.

The transfer area should be free from any condition or obstruction that could be hazardous during patient transfer.

Serious Injury

If a serious incident occurs in relation to this medical device, affecting the user, or the patient then the user or patient should report the serious incident to the medical device manufacturer or the distributor. In the European Union, the user should also report the serious incident to the Competent Authority in the member state where they are located.

Using the mattress

CAUTION

The upper and lower sections should both be the same version (Series 1 or Series 3); Series 3 mattresses have an identification label sewn into the seam.

arjo

LGMAT13 - Series 3

Only compatible with Series 3
Mattress / Base

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Observe the following precautions before and during patient transfer.

1. Apply the brakes on both the trolley and the platform to which the transfer is to be made.
2. The transfer process should be carried out by at least two people, using the handles provided on the upper section of the mattress.
3. Transfer forces should always be applied simultaneously and evenly along the full length of the mattress.
4. Only transfer laterally onto hard flat surfaces, e.g. X-ray tables.
5. Make sure that the trolley is as close to the other platform as possible. The transfer gap should not be more than 10cm.
6. To aid transfer, the platform being transferred from should be approximately 3cm higher than the platform being transferred to.
7. Adjust both platforms to an ergonomic transfer height.
8. The mattress will support a maximum patient weight of 180kg.

Decontamination

WARNING

Wear gloves and protective clothing when carrying out cleaning procedures.

Remove the lower mattress section from the trolley for cleaning by pulling the hook and loop tape strips apart.

To clean the mattress: wipe all surfaces with a disposable cloth soaked in a neutral detergent and hand hot water. Rinse with clean water and dry with disposable paper towels.

To disinfect the mattress: wipe all surfaces with a solution of sodium dichloroisocyanurate (NaDCC) at a concentration of 1000ppm (0.1%) available chlorine.

CAUTION

Do not use abrasive cleaners or phenol-based disinfectants.

End of life disposal

- Fabric material used on the mattresses or any other textiles, polymers or plastic materials etc. should be sorted as combustible waste.
- Mattresses at the end of life should be disposed of as waste according to the national or local requirements which may be landfill or combustion.

Product specification

Cover	Polyamide fabric with PVC (LGMAT/13) or polyurethane (LGMAT/13PN) coating. Sliding surfaces are polyurethane coated polyester.
Core	Polyurethane and closed cell polyethylene foams with PVC coated nylon reinforcing sheet.
Handles	Nylon webbing
Weight	3.2 kg

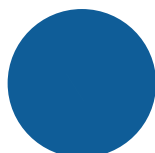
Symbols

CE marking indicating conformity with European Community harmonised legislation

Indicates the product is a Medical Device according to EU Medical Device Regulation 2017/745

Mandatory to read the Instructions for Use

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HONG KONG

Arjo Hong Kong Limited
Room 411-414, 4/F, Manhattan Centre,
8 Kwai Cheong Road, Kwai Chung, N.T.,
HONG KONG
Tel: +852 2960 7600
Fax: +852 2960 1711

ITALIA

Arjo Italia S.p.A.
Via Giacomo Peroni 400-402
IT-00131 ROMA
Tel: +39 (0) 6 87426211
Fax: +39 (0) 6 87426222
E-mail: Italy.promo@arjo.com

MIDDLE EAST

Arjo Middle East FZ-LLC
Office 908, 9th Floor,
HQ Building, North Tower,
Dubai Science Park,
Al Barsha South
P.O Box 11488, Dubai,
United Arab Emirates
Direct +971 487 48053
Fax +971 487 48072
Email: Info.ME@arjo.com

NEDERLAND

Arjo BV
Biezenwei 21
4004 MB TIEL
Postbus 6116
4000 HC TIEL
Tel: +31 (0) 344 64 08 00
Fax: +31 (0) 344 64 08 85
E-mail: info.nl@arjo.com

NEW ZEALAND

Arjo Ltd
34 Vestey Drive
Mount Wellington
NZ-AUCKLAND 1060
Tel: +64 (0) 9 573 5344
Free Call: 0800 000 151
Fax: +64 (0) 9 573 5384
E-mail: nz.info@Arjo.com

NORGE

Arjo Norway AS
Olaf Helset's vei 5
N-0694 OSLO
Tel: +47 22 08 00 50
Faks: +47 22 08 00 51
E-mail: no.kundeservice@arjo.com

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Address page - REV 24: 04/2019

At Arjo, we are committed to improving the everyday lives of people affected by reduced mobility and age-related health challenges. With products and solutions that ensure ergonomic patient handling, personal hygiene, disinfection, diagnostics, and the effective prevention of pressure ulcers and venous thromboembolism, we help professionals across care environments to continually raise the standard of safe and dignified care. Everything we do, we do with people in mind.

ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö, Sweden
www.arjo.com

arjo

CE

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Arjo Inc.