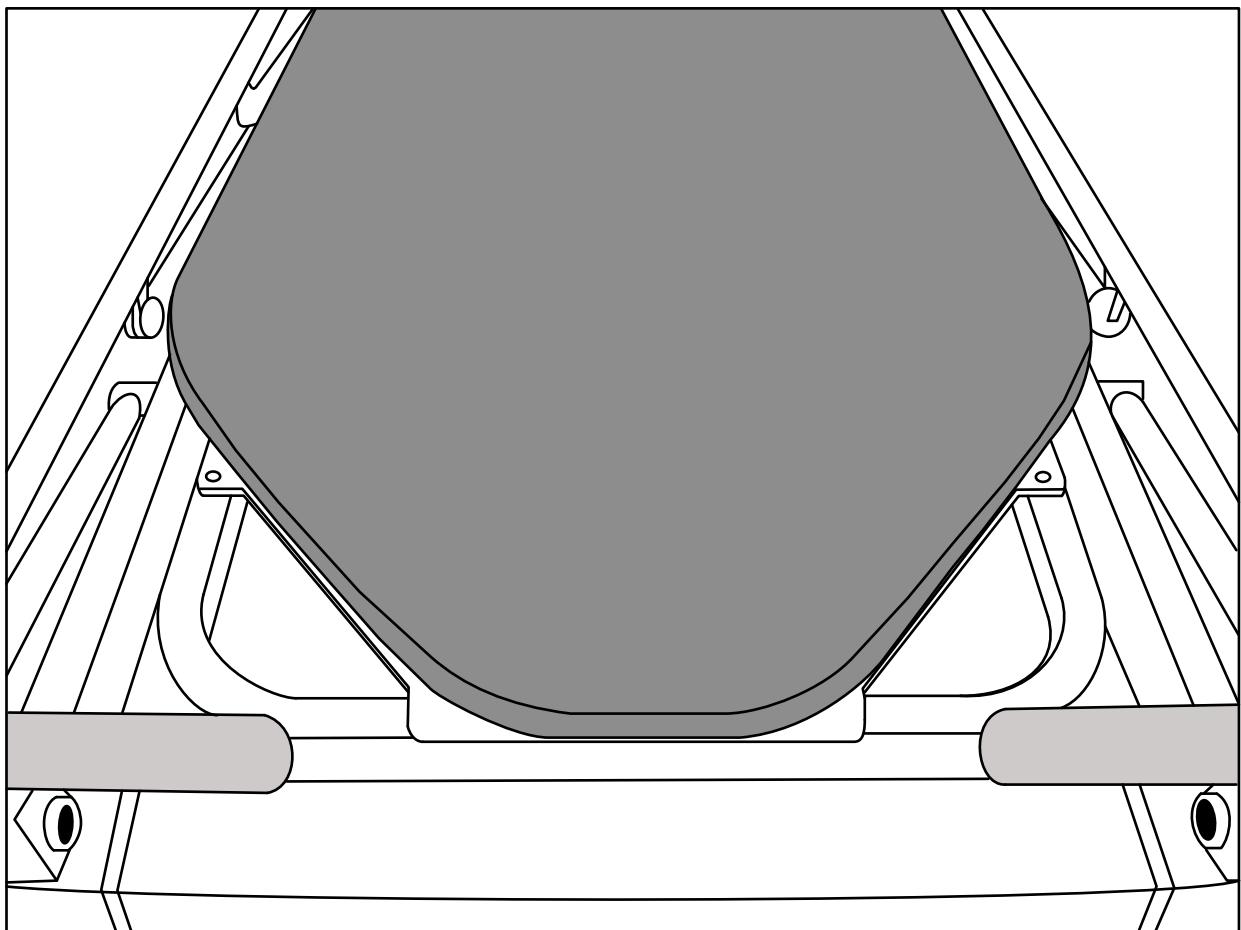
INSTRUCTIONS FOR USE

LGMAT/13 & LGMAT/13PN

EN · DE · NL · FR · JA

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WARNING

To avoid injury, always read this Instructions For Use and accompanied documents before using the product.

Design Policy and Copyright

® and ™ are trademarks belonging to the Arjo group of companies.

© Arjo 2023.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either whole or in part without the consent of Arjo.

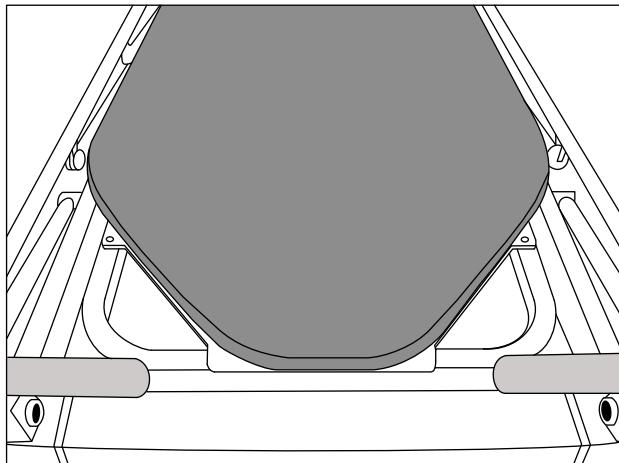
The LGMAT/13 transfer mattress is designed to allow patients to be moved easily and with minimal discomfort from a Lifeguard® trolley to an adjacent surface.

The mattress is supplied in two parts; the lower section is secured to the trolley and the upper section is free to slide over and off it during transfer, while still supporting the patient.

Fitting

Thoroughly clean the top surface of the mattress platform with a suitable volatile degreasing solvent, e.g. methylated spirits or isopropyl alcohol. Allow it to dry completely.

Turn the lower mattress section over and remove the backing tape from the two strips of hook and loop tape to expose the self-adhesive surface; do not separate the hook and loop tape at this stage.

Lower mattress section fitted to
mattress platform

Position the lower mattress section on the mattress platform, with the tapered part at the head end of the trolley. Press down firmly to secure the self-adhesive hook and loop tape strips to the trolley.

Place the upper mattress section on the lower mattress section with the blue sliding surfaces together.

WARNING

The clinically qualified person responsible should consider the size, age and condition of the patient before any movement is attempted.

The transfer area should be free from any condition or obstruction that could be hazardous during patient transfer.

Serious Injury

If a serious incident occurs in relation to this medical device, affecting the user, or the patient then the user or patient should report the serious incident to the medical device manufacturer or the distributor. In the European Union, the user should also report the serious incident to the Competent Authority in the member state where they are located.

Using the mattress

CAUTION

The upper and lower sections should both be the same version (Series 1 or Series 3); Series 3 mattresses have an identification label sewn into the seam.

Observe the following precautions before and during patient transfer.

1. Apply the brakes on both the trolley and the platform to which the transfer is to be made.
2. The transfer process should be carried out by at least two people, using the handles provided on the upper section of the mattress.
3. Transfer forces should always be applied simultaneously and evenly along the full length of the mattress.
4. Only transfer laterally onto hard flat surfaces, e.g. X-ray tables.
5. Make sure that the trolley is as close to the other platform as possible. The transfer gap should not be more than 10cm.
6. To aid transfer, the platform being transferred from should be approximately 3cm higher than the platform being transferred to.
7. Adjust both platforms to an ergonomic transfer height.
8. The mattress will support a maximum patient weight of 180kg.

Decontamination

WARNING

Wear gloves and protective clothing when carrying out cleaning procedures.

Remove the lower mattress section from the trolley for cleaning by pulling the hook and loop tape strips apart.

To clean the mattress: wipe all surfaces with a disposable cloth soaked in a neutral detergent and hand hot water. Rinse with clean water and dry with disposable paper towels.

To disinfect the mattress: wipe all surfaces with a solution of sodium dichloroisocyanurate (NaDCC) at a concentration of 1000ppm (0.1%) available chlorine.

CAUTION

Do not use abrasive cleaners or phenol-based disinfectants.

End of life disposal

- Fabric material used on the mattresses or any other textiles, polymers or plastic materials etc. should be sorted as combustible waste.
- Mattresses at the end of life should be disposed of as waste according to the national or local requirements which may be landfill or combustion.

Product specification

Cover	Polyamide fabric with PVC (LGMAT/13) or polyurethane (LGMAT/13PN) coating. Sliding surfaces are polyurethane coated polyester.
Core	Polyurethane and closed cell polyethylene foams with PVC coated nylon reinforcing sheet.
Handles	Nylon webbing
Weight	3.2 kg

Symbols

CE marking indicating conformity with European Community harmonised legislation

Indicates the product is a Medical Device according to EU Medical Device Regulation 2017/745

Refer to instruction manual/ booklet - Instructions for use should be read

Unique device identifier

UK Symbol explanation

This section is only applicable to United Kingdom (UK) market when UK marking is applied to the Arjo medical device labelling.

UK marking indicating conformity with UK Medical Devices Regulations 2002 (SI 2002 No 618, as amended)

UK Responsible Person & UK Importer:

Arjo (UK) Ltd, ArjoHuntleigh House, Houghton Regis. LU5 5XF

Is the appointed UK Responsible Person as defined in UK Medical Devices Regulations 2002 (SI 2002 No 618, as amended).

For Northern Ireland (NI) CE marking will still apply until further amendment to applicable regulations.

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Intentionally left blank

AUSTRALIA

Arjo Australia
Building B, Level 3
11 Talavera Road
Macquarie Park, NSW, 2113,
Australia
Phone: 1800 072 040

BELGIQUE / BELGIË

Arjo Belgium
Evenbroekveld 16
9420 Erpe-Mere
Phone: +32 (0) 53 60 73 80
Fax: +32 (0) 53 60 73 81
E-mail: info.belgium@arjo.com

BRASIL

Arjo Brasil Equipamentos Médicos Ltda
Rua Marina Ciufuli Zanfelice, 329 PB02
Galpão - Lapa
São Paulo – SP – Brasil
CEP: 05040-000
Phone: 55-11-3588-5088
E-mail: vendas.latam@arjo.com
E-mail: servicios.latam@arjo.com

CANADA

Arjo Canada Inc.
90 Matheson Boulevard West
Suite 350
CA-MISSIONSSAUGA, ON, L5R 3R3
Tel/Tél: +1 (905) 238-7880
Free: +1 (800) 665-4831
Fax: +1 (905) 238-7881
E-mail: info.canada@arjo.com

ČESKÁ REPUBLIKA

Arjo Czech Republic s.r.o.
Na Strzi 1702/65
140 00 Praha
Czech Republic
Phone No: +420225092307
E-mail: info.cz@arjo.com

DANMARK

Arjo A/S
Vassingerødvej 52
DK-3540 LYNGE
Tel: +45 49 13 84 86
Fax: +45 49 13 84 87
E-mail: dk_kundeservice@arjo.com

DEUTSCHLAND

Arjo GmbH
Peter-Sander-Strasse 10
DE-55252 MAINZ-KASTEL
Tel: +49 (0) 6134 186 0
Fax: +49 (0) 6134 186 160
E-mail: info-de@arjo.com

ESPAÑA

ARJO IBERIA S.L.
Polígono Can Salvatella
c/ Cabanyes 1-7
08210 Barberà del Valles
Barcelona - Spain
Telefono 1: +34 900 921 850
Telefono 2: +34 931 315 999

FRANCE

Arjo SAS
2 Avenue Alcide de Gasperi
CS 70133
FR-59436 RONCQ CEDEX
Tél: +33 (0) 3 20 28 13 13
Fax: +33 (0) 3 20 28 13 14
E-mail: info.france@arjo.com

HONG KONG

Arjo Hong Kong Limited
Room 411-414, 4/F, Manhattan Centre,
8 Kwai Cheong Road, Kwai Chung, N.T.,
HONG KONG
Tel: +852 2960 7600
Fax: +852 2960 1711

ITALIA

Arjo Italia S.p.A.
Via Giacomo Peroni 400-402
IT-00131 ROMA
Tel: +39 (0) 6 87426211
Fax: +39 (0) 6 87426222
E-mail: Italy.promo@arjo.com

MIDDLE EAST

Arjo Middle East FZ-LLC
Office 908, 9th Floor,
HQ Building, North Tower,
Dubai Science Park,
Al Barsha South
P.O. Box 11488, Dubai,
United Arab Emirates
Direct +971 487 48053
Fax +971 487 48072
Email: Info.ME@arjo.com

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At Arjo, we believe that empowering movement within healthcare environments is essential to quality care. Our products and solutions are designed to promote a safe and dignified experience through patient handling, medical beds, personal hygiene, disinfection, diagnostics, and the prevention of pressure injuries and venous thromboembolism. With over 6500 people worldwide and 65 years caring for patients and healthcare professionals, we are committed to driving healthier outcomes for people facing mobility challenges.

ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö, Sweden
www.arjo.com

arjo

CE

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Arjo Inc.