Lifeguard LG55

使用说明
Instructions for Use

ARJOHUNTLEIGH
GETINGE GROUP

EN

ZH

Lifeguard LG55

Instructions for Use

ARJOHUNTLEIGH
GETINGE GROUP

WARNING

To avoid injury, always read this Instructions for Use and accompanied documents before using the product.

Mandatory to read the Instructions for Use

Design Policy and Copyright

® and ™ are trademarks belonging to the ArjoHuntleigh group of companies.

© ArjoHuntleigh 2004.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either whole or in part without the consent of ArjoHuntleigh.

Contents

1. Introduction	3
2. Clinical Applications	4
3. Operation	5
4. Accessories	18
5. Decontamination	20
6. Care and Preventive Maintenance	21
7. Warranty and Service.....	23
8. Dimensions and Data.....	24

Warnings and Cautions

WARNINGS given in this manual identify possible hazards in procedures or conditions which, if not correctly followed, could result in death, injury or other serious adverse reactions.

Cautions given in this manual identify conditions or procedures which, if not correctly followed, could result in equipment failure or damage.

Notes given in this manual (indicated by ) are used to explain or amplify a procedure or condition.

General Warnings

WARNING

The age, size and condition of the patient should be assessed by a competent clinically qualified person before using the trolley, giving particular consideration to appropriate use of the safety sides.

Do not use accessories that are not designed or approved for use with the trolley.

Always apply the brakes when the trolley is stationary.

Raise the safety sides before moving the trolley with a patient.

Children and restless or disorientated patients should not be left unattended at any time.

Before turning (rolling) a patient on the trolley or performing a lateral transfer, adjust the mattress platform to a comfortable working height; be aware that the trolley may overbalance if the patient's weight is not evenly distributed across the mattress platform.

1. Introduction

This manual gives instructions for the use and maintenance of the ArjoHuntleigh Lifeguard® patient trolley, Model Number LG55.

These trolleys have X-ray translucent mattress platforms. The products also feature a sliding X-ray cassette tray facility.

Model LG55


2. Clinical Applications

Area of Use A & E departments, admissions, day surgery and general hospital areas.

Indications The trolleys may be used for general and emergency admissions, examination, diagnostic X-ray and initial treatment.

They are also appropriate for patient transfer between departments, wards and theatre.

Contraindications The maximum safe working load for these trolleys is 250 kg.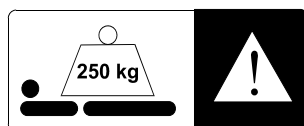

3. Operation

Safety

For the safety of operator, patient and equipment:


- Always use a mattress of the correct size and type for the trolley.
- Do not expose the equipment, especially the mattress, to naked flames, cigarettes etc.
- Before operating the trolley, make sure that the patient is positioned correctly to avoid entrapment and imbalance.
- When the trolley is operated, check that no obstacles are in its path.
- Keep these instructions handy for reference.

Mattress

The mattress is secured to the platform by means of hook and loop tapes on the mattress underside and the laminate top.

Push Bar Handles

To adjust the height of the handles, press the spring-loaded release button (1) and raise or lower the handle as required (2). Let go of the button to lock the handle in the high or low position.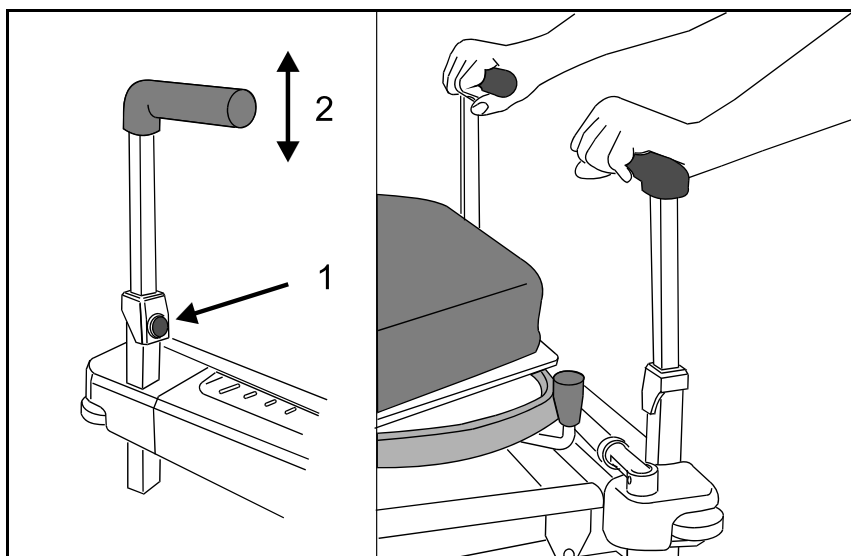


Push Bar Handles

Brakes and Steering

There is a pair of linked brake pedals at each end of the trolley. Push the pedals down to apply brakes on all castors.

When the pedals are horizontal, the castors are free to rotate and swivel.

A steering castor is engaged when an optional 5th wheel is fitted and the pedals are fully raised. In this situation the trolley will keep on a straight course in any direction and can be steered by pivoting about the 5th wheel. When the 5th wheel is not fitted the steering castor is at the foot-end and the trolley should be pushed from the head-end.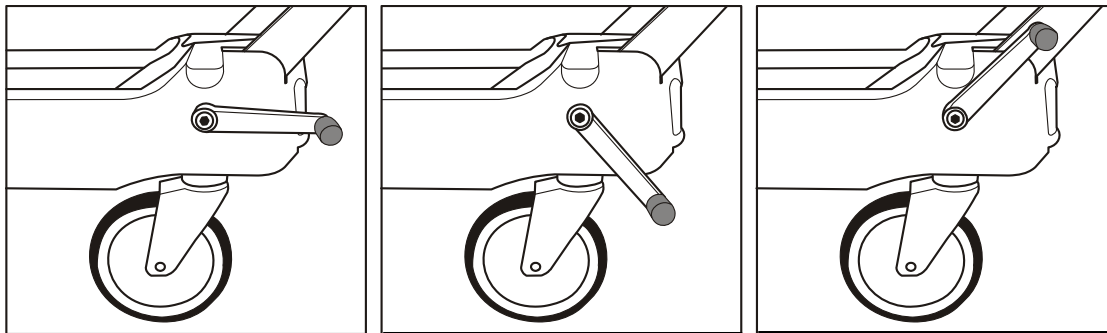

FREE


BRAKE

STEER


Brake and Steering Control

Height and Tilt Adjustment

The height and the tilt angle of the mattress platform are adjusted using a group of five foot pedals on either side of the trolley. The three pedals marked ▲ on the bottom row are used to raise the platform. The two pedals marked ▼ on the top row are used to lower the platform.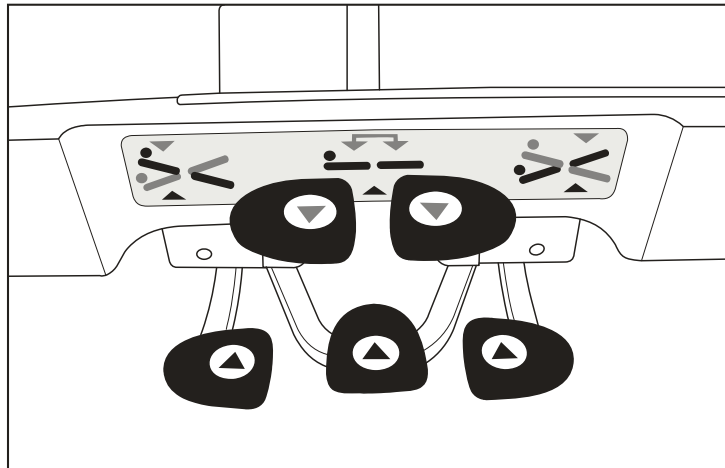

Height/Tilt Pedal Arrangement

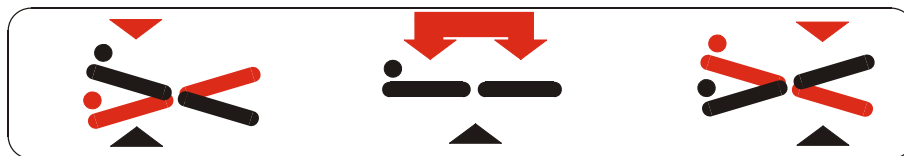Tilt Adjustment

To raise one end of the platform: Operate the ‘raise’ pedal (▲) nearest that end of the trolley with a pumping action.

To lower one end of the platform: Press down on the ‘lower’ pedal (▼) nearest that end of the trolley and keep it pressed.

There is an additional ‘lower’ pedal (▼) that enables head down tilt to be applied from the head-end of the trolley.

Height Variation

To raise the platform: Operate the central ‘raise’ pedal (▲) with a pumping action.

To lower the platform: Press down both ‘lower’ pedals (▼) and keep them pressed.


If the platform is raised or lowered when tilted, the tilt angle will be maintained until the leading end reaches the limit of travel, when the platform will return to the horizontal condition.

Backrest

The amount of effort needed to raise or lower the backrest will vary depending on patient weight and mobility. With a very heavy patient, the backrest may fall quickly if it is not supported when released.

To adjust the backrest: Grasp the handle firmly, press the release knob and lift the backrest assisted by the gas spring or push it down against the action of the gas spring. Let go of the release knob to lock the backrest at any angle.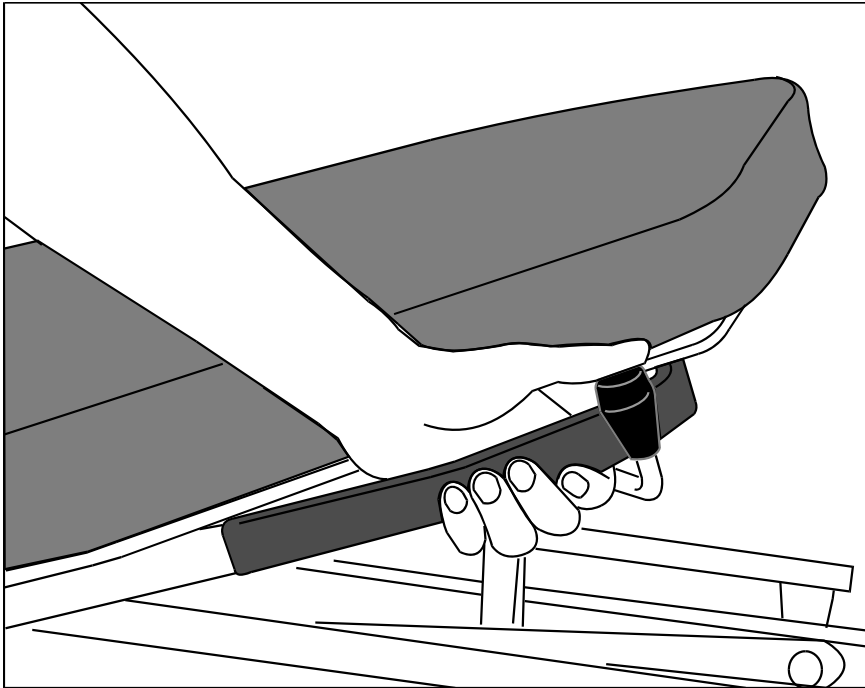


Backrest Adjustment

Safety Sides


To raise the safety side: Hold the top rail and lift until it locks into position at maximum height.

To lower the safety side: Hold the top rail with one hand, pull the release button at the trolley foot-end and lower the rail towards the head-end.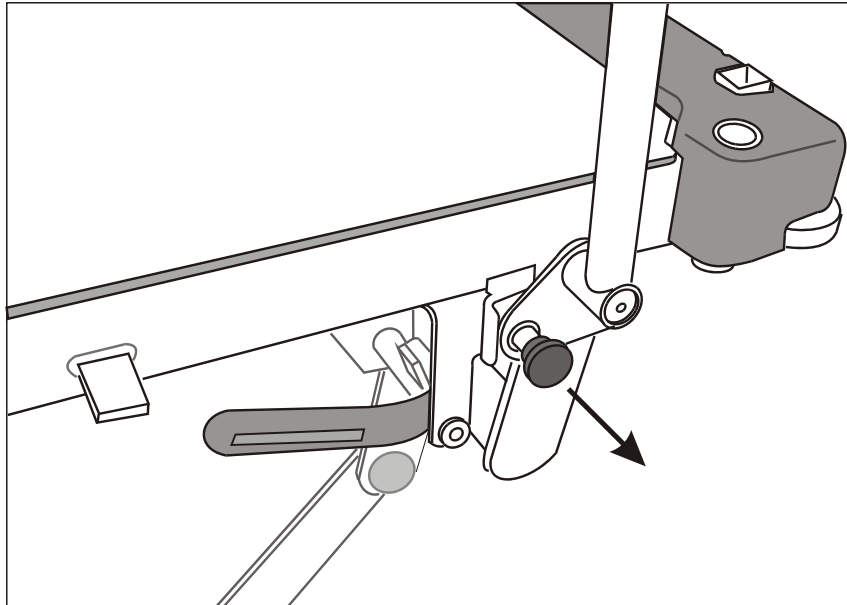


Safety Side Release Button

In order to accommodate “C” Arm X-ray vision when taking side-shot X-ray pictures, each safety side can be lowered even further than its initial resting position. When the safety side is folded down it rests on two stops that are located at each end of the mattress platform side-frame.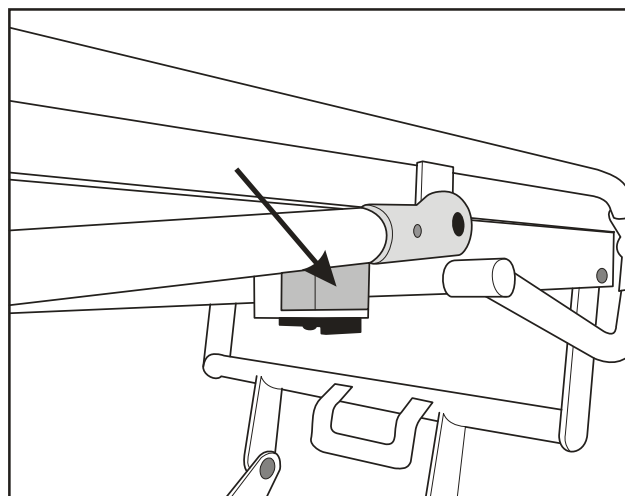

Safety Side Stop Positions

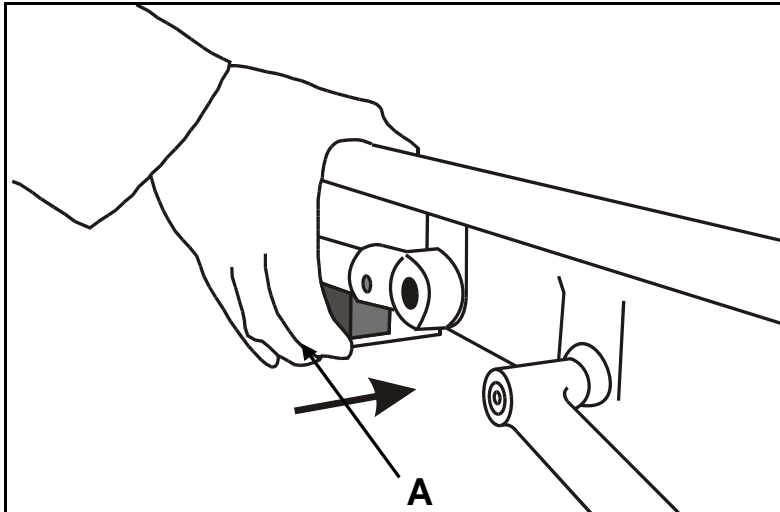Safety Side Stop Operation

WARNING

When the safety side is in the folded position, the upright bars rest on the two stops. Care should be taken to keep hands and fingers away from these areas.

Operation

Lower the safety side until the upright bars rest on two white plastic stops.

At the head-end of the trolley, position the thumb over the top rail of the safety side and use fingers to push the sliding catch (A), which is located beneath each side-rail white plastic stop, in towards the trolley frame as illustrated (avoid squeezing the rail and catch together).

Caution

Only use the sliding catch A to retract the safety side stop.

Caution

Do not use the safety sides to move the trolley. Only use the push bar handles to operate the trolley.


With the catch fully pushed in, ease the top rail down to the second position.

At the foot-end of the trolley use two hands, one to hold the top rail and the other to push the sliding catch (A). Again, once the catch is fully pushed in, ease the top of the rail down to its second position.


To reinstate the safety side to normal position: Lift the top rail of the safety side at a point approximately halfway along the trolley. The stops will automatically spring back to their original position.

X-Ray Photography

The mattress platform is X-ray translucent. The X-ray cassette drawer below the mattress platform can be positioned anywhere along the length of the platform for full patient coverage.

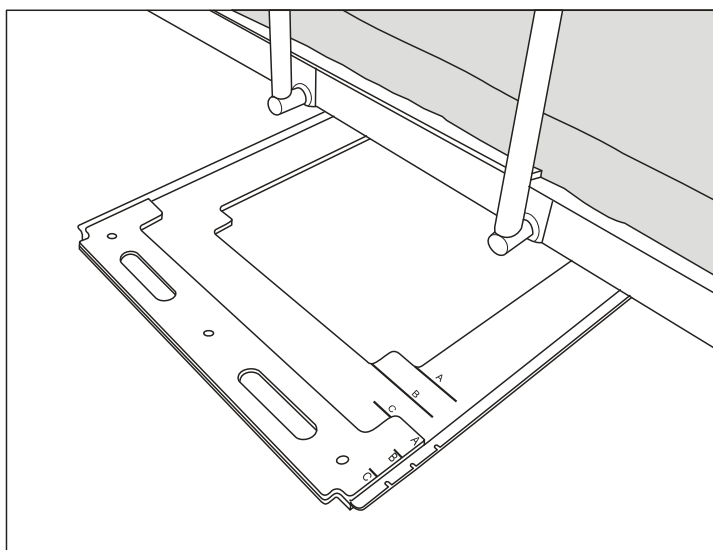
**X-Ray Cassette Drawer Shown Accessed from Patient's
Left-Hand Side**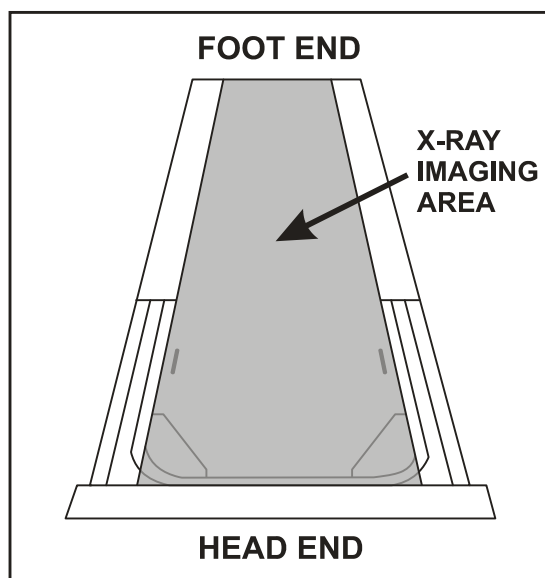


Mattress Platform Showing Imaging Area

To load or unload X-ray cassettes: press the red latch (A) down and use the slot (B) to pull the cassette drawer out until it reaches its stop position.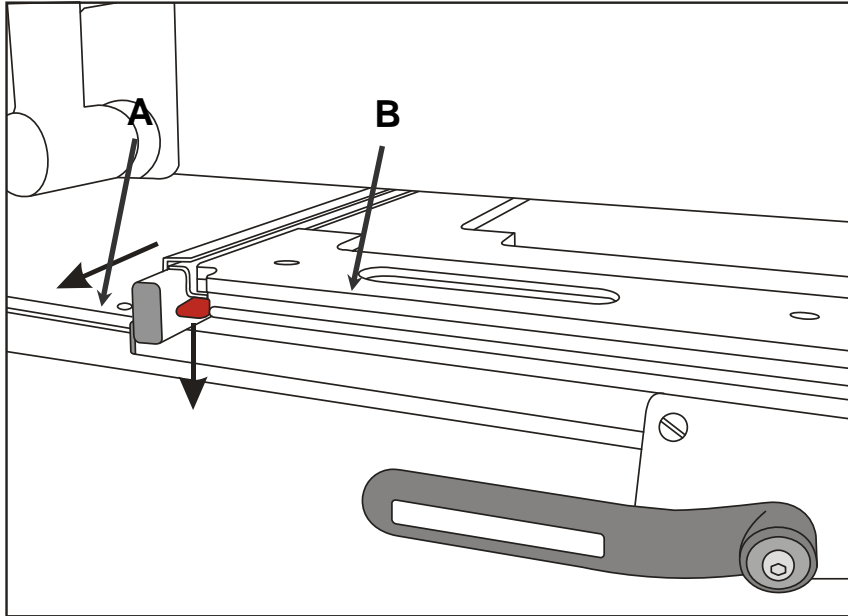

Cassette Drawer Latch


The X-ray cassette drawer can be accessed from either side of the mattress platform.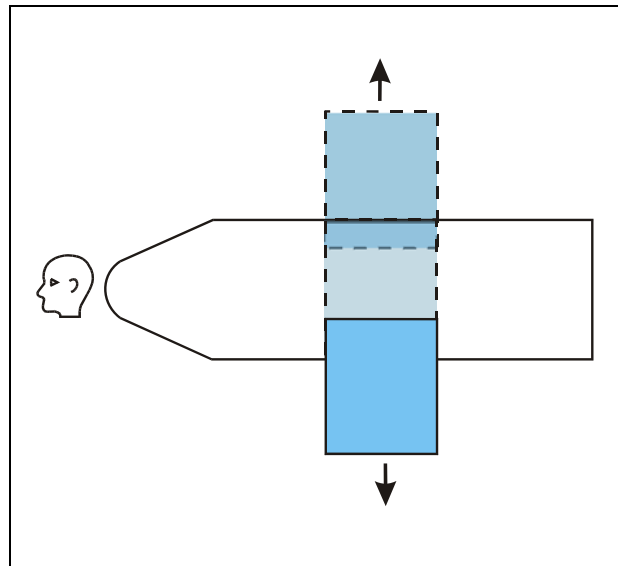

Cassette Drawer Access

The drawer has recesses to suit two sizes of X-ray cassette, **A** (390mm x 466mm) and **B** (273mm x 334mm).


For each size of X-ray cassette there are two possible positions, at 90° to each other.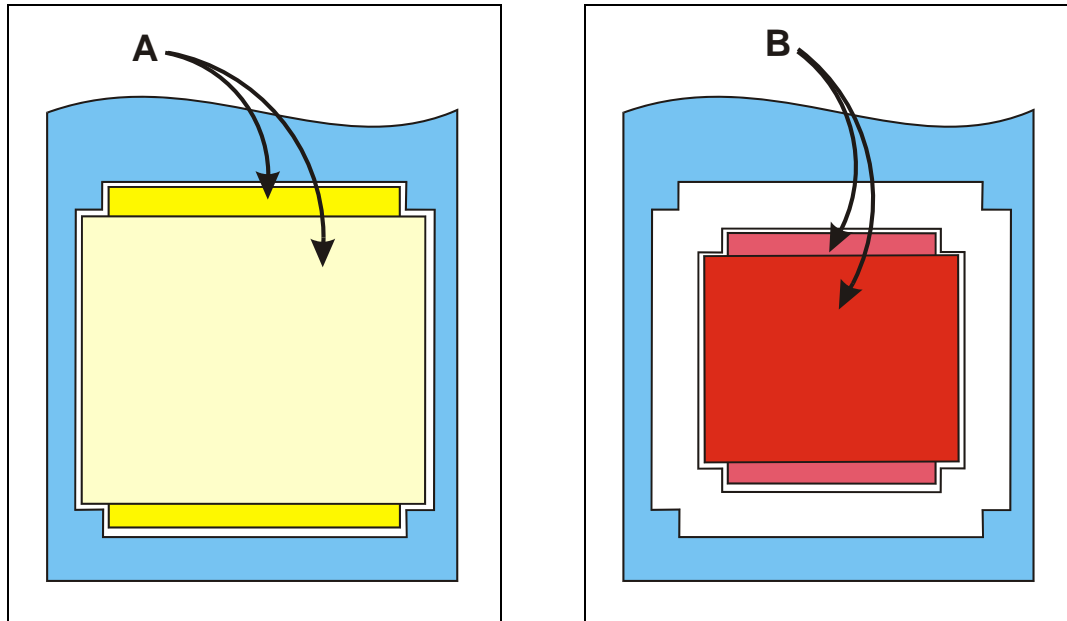

X-Ray Cassette Positioning Options

Place the X-ray cassette flat in the appropriate recess. Push the drawer back under the mattress platform until it latches into place.


When the cassette drawer is pushed all the way in to its latched position, it is centrally located within the imaging area, as shown below (1).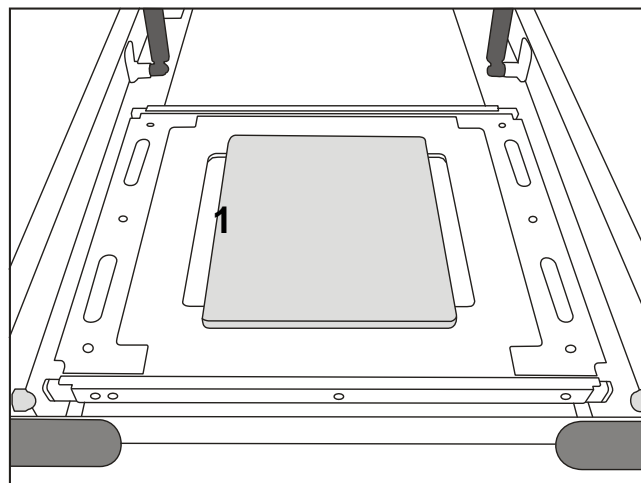


X-Ray Cassette Drawer in Central Position (Backrest Raised)

To offset the cassette drawer from the central position:

Pull out the drawer and line up one of the markers (A, B or C) with the black indicator bar (2). The markers correspond with the letters next to the X-ray cassette; e.g. when marker A is lined up with the indicator bar, edge A of the X-ray cassette is on the boundary of the imaging area.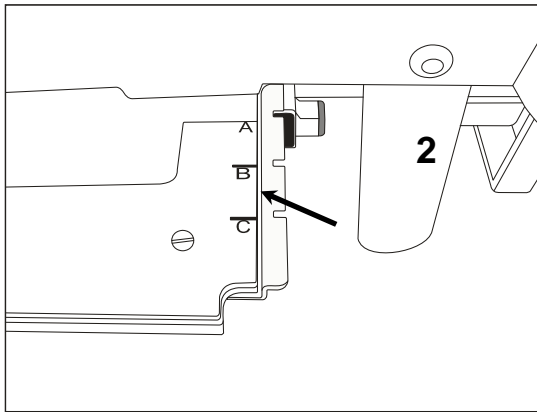


Cassette Drawer Markers on Patient's Left-Hand Side

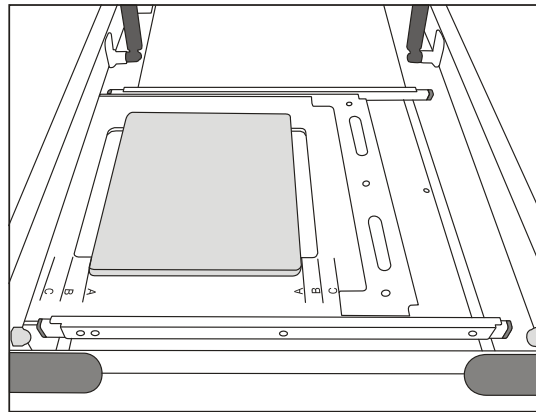
Cassette Drawer in Offset Position A (Backrest Raised)

To adjust the position of the X-ray cassette drawer along the length of the mattress platform: lift the lever and slide the drawer to the desired position. Release the lever to hold the drawer in place.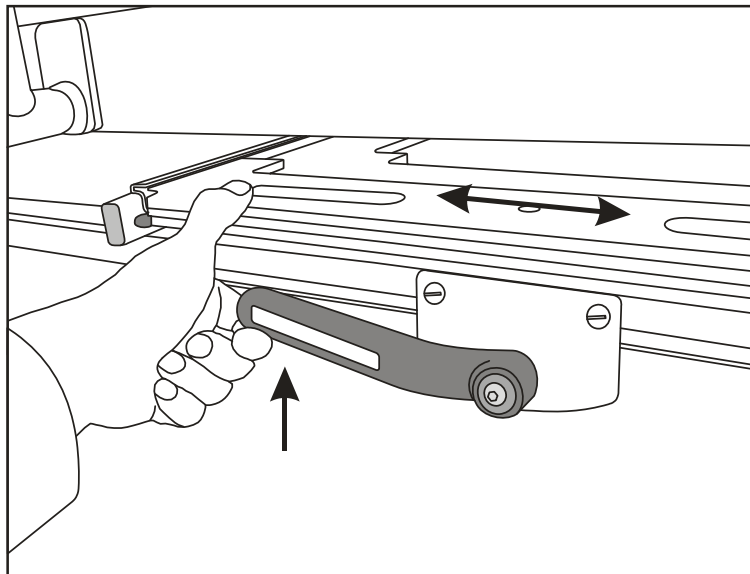


X-Ray Cassette Drawer Lever

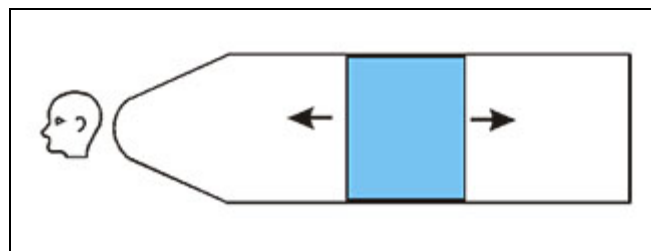Cassette Drawer Movement

Oxygen Therapy Equipment

A recess in the base of the trolley will accommodate a size D, E or F oxygen cylinder. Rubber pads cushion the cylinder to prevent unwanted movement. In addition, facilities are provided for the installation and use of oxygen therapy systems. A short rail (Medirail) is provided at the head-end of the trolley to accommodate the flowmeter and injection suction unit. A universal mounting bracket is installed in the base of the trolley to accept the suction receiver jar.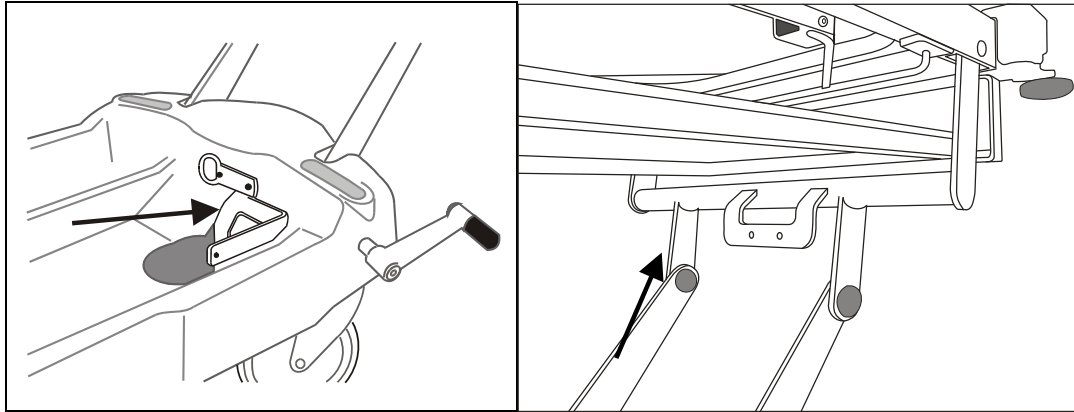

Mounting Bracket in Base of Trolley and Medirail at Head-End

WARNING

Ensure that the oxygen line does not become entangled or trapped which could cause damage to the line or loss of supply.

Ensure that when the trolley is operated it does not collide with the installed equipment, especially at low height.

If the mounting bracket is not already fitted, remove the plastic cover from the hole in the base cover moulding and attach the bracket with two 6mm Button Head screws and nuts using the pre-drilled holes in the base frame.


IV Drip Rod MB-ACC32

Install the pole in one of the four tubular sockets at the corners of the platform (1).


WARNING

The maximum total load on the hooks must not exceed 5 kg

The maximum load attached to the lower tube must not exceed 10 kg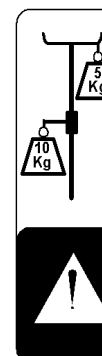

To adjust the height: Hold the telescoping rod firmly, loosen the locking collar (2), lift or lower the hooks as required (3) and re-tighten the locking collar firmly (4).
Fold out the hooks (5).

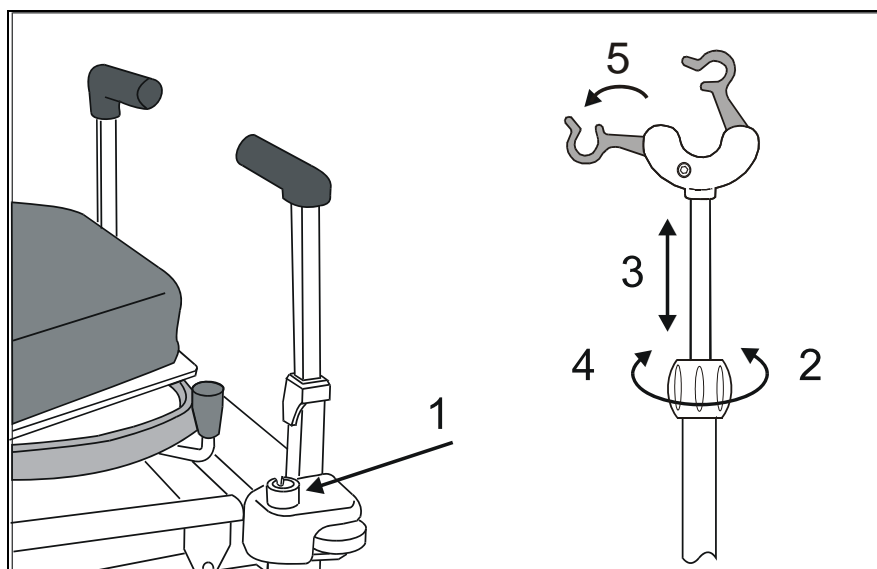IV Drip Rod Location and Operation


Be careful not to lift the pole out of the socket when raising the hooks.

Storage: Adjust the pole to minimum length and tighten the locking collar before storing it in the clips on the base of the trolley.

**Integral IV Pole
799.501**


The built-in IV pole is located at the head end of the trolley.

WARNING
Do not exceed the maximum safe load of 5 kg.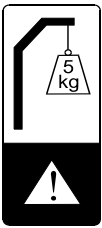

Lift the pole to the vertical position (1) and allow it to drop into the tubular socket on the platform (2).

To adjust the height: Hold the telescoping rod firmly, loosen the locking collar (3), lift or lower the hook as required (4) and re-tighten the locking collar firmly (5).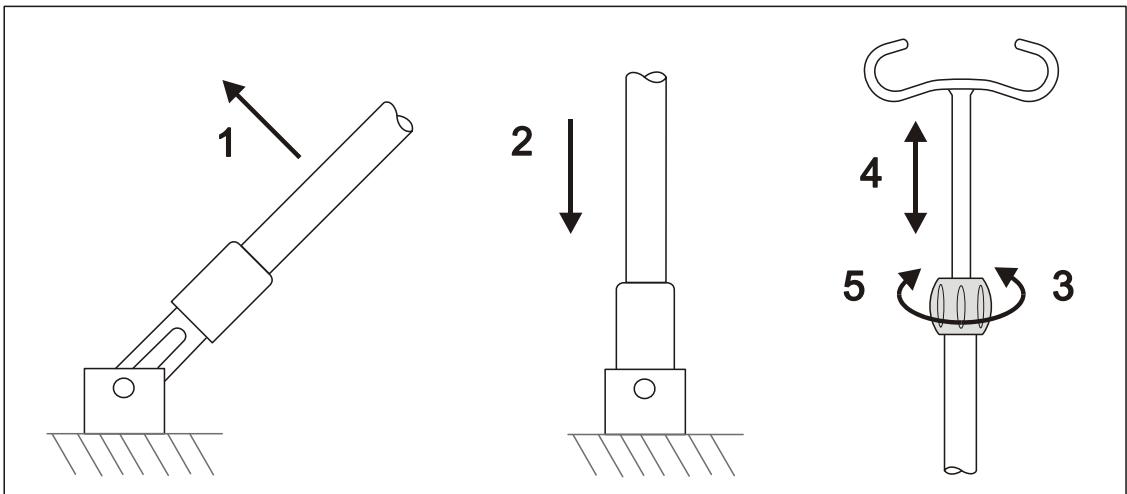

Integral IV Drip Rod Operation

Storage: Adjust the pole to minimum length and tighten the locking collar, then lift the pole out of the socket and fold it down.

4. Accessories

A range of accessories, including mattresses, is available from ArjoHuntleigh. Details are available on request.

Accessories currently available for the *Lifeguard* LG55 are shown in the table below.

ACCESSORY NAME	PRODUCT CODE
Loose drip rod	157003
Foot end panel & bow	157009
Foot panel with monitor table	157018
Clip-on table	157021
Arm rest	157006PE02
Safety side pads (grey)	99901PE16
DIN rail kit (head end)	LG/ACC03
Oxygen bottle holder (PD size)	LG/ACC04
DIN rail kit (foot end)	LG/ACC05
Mattress with two-way stretch cover	LGMAT/10
Two part transfer mattress	LGMAT/13
Two part transfer mattress “Pennyla”	LGMAT/13PN
Bi-Flex pressure reducing mattress	TRM2
Bi-Flex pressure redistribution mattress	TRM3

Some accessories may not be available in all markets. ArjoHuntleigh reserves the right to alter specifications and availability without notification.

The following information relates to accessories supplied with the trolley. Other accessories are supplied with instructions for use.

LGMAT/10

The LGMAT/10 is a one piece mattress with two-way stretch anti-static PU cover, used to provide the patient with adequate comfort during transport and to facilitate cleaning and disinfection.

LGMAT/13


The LGMAT/13 transfer mattress is designed to allow patients to be moved easily and with minimal discomfort from a *Lifeguard* trolley to an adjacent surface.

Foot Panel and Monitor Table Model 157018

Install the foot panel in the square sockets at the end of the mattress platform, with the clipboard on the outside.


WARNING

Maximum safe working load 12 kg evenly distributed.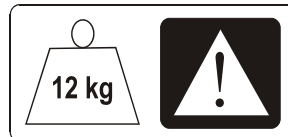

To use the monitor table, rotate the clipboard to the horizontal position. Use the straps to secure the monitor to the table.

To remove the foot panel, lift it out of the sockets.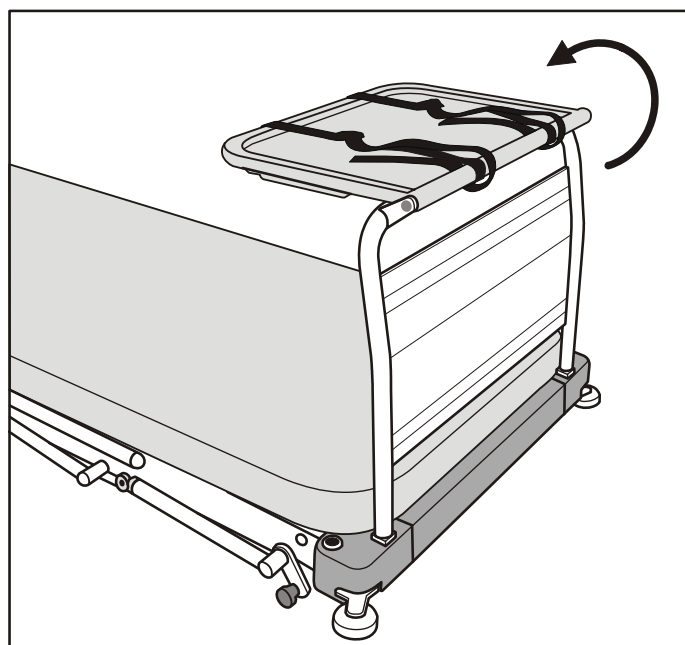

Monitor Table Installed

5. Decontamination

The following processes are recommended, but should be adapted to comply with the local or national guidelines (Decontamination of Medical Devices) which may apply within the Healthcare Facility or the country of use. If you are uncertain, you should seek advice from your local Infection Control Specialist.

The *Lifeguard* system should be routinely decontaminated between patients and at regular intervals while in use; as is good practice for all reusable medical devices.

WARNING

Protective clothing should always be worn when carrying out decontamination procedures.

Caution

Do not use Phenol-based solutions or abrasive compounds or pads during the decontamination process as these will damage the surface coating. Do not boil or autoclave the cover.

To Clean Clean all exposed surfaces and remove any organic debris by wiping with a cloth moistened with a simple (neutral) detergent and water. Dry thoroughly.

Chemical Disinfection To protect the integrity of the cover we recommend a chlorine-releasing agent, such as sodium hypochlorite, at a strength of 1,000ppm available chlorine (this may vary from 250ppm to 10,000ppm depending on local policy and contamination status).

Wipe all cleaned surfaces with the solution, rinse and dry thoroughly.

Alcohol based disinfectants (maximum strength 70%) may be used as an alternative.

Ensure the product is dry before storage.

If an alternative disinfectant is selected from the wide variety available we recommend that suitability for use is confirmed with the chemical supplier prior to use.

To clean the mattress: wipe all surfaces with a disposable cloth soaked in a neutral detergent and hand hot water. Rinse with clean water and dry with disposable paper towels.

To disinfect the mattress: wipe all surfaces with a solution of sodium dichloroisocyanurate (NaDCC) at a concentration of 1,000ppm (0.1%) available chlorine.

6. Care and Preventive Maintenance

This product is subject to wear and tear during use. To ensure that it continues to perform within its original specification, preventive maintenance procedures should be carried out at the intervals shown.

WARNING

This list indicates the minimum level of preventive maintenance recommended by the manufacturer. More frequent inspections should be carried out when the product is subjected to heavy use or aggressive environments, or where required by local regulations.

Failure to carry out these checks, or continuing to use the product if a fault is found, may compromise the safety of both the patient and carer/user. Preventive maintenance can help to prevent accidents.

<i>Actions to be done by carer/user</i>	<i>Daily</i>	<i>Weekly</i>	<i>Yearly</i>
Clean and disinfect as shown in Chapter 5	✓		
Check operation of safety sides		✓	
Visually check castors		✓	
Inspect the mattress as described on the next page	✓		
Check safety side pads (if fitted)		✓	

If the result of any of these tests is unsatisfactory, contact ArjoHuntleigh or an approved service agent.

WARNING

The procedures below must be carried out by suitably trained and qualified personnel. Failure to do so may result in injury or an unsafe product.

<i>Actions to be done by qualified personnel</i>	<i>Daily</i>	<i>Weekly</i>	<i>Yearly</i>
Check the operation of the lifting mechanism as described below			✓
Check the operation of the backrest raise and lower			✓
Check operation of castors, paying special attention to braking and steering functions			✓
Check the hydraulic pump for leaks.			✓
Check all nuts, bolts and other fasteners are present and correctly tightened			✓
Check all accessories, paying particular attention to fasteners and moving parts			✓

Mattress

Check the cover, especially around the seams, for cuts or splits that may allow fluid ingress. Replace the cover if necessary.

Unzip the cover and inspect the mattress core for signs of fluid ingress, e.g. staining or dampness. Replace the complete mattress if necessary.

Lifting Mechanism

Check the operation. A steady firm resistance should be felt over the full stroke of the ‘raise’ pedals. If the pedal pressure is variable or yielding, this may indicate the presence of air in the system and can be remedied as follows:

1. Raise the platform to full height. Continue pumping for four or five pedal strokes then lower the platform fully.
2. Repeat this action two or three times.
3. If firm and steady pedal operation is not restored, contact ArjoHuntleigh or an approved service agent.

X-ray Cassette Holder

Check the cassette holder operation. It should slide easily up and down the length of the trolley. It should also move freely in and out on both sides of the deck. If not, contact ArjoHuntleigh or an approved service agent.

7. Warranty and Service

ArjoHuntleigh standard terms and conditions apply to all sales.

A copy is available on request. These contain full details of warranty terms and do not limit the statutory rights of the consumer.

For service, maintenance and any questions regarding this product, please contact your local ArjoHuntleigh office or authorised distributor. A list of ArjoHuntleigh offices can be found at the back of this manual.


Product Lifetime

The lifetime of this product is typically ten (10) years. “Lifetime” is defined as the period during which the product will maintain the specified performance and safety, provided it has been maintained and operated in conditions of normal use in accordance with the requirements in these instructions.

8. Dimensions and Data

A & E Trolleys, models LG55 and NLG55

Overall length	211 cm
Overall width	77 cm
Height of mattress platform	93 cm (max.) 56 cm (min.)
Tilt (Max) head down	12°
Tilt (Max) foot down	12°
Backrest adjustment	0 to 87°
Castors (diameter)	20 cm
Product Weight (approximate)	177 kg
Safe working load	250 kg

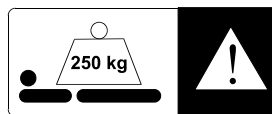Mattress size	195 x 67 x 6.5 cm
	195 x 67 x 7.5 cm
	195 x 67 x 13 cm

STORAGE

Handle with care. Do not drop. Avoid shock or violent impact.

This equipment should be stored in a clean, dry and well-ventilated area which meets the following conditions:

Temperature	-10°C to +50°C
Relative Humidity	20% to 90% at 30°C, non-condensing
Atmospheric Pressure	700hPa to 1060hPa

Lectrolite® is a registered trademark of Herculite Products Inc. .

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HONG KONG

ArjoHuntleigh (Hong Kong) Limited
Room 411-414, 4/F, Manhattan Centre,
8 Kwai Cheong Road, Kwai Chung, N.T.,
HONG KONG
Tel: +852 2960 7668
Fax: +852 2960 1711

ITALIA

Arjo Italia S.p.A.
Via Giacomo Peroni 400-402
IT-00131 ROMA
Tel: +39 (0) 6 87426211
Fax: +39 (0) 6 87426222
E-mail: Italy.promo@arjo.com

MIDDLE EAST

Huntleigh Middle East
Office G005 - Nucleotide Complex,
Biotechnology & Research Park,
P.O.Box 214742, Dubai,
United Arab Emirates
Tel: +971 (0)4 447 0942
E-mail: Info.ME@arjo.com

NEDERLAND

Arjo BV
Biezenwei 21
4004 MB TIEL
Postbus 6116
4000 HC TIEL
Tel: +31 (0) 344 64 08 00
Fax: +31 (0) 344 64 08 85
E-mail: info.nl@arjo.com

NEW ZEALAND

Arjo Ltd
34 Vestey Drive
Mount Wellington
NZ-AUCKLAND 1060
Tel: +64 (0) 9 573 5344
Free Call: 0800 000 151
Fax: +64 (0) 9 573 5384
E-mail: nz.info@Arjo.com

NORGE

Arjo Norway AS
Olaf Helset's vei 5
N-0694 OSLO
Tel: +47 22 08 00 50
Faks: +47 22 08 00 51
E-mail: no.kundeservice@arjo.com

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ArjoHuntleigh is a world-leading provider of integrated products and solutions that improve the lives of patients and residents with reduced mobility. We help healthcare facilities deliver wellness and effective everyday care, early mobilisation, safe patient handling, venous thromboembolism prevention, pressure injury prevention, hygiene routines, bariatric care and diagnostics. With extensive knowledge and experience, we strive to improve efficiency and ensure a safer and dignified environment for caregivers and their patients.

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ArjoHuntleigh AB