

Standard hand control

Press and hold for chair position
 Return the mattress platform to a flat and level position
 Raise/lower the backrest
 Flatten the mattress platform and lower the bed
 Power indicator
 Raise/lower the mattress platform
 Raise/lower the thigh section
 Function LED indicator
 Lockout indicator and magnetic sensor area

Temporary unlock

- Hold the magnetic sensor over a magnet for 1 second.
- All LEDs are flashing. Use the functions as needed.
- To exit, hold the magnetic sensor over a magnet for 1 second.

Setting lockout configuration

- Hold the magnetic sensor over a magnet for 5 to 10 seconds until there is a beep.
- Press the function buttons to lock or unlock each function.
 LED on = function locked
 LED off = function unlocked
- To exit, hold the magnetic sensor over magnet for 1 second.