

IMPORTANT!


This Quick Reference Guide (QRG) is intended to assist the user in understanding the basic functions of this product. The QRG does not replace the safety warnings, cautions and other important information provided in the Instructions for Use (IFU) supplied with the equipment; copies of the IFU are available on request from Arjo.

Some of the functions should only be used by clinically-qualified personnel. The carer should make sure they have read and understood the IFU before using this equipment.

A clinically-qualified person should assess the age, height and medical condition of the patient to ensure they can use the product safely.

Place the mattress platform in the low height position when the patient is unattended.


Product Overview

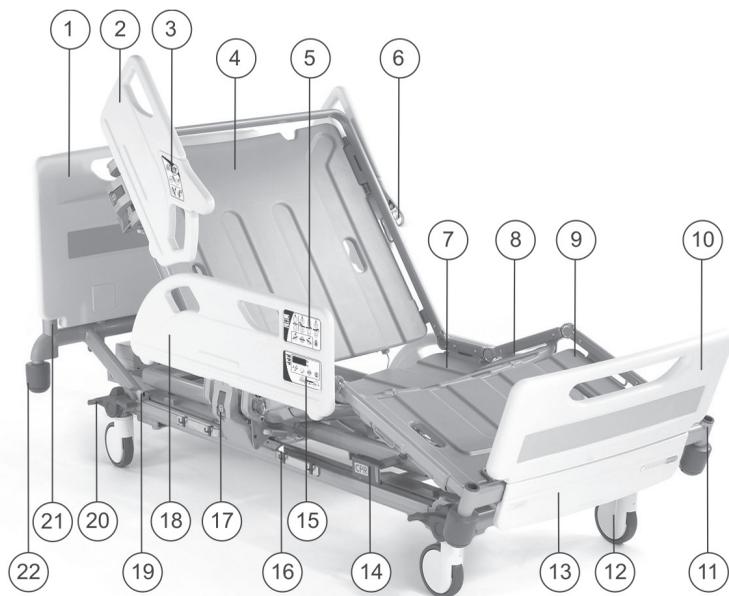


- | | |
|----------------------------------|-----------------------------------|
| 1. Head end panel | 12. Caster |
| 2. Head end side rail | 13. Linen shelf |
| 3. Caregiver controls | 14. CPR release handle |
| 4. Backrest section | 15. Weighing system controls |
| 5. Attendant Control Panel (ACP) | 16. Drainage bag rail |
| 6. Patient controls | 17. Side rail release lever |
| 7. Seat section | 18. Foot end side rail |
| 8. Thigh section | 19. Anti-entrapment sensor |
| 9. Calf section | 20. Brake/steer pedal |
| 10. Foot end panel | 21. Lifting pole/accessory socket |
| 11. Accessory socket | 22. Roller buffer |

Technical Data

Safe working load (SWL):.....250kg (550lbs)
Product weight (without mattress):185kg (407lbs)
Operating temperature:5°C to 40°C (41°F to 104°F)

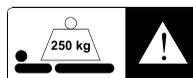

CPR Release

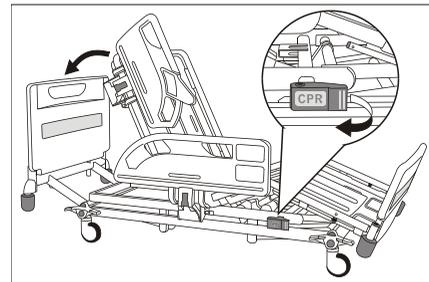

Manual CPR release handles are situated on either side of the bed.

Pull the red CPR release handle to lower the backrest.

Alternatively, press and hold CPR button on the Attendant Control Panel.



The backrest will lower quicker when using the manual CPR release handle.


Brakes and Steering

WARNING

Operate the brake pedals with your feet while wearing suitable shoes. Do not operate the pedals with your hands.

The pedals have three positions as shown below:

- BRAKE: brakes are applied on all four castors.
- FREE: all four castors are free to rotate and swivel.
- STEER: all four castors can rotate, but the steering castor (see below) is locked so that it cannot swivel. This helps to keep the bed on a straight line.

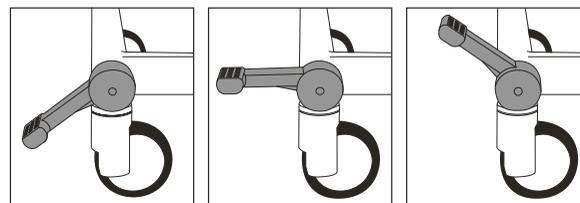

BRAKE

FREE

STEER

Using the Steering Castor

Position the bed so that all the castors line up in the direction of travel. Raise the pedals to lock the steering castor and move the bed by pushing it from the opposite end of the steering castor.



The steering castor may be at either end of the bed, as specified by the customer.

Head End Brake Pedals

Brake pedals can be fitted at the head end of the bed. These operate in the same way as the foot end pedals.

Troubleshooting

If you experience any problems when using this product:

1. Refer to the Instructions for Use (part number 746-579) for further information.
2. Check the troubleshooting guide in the product service manual (part number 746-581).
3. For additional product information contact Arjo.

® and ™ are trademarks belonging to the Arjo group of companies.

© Arjo 2021

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either wholly or in part without the consent of Arjo.



ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö, Sweden
www.arjo.com



815-513EN_04: 07/2021

QUICK REFERENCE GUIDE

Enterprise 9600


One touch controls



Attendant Control Panel (ACP)


Attendant Control Panels are located on the outside panels of both foot end side rails.

1. Power indicator
2. Battery indicator
3. Anti-entrapment indicator
4. Raise / Lower mattress platform
5. Raise / Lower backrest
6. Raise / Lower thigh section
7. Raise / Lower calf section (adjust kneebreak angle)
8. Head down tilt (Trendelenburg)
9. CPR - Hold down the button to return the bed to a flat position. CPR button works even if functions are locked out.
10. Foot down tilt (Reverse Trendelenburg)
11. Lock / unlock function button. Press the lock button then the function(s) you want to lock out. To unlock use the same sequence.

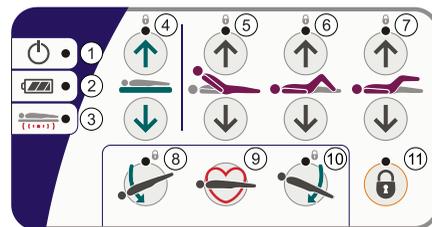

Caregiver Controls


The caregiver controls are located on the outside panel of both head end side rails.

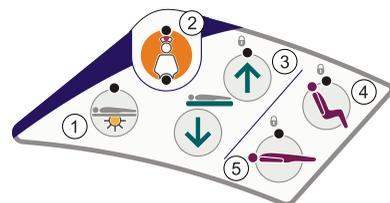

1. Underbed lights on / off
2. Nurse Call
3. Raise / Lower mattress platform
4. Bio-Contour / Auto-Chair
5. Return to flat horizontal position

Patient Controls

The patient control panel is located on the inside panel of both head end side rails.

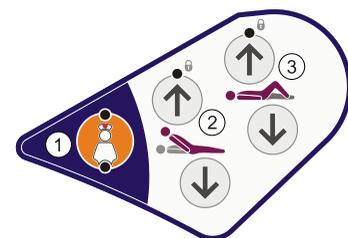

1. Nurse Call
2. Raise / Lower backrest
3. Raise / Lower thigh section

To operate controls

1. Select a function button for example **Raise mattress platform** or **Lower mattress platform**.
 2. The indicator light on the selected function will turn green and the bed will immediately begin to move accordingly.
 3. To lock or unlock bed functions:
 - Press the Function Lock button, then press the Attendant Control Panel button(s) corresponding to the function(s) to be locked or unlocked.
 - The "lock" indicator above the function button will light yellow when the function is locked and go out when it is unlocked.
- Green light indicates function selected. ● Yellow light indicates function is locked out.

Underbed lights

The underbed light illuminates the floor on either side of the bed. Press this button to turn the underbed lights on or off. The indicator above the button will light when the underbed lights are on.


Nurse call

Press the **Nurse Call** button to request assistance. The indicator above the button will light to confirm operation. At the nurse's station, the call location will be displayed and/or an audible alert will sound, depending on the type of nurse call system. Procedures to reset nurse call vary from one system to another.

The nurse call function should be checked periodically for correct operation; as a minimum, Arjo recommends that this should be done before each new patient uses the bed and whenever the nurse call cable has been disconnected. Failure to carry out this check may result in alarms not being indicated at the nurse's station.

Patient Weighing and VariZone Movement Detection

The controls for the weighing system and VariZone patient movement detection are located on the foot end side rails.

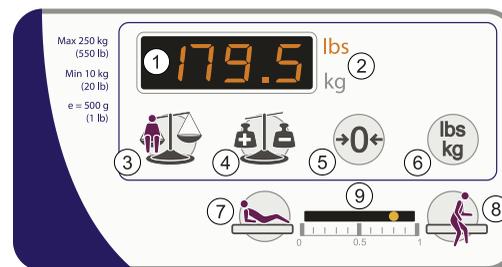

1. Display (shows patient weight and bed section angles.)
2. Pounds/kilograms indicators.
3. Weigh.
4. Auto Compensation.
5. Zero.
6. Select Units.
7. In Bed.
8. Egress.
9. VariZone movement detection threshold indicator.

Patient Weighing System Operating Instructions

Initialization

After connecting the bed to the electrical supply, the weighing system must be initialised as follows: Place the mattress, bed linen and all necessary accessories on the bed. The patient must not be on the bed at this point.

1. Press the **Zero** key once.
2. After a few seconds, the display will read zero.
3. The patient can now get onto the bed.



The zero point can only be set if the load on the bed is less than 77lbs or 35kg above the factory calibrated zero.

Weighing

To determine the patient's weight, press the **Weigh** key once.

The display will show a moving circular pattern until a stable weight reading is obtained.

The patient's weight will be displayed for ten seconds then the display will go blank.

Weighing Units

Weight readings can be viewed in pounds or kilograms by pressing the **Select Units** key.

To lock the **Select Units** key so that the display always show readings in either pounds or kilograms:

Hold down the **Function Lockout** key. Then press and hold the **Select Units** key.

Keep both keys pressed until you hear a "beep". Check that the required indicator is lit (lbs or kg). Normal operation of the **Select Units** key can be restored by repeating this procedure.

In kg mode, weights are shown correct to the nearest 500g.

In lbs mode, weights are shown correct to the nearest 0.5 lb.

Auto Compensation

The Auto Compensation facility allows weight to be added to or removed from the bed, without affecting the indicated patient weight.

With the patient on the bed, press the **Auto Compensation** key once.

The display will show a moving circular pattern until a stable weight reading is obtained.

The display will now indicate **Auto**.

Add or remove accessories, bed linen, etc., as required.

Press the **Auto Compensation** key again.

The display will show a moving circular pattern for a few seconds then revert to showing the patient's weight.

Auto Compensation allows the addition or removal of weights up to 220 lbs or 100kg.

VariZone Patient Movement Detection

Before setting and activating movement detection, make sure the patient has been weighed, all additional accessories have been accounted for using the Auto Compensation function, and the weighing display is blank.



The display will show a moving circular pattern for a few seconds then revert to showing the patient's weight. Press and hold either the **In Bed** key or the **Egress** key for two seconds.



One of the indicators between the keys will flash to show the selected movement detection threshold.



Increasing movement detection threshold.



To increase the threshold, press and hold the **Egress** key. The flashing indicator will move to the right.



To decrease the threshold, press and hold the **In Bed** key. The flashing indicator will move to the left.

When the required threshold is reached, release the key.



After a few seconds the indicator will stop flashing and remain lit to show that movement detection is activated.

If movement detection is triggered, a local alarm will sound and the threshold indicator will flash. If the bed is connected to a suitable nurse call system, the alarm will also be indicated at the nurse's station.



To cancel the patient movement alarm, or to deactivate movement detection without triggering the alarm, press the **In Bed** or **Egress** key once.



All the threshold indicators will go out to show that movement detection is deactivated.



The degree of mattress platform profile adjustment before triggering the alarm depends on the threshold setting.

The Auto Compensation function can be used while movement detection is activated.

WARNING

The movement detection function should be checked periodically for correct operation; as a minimum, Arjo recommends that this should be done before each new patient uses the bed and whenever the nurse call cable has been disconnected. Failure to carry out this check may result in alarms not being indicated at the nurse's station.