



Citadel Bed Frame System




Attendant Control Panel

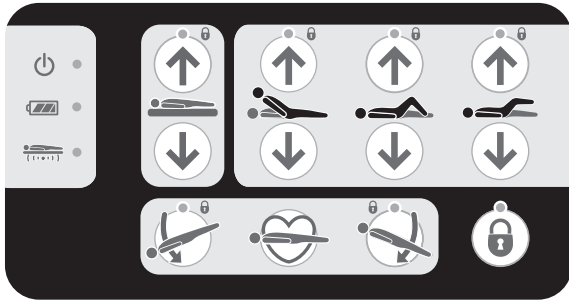


Nurse Control Panel

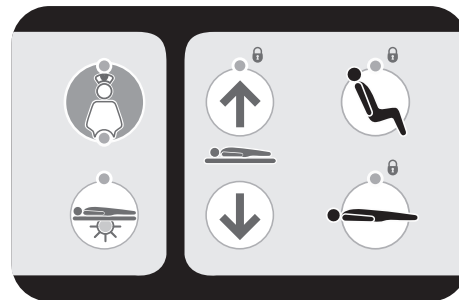



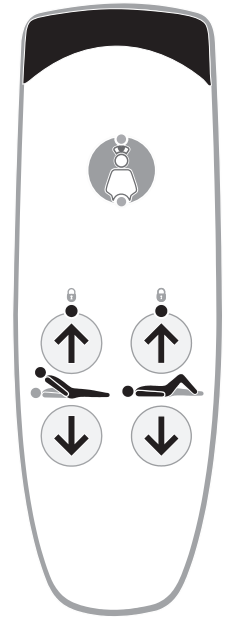



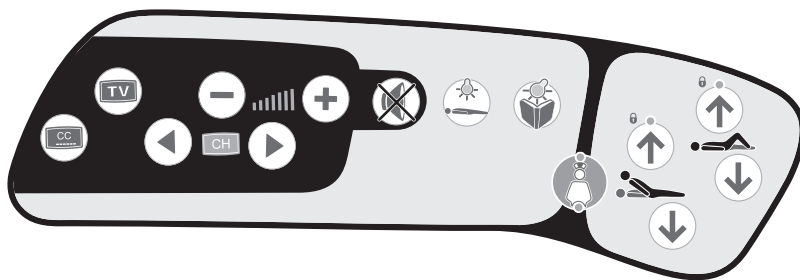







Patient Handset



Patient Control Panel



- 
 - **Power On Indicator**
Green illumination when power on.
- 
 - **Battery Status LED Indicator**
 -  ● fully charged
 -  ● charging
 -  ● discharged
- 
 - **Anti-Entrapment System (AES) Indicator**
Red illumination: AES system has been activated and obstruction needs to be removed before the frame will lower.
- 
 - **Thigh Section** - Press to raise and lower the thigh section.
- 
 - **Calf Section**
Press to raise and lower the calf section.
- 
 - **Trendelenburg**
Press to lower the head end of the deck.
- 
 - **Reverse Trendelenburg**
Press to lower the foot end of the deck.
- 
 - **CPR Position**
Press to flatten the mattress platform.
- 
 - **Deck Height**
Press to raise and lower the deck.
- 
 - **Backrest Angle**
Press to raise and lower the backrest.
- 
 - **Function Lockout**
Press the button, then press selected Attendant Control Panel button(s) to lock/unlock function.
Yellow light = locked



Auto-Chair

Press and hold for chair position (nurse control panel only).



Auto-Chair Down

Press and hold to return to flat level position (nurse control panel only).



Under the Bed Light

(nurse control panel only).



Nurse Call

Press to request assistance (available on select models).



Reading lamp

Press to turn on light (available on select models).



Reading lamp on/off

(available on select models).



Sound on/off (mute)

(available on select models).



Volume down

(available on select models).



Volume up

(available on select models).



Channel down

(available on select models).



Channel up

(available on select models).



Television on/off



(available on select models).



Closed caption on/off

(available on select models).

Weigh Scale Control Panel

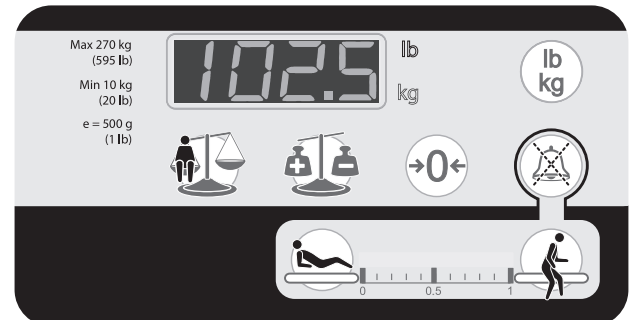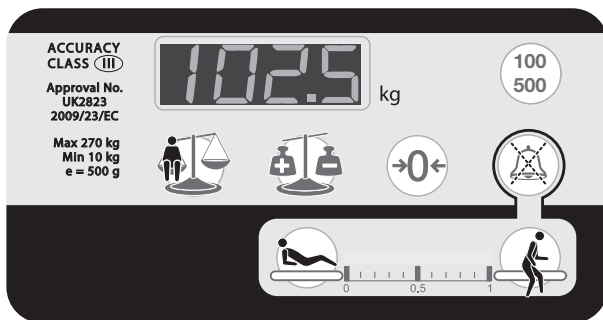

Weigh Scale Panels (dependent on country)



Zero

Press to reset weighing system when bed is first set up and before a new patient is placed on the bed.



Auto Compensation (tare)

Press when adding or removing items from the bed without affecting the indicated patient weight.



Weigh

Press to display weight.



Select Weight Display

Press to round up to nearest 100g or 500g.



Select Units

Press to select weighing unit.

Patient movement detection threshold
An indicator shows the selected sensitivity of patient movement.



In Bed

Press to reactivate/deactivate patient movement detection and increase sensitivity.



Egress

Press to reactivate/deactivate patient movement detection and decrease sensitivity.



VariZone™ Mute Button

Press to mute exit alarm for three minutes.