


# LOW AIR LOSS ALTERNATING MATTRESS REPLACEMENT SYSTEM **ARJO**

## Sapphire 1100EC

The following Product Specification is intended to assist the user in understanding the basic functions of the product. This information does not replace the safety warnings, cautions and other important information provided in the Instructions for Use (IFU) supplied with the equipment; copies of the IFU are available electronically on request from Arjo. Refer to the product instructions for use before using the product.

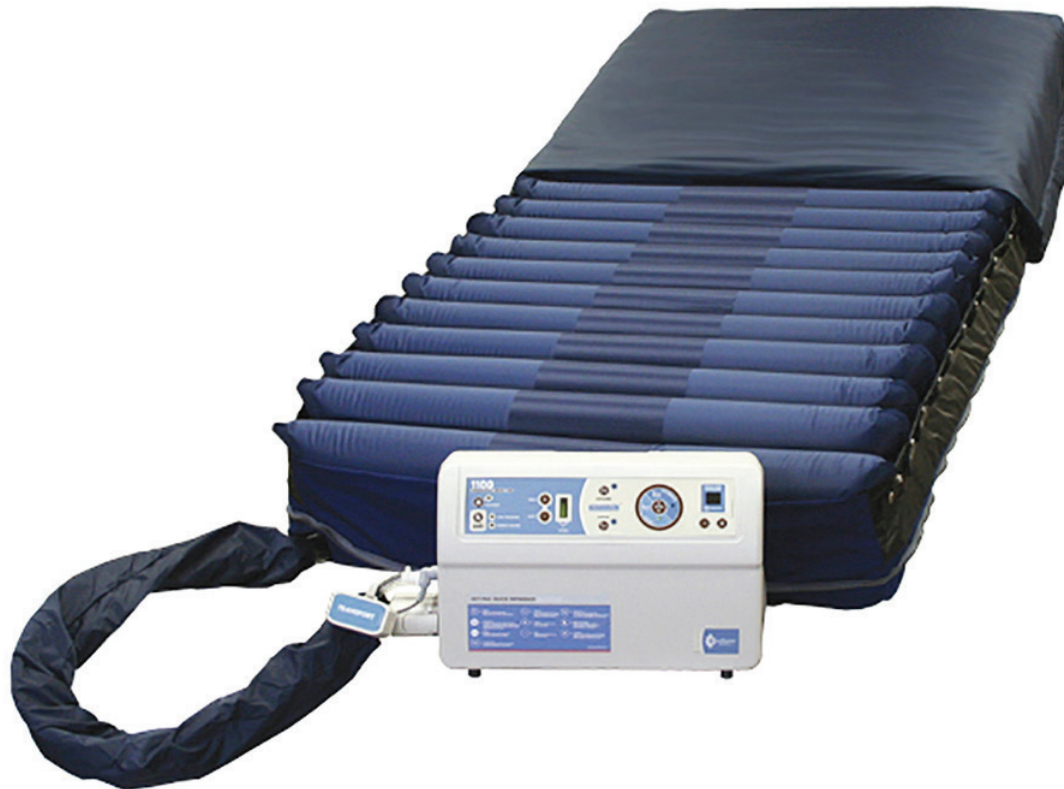

The Sapphire 1100EC is a therapeutic mattress that provides either active or reactive pressure redistribution and low air loss therapy. When mobility, moisture and/or inactivity are healthcare concerns, it is indicated for the prevention and treatment of pressure ulcers and other skin related injuries.

- Static mode with adjustable comfort level setting
- Alternating mode with cycle time options (3-20 minutes) and adjustable comfort level
- Fowler mode which is used when head section of the bed is elevated and increases airflow to the mattress for seat inflation
- Autofirm - quickly inflates mattress to maximum firmness
- Lockout - locks all functions automatically after 3 mins, to disable, press and hold lockout button for 3 seconds
- Quick Deflation or CPR Use
- Low Pressure - light will indicate when pressure is getting too low
- Power Failure - light will indicate when power is no longer being provided to unit
- Alarm Silence - mutes the audible alarm. (Visual light will not turn off until failure is resolved)
- Several width options: 39", 42", 48" and 60"
- Maximum User Weight - 453kg



DISTRIBUTED BY


**ARJO**









MANUFACTURED BY

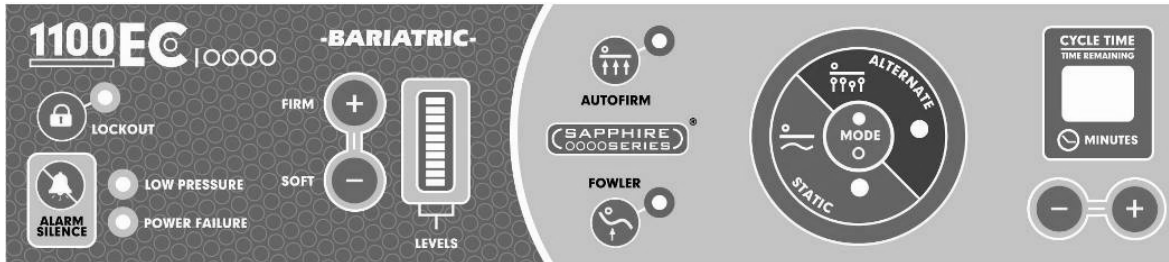

# Sapphire 1100EC

This Quick Reference Guide (QRG) is intended to assist the user in understanding the basic functions of this product. The QRG does not replace the safety warnings, cautions and other important information provided in the Instructions for Use (IFU) supplied with the equipment; copies of the IFU are available on request from Arjo. Refer to the product instructions for use before using this product.

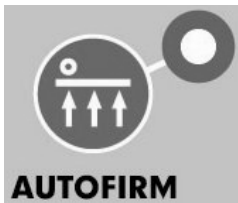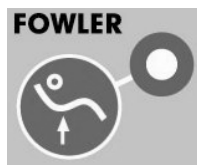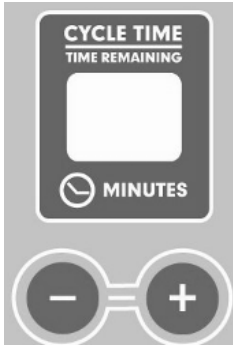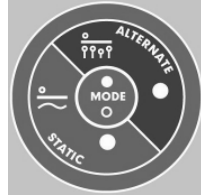

 <b>LOCKOUT</b>	<p>Lockout – Locks all functions automatically after 3 mins. To disable, press and hold lockout button for 3 sec.</p>	 <b>ALARM SILENCE</b>	<p>Alarm Silence – Mutes the audible alarm. (Visual light will not turn off until failure is resolved.)</p>
 <b>FIRM</b>	<p>Firm – Increases airflow for a firmer setting.</p>	 <b>SOFT</b>	<p>Soft – Decreases airflow for a softer setting.</p>
 <b>AUTOFIRM</b>	<p>Autofirm – Quickly inflates mattress to maximum firmness.</p>	 <b>FOWLER</b>	<p>Fowler – Used when head section of bed is elevated. Increases airflow to the mattress.</p>
 <b>CYCLE TIME</b>	<p>Cycle Time – Increases or decrease the time of cycle between 3 to 20 minutes.</p>	 <b>MODE</b>	<p>Mode – Press the MODE button to switch between Alternate and Static modes.</p>

This chart is to be used as a reference ONLY. Final patient settings must be completed by the patient's caregiver.

Reference: Doc. #8085, Rev. 8.0, Sunflower, Date: 10/05/2015